seca 207

Deutsch	Seite 3
Englisch	Page 5
Français	Page 7
Español	Página 9
Italiano	Pagina 11
Dansk	Side 13
Svenska	Sida 15
Norsk	Side 17
Suomi	Sivu 19
Nederlands	Pagina 21
Português	Página 23
Ελληνικά	Σελίδα 25
日本語	27
Polski	Strona 29

Herzlichen Glückwunsch

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Congratulations!

In the seca 207 measuring rod, you have acquired an accurate and at the same time sturdy piece of equipment for measuring the length of babies.

The measuring rod serves both to determine the length at birth of a newborn and to check body length during the growth process. It is made of rigid, easy-care aluminium profile. The plastic measuring slides can be folded, so the measuring rod does not take up much space in storage.

If you want to attach it to the wall, use the screws and spacers supplied.

The measuring rod can thus be attached horizontally over the changing table, for example, and be used as a stationary measuring device. The seca 207 measuring rod is used in hospitals and doctors' surgeries.

Safety

Before using the new measuring rod, please take a little time to read the following safety instructions:

- Follow the operating instructions.
- Never leave the baby lying unsupervised with the measuring rod, otherwise there is a risk of injury.
- Only measure length on surfaces from which the baby cannot fall.
- Ensure that the measuring rod and scale are cleaned and disinfected regularly to prevent infectious diseases being transmitted.

- A second person is ideally required to enable length to be measured exactly.

Operation

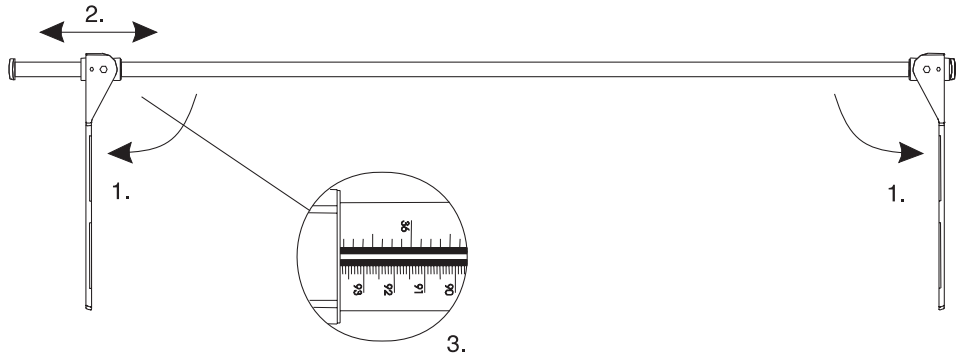
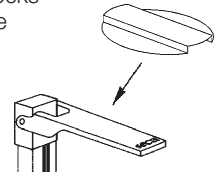
Measurement process:

1. Place the baby on a flat surface. Position the measuring rod with the measuring slide (1) at the baby's head.
2. Using your right hand, slide the measuring slide (2) in the direction of the baby's feet. The left hand holds the baby steady.
3. Stretch the baby's legs, using the left hand to push the baby's knees carefully and gently down onto the surface.
4. Use your right hand to place the measuring slide (2) at right angles to the soles of the baby's feet.

5. Read off body length on the marking on the inside of the measuring rod (3).
If the above-mentioned instructions are followed, accuracy greater than ± 10 mm can be achieved.

Attaching the measuring tongue extension...

- There is provision for enlarging the head support surface.
- Press the measuring tongue extension onto the measuring tongue from above so that it locks audibly in position. The extension can then be moved on the measuring tongue with little effort.

Cleaning

Clean the measuring rod as required using a domestic cleaning agent or commercially-available disinfectant. Follow the manufacturer's instructions.

Technical data

Measurement range: 0-990 mm /
0-39 inches
Graduations: 1 mm / 1/8 inch
Accuracy: greater than ± 10 mm
Dimensions (W x H x D): 55 mm x 1050 mm x
34 mm
Weight: approx. 700 g
Temperature range: +10 °C to +40 °C
Medical product in
accordance with
Directive 93/42/EEC: Class I with measuring
function

Disposal

Should the measuring rod no longer be able to be used, the responsible waste disposal association will be glad to give you information about the measures necessary to dispose of it properly.

Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts – batteries, cables, mains units, rechargeable batteries etc. – are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the merchandise secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

Declaration of conformity

Mechanical equipment for measuring length

seca model 207

Batch no.: see back of measuring rod

The measuring rod meets the applicable requirements of Directive 93/42/EEC on medical products.

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