

# seca mBCA 555/seca TRU


Tenant connection

Connect your  
tenant in a  
few steps



# seca mBCA 555/seca TRU

## Tenant connection

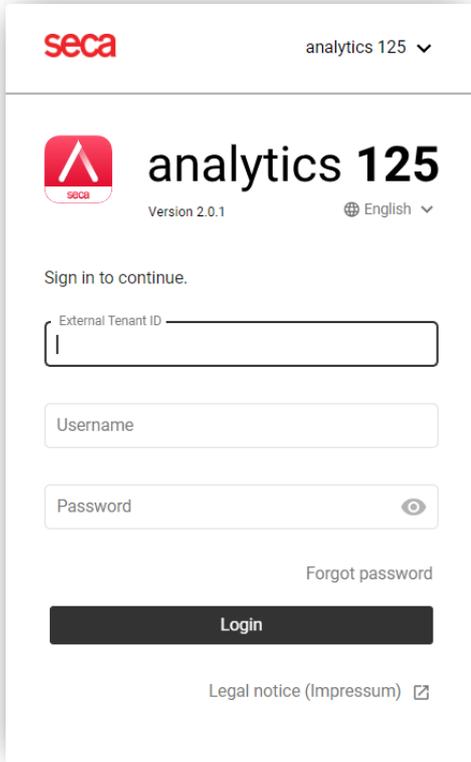

The screenshot shows the login interface for 'analytics 125'. At the top left is the 'seca' logo, and at the top right is 'analytics 125' with a dropdown arrow. Below this is the 'analytics 125' logo and 'Version 2.0.1' on the left, and a language selector 'English' with a dropdown arrow on the right. The main text says 'Sign in to continue.' Below this are three input fields: 'External Tenant ID' (with a small '1' in the first position), 'Username', and 'Password' (with an eye icon for visibility). A 'Forgot password' link is located below the password field. A black 'Login' button is at the bottom of the form. At the very bottom, there is a link for 'Legal notice (Impressum)' with an external link icon.

1. Use a browser (e.g. Chrome) on your computer or mobile device to open the following website: <https://secacloud.com>
2. Log-in with your log-in details.

## seca analytics 125

 Home

 Customers

 Measurements

 Insights

### ADMINISTRATION

 Users

 Devices


 Data import/export

 Settings

3. Open the menu at the top left. Then select "Devices"

# seca mBCA 555/seca TRU

4. No devices have been added to the device list yet. To add your device, click on “Add new device” button at the top right.

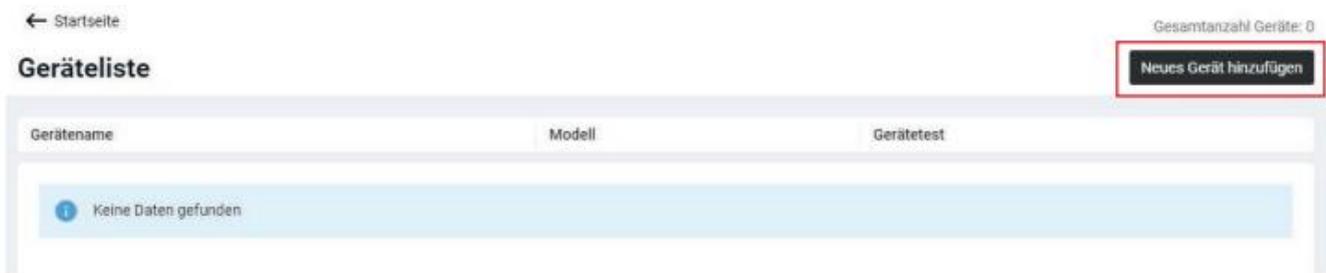

5. Enter the serial number of your device in the following mask and click the “Add” button.  
You can find the serial number of your device on a label at the bottom left of the device platform.

### Neues Gerät hinzufügen

Seriennummer:

6. Now the device will be automatically added to your personal database (tenant).  
A certificate will be installed in the background. Wait until the message “Certificate installed successfully” is displayed beneath the added device.
7. Your device is now connected to your tenant and ready to use.