

seca mBCA Alpha (545)

seca TRU Alpha (542)

Tenant-connection

Connect your
tenant in a few
steps



seca mBCA Alpha (545) seca TRU Alpha (542)

seca analytics 125

Log in to your business account

Tenant ID

User name

Password

[Forgot password?](#)


Login

[SSO login](#)

Don't have a business account? [Sign up](#)

1. Open a browser (e.g., Chrome) on a device and open the following website:
<https://eu.secacloud.com/guest/login?accountType=analytics125>

2. Log in with your access data or create your own tenant using the “Register” button.

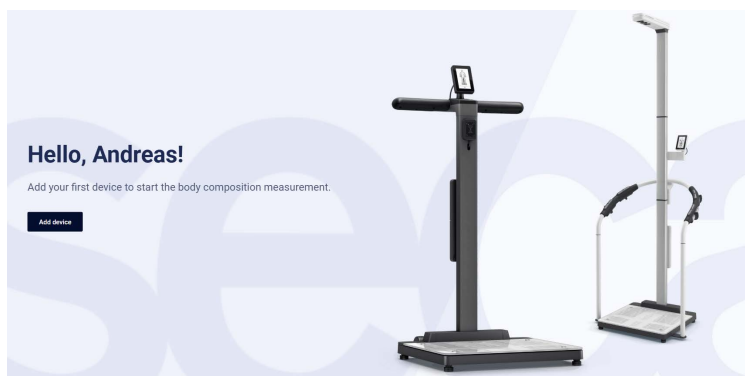

3. After logging in for the first time, this start page will appear if no device has been added yet. Click on “Add device.”

Add device [X]

Serial number *

Registration key *

Please enter address data

Street address *

City

State / Province / Region

ZIP / Postal code

Country *

Please enter contact details

First name *

Last name *

Email *

Phone number *


[Cancel](#) **Add**

4. Please fill in all mandatory fields marked with an asterisk (*). The serial number can be found on the type plate at the bottom left of the platform. You will have received the registration key in advance by email.

seca mBCA Alpha (545)


seca TRU Alpha (542)

seca analytics 125

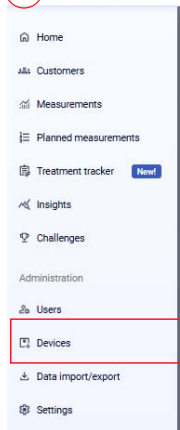

- If you already have a device in your cloud, you can add more devices. To do this, open the menu in the top left corner and select “Devices.”

- To add your device, click “Add New Device” in the upper right corner.

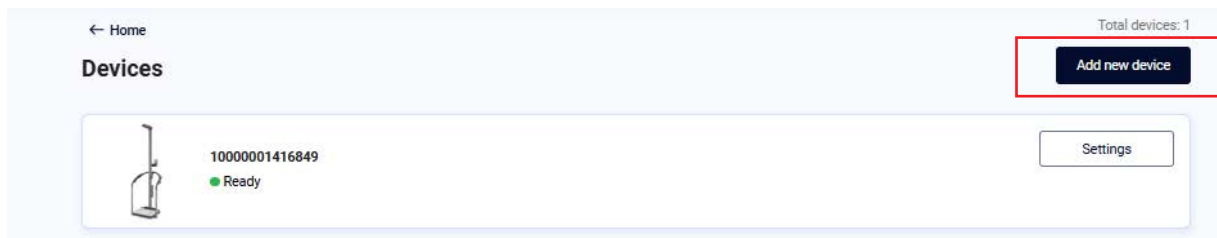

Add device x

Serial number *

Registration key *

Please enter address data

Street address *

City

State / Province / Region

ZIP / Postal code

Country *

Please enter contact details

First name *

Last name *

Email *

Phone number *

Cancel **Add**

- Please enter all mandatory information marked with an asterisk (*) in the following form and then click on “Add.”
The serial number of your device can be found on the type plate at the bottom left of the platform. You will have received the registration key in advance by email.

- The device will now be automatically added to your personal database (tenant). A certificate will be installed in the background. The device will then be ready for use.

