


Assembly instructions seca 406


Overlay ramps for seca 604

(standard ramps seca 404 required)


1. INTENDED USE

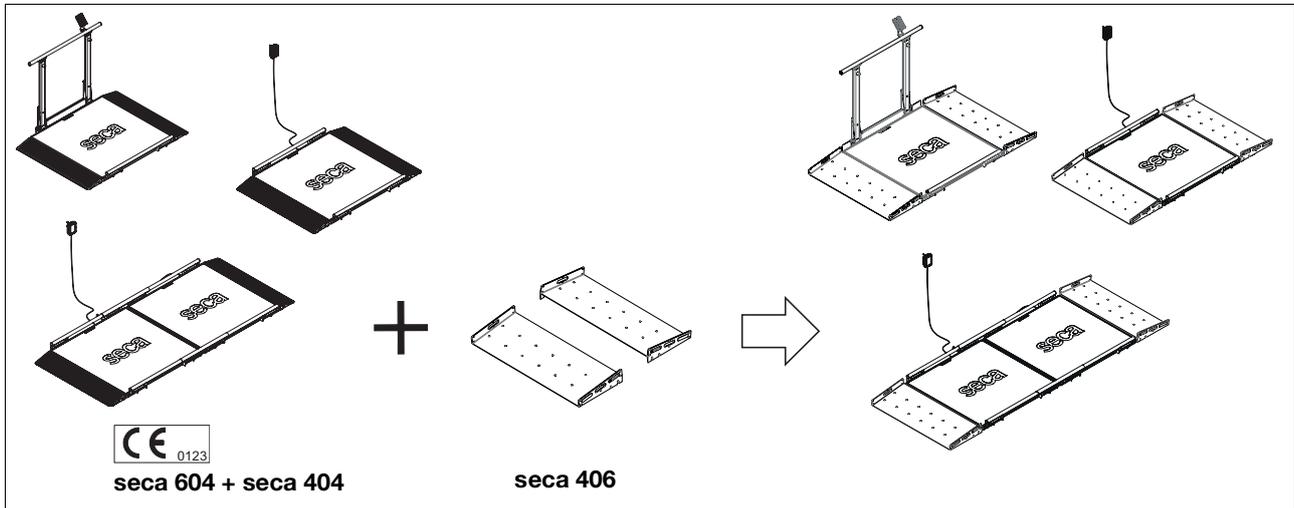


2. SAFETY PRECAUTIONS

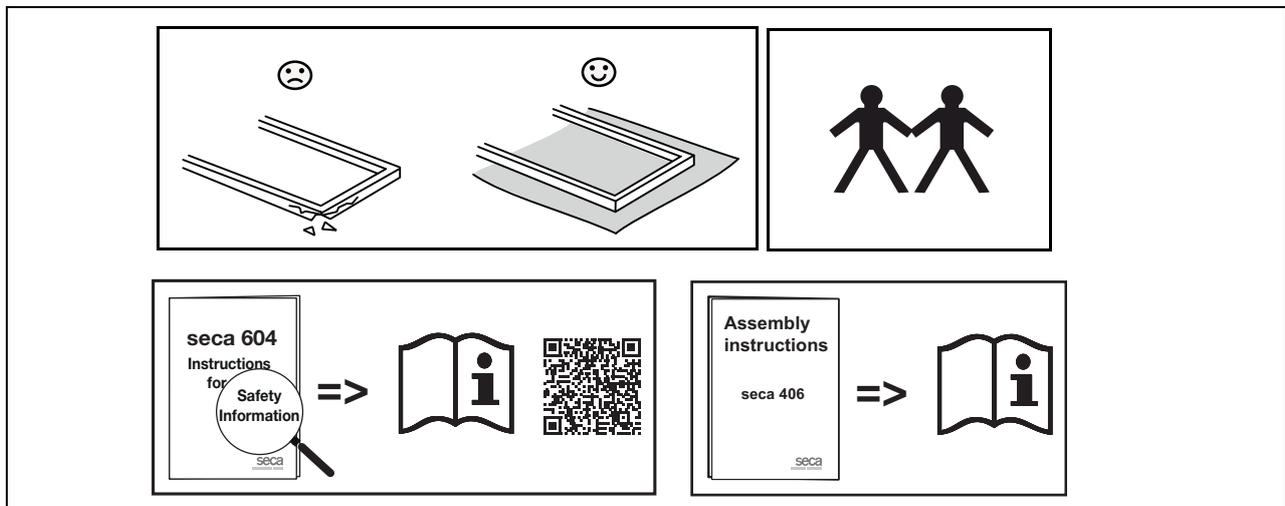


3. SCOPE OF DELIVERY

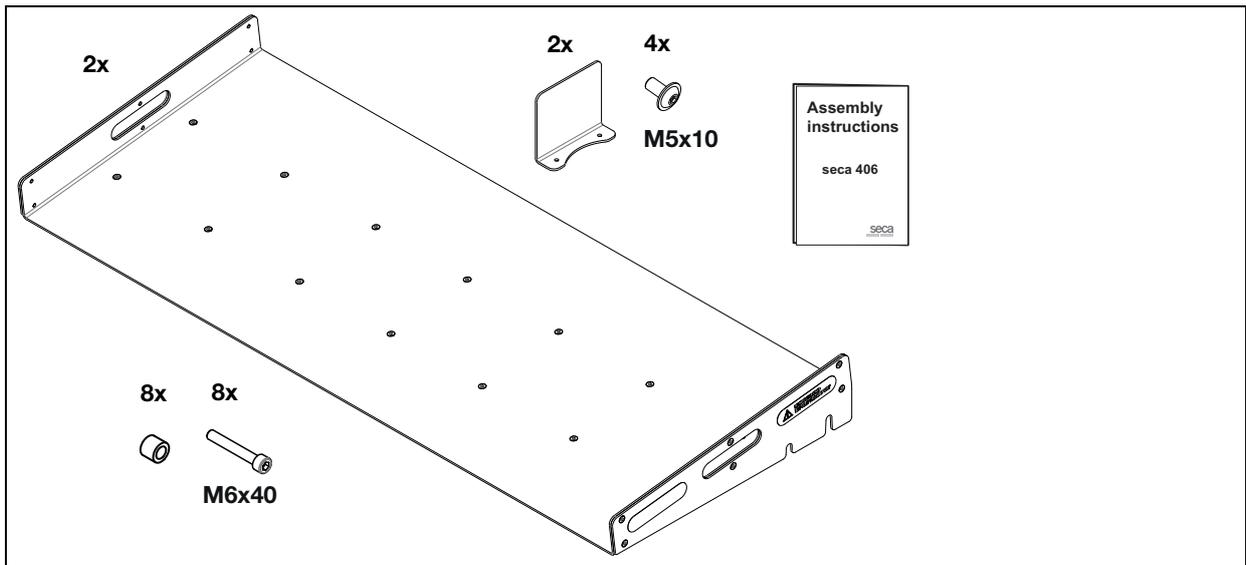

4. MOUNTING GUIDING SCREWS FOR OVERLAY RAMPS SECA 406

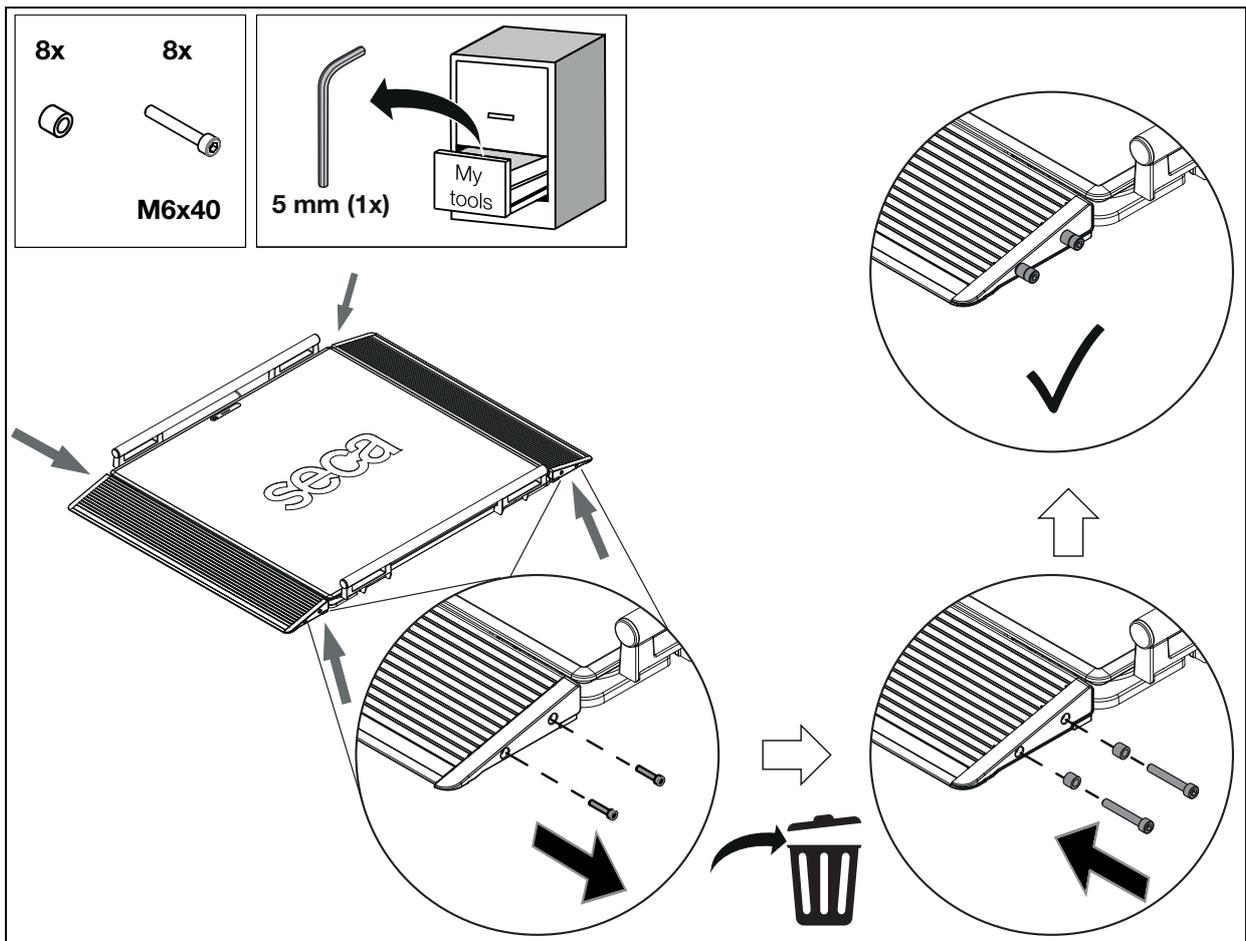


5. MOUNTING GAP SHEET (ONLY FOR DEVICES WITH HANDRAIL SECA 446)

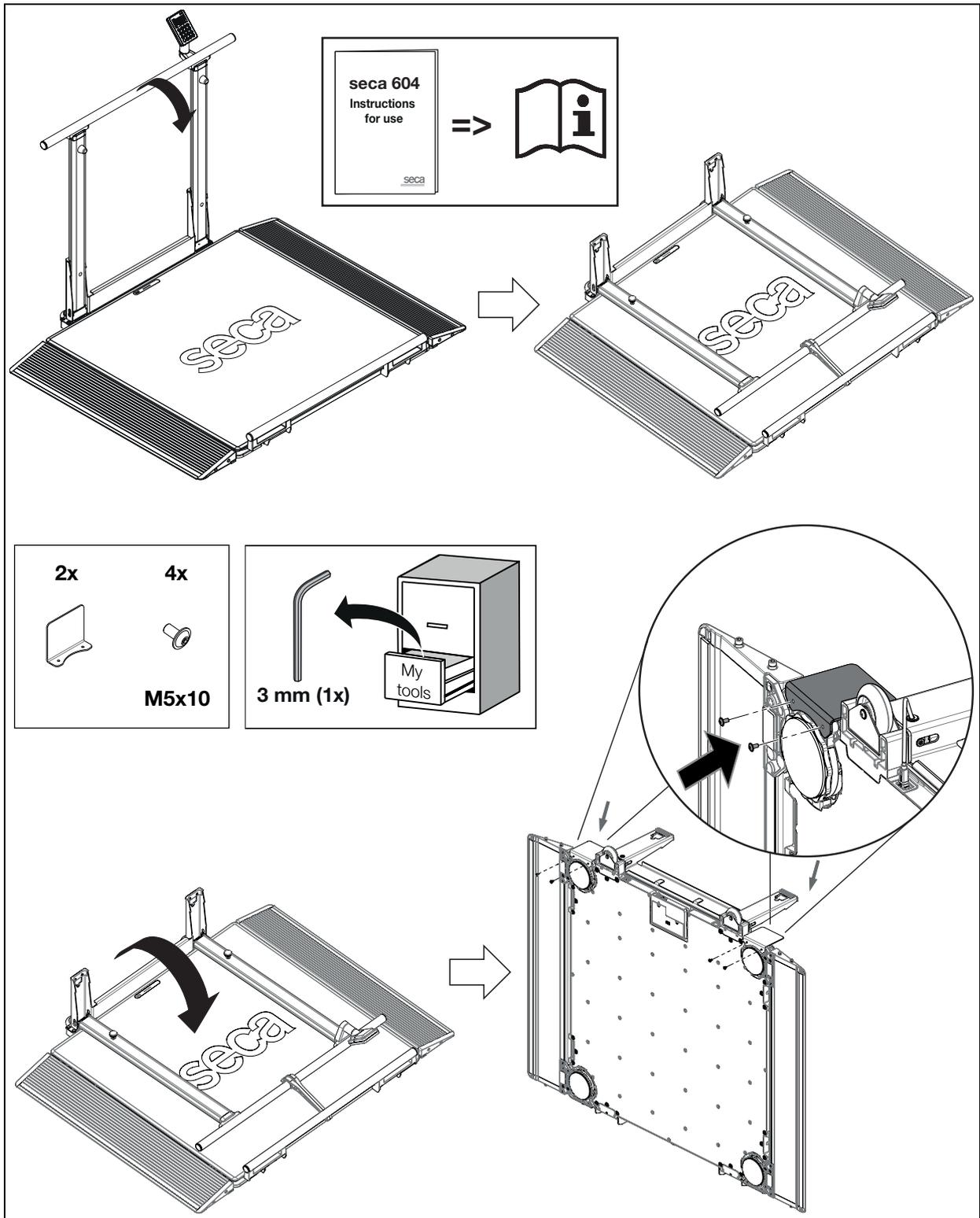

6. PLACE OVERLAY RAMPS SECA 406 ON GUIDING SCREWS

NOTE


- ▶ Place the device at the desired setup location first. After that, position the ramps seca 406 on the device.

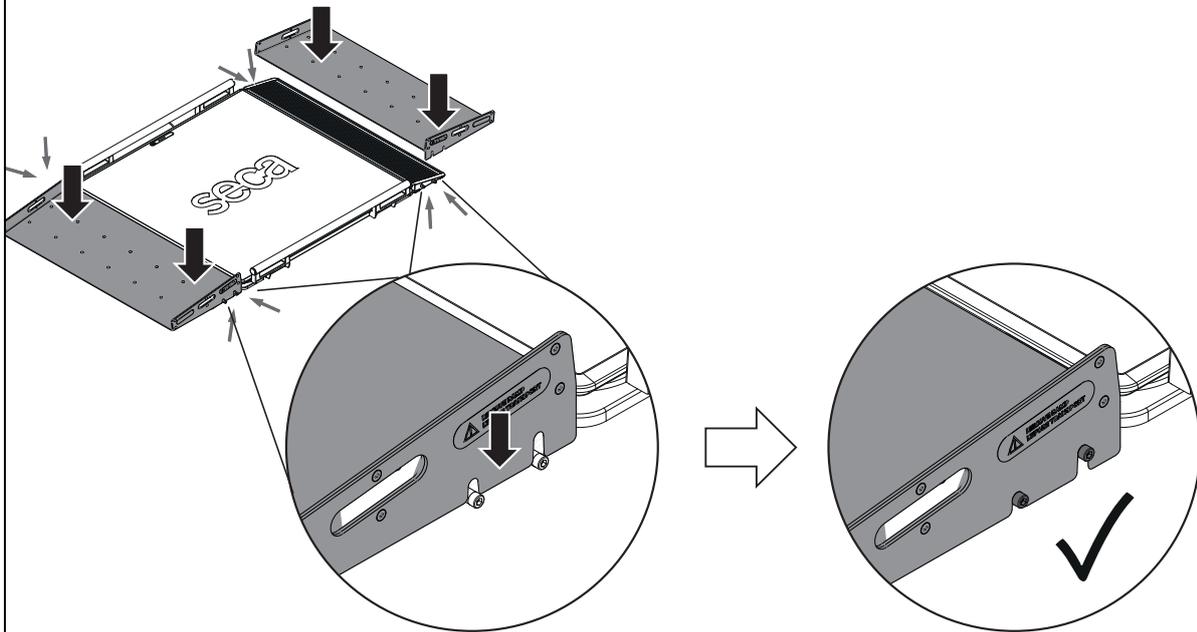

WARNING!

Injury from falling

- ▶ Make sure to position the ramps seca 406 securely to the guiding screws.

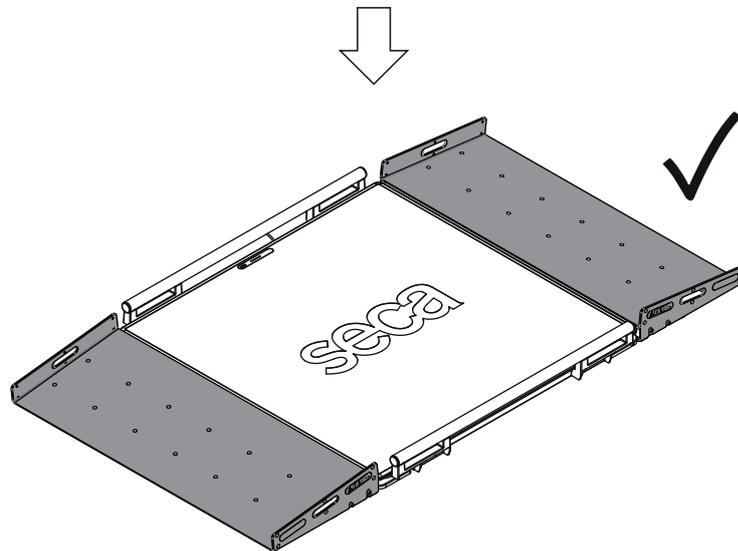


7. PRIOR TO TRANSPORT: REMOVE OVERLAY RAMPS SECA 406

CAUTION!

Injury from moving parts, damage to device

- ▶ Before lifting the scale for transport, remove the ramps seca 406.

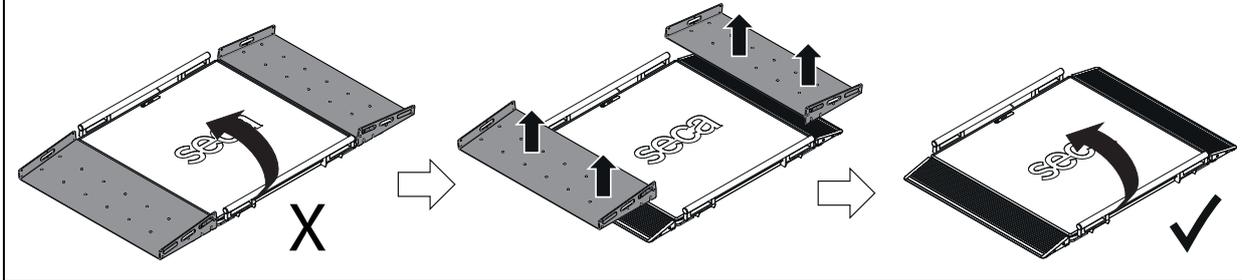

8. TECHNICAL DATA

Technical data	
Dimensions	
• Length:	1106 mm (43.54 in)
• Width:	441 mm (17.36 in)
• Height:	112 mm (4.41 in)
Weight:	16,3 kg (35.27 lbs)

Medical Measuring Systems and Scales since 1840

seca gmbh & co. kg
Hammer Steindamm 3–25
22089 Hamburg · Germany
T +49 40 20 00 00 0
F +49 40 20 00 00 50
E info@seca.com

seca operates worldwide with headquarters
in Germany and branches in:

[seca france](#)
[seca united kingdom](#)
[seca north america](#)
[seca schweiz](#)
[seca zhong guo](#)
[seca nihon](#)
[seca mexico](#)
[seca austria](#)
[seca polska](#)
[seca middle east](#)
[seca suomi](#)
[seca américa latina](#)
[seca asia pacific](#)
[seca danmark](#)
[seca benelux](#)
[seca lietuva](#)

and with exclusive partners in
more than 110 countries.

All contact data at [seca.com](https://www.seca.com)