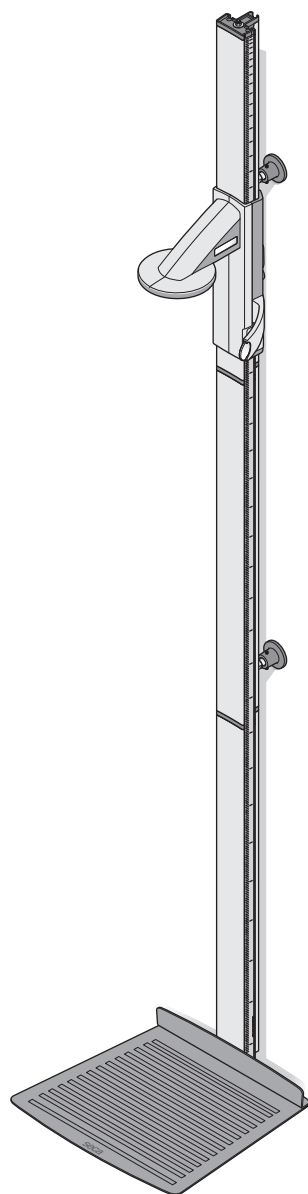
seca 264

Instructions for use

17-10-02-299-100b1_2025-02S

Deutsch – Gebrauchsanweisung	3
English – Instructions for use	29
Français – Mode d’emploi.....	54
Italiano – Istruzioni per l’uso	80
Español – Instrucciones de uso	106
Português – Manual de instruções de utilização	132
Polski – Instrukcja obsługi.....	159

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TABLE OF CONTENTS

1 About this document	30	6 Operation.....	46
1.1 Representation in text	30	6.1 Measuring height.....	46
1.2 Representation in diagrams	30	Switching on head slide	46
2 Description of device	30	Starting height measurement	46
2.1 Intended use	30	Displaying measuring result permanently (Hold)	47
2.2 Contraindications	30	Performing relative measurements (ZERO)	47
2.3 Clinical benefit	31	Switching off head slide	47
2.4 Patient target group.....	31	6.2 Other functions (menu)	48
2.5 User qualification.....	31	Setting display backlighting (LCd).....	48
Administration/network operation	31	Activating beeps (bEEP)	48
Operation.....	31	Switching unit of height (Unit)	48
Assembly	31	Restoring factory settings (rESet)	49
2.6 Measuring height	31	7 Hygiene treatment	49
3 Safety information	31	7.1 Cleaning	49
3.1 Safety information in these instructions for use.....	31	7.2 Disinfecting.....	49
3.2 Basic safety information	32	7.3 Sterilizing.....	50
Handling device	32	8 Function check	50
Preventing injuries and infections.....	33	9 Troubleshooting.....	50
Preventing damage to device.....	33	9.1 General faults	50
Handling measuring results	34	9.2 Changing the batteries	51
Handling packaging material	34	10 Servicing	51
Handling batteries and rechargeable batteries.....	34	11 Technical data.....	52
4 Overview	35	11.1 General technical data	52
4.1 Controls	35	11.2 Height measurement	52
4.2 Symbols in the display	36	12 Optional accessories and spare parts.....	53
4.3 Markings.....	36	13 Disposal	53
4.4 Menu structure	38	13.1 Disposing of the device	53
4.5 Navigating in the menu.....	38	13.2 Disposing of batteries and rechargeable batteries.....	53
5 Starting up device.....	39	14 Warranty	53
5.1 Scope of delivery.....	39	15 Declaration of conformity	53
5.2 Assembling device	40		
Introduction	40		
Fitting second column element	40		
Fitting Frankfurt measure	41		
Mounting the head slide	41		
Fitting third column element.....	41		
Fitting push-in scale	42		
5.3 Mounting device on a wall.....	42		
Selecting installation location.....	42		
Installing wall brackets	43		
Installing the slot bolts.....	43		
Positioning the device in the wall brackets.....	44		
Installing the foot mat.....	44		
5.4 Inserting batteries.....	44		
5.5 Calibrating height measurement	45		
Calibrating automatically.....	45		
Calibrating manually	45		
5.6 Disinfecting device	46		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

1.2 Representation in diagrams

Symbol	Description
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays
	• Correct action • Correct result of action
	• Incorrect action • Incorrect result of action
	End of a procedure, e.g. the installation of a part

2 DESCRIPTION OF DEVICE

2.1 Intended use

The electronic stadiometer supports physicians in decision-making regarding diagnosis or therapy, based on the patient's height or growth.

2.2 Contraindications

No contraindications are known.

2.3 Clinical benefit

The electronic stadiometer supports physicians in decision-making regarding diagnosis or therapy, based on the measured height.

By that, the electronic stadiometer only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

2.4 Patient target group

The device is intended for persons of any gender and age, with the exception of babies. Restrictions exist in relation to the height (refer to information in the "Technical data" section).

2.5 User qualification

Administration/network operation	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.
Operation	The device may only be operated by persons with formal training in the healthcare sector or in medicine.
Assembly	Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).

2.6 Measuring height

Height is recorded by scanning a digital scale.

3 SAFETY INFORMATION

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

CAUTION!

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

DANGER!

Risk of explosion

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
 - oxygen
 - flammable anesthetics
 - other flammable substances/air mixtures

CAUTION!

Patient hazard, damage to device

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

CAUTION!

Patient hazard, damage to device

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

CAUTION!

Patient hazard, malfunction

- ▶ Keep other medical electrical devices, e.g. high-frequency surgical devices, a minimum distance of approx. 1 meter away to prevent faulty measurements or wireless transmission interference.
- ▶ Keep HF devices such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements or wireless transmission interference.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to www.seca.com.

Preventing injuries and infections

WARNING! **Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited mobility, e.g. when they are getting up from a wheelchair.
- ▶ Ensure that the patient does not step straight onto or off the edges of the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

WARNING! **Risk of slipping**

- ▶ Ensure that the patient standing area is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the patient standing area.
- ▶ Ensure that the patient steps onto and off the patient standing area slowly and safely.

WARNING! **Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Subject the device to a hygiene treatment at regular intervals as described in the relevant section of these instructions for use.
- ▶ Ensure that the patient does not have any infectious diseases.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.

Preventing damage to device

NOTICE! **Damage to device**

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ If you are not going to use the device for an extended period of time, remove the batteries or battery packs. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants which match the details in the section entitled "Hygiene treatment".
- ▶ For scales: Ensure that maximum capacity is not exceeded.

Handling measuring results

CAUTION! **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

NOTICE! **Inconsistent measuring results**

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

Handling packaging material

WARNING! **Risk of suffocation**

Packaging materials made of plastic film (bags) present a risk of suffocation.

- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

Handling batteries and rechargeable batteries

WARNING! **Injury**

Batteries and rechargeable batteries contain harmful substances which may explode and so escape if not handled properly.

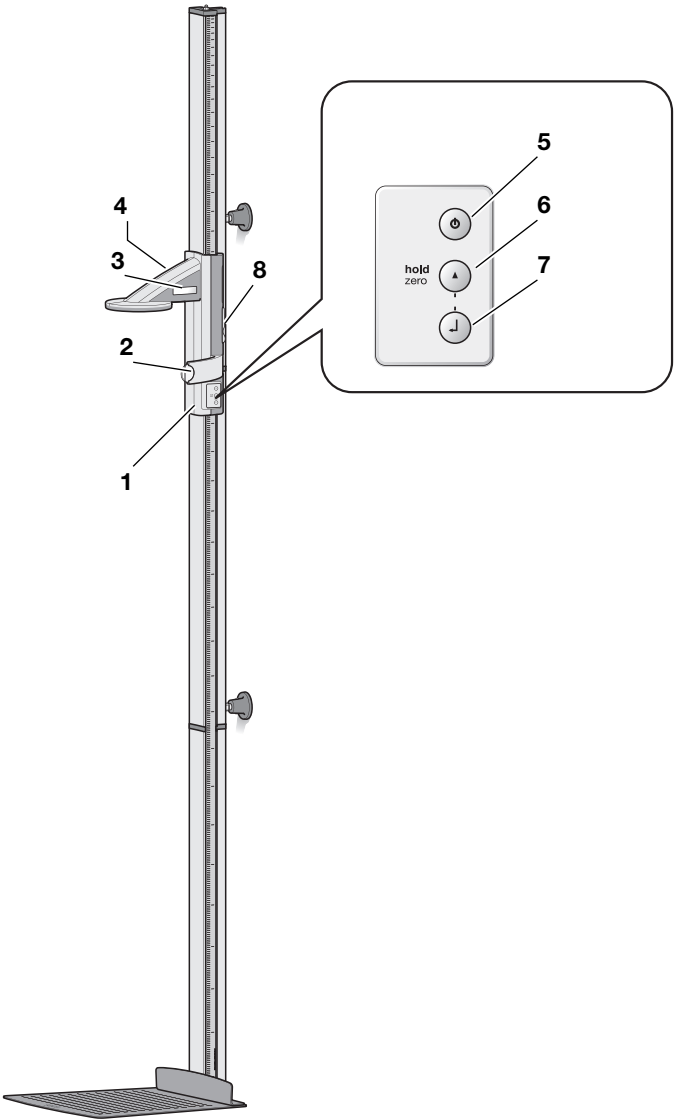
- ▶ Do not try to recharge batteries.
- ▶ Do not expose batteries/rechargeable batteries to heat.
- ▶ Do not burn batteries/rechargeable batteries.
- ▶ If harmful substances are leaking out, avoid contact with the skin, eyes, and mucous membranes. Rinse affected areas of the body with plenty of clean water and seek medical help at once.

NOTICE! **Damage to device and malfunction due to incorrect handling**

- ▶ Use only the type of battery/rechargeable battery quoted in this document.
- ▶ Always replace all batteries/rechargeable batteries at the same time.
- ▶ Do not short-circuit batteries/rechargeable batteries.
- ▶ If you do not use the device for an extended period of time, remove the batteries/rechargeable batteries. This prevents acid from leaking into the device.
- ▶ If harmful substances have penetrated the device, do not continue using it. Have the device checked by an authorized seca service partner and repaired if necessary.

4 OVERVIEW

4.1 Controls


Item	Control	Function
1	Head slide	For determining height
2	Frankfurt measure	Pull-out measure for aligning the head in accordance with the Frankfurt horizontal plane.
3	Display	Display element for measuring results and for configuration
4	Battery compartment	Takes battery holder with 4 AA-type batteries, 1.5 V
5		Switch the head slide on or off

17-10-02-299-100b1_2025-02S

Item	Control	Function
6		- During measuring: - Press briefly: Activate Hold function - Press and hold: Set zero point - In the menu: - Select submenu, select menu item - Set value (press briefly: Value is changed by 1 increment, press and hold: Value is changed until key is released)
7		- Confirm selected menu item - Save set value
8	Brake button	- Holds the head slide in position - Press: Move head slide

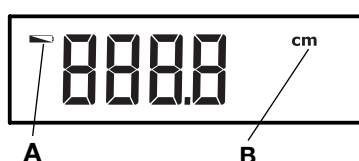4.2 Symbols in the display

Item	Symbol	Meaning
A		Batteries are low
B	cm	Unit in which height is displayed

4.3 Markings


Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identification (product identification number as per Directive (EU) 2017/745)
	Article number
	Serial number
ProdID	seca product identification number
	Follow instructions for use
	Medical electrical equipment, type B
IP20	Type of protection in line with IEC 60529: - Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm - Protection against access with fingers - No protection from ingress of water

17-10-02-299-100b1_2025-02S

Markings on the device and on the type plate	
Symbol	Meaning
	Device complies with EU directives 0123 : Notified Body for Medical Devices
	Medical device in accordance with Regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	Device complies with United Kingdom directives XXXX : Notified Body for Medical Devices of the United Kingdom
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönmatt Str. 2 CH-4153 REINACH
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

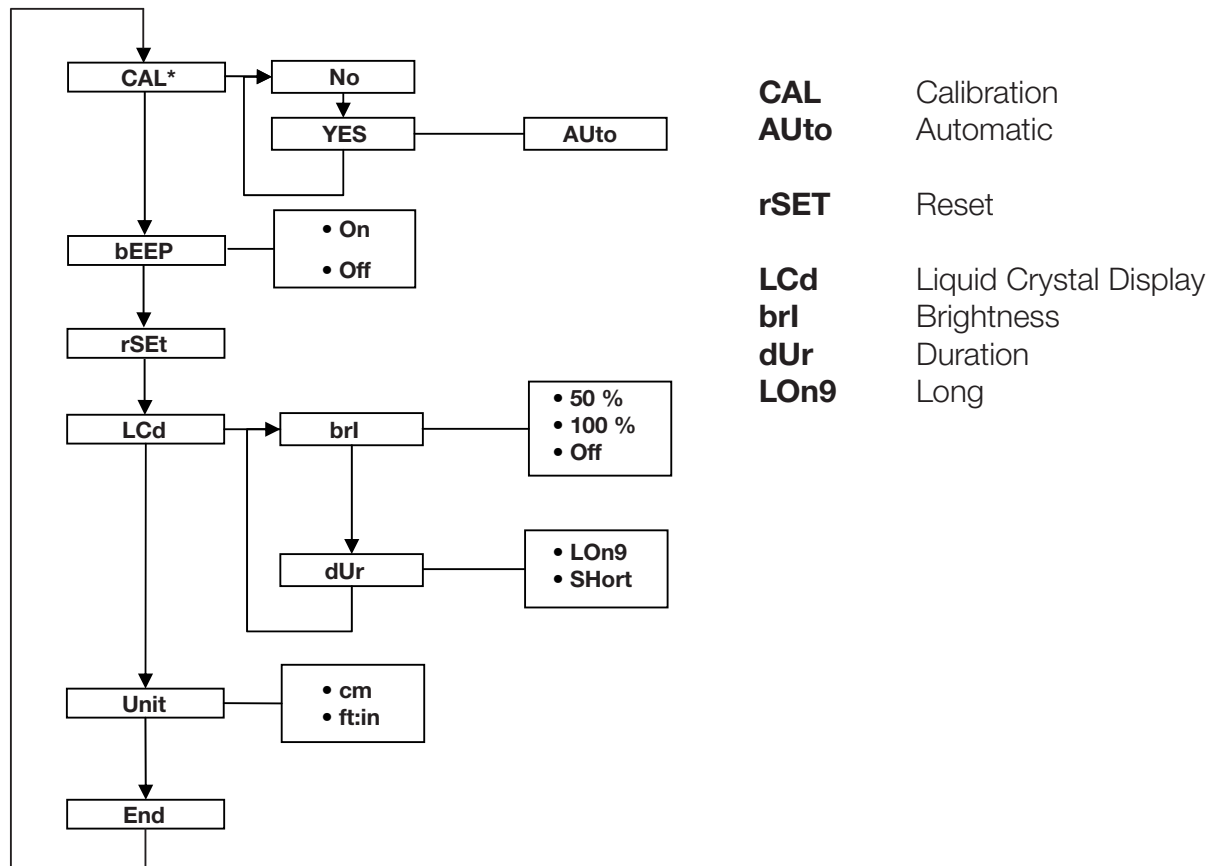4.4 Menu structure

* If measured value is shown in the display

4.5 Navigating in the menu

- Press the key of the head slide.
⇒ ---- is shown in the display.
- Press and hold the and keys simultaneously.
⇒ The last menu item selected is displayed (in this case: **bEEP**).
- Press the key until the desired menu item is displayed (in this case: **LCd** display settings).
- Confirm the selection by pressing the key.
⇒ The current setting for the menu item or a submenu are displayed (in this case: Duration of display backlighting **dUr**).
- To amend the setting or call up a different submenu, press the key until the desired setting is displayed (in this case: Display brightness **brl**).
- Confirm the selection by pressing the key.
⇒ The current setting for the menu item or a submenu are displayed (in this case: Display brightness **50%**).
- To amend the setting or call up a different submenu, press the key until the desired setting is displayed (in this case: Display brightness **100 %**).

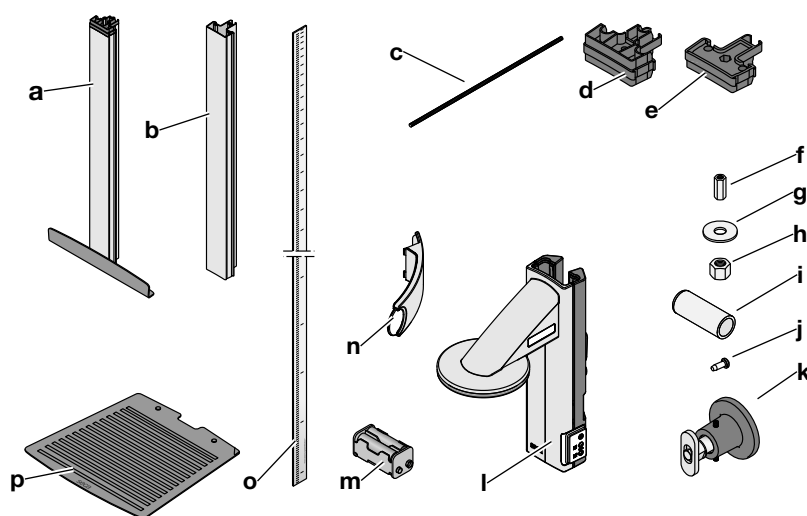
8. Confirm the setting by pressing the  key.
- ⇒ The setting is adopted.
 - ⇒ The menu closes.

NOTE

If no key is pressed for about 30 seconds, you exit the menu automatically.

5 STARTING UP DEVICE

5.1 Scope of delivery


Item	Component	Pcs.
a	First column element with heel stop	1
b	Column element	2
c	Threaded rod	2
d	Column connector	1
e	End cap	1
f	Coupling nut	1
g	Plain washer	1
h	Nut	1
i	Decorative cap	1
j	Cross-head screw	2
k	Wall installation set: • Wall brackets • Slot bolts • M6x60 screws • Standard rawl plugs	2
l	Head slide	1
m	Battery holder with 4 batteries (AA-type)	1
n	Frankfurt measure	1
o	Push-in scale	1
p	Foot mat	1

Item	Component	Pcs.
–	Calibration rod	1
–	Instructions for use	1
–	Cover for Frankfurt measure	1
–	Tool kit: • Hex socket wrench, 2 mm • Open-ended spanner, 10 mm • Open-ended spanner, 13 mm	1


5.2 Assembling device

Introduction The heel stop is fitted as standard to the first column element on delivery.

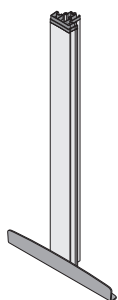- Use an assistant to perform the assembly.

Fitting second column element

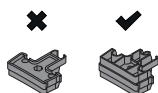NOTICE!

Damage to device as a result of assembly errors

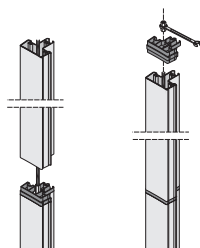
If the column connector and the end cap are switched, it will not be possible to assemble the device so that it is fit for purpose.

- Ensure that you use the column connector when assembling the second column element.


1. Screw a threaded rod (approx. 3 Nm) into the coupling nut of the first column element.


2. Push a column element over the threaded rod onto the first column element.
3. Place the column connector on the second column element so that the threaded rod protrudes out of the bore in the column connector.
4. Screw a coupling nut onto the threaded rod.
5. Tighten the coupling nut.

Fitting Frankfurt measure

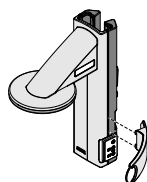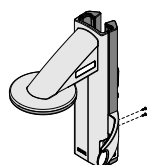
1. Hold the Frankfurt measure up against the head slide as shown in the illustration.

2. Insert the two cross-head screws in the bores.
3. Tighten the cross-head screws (2 Nm).

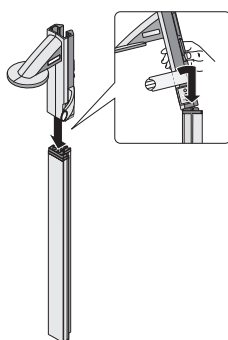Mounting the head slide

Proceed as follows to fit the head slide onto the column:

1. Tilt the head slide slightly forward.
2. Place the tilted head slide carefully onto the column element.
3. Align the head slide so that it is positioned vertically on the column element.
4. Press and hold the brake button and push the head slide carefully onto the column element.

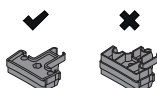
NOTICE!

Damage to device from incorrect handling

Incorrect handling can damage the claw guide of the head slide.

- ▶ Do not position the head slide vertically or tilted backward onto the column element.
- ▶ Do not apply any excessive force during the installation.

Fitting third column element


NOTICE!

Damage to device as a result of assembly errors


If end cap and column connector are switched, it will not be possible to assemble the device so that it is fit for purpose.

- ▶ Ensure that you use the end cap when assembling the third column element.

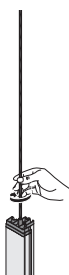1. Screw a threaded rod (approx. 3 Nm) into the coupling nut of the second column element.

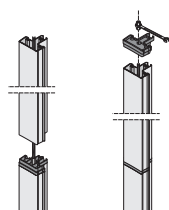2. Push the column element over the threaded rod onto the column connector of the second column element.
3. Place the end cap on the third column element so that the threaded rod protrudes out of the bore in the end cap.
4. Position the plain washer on the free end of the threaded rod.
5. Position the nut on the free end of the threaded rod.
6. Tighten the nut (3 Nm).

NOTICE!

Damage to device and malfunction as a result of assembly errors

If the nut on the end cap is not tightened enough, the column elements will be too loosely connected. The device will be unstable and faulty measurements will occur.

- ▶ Tighten the nut to the specified torque so that the columns are located firmly on one another and there is no gap between the column connector and the third column element.

7. Put the decorative cap on the nut.

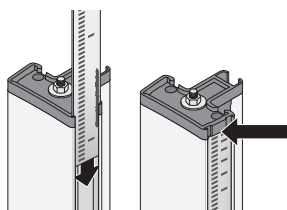
Fitting push-in scale

NOTICE!

Faulty measurements as a result of assembly errors

If the push-in scale is assembled incorrectly, the head slide cannot determine any measured values which can be used.

- ▶ Align the push-in scale so that the printing remains visible when it is pushed into the groove.
 - ▶ Align the push-in scale so that the area with the article number forms the bottom end.
 - ▶ Ensure that the push-in scale engages underneath the end cap.
- ▶ Push the push-in scale into the groove on the side of the column elements until the scale engages under the end cap.

5.3 Mounting device on a wall

Selecting installation location

Choose an adequately strong, load-bearing wall to mount the device. Standard rawl plugs are included in the scope of delivery for installation on solid brick walls. seca recommends use of appropriate special rawl plugs for other types of walls.

The installation height of the wall brackets can be freely selected. For optimum stability, seca recommends choosing the spacing between the drill holes roughly as shown in the drawing on the left.

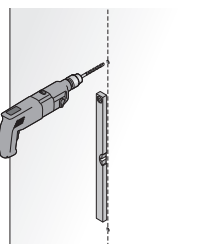
CAUTION!

Faulty measurement due to unsuitable installation location


Soft floors (wooden boards, for example) give under the patient's weight and distort the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select an installation location with a level, stable floor in order to achieve precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Before using the device, make sure it is vertical and securely fixed.

Installing wall brackets

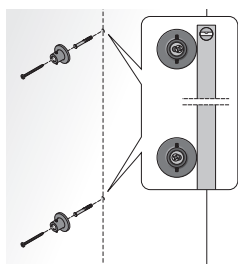1. Mark 2 drill holes for the wall brackets perpendicular to each other.
2. Drill the holes using a drill bit ($\varnothing$ 10 mm).
3. Insert the rawl plugs in the drill holes.


4. Remove the slot bolts from the wall brackets.
5. Mount the wall brackets using the screws (M6x60).
6. Use a spirit level to align the wall brackets perpendicularly with each other.
7. Tighten the screws of the wall brackets.

Installing the slot bolts

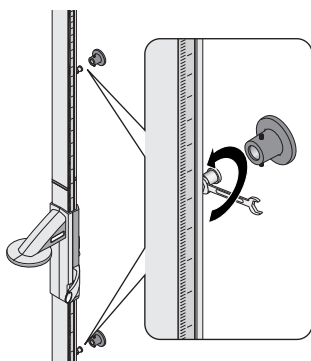
1. Insert the slot bolts into the groove of the column elements.
2. Position the slot bolts in the groove:
 - The distance between the slot bolts must correspond to the spacing between the wall brackets.
 - The heel stop must be on the floor when the device is positioned in the wall brackets.


3. Use an open-ended spanner (size 13) to tighten the slot bolts in the groove.

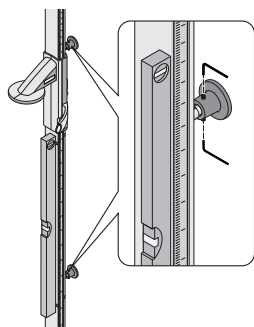Positioning the device in the wall brackets


1. Position the device with the slot bolts in the wall brackets.
2. Ensure that the heel stop is on the floor.
3. Use a spirit level to align the device perpendicularly.
4. Tighten the grub screws of the wall brackets.

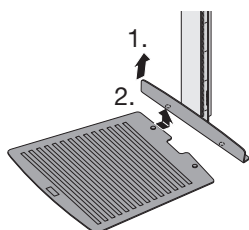Installing the foot mat

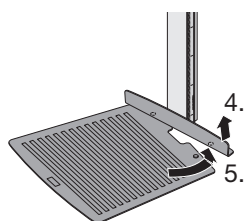
1. Lift up the heel stop on left-hand side.
2. Position the foot mat under the heel stop so that the anti-slip mat is located exactly under the hole of the heel stop.
3. Release the heel stop.
⇒ The anti-slip mat sits in the hole of the heel stop.


4. Lift up the heel stop on the right-hand side.
5. Swing the foot mat under the heel stop so that the anti-slip mat is located exactly under the hole of the heel stop.
6. Release the heel stop.
⇒ The anti-slip mat sits in the hole of the heel stop.
⇒ The foot mat is secured against slipping.

5.4 Inserting batteries

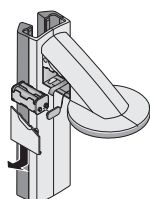The battery holder supplied already contains 4 AA-type batteries (1.5 V). To establish the power supply for the head slide, proceed as follows:

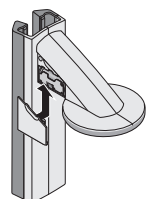1. Take off the cover of the battery compartment.
2. Remove the connector cable from the battery compartment.

3. Connect the battery holder to the connector cable.
4. Insert the battery holder in the battery compartment.

NOTICE!

Damage to device and malfunction

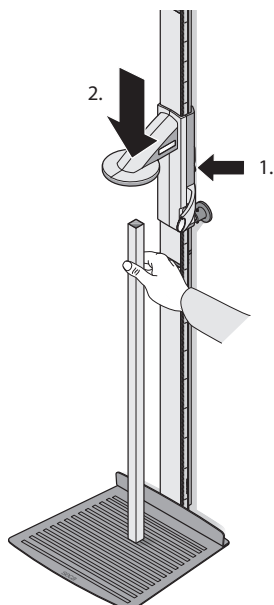
If the battery compartment is not closed correctly, the battery cable may be damaged.

- When closing the battery compartment, ensure that the battery cable is not trapped between the cover and the battery compartment.

5. Push the cover onto the battery compartment until you hear it engage.


5.5 Calibrating height measurement

Calibrating automatically


You can perform automated calibration with the aid of the calibration rod supplied.

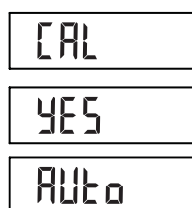1. Switch on the device.
⇒ ---- is displayed.
2. Press and hold the brake button and move the head slide until a height value is shown in the display.
3. Place the calibration rod supplied (0.8 m long) vertically against the lower part of device and keep it in this position.
4. Press and hold the brake button and push the head slide onto the calibration rod.
5. From the menu, select the **CAL** menu item.

6. Confirm your selection.
7. Select the setting **YES**.
8. Confirm your selection.
⇒ The **Auto** setting is shown in the display.
9. Confirm the setting.
⇒ The device is calibrated.
⇒ You can perform height measurements.

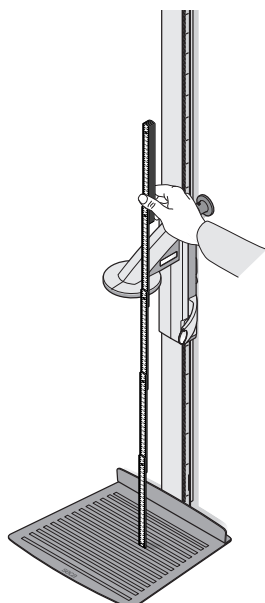Calibrating manually


You can perform calibration manually using a folding rule (alternative to the calibration rod supplied).

1. Switch on the device.
⇒ ---- is displayed.
2. Press and hold the brake button and move the head slide until a value is shown in the display.
3. Push the head slide to a position specified using the folding rule (e.g. 83.5 cm).

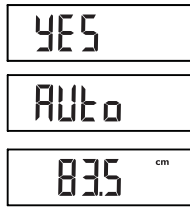4. From the menu, select the **CAL** menu item.
5. Confirm your selection.

6. Select the setting **YES**.
7. Confirm your selection.
⇒ The **Auto** setting is shown in the display.
8. Enter the position of the head slide (in this case: 83.5 cm).
9. Confirm the entry.
⇒ The device is calibrated.
⇒ You can perform height measurements.

5.6 Disinfecting device

- Disinfect the following components **before** initial commissioning (→ [Disinfecting, page 49](#)):
 - Foot mat
 - Column
 - Head slide with Frankfurt measure

6 OPERATION

CAUTION!

Personal injury/faulty measurement

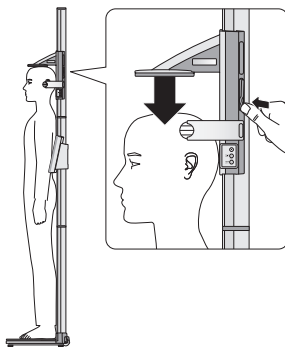
- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

6.1 Measuring height


Switching on head slide

- Press the  key of the head slide.
⇒ --- is shown in the display.
⇒ The head slide is operational.

Starting height measurement

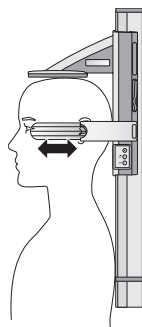1. Press and hold the brake button and move the head slide until a measuring result is shown in the display.
2. Press and hold the brake button and move the head slide upwards so that the patient can position him or herself underneath it comfortably.
3. Ask the patient to stand under the head slide:
 - Back to the column of the device
 - Upright posture: Back and head straight
 - Heels against the heel stop
4. Press and hold the brake button and push the head stop down until it is in contact with the patient's head.

5. Take the Frankfurt measure out of the head slide.
6. Align the patient's Frankfurt horizontal planes with the three lines of the Frankfurt measure.
7. Press and hold the brake button and correct the position of the head slide.
8. Read off the measuring result.

Displaying measuring result permanently (Hold)

If you activate the **Hold** function, the measured value continues to be displayed after the measurement. This allows you to move the head slide before you note the measured value.

1. Position the head slide.
2. Briefly press the  key.
 ⇒ The **HOLD** message is displayed.
 ⇒ The head slide can be moved without the displayed measured value changing.
3. To deactivate the **Hold** function, press the  key.
 ⇒ The **HOLD** message is no longer displayed.

Performing relative measurements (ZERO)

The head slide display can be zeroed in any position. This function facilitates relative measurements – for measuring extremities, for example.

1. Press the  key of the head slide.
 ⇒ --- is shown in the display.
2. Press and hold the brake button and move the head slide until a height value is shown in the display.
3. Position the head slide at the point at which you would like to zero the head slide display.
4. Press and hold the  key until the **ZERO** message is displayed.
 ⇒ The display is zeroed at the current position of the head slide.
5. Reposition the head slide.
 ⇒ Height is measured relative to the set zero point.
 ⇒ If the zero point is undershot, the measured values will be shown with a minus sign in front.
6. To deactivate the **zero** function, press and hold the  key until the **ZERO** message is no longer displayed.

NOTE

If for documentation purposes, you send results of relative measurements to devices which calculate Body Mass Index or Body Fat Ratio automatically, implausible values will result for these parameters.

Switching off head slide

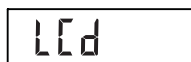
- Press the  key of the head slide.
 ⇒ The head slide display goes out.

NOTE

The head slide switches off automatically after a short time if it is not moved.

6.2 Other functions (menu)

Setting display backlighting (LCd)

You can change the duration and brightness of display backlighting.

1. From the menu, select the **LCd** menu item.
2. Confirm your selection.
3. Select a menu item:
 - Duration (**dUr**)
 - Brightness (**brl**)
4. Confirm your selection.
⇒ The current setting is displayed.
5. Select the desired setting.

Function	Setting
Duration (dUr)	• SHrt (approx. 15 sec.) • LOnG (approx. 45 sec.)
Brightness (brl)	• 50 % • 100 % • OFF

6. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Activating beeps (bEEP)

You can set whether a beep is heard every time a key is pressed.

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Switching unit of height (Unit)

You can select the unit in which height is to be displayed.

WARNING!

Patient hazard

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (weight: kilograms/grams, height: meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- Only use measuring results in SI units.
- The user takes sole responsibility for the use of measuring results in non-SI units.

NOTE

Height is sent to the multifunctional display and displayed there in the unit set here.

1. From the menu, select the **Unit** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the unit in which height is to be displayed:
 - **FEET**: Feet, inches

- **Mtr**: Centimeters

- Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Restoring factory settings (rSEt)

You can restore the factory settings for the functions below:

Function	Factory setting
Beep (bEEP)	On
Display backlighting brightness (brl)	50 %
Display backlighting duration (dUr)	SHrt
Unit of height (Unlt)	Mtr (cm)

- From the menu, select the **rSEt** menu item.
- Confirm your selection.
 - ⇒ You exit the menu automatically.
- Switch off the device.
 - ⇒ Factory settings are restored and are available when the device is switched back on.

7 HYGIENE TREATMENT

WARNING!

Electric shock

The device is not de-energized when the on/off button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Before each hygiene treatment, ensure that the device is switched off.
- ▶ Take the batteries/battery packs out of the device before every hygiene treatment.
- ▶ Ensure that no fluids penetrate the device.

WARNING!

Risk of infection

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and lead to transparent components becoming clouded.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

7.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

7.2 Disinfecting

- Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).

2. Follow the instructions for use for the disinfectant.
3. Disinfect the device:
 - Moisten a soft cloth with disinfectant and wipe down the device with it.
 - Comply with the intervals, see table:

Interval	Component
Before and after each measurement	• Foot mat • Head slide
As required	• Column

7.3 Sterilizing

The device must not be sterilized.

8 FUNCTION CHECK

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the “Troubleshooting” section in this document.

CAUTION!

Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled “Servicing” in this document.

9 TROUBLESHOOTING


9.1 General faults

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service.

Fault	Cause	Remedy
Calibration of height measurement cannot be performed. The CAL menu item is not displayed in the head slide menu.	The head slide was not moved before the menu was called up.	• Move the head slide until a measured value is displayed • Call up menu • Perform calibration (→ Calibrating height measurement, page 45)

Fault	Cause	Remedy
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.
The display backlighting in the head slide display goes out and cannot be switched back on.	Battery voltage is dropping. Display backlighting has been switched off automatically to save energy. You can perform measurements and send data for approximately another 12 h.	→ Changing the batteries, page 51
 is displayed.	Battery voltage is dropping.	→ Changing the batteries, page 51
bAtt is displayed.	The batteries have discharged.	→ Changing the batteries, page 51
StOP is displayed.	Maximum capacity has been exceeded.	Remove the load from the device

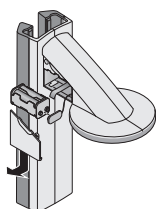9.2 Changing the batteries


✓ You need 4 AA-type batteries, 1.5 V.

1. Take off the cover of the battery compartment.
2. Remove the battery holder from the battery compartment.
3. Take the discharged batteries out of the battery holder.
4. Put the new batteries in the battery holder.
5. Insert the battery holder in the battery compartment.

NOTE

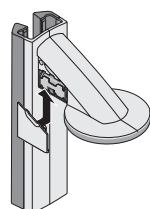- Dispose of empty batteries as described in the section entitled “Disposal”.
- Ensure the correct polarity of the batteries (markings on the battery holder). If **bAtt** is shown in the display, you have inserted one of the batteries the wrong way round or the batteries have discharged. If batteries are inserted the wrong way round, they must be removed immediately.

NOTICE!

Damage to device and malfunction

If the battery compartment is not closed correctly, the connector cable may be damaged.

- When closing the battery compartment, ensure that the connector cable is not trapped between the cover and the battery compartment.

6. Push the cover onto the battery compartment until you hear it engage.

10 SERVICING

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find your local service partner at www.seca.com.

11 TECHNICAL DATA

11.1 General technical data

seca 264	
Dimensions	
• Depth	474 mm
• Width	428 mm
• Height (standard model)	2318 mm
• Height (short model)	2118 mm
Net weight	
• Standard model	Approx. 5 kg
• Short model	Approx. 4.75 kg
Ambient conditions, operation	
• Temperature	+10 °C to +40 °C / +50 °F to 104 °F
• Air pressure	700 – 1060 hPa
• Humidity	30 % – 80 %, no condensation
Ambient conditions, storage and transport	
• Temperature	-10 °C to +65 °C / +14 °F to 149 °F
• Air pressure	700 – 1060 hPa
• Humidity	0 % – 95 %, no condensation
Power supply	
Batteries	
• Supply voltage	6 V
• Battery type	4 x type AA, 1.5 V
Current consumption	
• Wireless module deactivated, display backlighting off	Approx. 20 mA
• Wireless module activated, display backlighting permanent (brightness: 100%)	
Maximum running time	Approx. 3,800 minutes
Medical device in line with directive (EU) 2017/745	Class I with measuring function
EN 60601-1: Medical electrical equipment, type B	
Type of protection	IP20
Duty cycle	Continuous duty
Application parts in line with IEC 60601-1	Foot mat, head slide

11.2 Height measurement

Measuring range, graduation, accuracy	
Measuring range, standard model	30 – 220 cm
Measuring range, short model	30 – 200 cm
Graduation	1 mm
Accuracy	± 2 mm

12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Spare parts kit, short column	19 17 01 079 509

13 DISPOSAL

13.1 Disposing of the device

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service at service@seca.com.

13.2 Disposing of batteries and rechargeable batteries

Do not dispose of used batteries and rechargeable batteries in household waste, regardless of whether they contain harmful substances or not. As a consumer, you are legally obliged to dispose of batteries and rechargeable batteries via local authority collection points or trade collection points. Only dispose of batteries and rechargeable batteries once they are completely discharged.

14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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