## 1. Herzlichen Glückwunsch

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## 1. Congratulations

By purchasing the **seca 460** RS232 adapter kit with printer interface you have acquired a high-quality accessory.

seca has been putting its experience to the service of health for over 170 years now, and as market leader in many countries of the world, is always setting new standards.

The **seca 460** adapter kit with printer interface is an add-on unit to provide seca scales with a serial RS232 interface and a printer port. An “EPSON-compatible” serial printer can be connected to this printer port.

**The RS232 interface and the printer port cannot be used simultaneously.**

## 2. Safety

- Follow the safety instructions in instruction manual.
- Keep the operating manual in a safe place.
- Only authorized persons should carry out repairs.

## 3. Connecting the adapter kit

See drawing

**Note: Before the adapter kit is connected to the scale, it should be checked that**

- the power supply to the scale is not connected
- the printer/PC is not connected to the adapter kit.

## 4. Configuring the printer

In order to be able to use a printer, the printer first needs to be set to the following parameters:

- Baud rate:	9600 bps
- Databit:	8 bits
- Parity check:	enabled / yes
- Parity:	even
- Handshaking:	XON / XOFF
- Receive error:	prints ‘?’
- Buffer Capacity:	maximum
- Handshaking Operation:	receive buffer full

## 5. RS232 interface

See “RS232 interface description”.

Document no. 30-34-00-605

## 6. Technical data

Dimensions (width, length, height)	61.5 x 102.5 x 26 mm
Temperature range	+10 °C to +40 °C
Power supply	via connected scales (typically 2 mA; max. 5 mA)

## 7. Warranty

A two-year warranty from date of delivery applies to defects attributable to poor materials or workmanship. All moveable parts - batteries, cables, mains units, rechargeable batteries etc. - are excluded. Defects which come under warranty will be made good for the customer at no charge on production of the receipt. No further claims can be entertained. The costs of transport in both directions will be borne by the customer should the equipment be located anywhere other than the customer's premises. In the event of transport damage, claims under warranty can be honoured only if the complete original packaging was used for any transport and the scale secured and attached in that packaging just as it was when originally packed. All the packaging should therefore be retained.

A claim under warranty will not be honoured if the equipment is opened by persons not expressly authorised by seca to do so.

We would ask our customers abroad to contact their local sales agent in the event of a warranty matter.

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