



# seca mVSA 535 seca mBCA 525

# Instructions for use

Software version 1.1 from Build 104 Art no: 17-10-05-353-002h\_2023-01 B

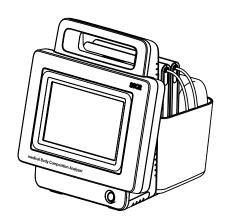



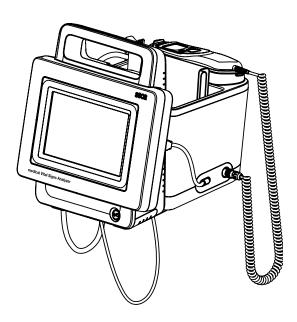

# CONTENTS

Instructions for use for seca mVSA 535/ seca mBCA 525				
1.	1. About this document			
	Description of device			
Ζ.	2.1 Intended use for the seca mVSA 535			
	2.2 Intended use for the seca mBCA 525			
	2.3 Description of function			
	Device components			
	Power supply			
	Bioimpedance measurement			
	Vital signs measurement			
	Alarms			
	Weight and height			
	seca user accounts	8		
	User IDs of EMR systems	8		
	seca patient files	8		
	Patient data from EMR systems			
	Measuring results			
	Data transmission and network functions			
	Compatibility			
	2.4 User qualification	10		
	Administration/network connection	10		
	Measuring mode	10		
	2.5 Contraindications	10		
3.	Safety precautions	11		
	3.1 Safety precautions in these instructions for			
	use	11		
	3.2 Basic safety precautions	11		
	Handling the device	11		
	Handling a wheeled stand	12		
	Preventing electric shock	13		
	Prevent injuries and infections	13		
	Preventing damage to device	14		
	Handling measuring results	14		
		15		
	Handling batteries and rechargeable batteries	15		
		15		
4.	Device overview	16		
	4.1 Monitor controls for seca mVSA 535/			
	seca mBCA 525	16		
	4.2 In-ear thermometer controls	18		
	4.3 Measuring mat controls (bioimpedance	10		
	analysis).	19		
	4.4 Fields in the touchscreen display	20		
	4.5 Login/navigation: Buttons and symbols in the	20		
	touchscreen display	20		
	4.6 Operating state: Symbols	21		
	display	21		
	Operating state: Symbols on the measuring	21		
	mat	23		
	4.7 Measuring: Buttons and symbols	23		
	Patient tab	23		
	Vital signs tab	24		
	BIA tab	25		
	Measuring mat	26		
	Analysis tab	26		
	4.8 Unsubmitted measurements list: Symbols	27		
	4.9 Markings on the device and on the type			
	plate	28		

4.10 Markings on the blood pressure cuff	29
4.11 Markings on the packaging	30
4.12 Other symbols	31
Starting up the device	32
5.1 Scope of delivery	32
5.2 Setting up device	33
Connecting the SmartBucket	33
Connecting the blood pressure cuff	34
Connecting the temperature probe	34
Connecting the in-ear thermometer	35
Connecting the seca SpO <sub>2</sub> sensor	35
Connecting the	
Masimo SET <sup>®</sup> SpO <sub>2</sub> sensor	36
Suspending the measuring mat in the	
magnetic catch	37
5.3 Establishing a power supply	38
5.4 Charging rechargeable batteries	38
5.5 Setting the date and time	39
5.6 Initial login	40
5.7 Configuration options	40
Operation	41
6.1 Switching the system on/off	41
Switching on	41
	42
Logging out/switching user	43
Saving power/switching off	43
6.2 Managing seca patient files	44
Creating a seca patient file	44
Calling up a seca patient file	47
Editing a seca patient file	47
Exporting a seca patient file	48
Deleting a seca patient file	48
6.3 Measuring vital signs	49
	49
Measuring blood pressure	50
Measuring temperature, oral/axillary	
(COVIDIEN <sup>TM</sup> FILAC <sup>TM</sup> 3000, blue)	55
Measuring temperature, rectal	
(COVIDIEN <sup>TM</sup> FILAC <sup>TM</sup> 3000 red)	58
Measuring in-ear temperature	
(COVIDIEN <sup>TM</sup> GENIUS <sup>®</sup> 3)	61
Entering temperature manually	65
Reading off pulse rate	66
Measuring oxygen saturation (SpO <sub>2</sub> )	68
Weight and height	72
Stopping a measurement procedure	73
Assigning anonymous measurement	
procedure to a seca patient file	73
6.4 Measuring bioimpedance	73
User qualification for bioimpedance	
measurement	74
Preparing a bioimpedance measurement	74
Performing a measurement (with WiFi	
direct connection to the measuring mat)	75
Performing a measurement (without WiFi	
direct connection to the measuring mat)	78
Connecting the measuring mat	80
6.5 Adding weight and height	81
Receiving weight and height	
(seca 360° proximity)	82

5.

6.

Receiving weight and height	
(seca 360° wireless)	. 83
Entering weight and height manually	
6.6 Analyzing measurement	
Viewing current measuring results	. 85
Analyzing vital signs (history)	
Analyzing a bioimpedance measurement	. 88
7. Operation when connected to an EMR system	. 92
7.1 Switching on the device	
7.2 Measuring	. 93
Measuring with an active connection to an	00
EMR system	
Measuring with an interrupted connection to an EMR system	
7.3 Logging in with EMR user ID	
Scanning EMR user ID (recommended)	
Entering EMR user ID manually	
7.4 Calling up patient data from the EMR	
system	
Scanning patient ID (recommended)	
Manually entering patient ID	
7.5 Saving measurements in the EMR system	. 98
Submitting the measurement directly to the	00
EMR system	. 98
submitting it later	. 98
Using the Unsubmitted measurements list .	
-	103
8. Hygiene treatment	103
8.2 Disinfecting	105
8.3 Sterilizing	105
8.4 Removing/fitting probe holder (devices with	
temperature probe)	106
Removing the probe holder	106
Fitting the probe holder	106
8.5 Removing/fitting the magazine holder	107
(devices with in-ear thermometer)	107 107
Fitting the magazine holder	107
9. Function check	<b>108</b> 108
9.1 Device	100
thermometer	108
10. Servicing	
10.1 Device	<b>108</b> 108
10.2 COVIDIEN <sup>TM</sup> GENIUS <sup>®</sup> 3 in-ear	100
thermometer	109
11. Troubleshooting	110
11.1 Monitor	110
11.2 Measuring mat.	111
11.3 Bioimpedance measurement	112
General	112
Deviating measuring results	113
11.4 Vital signs measurement	115
	115
Blood pressure measurement COVIDIEN <sup>TM</sup> FILAC <sup>TM</sup> 3000 temperature	115
	116
COVIDIEN <sup>TM</sup> GENIUS <sup>®</sup> 3 temperature	110
measurement	116

Manual temperature input SpO <sub>2</sub> measurement 11.5 Network connection	118 118 119
<ul><li>11.6 Connection to the seca analytics 115</li><li>software</li><li>11.7 EMR system connection via</li></ul>	119
seca connect 103 software	120 122 122
<b>12. Technical data</b> 12.1 Monitor         12.2 Monitor interfaces and network ports         Interfaces and network ports         Recommended WiFi settings	<b>123</b> 123 124 124 124
<ul> <li>12.3 Measuring mat.</li> <li>12.4 Bioimpedance measurement.</li> <li>Measuring method .</li> <li>Clinical studies .</li> <li>Accuracy of predictive formulas .</li> <li>12.5 Vital signs measurement .</li> <li>Blood pressure measurement .</li> <li>COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 temperature</li> </ul>	125 125 125 126 126 127 127
COVIDIEN <sup>TM</sup> FILAC <sup>TM</sup> 3000 temperature measurement COVIDIEN <sup>TM</sup> GENIUS <sup>®</sup> 3 temperature	128
<ul> <li>measurement</li> <li>Manual temperature input</li> <li>Masimo SET<sup>®</sup> SpO<sub>2</sub> measurement</li> <li>12.6 Analysis parameters.</li> <li>12.7 Analysis modules.</li> <li>12.8 seca 360° wireless system.</li> <li>12.9 Standards and directives.</li> </ul>	129 129 131 132 133 133 133
13. Optional accessories and spare parts	134
14. Compatible seca products	135
<b>15. Disposal</b> 15.1 Measuring mat and device.         15.2 Batteries and rechargeable batteries         15.3 Consumables	<b>136</b> 136 136 136
16. Warranty	137
17. Declaration of conformity	137

# For administrators: Configuring

seca mVSA 535 /seca mBCA 525	138
1. Preparing configuration	138
1.1 Administrator login	138
1.2 Configuration options	140
Network functions	140
Overview of access rights	140
2. User accounts	141
2.1 Activating initial user accounts	141
Changing the password for the Admin user	
account	141
Assigning a password to the User user	
account	142
2.2 Working with user accounts	143
Setting up user accounts	143
Editing user accounts	143
Deleting user accounts	144
3. Making settings for measuring mode	145
3.1 Making regional settings	145
3.2 Setting display brightness and volume	146
3.3 Calibrating the touchscreen display	147
3.4 Setting units of measurement.	149
3.5 Deactivating analysis modules	150
3.6 Making presets for vital signs measurement.	151
Presets for blood pressure	151
Presets for pulse rate (seca measuring	
equipment only)	155
Presets for SpO <sub>2</sub>	156
Presets for temperature	
(COVIDIEN <sup>TM</sup> FILAC <sup>TM</sup> 3000 only)	157
Selecting color mode for Vital signs tab	159
4. Managing system components	160
4.1 Viewing system information	160
4.2 Updating the monitor software	161
4.3 Updating the measuring mat software	162
4.4 Retrofitting the SmartBucket (seca mBCA 525	
only)	163
4.5 Retrofitting the measuring mat	
(seca mVSA 535 only)	164
4.6 Retrofitting the in-ear thermometer	
(seca mVSA 535 only)	165
4.7 Data export and backup	166
Exporting patient and user data manually	166
Restoring patient and user data manually .	167
4.8 Power management	168
5. Setting up peripherals	169
5.1 Setting up a LAN connection to the	
network (stationary operation)	169
Introduction	169
Activating the LAN connection	170
Deactivating the LAN connection	172
5.2 Setting up a WiFi connection (mobile	
operation)	173
	173
Activating the WiFi connection	174
Deactivating the WiFi connection	175
5.3 Setting up a connection to the seca	170
analytics 115 PC software	176
Introduction	176

Connecting the device automatically (UDP) Connecting the device manually (TCP) Activating automatic synchronization Setting up a utomatic export 5.4 Setting up a seca 360° proximity connection . 5.5 Setting up a seca 360° wireless connection . Introduction Activating/deactivating the seca 360° wireless module Setting up a seca 360° wireless connection	176 177 178 180 181 184 184 185 185
<ul> <li>6. Connecting to an EMR system</li> <li>6.1 Information about user and patient data</li> <li>6.2 Assigning a clear device name</li> <li>6.3 Setting up a connection to the seca connect 103 software</li> <li>Entering connection data</li> <li>6.4 Configuring data transfer to the EMR system</li></ul>	<b>188</b> 188 189 190 190 191 193 193
6.6 Summary: Device characteristics when connected to EMR system	193
<ul> <li>7. Factory settings</li> <li>7.1 Overview of factory settings</li> <li>7.2 Resetting the device</li> <li>7.3 Resetting the user interface</li> <li>7.4 Exporting the system log/audit trail</li> <li>7.5 Enabling VNC access</li> </ul>	<b>194</b> 195 196 196 197

# **INSTRUCTIONS FOR USE FOR seca mVSA 535/seca mBCA 525**

- → About this document
- → Description of device
- → Safety precautions
- → Device overview
- → Starting up the device
- → Operation
- $\rightarrow$  Operation when connected to an  $\rightarrow$  Warranty EMR system
- → Hygiene treatment
- → Function check

- → Servicing
- → Troubleshooting
- → Technical data
- → Optional accessories and spare parts
- → Compatible seca products
- → Disposal
- → Declaration of conformity
- → For administrators: Configuring seca mVSA 535 / seca mBCA 525

Software version: 1.0 from Build 104 Article number of this document: 17-10-05-353-002h\_2023-01 B

# 1. ABOUT THIS DOCUMENT

## NOTE

This document describes the maximal equipment of the seca mVSA 535/seca mBCA 525 product family: measurement of blood pressure, temperature, oxygen saturation and bioimpedance. Depending on the actual equipment of your device, some of this information may not be relevant to your device. Pay attention to the information in this document which is relevant to your device.

Display conventions		
Symbol	Description	
$\checkmark$	Requirement for action	
•	Handling instruction	
<ol> <li>Handling instructions which have to be performed.</li> <li>the specified sequence</li> </ol>		
a) b)	Steps of a handling instruction which have to be performed in the specified sequence	
•	First level of a list	
	Second level of a list	
	Indicates points on the device or on device components which require particular attention	
Indicates areas on the display which require particula         attention		
	Indicates directions in overview graphics	
	Correct handling method Correct handling result	
*	Incorrect handling method Incorrect handling result	

# 2. DESCRIPTION OF DEVICE

- → Intended use for the seca mVSA 535
- → Intended use for the seca mBCA 525
- → Description of function
- → User qualification
- → Contraindications

## 2.1 Intended use for the seca mVSA 535

The medical Vital Signs Analyzer **seca mVSA 535** is mainly used in inpatient facilities (hospitals, medical practices and care facilities) in accordance with national regulations.

The medical Vital Signs Analyzer **seca mVSA 535** is used for non-invasive, discontinuous determination of arterial blood pressure and/or non-invasive determination of oxygen saturation of arterial hemoglobin and/or determination of body temperature and pulse rate and determination of weight and height.

When the Bioimpedance analysis function is included, the medical device is also used to record bioelectric impedance measurements and to calculate automatically parameters such as fat-free mass (FFM) which can be derived from these measurements. The results are displayed in graphical form and assist the attending physician with the following medical issues:

- Determining energy expenditure and energy reserves as a basis for nutritional advice
- Assessing metabolic activity and the success of a training program, e.g. within the framework of rehabilitation or physiotherapy
- Determining a patient's fluids status

The medical Vital Signs Analyzer **seca mVSA 535** is intended for use in children 3 years and older and adults. The Bioimpedance analysis function is not intended for use on children.

## 2.2 Intended use for the seca mBCA 525

The medical Body Composition Analyzer **seca mBCA 525** is mainly used in inpatient facilities (hospitals, medical practices and care facilities) in accordance with national regulations.

The medical Body Composition Analyzer **seca mBCA 525** records weight, height and bioelectric impedance measurements and automatically calculates parameters such as fat-free mass (FFM) which can be derived from these measurements. The results are displayed in graphical form and assist the attending physician with the following medical issues:

- Determining energy expenditure and energy reserves as a basis for nutritional advice
- Assessing metabolic activity and the success of a training program, e.g. within the framework of rehabilitation or physiotherapy
- Determining a patient's fluids status

The device is **not** intended for use on children.

# 2.3 Description of function

	→ Device components
	→ Power supply
	→ Bioimpedance measurement
	→ Vital signs measurement
	→ Alarms
	$\rightarrow$ Weight and height
	→ seca user accounts
	→ User IDs of EMR systems
	→ seca patient files
	$\rightarrow$ Patient data from EMR systems
	→ Measuring results
	$\rightarrow$ Data transmission and network functions
	→ Compatibility
Device components	The seca mVSA 535 consists of a monitor and a SmartBucket.
	The seca mBCA 525 consists of a monitor and a storage compartment.
	The monitor is for managing patient and user data and for preparing and ana- lyzing measurements. The monitor is equipped with a touchscreen display.
	The SmartBucket includes the measuring equipment for recording vital signs and storage facilities for the measuring accessories.
	In the maximum configuration, the vital signs blood pressure, body tempera- ture, pulse rate and oxygen saturation can be recorded and bioimpedance measurements performed. Your version of the device may have a more lim- ited functional scope.
Power supply	The monitor is powered by a connection to the power supply. The monitor has a lithium-ion rechargeable battery to provide a mobile power supply.
	The SmartBucket is supplied with power via a USB connection from the monitor.
Bioimpedance measurement	Bioimpedance measurement is performed with a measuring mat developed by seca.
	Bioimpedance measurement is carried out with the patient lying down, using the 8-point method. The 4-point method (measuring the right half of the body) is likewise possible. The low alternating current is provided and impedance is measured by the electrode cables of the measuring mat. The electrode cables are connected to two pairs of electrodes for each half of the body. The electrodes are affixed to the patient's hands and feet.
Vital signs measurement	Blood pressure is measured non-invasively with seca measuring equipment and seca blood pressure cuffs.
	Temperature is measured with COVIDIEN <sup>™</sup> measuring equipment and, depending on the device version, as an oral/axillary (blue temperature probe), rectal (red temperature probe) or in-ear thermometer procedure.
	On device versions without integrated temperature measurement, temperature values can be entered manually.

	Depending on the device version, oxygen saturation is measured with Masimo SET <sup>®</sup> or with seca measuring equipment and with the corresponding SpO <sub>2</sub> sensors and patient cables.
	Depending on device version, pulse rate is determined using either oxygen saturation or blood pressure.
Alarms	The device is intended for discontinuous measurement of vital signs, so it does <b>not</b> have an alarm function.
Weight and height	Scales and measuring rods from the <b>seca 360° wireless</b> system can transmit measuring results to the device wirelessly.
	Scales and measuring rods with an internal interface module or a <b>seca 452</b> external interface module can transmit measuring results to the device via LAN or WiFi.
	Alternatively, the patient's weight and height can be recorded manually.
seca user accounts	The user accounts for the device can be synchronized automatically with the seca user accounts of the <b>seca analytics 115</b> PC software. This allows access to both the device and the PC software with the same seca user account.
User IDs of EMR systems	If the device is connected to an EMR system via the <b>seca connect 103</b> soft- ware, the user IDs of the EMR system can be used for measurements. Local user accounts on the device are required for administration and service tasks.
seca patient files	seca patient files and seca patient databases contain exclusively data neces- sary for working with seca products or which were determined using seca products.
	In standalone mode or when connected to the <b>seca analytics 115</b> PC soft- ware, measuring results are managed in seca patient files. seca patient files can be set up, edited, exported and deleted directly on the device.
	The device can be configured so that the seca patient files of the device are automatically synchronized with the seca patient files of the <b>seca analytics 115</b> PC software.
Patient data from EMR systems	Depending on the EMR system in question, patient data from EMR systems may be used via the export and import functions of the <b>seca analytics 115</b> PC software or via a connection to the <b>seca connect 103</b> software.
Measuring results	Measured results for the vital signs blood pressure (NIBP), temperature (TEMP), pulse rate (PR) and oxygen saturation (SpO <sub>2</sub> ) are displayed in graphical form.
	Bioimpedance measurements are analyzed in graphical form based on scientifically-validated formulas. In-house studies by seca generated predictive formulas for determining the parameters total body water (TBW), extracellular water (ECW), fat-free mass (FFM) and skeletal muscle mass (SMM) for arms, legs, torso and the whole body. Further studies generated normal ranges for the following parameters: Bioelectric impedance vector analysis (BIVA), mass indices (FMI, FFMI), phase angle ( $\phi$ ), fat mass (FM) and skeletal muscle mass (SMM).
Data transmission and network functions	The device can be integrated in a network via a LAN interface or via WiFi in order to use the following functions:
	<ul> <li>order to use the following functions:</li> <li>Connection of the device to the seca analytics 115 PC software available as an option: Communication with the databases of the seca analytics 115 PC software and use of the seca directprint function</li> <li>Connection of the device to the seca connect 103 software available as</li> </ul>
	<ul> <li>Connection of the device to the seca connect 103 software available as an option: Using the user and patient data of an EMR system and submitting measuring results to the EMR system</li> </ul>

• seca 360° proximity connection: Receiving weight and height from scales and measuring rods with an internal interface module or a seca 452 external interface module

Monitor and measuring mat – if there is one – communicate with each other via a WiFi direct connection or via an infrared interface.

Measuring results for the parameters blood pressure, body temperature, pulse rate and oxygen saturation are transmitted from the SmartBucket to the monitor via a USB connection.

The device has a **seca 360° wireless** module and can receive measuring results from compatible seca scales and measuring rods.

#### seca analytics 115 PC software

This device (software version 1.0, Build 104 or higher) is only compatible with version 1.4 (Build 6657 or higher) of the **seca analytics 115** PC software. There is no downward compatibility with older versions of **seca analytics 115**.

#### seca connect 103 software

This device (software version 1.0, Build 104 or higher) is only compatible with version 2.2 or higher of the **seca connect 103** software. There is no downward compatibility with older versions of the **seca connect 103** software.

#### Internal interface module/seca 452 interface module

This device (software version 1.0, Build 104 or higher) is only compatible with interface modules on which firmware version Q2\_2020\_Build\_615 or higher is installed.

#### seca mVSA 526 SmartBucket

medical Body Composition Analyzer **seca mBCA 525** from serial number 1000000090505 can be retrofitted with the SmartBucket **seca mVSA 526** (accessory kit for measuring vital signs).

Compatibility

## 2.4 User qualification

	→ Administration/network connection
	→ Measuring mode
Administration/network connection	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.
Measuring mode	The device may only be used by persons with sufficient expertise.
	Basic knowledge required for measuring vital parameters is <b>not</b> the subject of these instructions for use.
	Connecting the device to an EMR system affects the measuring sequence and operation of the device: $\rightarrow$ Operation when connected to an EMR system. People operating the device must be informed about these effects.
2.5 Contraindications	
	The device is <b>not</b> intended for permanent monitoring of patients.
	The device is <b>not</b> intended for patient monitoring during either transport (for example, in ambulances or helicopters) or transfer within an institution.
	The device is <b>not</b> intended for operation in the vicinity of an MRI device or in a pressurized chamber.
	The SpO <sub>2</sub> measuring function of the device is <b>not</b> intended for monitoring apneas, detecting arrhythmias or for use during defibrillation or electrocauterization.
	This device is not suitable for persons with the following characteristics:
	<ul><li>Cramps</li><li>Tremors</li></ul>
	Bioimpedance measurements may <b>not</b> be performed on individuals exhibiting the following characteristics:
	<ul><li>electronic implants, e.g. cardiac pacemakers</li><li>active prostheses</li></ul>

Bioimpedance measurements may **not** be performed on persons who are connected to one of the following devices:

- electronic life-support systems, e.g. artificial heart, artificial lung
- portable electronic medical devices, e.g. ECG devices or infusion pumps

Impedance measurements may only be performed on persons exhibiting the following characteristics after discussion with the attending physician:

- cardiac arrhythmias
- pregnancy

# 3. SAFETY PRECAUTIONS

- → Safety precautions in these instructions for use
- → Basic safety precautions

#### Safety precautions in these instructions for use 3.1



## DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.



## WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.



#### **CAUTION!**

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

## NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measured results may be incorrect.

#### NOTE

Includes additional information about use of the device.

#### 3.2 **Basic safety precautions**

- → Handling the device
- → Handling a wheeled stand
- → Preventing electric shock
- → Prevent injuries and infections
- → Preventing damage to device
- → Handling measuring results
- → Handling packaging material
- → Handling batteries and rechargeable batteries
- Please take note of the information in these instructions for use.
- Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and the authority responsible in your country.



**Risk of explosion** 

Do not use the device in an environment in which one of the following gases has accumulated:

- oxygen
- flammable anesthetics
- other flammable substances/air mixtures

Handling the device

Safety precautions • 11

## CAUTION! Patient hazard, damage to device

- Additional devices which are connected to electrical medical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1, respectively). Anyone connecting additional devices to electrical medical devices is considered a system configurer and is therefore responsible for ensuring that the system complies with the requirements of standards for systems. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.
- Please have servicing and measuring technology checks performed every two years.
- Technical modifications may not be made to the device. The device does not contain any parts for servicing by the user. Only have servicing and repairs performed by an authorized seca Service partner. You can find service partners in your area at www.seca.com or by sending an e-mail to service@seca.com.
- Only use original seca accessories and spare parts, otherwise seca will not grant any warranty.

# 

## **!!** Patient hazard, malfunction

- Keep other electrical medical devices, e.g. high-frequency surgical devices, a minimum distance of approx. 1 meter away to prevent incorrect measurements or wireless transmission interference.
- Keep HF devices such as cell phones a minimum distance of approx. 1 meter away to prevent incorrect measurements or wireless transmission interference.
- The actual transmission output of HF equipment may require minimum distances of more than 1 meter. Details can be found at www.seca.com.

## WARNING!

#### old L Injury from falling, damage to device

When transporting the device on a wheeled stand, make sure that all cables and tubes are stowed properly directly on the machine or in the basket of the stand.

# 

- Damage to device
- Do not pull on cables and tubes to move the equipment or wheeled stand.
- Do not move the wheeled stand when the power cord of the device is plugged into an electrical outlet.

Handling a wheeled stand

## **Preventing electric shock**

#### WARNING! Electric shock

- Set up devices which can be operated with the electricity supply so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- Ensure that your local power supply matches the details on the device.
- Connect this device only to a power supply with a protective earth facility.
- Do not connect the device to a power supply network if there is any uncertainty about whether the protective earth is functioning. In this case, use the device exclusively in rechargeable battery mode.
- Do not connect the device to sockets that are switched by an on/off switch or a dimmer.
- Never touch the power supply cable with wet hands.
- Do not use extension cables or power strips.
- Make sure that cables are not pinched or damaged by sharp edges.
- Make sure that cables do not come into contact with hot objects.
- Do not operate the device at an altitude of more than 3000 m above sea level.

#### WARNING! Injury from falls

- Ensure that the device is positioned firmly and level.
- Route connecting cables (if present) in such a way that neither user nor patient can trip over them.

#### WARNING! Risk of infection

- Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- Hygienically reprocess the device regularly as described in the respective section in this document.
- Make sure that the patient has no infectious diseases.
- ► Make sure that the patient has no open wounds or infectious skin alterations, which may come into contact with the device.

Prevent injuries and infections

#### Preventing damage to device

#### NOTICE! Damage to device

- If liquids have penetrated the device, the device may not be operational for a short period. Allow the device to dry for an extended period (e.g. overnight), before starting it up again.
- Switch off the device before you take the power supply connector out of the power supply socket.
- If you are not going to use the device for an extended period, disconnect the power supply connector from the power supply socket and remove the rechargeable battery (if present and removable). Only then is the device de-energized.
- Make sure not to drop the device.
- Do not expose the device to any impacts or vibrations. ►
- Ensure that there is no heat source in the immediate vicinity. Do not expose to direct sunlight. The excessive temperature could damage the electronics.
- Ensure that the air openings of the device (if present) are not covered.
- Perform function controls regularly as described in the relevant section in this document. Do not operate the device if it is damaged or not working properly.
- ► Avoid rapid temperature fluctuations. When the device is transported so that a temperature difference of more than 20 °C occurs, it must stay turned off for at least 2 hours before it can be turned on again. Otherwise, condensation water will form which can damage the electronics.
- Use the device only in the intended ambient conditions.
- Store the device only in the intended storage conditions.
- Use only disinfectants free of chlorine and alcohol which are explicitly suitable for acrylic sheet and other sensitive surfaces (active ingredient: guaternary ammonium compounds, for example).
- Do not use aggressive or abrasive cleaning agents.
- Do not use organic solvents (e.g. white spirit or petroleum spirit).
- ► Use disinfectants including the active ingredient 70 % isopropanol only for measuring accessories for measuring vital signs.



# Patient hazard

In order to avoid misinterpretations, measuring results for medical use must be displayed and used in SI units (weight: kilogrammes, length: metres) only. Some devices offer the ability to display measuring results in other units. This is only an additional function.

- Use the results exclusively in SI units.
- The use of measuring results in non-SI units is the sole responsibility of the user.

#### NOTICE!

#### Inconsistent measuring results

- Before you electronically save measured values determined using this device and use them further (e.g. in seca PC software or in an EMR system), make sure that the measured values are plausible.
- If measured values are transmitted to seca PC software or an EMR system, make sure prior to further use that the measured values are plausible and are assigned to the correct patient.

## Handling measuring results

## NOTICE!

#### Results not comparable to other devices

Results of bioelectric impedance measurements are not interchangeable with measurements obtained from different manufacturers' devices. Follow-up measurements not performed on a seca device may lead to inconsistent data and to misinterpreted measuring results.

Ensure that follow-up measurements are also performed on a seca device.

## NOTE

For an overview of the parameters which can be determined using this device, see the section entitled "Technical data\analysis parameters". If necessary, you can print out this overview and give it to your patients (print-out from the device not possible).



#### WARNING! **Risk of suffocation**

Packaging material made of plastic foil (bags) is a choking hazard.

- Keep packaging material out of reach of children.
- In the event that the original packing material may not be available anymore, only use plastic bags with security holes in order to reduce the risk of suffocation. Use recyclable materials if possible.

#### NOTE

Keep the original packing material for future use (e.g. returning for servicina).



## Personal injury as a result of improper handling

Batteries and rechargeable batteries contain harmful substances which may explode if not handled properly.

- Do not try to recharge batteries.
- Do not expose (rechargeable) batteries to heat.
- Do not burn (rechargeable) batteries.
- If acid is leaking out, avoid contact with the skin, eyes and mucous membranes. Rinse affected areas with plenty of clean water and seek medical help at once.

#### NOTICE!

#### Damage to device and malfunctions with improper handling

- Only use the type of (rechargeable) battery specified in this document.
- When replacing (rechargeable) batteries, always replace a complete set at a time.
- Do not short-circuit (rechargeable) batteries.
- If you do not use the device for a long period of time, remove the batteries (incl. rechargeable batteries). This prevents acid from leaking into the device.
- If acid leaked into the device, discontinue use. Have the device checked by an authorised seca Service partner and repaired if necessary.

Handling packaging material

Handling batteries and rechargeable batteries


## 4. Device overview

- → Monitor controls for seca mVSA 535/seca mBCA 525
- → In-ear thermometer controls
- → Measuring mat controls (bioimpedance analysis)
- → Fields in the touchscreen display
- → Login/navigation: Buttons and symbols in the touchscreen display
- → Operating state: Symbols
- → Measuring: Buttons and symbols
- → Unsubmitted measurements list: Symbols
- → Markings on the device and on the type plate
- → Markings on the blood pressure cuff
- → Markings on the packaging


## 4.1 Monitor controls for seca mVSA 535/seca mBCA 525

## NOTE

This section shows product versions. The functional scope of your device may deviate from this.

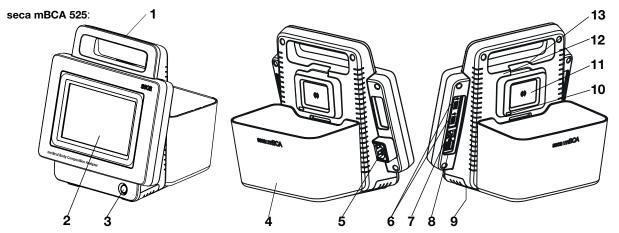

seca mVSA 535 (additional/deviating controls):

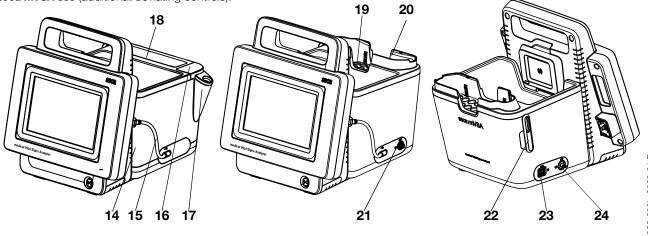


17-10-05-353-002h\_2023-01 B

	Handle Touchscreen display ON/OFF button with LED	Transporting the device Central control/display element
3		
	ON/OFF button with LED	
	ON/OFF button with LED	LED white: Device on
4		LED green: Device on standby
4		LED off: Device off
4	Storage compartment	Transporting/storing the measuring mat (bioimpedance analysis)
	(seca mBCA 525 only)	Data transmission: Monitor/measuring mat
		Charge measuring mat rechargeable battery
5	Power supply connection socket	For connecting the power supply cable
		For transmitting data via a USB memory stick
6	USB interface, 2 pcs	For transmitting data between monitor and SmartBucket
-		For supplying SmartBucket with power
+		Connecting the scanner
7	ISIS interface	Advance feature for future system upgrade (currently no function)
		For integrating the device in a network, using with the
8	LAN interface	seca analytics 115 PC software or with the seca connect 103 software
	Dechargeeble better	for connection to an EMR system
9	Rechargeable battery compartment	To take lithium-ion rechargeable battery supplied
	•	Automatic data transmission: Monitor/measuring mat (alternative if no
10	Infrared interface	WiFi available)
	Inductive charging interface with	
11	magnetic catch	For charging measuring mat rechargeable battery (version-dependent)
12	seca 360° internal wireless module	For connecting scales and length measuring devices from the seca 360° wireless system
		For integrating the device in a network (using with the
		seca analytics 115 PC software or with the seca connect 103 software
13	Internal WiFi module	for connection to an EMR system)
		• For automatic transmission of data, monitor/measuring mat
		For transporting/storing measuring equipment (bioimpedance analysis,
	SmartBucket (seca mVSA 535	vital signs)
14	only)	For storing consumables
	Silly)	Data transmission: Monitor/measuring mat
		Charge measuring mat rechargeable battery
15	Connecting cable with	For power supply and data transmission between monitor and
	USB connector	
16	Connection for temperature	For COVIDIEN™ FILAC™ 3000 temperature probes
10	measurement	<ul> <li>Blue: Oral/axillary measurement</li> <li>Red: Rectal measurement</li> </ul>
		For COVIDIEN <sup>™</sup> FILAC <sup>™</sup> 3000 temperature probes
17	Probe holder	Blue: Oral/axillary measurement
		Red: Rectal measurement
	Storage compartment for probe	
18	covers	Capacity: 2 packs for COVIDIEN™ FILAC™ 3000
19	Removable magazine holder for	Capacity: 2 magazines for COVIDIEN™ GENIUS®3
	probe covers	
20	Thermometer compartment	For COVIDIEN™ GENIUS®3 in-ear thermometer
21	Connection for temperature measurement	For COVIDIEN <sup>™</sup> GENIUS <sup>®</sup> 3 in-ear thermometer
22	Sensor holder	For SpO <sub>2</sub> sensor
		Depending on version, to fit:
23	Connection for SpO <sub>2</sub>	<ul> <li>Masimo SET<sup>®</sup> patient cables and sensors (not illustrated)</li> </ul>
-	measurement	<ul> <li>seca patient cables and sensors (shown in illustration)</li> </ul>
04	Connection for blood pressure	
24	measurement	For seca blood pressure cuffs


## NOTE

The illustration shows one equipment example. The actual functional scope of your device may deviate from this.

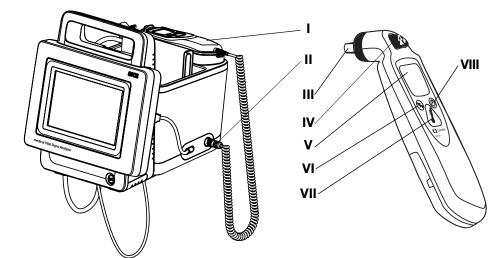


No.	Control	Function
I	Thermometer	COVIDIEN <sup>™</sup> GENIUS <sup>®</sup> 3 in-ear thermometer
П	Connection for temperature measurement	For COVIDIEN™ GENIUS®3 in-ear thermometer
111	Measuring head	For measuring temperature in ear
IV	Discard button	Discard probe cover
V	In-ear thermometer display	Serves as a secondary display. seca mVSA 535 display takes priority
VI	Switch unit button	Switch between °C and °F
VII	Measure button	Press button to start measurement
VIII	Pulse timer button	Not relevant for <b>seca mVSA 535</b> . Pulse rate is determined automatically by the <b>seca mVSA 535</b>

# 4.3 Measuring mat controls (bioimpedance analysis)

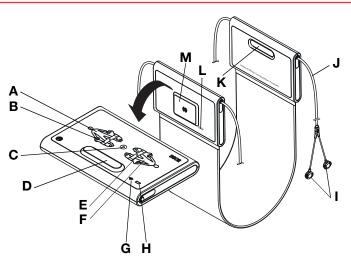

No.	Control	Function	
Α	LEDs, electrodes on left	LED green: Electrode contact good	
		LED green, flashing: Measurement in progress	
		LED red: Electrode contact poor	
		Enter patient position	
в	Button with LED,	LED green: seca patient file loaded, patient position selected	
D	patient position left	LED green, flashing: Measurement stopped	
		LED red: No seca patient file loaded or faulty measurement	
С	Start button	Start measurement	
D, K	Magnetic catch	Fold up measuring mat for transport/storage	
		Enter patient position	
Е	Button with LED,	LED green: seca patient file loaded, patient position selected	
-	patient position right	LED green, flashing: Measurement stopped	
		LED red: No seca patient file loaded or faulty measurement	
		LED green: Electrode contact good	
F	LEDs, electrodes on right	LED green, flashing: Measurement in progress	
		LED red: Electrode contact poor	
	LED, WiFi	LED green: WiFi connection to monitor OK	
G		LED green, flashing: WiFi connection being set up	
, u		LED red: No WiFi connection to monitor	
		LED off: WiFi deactivated	
		LED green: Rechargeable battery full	
н	LED, charge status	LED green, flashing: Rechargeable battery almost discharged, charge as	
••		soon as possible	
		LED red: Rechargeable battery discharged	
	Electrode connections	Connect to electrodes	
J	Electrode cables	Transmit signals from electrodes to the measuring mat	
L	Infrared interface	Automatic data transmission: Monitor/measuring mat	
-		Alternative if no WiFi available	
М	Inductive charging interface with magnetic catch	Charge measuring mat rechargeable battery	

## 4.4 Fields in the touchscreen display

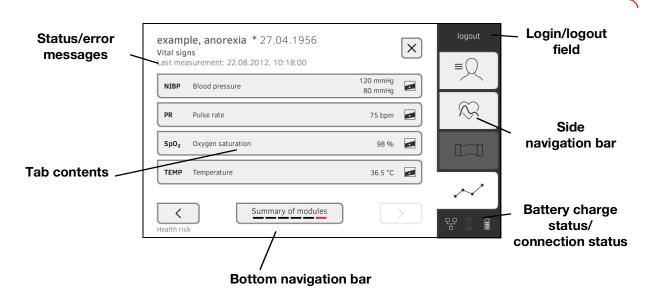

4.5 Login/navigation: Buttons and symbols in the touchscreen display

Button/symbol	Meaning
≡	Patient tab
$\bigotimes$	Vital signs tab
	BIA tab
$\sim$	Analysis tab
i	Open instructions for use
	Instructions for use: Return to section summary
	Enter text or numbers
	No input or input faulty
	Select user account
9	Enter password

17-10-05-353-002h\_2023-01 B

Button/symbol	Meaning
$\checkmark$	<ul> <li>Navigation: Confirm entry</li> <li>Measuring: Save measurement procedure</li> </ul>
	Process running
create	Button available
create	Button pressed
create	Button not available
$\langle \rangle$	Navigate to the left/right
$\sim$ $\sim$	Navigate up/down
✓	One or more items from list selected/not selected
	Alternative from list selected/not selected
×	Return to previous screen
Login	Log in user
Logout	Log out/switch user

## 4.6 Operating state: Symbols

- → Operating state: Symbols in the touchscreen display
- → Operating state: Symbols on the measuring mat

# Operating state: Symbols in the touchscreen display

Symbol	Operating state
	Monitor: Controls permanently on: Rechargeable battery charge status Controls flashing: Rechargeable battery charging
	Measuring mat: Controls permanently on: Rechargeable battery charge status Controls flashing: Rechargeable battery charging

Symbol	Operating state
Î	Monitor: Controls permanently on: Rechargeable battery full Controls flashing consecutively: Rechargeable battery charging
Î	Monitor: Rechargeable battery discharged
	Measuring mat: Controls permanently on: Rechargeable battery full Controls flashing consecutively: Rechargeable battery charging
	Measuring mat: Rechargeable battery discharged
몽몽	LAN connection set up: on/off
() ()	WiFi connection set up: on/off
琚 📀	seca connect 103 (EMR)/seca analytics 115: connection active
무 资	seca connect 103 (EMR)/seca analytics 115: connection interrupted
묻 📚	Connection to <b>seca analytics 115</b> : Synchronization in progress
	Measuring mat detected
	Measuring mat not detected
í	Pop-up window: Information for the user
$\overline{\times}$	Pop-up window: Error message
0	Pop-up window: Setting option for the user
	Error message during vital signs measurement

# Operating state: Symbols on the measuring mat

Symbol	Operating state
	LED green: Rechargeable battery full Green, flashing: Charge rechargeable battery as quickly as possible
	LED red: Rechargeable battery discharged
•(((+	LED green: WiFi available Green, flashing: WiFi connection being set up
•(((+	LED red: WiFi not available LED off: WiFi deactivated

# 4.7 Measuring: Buttons and symbols

- → Patient tab
- → Vital signs tab
- → BIA tab
- → Measuring mat
- → Analysis tab

Patient tab

$-\bigcirc$	Button/symbol	Meaning
=	9	Search for seca patient file
	• •	Switch sorting direction
		Edit seca patient file
		Export seca patient file
		Delete seca patient file
		Weight (W)
	<u>†</u>	Height (H)
	Ĵĸ	Waist circumference (WC)
	×.	Physical Activity Level (PAL)
	360°	Adopt seca 360° wireless value

Button/symbol	Meaning
$\checkmark$	Save value
Q	Adopt value from previous measurement
×	Close seca patient file, changes will not be saved

# Vital signs tab

 $\mathcal{N}$ 

Button/symbol	Meaning
NIBP	Blood pressure measured non-invasively
$\bigcirc$	Start blood pressure measurement
	Blood pressure measurement in progress
1. <del>117/77</del> 2.116/63 3.123/80	Blood pressure: Multiple measurement: First measurement will be discarded
	Blood pressure: Multiple measurement: Waiting time until next part-measurement running
SYS/DIA	Blood pressure: Systolic/diastolic pressure
МАР	Blood pressure: Mean arterial pressure
* *	Blood pressure: Inflation, deflation
• :	Blood pressure: Single measurement, multiple measurement
TEMP	Temperature
▶	Temperature measuring mode: Predictive, direct
Ŀ	Manual temperature input (Only for device versions without integrated temperature measurement)
°m° mm° mm°	Temperature input, measuring method oral, axillary, rectal
ີງ <u>~</u> ⊮••	in-ear, on the skin, contactless
<b>.</b>	COVIDIEN <sup>™</sup> FILAC <sup>™</sup> 3000 temperature probe Measuring position: Oral, axillary, rectal

17-10-05-353-002h\_2023-01 B

Button/symbol	Meaning
9	COVIDIEN <sup>™</sup> GENIUS <sup>®</sup> 3 in-ear thermometer Measuring position: in-ear
PR	Pulse rate
♥ ≑♥	Measuring range for pulse rate: Standard, extended
SpO <sub>2</sub>	Oxygen saturation in %
РІ	Devices with Masimo SET <sup>®</sup> measuring equipment: Perfusion index, information on the quality of perfusion (min: 0.02 %, max: 20 %)
	Devices with Masimo SET <sup>®</sup> measuring equipment, oxygen saturation measuring mode: Normal, APOD, maximum
<u>^</u> ^^*	Devices with seca measuring equipment, oxygen saturation measuring mode: Stable, standard, sensitive
$\checkmark$	Save measurement procedure
×	Discard measurement

BIA tab

Button/symbol	Meaning
•	Enter patient position
	Start measurement
$\bigcirc$	Measurement in progress
	Permanently on: Electrode OK Flashing: Measurement in progress
×	Electrode not OK or implausible measured values
8	Skin contact error black electrodes
$\overline{\mathbf{x}}$	Skin contact error red electrodes
$\bigcirc$	Measurement successful

Button/symbol	Meaning
	Write comment
$\checkmark$	Save measurement procedure
×	Discard measurement

# Measuring mat

Button/symbol	Meaning
ŀ	Enter patient position LED green: seca patient file loaded, patient position selected LED green, flashing: Measurement stopped LED red: No seca patient file loaded
$\odot$	Start measurement
•	Electrode LEDs: LED green: Electrode OK LED green, flashing: Measurement in progress
•	Electrode LEDs: LED red: Electrode not OK

## Analysis tab

	Button/symbol	Meaning
$\sim$		View history
		Forward results report for printing (seca directprint: Function of the seca analytics 115 PC software)
		Position indicator for analysis modules, in this case: Second module of 5
		Position indicator for analysis parameters, in this case: Second analysis parameter of 4
		Detail view available for analysis parameter: Bar chart
		Detail view available for analysis parameter: Percentile curve
	•	Detail view available for analysis parameter: Tolerance ellipse
	• •	Parameter-dependent symbols, red: Value outside normal range
	• •	Parameter-dependent symbols, gray: Value within normal range

17-10-05-353-002h\_2023-01 B

Button/symbol	Meaning
28.6 kg/m <sup>2</sup>	Text red: Value outside normal range
15.3 kg/m²	Text black: Value within normal range
3.5 low norms increased high 1.5 5.7 9.0	Green: Value within normal range
7.6 kow nomma horsened high 15 57 9.0	Orange: Value elevated
12.9 kw normal increased bigh 3.8 8.7 12.4	Red: Value outside normal range
Average (NIBP)	Blue marking: Mean value
ď	Male
Ŷ	Female

# 4.8 Unsubmitted measurements list: Symbols

Button/symbol	Meaning
凸	Weight (W)
$\uparrow$	Height (H)
	Waist circumference (WC)
×.	Physical Activity Level (PAL)
$\stackrel{\uparrow}{\longrightarrow}$	Body mass index (BMI)
-\/-	Pulse rate (PR)
$(\cdot)$	Blood pressure (NIBP)
( <sup>0</sup> 2)	Oxygen saturation (SpO <sub>2</sub> )
J	Temperature (Temp)
Average (NIBP)	Blue marking: Mean value
FN222225852 FN222225852	Yellow marking (when connected to EMR system): Unconfirmed offline measurement

# 4.9 Markings on the device and on the type plate

Text/symbol	Meaning
	Name and address of manufacturer, date of manufacture
REF	Model number
SN	Serial number, consecutive
ProdID	Product identification number, consecutive
Mat.No.	Variant number
	Follow Instructions for Use
	Do not use device on individuals with cardiac pacemakers or implanted defibrillators
<b>†</b>	Medical electrical device, type BF
⊣★⊦	Medical electrical device, type BF (defibrillation-protected)
$\bowtie$	No alarm function
Li-ion	Lithium-ion rechargeable battery
<b>CE</b> 0123	Device complies with EC directives • 0123: appointed office for medical devices
F©	Symbol of the US Federal Communications Commission (FCC)
FCC ID	Device license number from the Federal Communications Commission (FCC)
IC	Device license number from Industry Canada
xxx-yyy V ~ min xx-yy Hz xx A	<ul> <li>Type plate for power supply socket:</li> <li>Permitted supply voltage</li> <li>Permitted power supply frequency</li> <li>Power consumption</li> </ul>
⊙/Ċ	ON/OFF key
	Inductive charging interface
모모	LAN interface
●	USB interface
NIBP	Connection of blood pressure cuff

17-10-05-353-002h\_2023-01 B

Text/symbol	Meaning
TEMP	Connection of temperature sensor
SpO <sub>2</sub>	Connection of SpO <sub>2</sub> sensor
X	Do not dispose of device with household waste

# 4.10 Markings on the blood pressure cuff

Text/symbol	Meaning
i	Follow instructions for use
ÊÊÊÊÊ XSSM () XL XXL	Cuff size (here: L)
size L © 32-42 cm	Cuff suitable for specified arm size
	Artery position: These arrows must be on the brachial or femoral artery when blood pressure cuff is put on.
	Cuff end: These markings must be within the adjustment range when closing the blood pressure cuff.
	Adjustment range: The marking "Cuff end" must be in this range when closing the blood pressure cuff. This area also contains the cuff size (here: L).
LATEX	Latex-free
<b>(</b> €	The blood pressure cuff complies with EU directives

# 4.11 Markings on the packaging

Ť	Protect from moisture
<u><u><u></u></u><u></u><u></u><u></u><u></u></u>	Arrows indicate top of product Transport and store in an upright position
	Fragile Do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. moisture for transport and storage
NON	Not sterile
$\otimes$	Do not reuse
<b>†</b> †	Open packaging here
Ó	Packaging material can be disposed of through recycling programs

# 4.12 Other symbols

Text/symbol	Meaning
NON	Not sterile
$\otimes$	Do not reuse
R	Small parts present a danger of suffocation if swallowed.
*	Keep out of sunlight
	Only use indoors
$(( ( \bullet )))$	Non-ionizing radiation
DEHP	DEHP-free
LATER	Latex-free
MR	Not resistant to magnetic resonance
	Do not use if packaging damaged
LOT	Lot number
EC REP	Person with authority in the EU

Depending on device version, the symbols below may be applied to accessories and consumables and their packaging.


# 5. STARTING UP THE DEVICE

- → Scope of delivery
- → Setting up device
- → Establishing a power supply
- → Charging rechargeable batteries
- → Setting the date and time
- → Initial login
- → Configuration options

#### 5.1 Scope of delivery

## NOTE

This section shows version 535-3110-001 by way of an example. The scope of delivery of your device may deviate from this. An overview of versions can be found at www.seca.com.

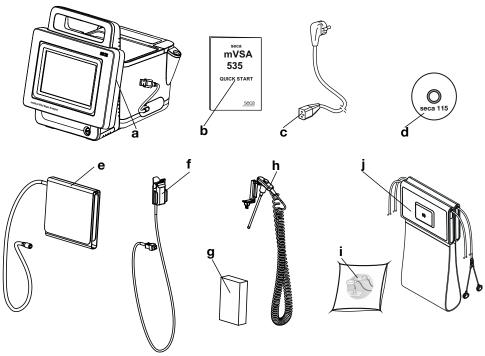

No.	Standard scope of delivery	Pcs.
а	Monitor to suit the version ordered	1
b	Quick Start brief instructions, printed	1
С	Power supply cable (country-specific)	1-3
-	Accessories to suit the version ordered	-

No.	Accessory for the version shown	Pcs.
d	DVD with seca analytics 115 PC software and license for	1
u	one permanent workstation	I.
е	seca blood pressure cuff, size M	1
£	<ul> <li>SpO<sub>2</sub> sensor (Masimo SET<sup>®</sup> or seca)</li> </ul>	1
•	<ul> <li>Patient cable (Masimo SET<sup>®</sup> or seca), not illustrated</li> </ul>	1
a	COVIDIEN <sup>™</sup> FILAC <sup>™</sup> 3000 probe covers	1
g	(pack of 20)	

17-10-05-353-002h\_2023-01 B

No.	Accessory for the version shown	Pcs.
h	COVIDIEN™ FILAC™ 3000 temperature probe	1
i	Push-button electrodes for affixing to patient (pack of 100)	1
j	Measuring mat with rechargeable battery and electrode cables	1

## 5.2 Setting up device

- → Connecting the SmartBucket
- → Connecting the blood pressure cuff
- $\rightarrow$  Connecting the temperature probe
- → Connecting the in-ear thermometer
- $\rightarrow$  Connecting the seca SpO<sub>2</sub> sensor
- → Connecting the Masimo SET<sup>®</sup> SpO<sub>2</sub> sensor
- → Suspending the measuring mat in the magnetic catch

#### NOTE

This section shows how to assemble all of the measuring accessories available for this device. The actual scope of delivery of your device may be less than this.

#### NOTICE!

#### Damage to device, malfunction

Excessive use of force may damage tubes and cables.


- In order to connect measuring accessories to the device or remove them from it, take hold of tubes only by the tube coupling.
- In order to connect measuring accessories to the device or remove them from it, take hold of cables only by the connectors.
- Use only measuring accessories which have no externally visible damage.

## NOTICE!

## Malfunction

The SmartBucket needs one of the USB interfaces for communication and power supply. If the USB connection is removed, no vital signs can be measured.

- Always keep the SmartBucket connected to the USB interface.
- Only connect accessories, for example a USB memory stick, to the other USB interface.

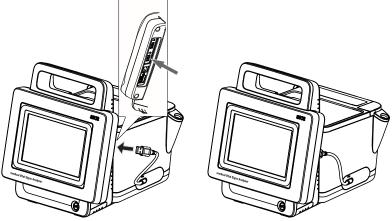


 Connect the USB cable of the SmartBucket to a free USB interface of the monitor.


## **Connecting the SmartBucket**

## Connecting the blood pressure cuff

1. Connect the tube coupling for the blood pressure cuff to the compressed air connection of the device until you hear the tube coupling engage.

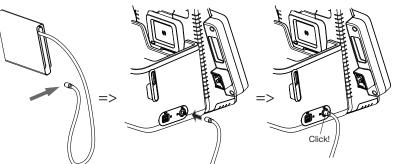


2. Stow the blood pressure cuff in the SmartBucketas shown in the illustration below.

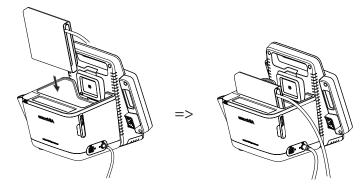


## Connecting the temperature probe

- 1. Open the cover cap of the connection compartment.
- 2. Insert the connector for the temperature probe in the probe connection completely as shown in the illustration below.
- 3. Close the cover cap of the connection compartment.

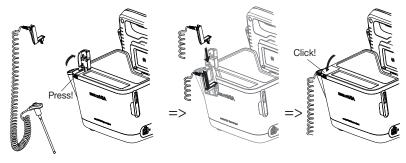

4. Push the temperature probe into the probe holder completely as shown in the illustration below.

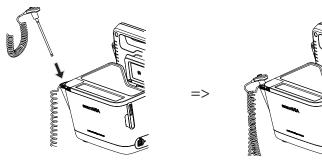

# NOTICE!

## Damage to device, malfunction


The temperature probe can only be pushed right into the probe holder if it does not have a probe cover on.

• Ensure that there is no probe cover on the temperature probe.

#### NOTE

The storage compartment on the probe connection provides space for two packs of probe covers.

#### Connecting the in-ear thermometer

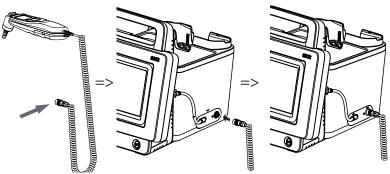


1. Put the connector for the in-ear thermometer in the socket on the Smart-Bucket until you feel the connector engage.

#### NOTE

The magazine holder in the thermometer compartment provides space for two probe cover magazines.

2. Place the in-ear thermometer in the thermometer compartment as shown in the illustration below.

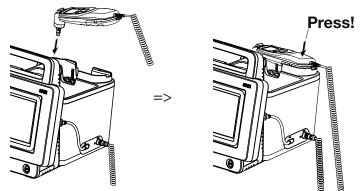

## Connecting the seca $SpO_2$ sensor


## NOTICE!

## Damage to device, malfunction

The  $\mbox{SpO}_2$  sensor must be compatible with the seca  $\mbox{SpO}_2$  measuring equipment fitted.

- Ensure that the SpO₂ sensor is compatible with the SpO₂ measuring equipment fitted in your device → Optional accessories and spare parts.
- ► Follow the user documentation from the sensor manufacturer.
- 1. If necessary, connect a patient cable to the SpO<sub>2</sub> sensor as described in the user documentation from the sensor manufacturer.

2. Put the connector for the SpO<sub>2</sub> sensor/the patient cable in the socket on the SmartBucket until you feel the connector engage.

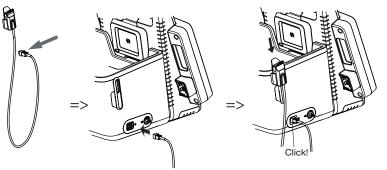

#### NOTE


The holder above the  $SpO_2$  connection is for storing the  $SpO_2$  sensor.

#### NOTICE!

#### Damage to device, malfunction

The SpO<sub>2</sub> sensor must be compatible with the SpO<sub>2</sub> measuring equipment fitted.

- Ensure that the SpO₂ sensor is compatible with the SpO₂ measuring equipment fitted in your device → Optional accessories and spare parts.
- ► Follow the user documentation from the sensor manufacturer.
- 1. If necessary, connect a patient cable to the SpO<sub>2</sub> sensor as described in the user documentation from the sensor manufacturer.
- 2. Put the connector for the SpO<sub>2</sub> sensor/the patient cable in the socket on the SmartBucket until you feel the connector engage.

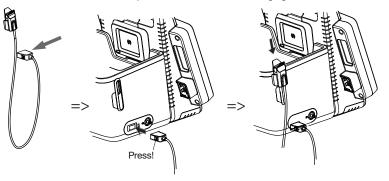

NOTE

The holder above the  $SpO_2$  connection is for storing the  $SpO_2$  sensor.

Connecting the Masimo SET<sup>®</sup> SpO<sub>2</sub>

sensor


Suspending the measuring mat in the magnetic catch

# NOTICE!


# Malfunction

The measuring mat rechargeable battery is only charged via the inductive charging interface of the monitor.

- Ensure that the inductive charging interface is not covered by other measuring accessories such as a blood pressure cuff, for example.
- After every measurement, suspend the measuring mat back in the magnetic catch. This ensures that the measuring mat rechargeable battery is always adequately charged.
- 1. Fold up the measuring mat as shown in the illustration below.

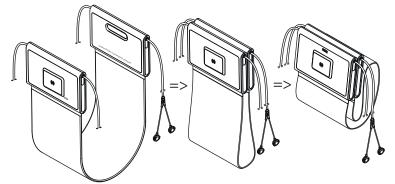

2. Suspend the measuring mat in the magnetic catch as shown in the illustration below.

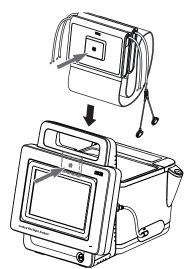

3. Ensure that the measuring mat is correctly located in the magnetic catch of the monitor.





- 1. Plug the device connector of the power supply cable into the connecting socket of the device.
- 2. Plug the power supply connector into a power supply socket.

# 5.4 Charging rechargeable batteries

Before you start up the device for the first time, the rechargeable batteries for the monitor and the measuring mat – if the measuring mat is present– must be fully charged.



- Ensure that the measuring mat if the measuring mat is present– is correctly suspended in the magnetic catch of the monitor → Suspending the measuring mat in the magnetic catch.
   Connect the device to the power supply → Establishing a power supply.
- 2. Connect the device to the power supply  $\rightarrow$  Establishing a power supply.
- Press the ON/OFF button of the monitor. The LED of the ON/OFF button is white. The charging process starts. Current charge status is displayed for approx. 15 seconds:





After approx. 5 minutes, the device switches to standby. The screen goes off.

The LED of the ON/OFF button flashes green.


When the rechargeable batteries are fully charged, the device switches off.

The LED of the ON/OFF button goes off.

# NOTE

Leave the device connected to the power supply for approx. 4 hours when starting it up for the first time. This ensures that the rechargeable batteries for the monitor and the measuring mat are fully charged.

# 5.5 Setting the date and time

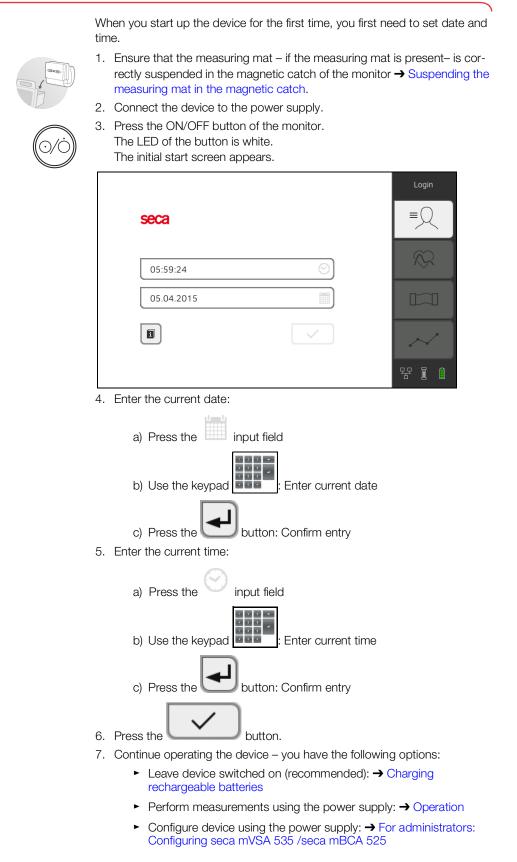

The following user accounts are available on the device initially:

- Admin: Configure and manage device
- User: Perform and manage measurements
- Service: For authorized service technicians

On initial login, these user accounts must be activated and configured.

Additional user accounts can only be set up in the **seca analytics 115** PC software and must be synchronized with the device.

Further information is available here:  $\rightarrow$  For administrators: Configuring seca mVSA 535 /seca mBCA 525  $\rightarrow$  User accounts

# 5.7 Configuration options

The device can only be configured by users with administrator rights. Further information is available here:  $\rightarrow$  For administrators: Configuring seca mVSA 535 /seca mBCA 525

# 6. OPERATION

- → Switching the system on/off
- → Managing seca patient files
- → Measuring vital signs
- → Measuring bioimpedance
- → Adding weight and height
- → Analyzing measurement

# 6.1 Switching the system on/off

- → Switching on
- → Logging in
- → Logging out/switching user
- → Saving power/switching off

# Switching on



- Ensure that the measuring mat if the measuring mat is present– is correctly suspended in the magnetic catch of the monitor → Suspending the measuring mat in the magnetic catch.
- Press the ON/OFF button of the monitor. The LED of the ON/OFF button is white. The device starts up. This takes a few seconds.

No patient file lo	baded	Login
No user logged in SpO <sub>2</sub> No Sensor compared		
	NIBP mmHg SYS/DIA ∗ ŝ	
	MAP	,∕≫
TEMP °C		
		$\sim$
<b>▲</b> kg	[ <u>t</u> m]	뭥 (1

The Vital signs tab is displayed.

The measuring mat – if there is one – switches on automatically. You have the following options:

- ► Measure vital signs (without patient identification) → Measuring vital signs
- ► Use full functional scope: Log in → Logging in

# Logging in

You must log in to the device if you want to do one of the following:

- Assign vital signs measurement to a seca patient file
- Perform a bioimpedance measurement
- Analyze measuring results
- Manage the system

# 1. Press the **Login** button.

The login window is displayed:

seca		Login
		$\sim$
	0	
	$\checkmark$	$\sim$
		물 (1)

2. Press the \_\_\_\_\_ input field. The list of user accounts is displayed.

### NOTE

If you have no user account, please contact your administrator.

- 3. Press your user account. Your user account is displayed in the input field.
- input field. 4. Press the



- Enter your password using the keypad
   The Patient tab is displayed
- 6. Continue with  $\rightarrow$  Managing seca patient files.

# Logging out/switching user

• Press the **Logout** button.

	example, weight reduction * 01.08.1961			
	asurement: 12.01.2016, 16:01:00		≡◯	
NIBP	Blood pressure	129 mmHg 70 mmHg		
PR	Pulse rate	70 1/min	$\sim$	
Sp02	Oxygen saturation	-		
TEMP	Temperature	- 🜌		
K Health ris	Summary of modules	>	·····································	

You will be logged out.

The login window is displayed.

Another user can log in  $\rightarrow$  Logging in.

# Saving power/switching off



- 1. Briefly press the ON/OFF button of the monitor.
- 2. The **Save energy** dialog window appears.

í	Save energy
	Screen off
	Standby
	Shut down
	Cancel

3. Select an option in accordance with the table:

# NOTE

The device automatically switches to standby/shuts down if it is not used for an extended period (phased by idle time)

Idle time button	Device characteristics	To remove setting
<ul> <li>Display switches off</li> <li>The LED of the ON/OFF button is white.</li> <li>The measuring mat – if present – stays switched on</li> <li>Entries are retained</li> <li>User remains logged in</li> <li>Patient file remains active</li> <li>Measurements continue to run</li> </ul>		Press screen
<ul> <li>Display switches off.</li> <li>The LED of the ON/OFF button is green</li> <li>Data which have not been saved are lost</li> <li>User is logged out</li> <li>Patient file is deactivated</li> <li>The measuring mat – if there is one – switches off</li> </ul>		Press screen

Idle time button	Device characteristics	To remove setting
Shut down 20 minutes	<ul> <li>Power supply operation:</li> <li>The charging process for the rechargeable batteries for the device and the measuring mat – if the measuring mat is present – starts automatically</li> <li>When the charging process is complete: Device shuts down and switches off</li> <li>Rechargeable battery operation:</li> <li>Device shuts down</li> <li>The device switches off</li> </ul>	Press the ON/OFF button

# NOTE

If rechargeable battery charge status undershoots a preset limit value [%], the device shuts down and switches off. It behaves in this way regardless of the option you select in the **Save energy** dialog. The limit value for rechargeable battery charge status is specified by your administrator:  $\rightarrow$  Power management.

# 6.2 Managing seca patient files

- → Creating a seca patient file
- → Calling up a seca patient file
- → Editing a seca patient file
- → Exporting a seca patient file
- → Deleting a seca patient file

# NOTICE!

#### Inconsistent measuring results

If you create seca patient files several times, this can lead to incorrect assignment of measuring results and falsify the analysis.

- If there is no seca patient file on the device for a patient, check whether there is already a seca patient file in the seca analytics 115 PC software.
- ► If a seca patient file for this patient exists in the seca analytics 115 PC software, check the synchronization settings → Activating automatic synchronization.
- Only create a new seca patient file on the device if you are sure that there is no seca patient file for the patient in the seca analytics 115 PC software.

# Creating a seca patient file

- 1. Log in  $\rightarrow$  Logging in.
- 2. Press the Patient tab.

The patient list is displayed.

Pati	ent list	Create patient file	Logout
	arch	P	] ≡Ջ
-	Name	Date of birth	
ę	example, healthy Asian	15.11.1977	
ď	example, healthy Afro-American	27.06.1971	
ď	example, weight reduction	01.08.1961	
ę	example, anorexia	27.04.1956	$\sim$
ď	example, obesety	14.02.1967	·] · · · · · · · · · · · · · · · · · ·

3. Press the Create patient file button.

Create patient file Page 1 of 2		×
Anonymous patient		
First name	Attending physician	
	Not assigned	-
Surname	ID	
		>

#### NOTICE!

#### **Restricted function**

If you press the Anonymous patient field, the first input screen is skipped and an anonymous patient file is created. It is not possible to display any measured value graphs (histories) in anonymous patient files.

4. Enter the first name and surname of the patient: ×.

a)	Press thei	nput field
b)	Use the keypad	Enter text
C)	Press the	outton: Confirm entry

#### NOTICE!

#### Data access by unauthorized persons

If you do not enter an attending physician, the patient file is accessible to all physicians with access to the device.

- ► Enter an attending physician to prevent access by unauthorized persons. If the attending physician has not been determined at the time of the measurement, make the entry as soon as possible afterwards.
- 5. Enter the attending physician:



- b) Press the name of the attending physician in the drop-down menu
- c) Enter patient ID

### NOTE

You only need to assign a patient ID if a specific structure is provided for this in your institution. If you leave the ID input field empty, the device will automatically assign an ID when the data are saved.

6.	Press the button. The second page of the <b>Create pati</b>	<b>ent file</b> dialog is displayed.			
	Create patient file Page 2 of 2	×			
	Date of birth	Ethnicity			
	DD.mm.YYYY	O Caucasian			
	Gender	O Asian			
	O d' Male	O African American			
	○ ♀ Female	O South and Central American			
		O Other			
	<				
7.	Enter the date of birth:				
	a) Press the piput field				
	b) Use the keypad	r date			
	c) Press the button: Cont	firm entry			
8.	3. Enter the appropriate gender.				
9.	9. Enter the appropriate ethnicity.				
10.	Press the <b>Save</b> button.				
	The seca patient file has been created	d and is displayed.			



You have the following options for continuing:


- ► Measure vital signs: → Measuring vital signs
- ► Measure bioimpedance: → Measuring bioimpedance

# Calling up a seca patient file

- 1. Log in  $\rightarrow$  Logging in.
- Press the Patient tab. The patient list is displayed.

Patie	ent list	Create patient	file	Logout
sea	arch		9	≡
-	Name	Date of birth		
ę	example, healthy Asian	15.11.1977		
ď	example, healthy Afro-American	27.06.1971		
ď	example, weight reduction	01.08.1961		
ę	example, anorexia	27.04.1956		$\sim$
ď	example, obesety	14.02.1967	$\sim$	
				명 🌔

- 3. Select a seca patient file:
  - Desired entry not visible: Continue at step 4.
  - Desired entry visible: Continue at step 5.
- 4. Search for the desired seca patient file in the list:

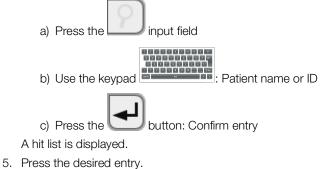

- The selected seca patient file appears in the **Patient information** dialog field.
- 6. Press the **Confirm** button.

The seca patient file is opened.

You have the following options for continuing:

- ► Measure vital signs: → Measuring vital signs
- ► Measure bioimpedance: → Measuring bioimpedance
- 1. Call up a seca patient file  $\rightarrow$  Calling up a seca patient file.

3. Edit the entries on the first page.



- 5. Edit the entries on the second page.
- 6. Press the **Save** button. The changes are saved.

Editing a seca patient file

# Exporting a seca patient file

- 1. Connect a USB memory stick to the monitor.
- 2. Call up a seca patient file  $\rightarrow$  Calling up a seca patient file.



4. Press the **Confirm** button. The patient file is exported.

# Deleting a seca patient file

1. Call up a seca patient file  $\rightarrow$  Calling up a seca patient file.



- A confirmation dialog opens.
- 3. Press the **Delete** button. The patient file is deleted.

# 6.3 Measuring vital signs

- → Introduction
- → Measuring blood pressure
- → Measuring temperature, oral/axillary (COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000, blue)
- → Measuring temperature, rectal (COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 red)
- → Measuring in-ear temperature (COVIDIEN<sup>TM</sup> GENIUS<sup>®</sup>3)
- → Entering temperature manually
- → Reading off pulse rate
- → Measuring oxygen saturation (SpO<sub>2</sub>)
- → Stopping a measurement procedure
- → Assigning anonymous measurement procedure to a seca patient file


# WARNING!

# A Hazard to patient, malfunction, damage to device

- Set up the device so that it cannot fall on patients.
- Route the cables and tubes of the measurement accessory so that the patient cannot become entangled or be strangled.
- The device has **no** alarm function. Never leave the patient unobserved during a measurement.
- Always connect only one individual patient to the device for each measurement procedure.

The Vital signs measurement function is available after the device is switched on. You can perform measurements without login and without patient identification.

If you want to assign the measurement procedure to a patient file, we recommend calling up  $\rightarrow$  Calling up a seca patient file or creating  $\rightarrow$  Creating a seca patient file a patient file **before** the measurement. This applies in particular when repeat measurements for a patient are likely.

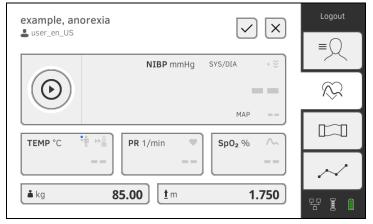

You can measure one or more vital signs in one measurement procedure. Measurement starts automatically after the measuring accessories have been attached to the patient (exception: blood pressure measurement and in-ear temperature measurement). Depending on device configuration, pulse rate is determined using either oxygen saturation or blood pressure.

Weight and height can be recorded by compatible seca measuring devices or entered manually:  $\rightarrow$  Adding weight and height.

Introduction

### Measuring blood pressure

- → Preparing the blood pressure measurement
- $\rightarrow$  Starting a single measurement
- $\rightarrow$  Starting a multiple measurement
- → Modifying presets



- Do not use luer lock adapter on the blood pressure measurement tubes. The use of luer lock adapters can lead to accidental connection of the blood pressure tubes to intravenous lines and thus lead to an infusion of air into the bloodstream of the patient.
- The decision to use this device with pregnant or preclamptic patients is the responsibility of the user.
- ► The device has **no** alarm function. Never leave the patient unobserved during a measurement.
- Frequent measurements may lead to perfusion disorders and consequently to severe harm to the patient.
- Route the compressed air tube so that it cannot kink. A kinked compressed air tube leads to sustained cuff pressure. This may lead to perfusion disorders and consequently to severe harm to the patient.
- Do not place the blood pressure cuff over open wounds. This may lead to further harm to the patient.
- Do not apply any external pressure onto the blood pressure measurement tubes or onto the blood pressure cuff.
- For patients with moderate to severe cardiac arrhythmias, inaccurate measurements may result with the blood pressure measurement.
- ► The following factors may affect the measurement result:
  - Measuring location (e.g. altitude)
  - Patient position (standing, sitting, lying)
  - Physiological state of the patient (e.g. exertion, movement, shaking, shivering)
  - Patient's age
  - Arteriosclerosis
  - Poor perfusion
  - Diabetes
  - Kidney disease
- If the measurements seem implausible, check and evaluate the vital signs of the patient using alternative means. Then check the measurement function of the device with the help of the section "Troubleshooting".

## WARNING!

#### Patient hazard, incorrect measurement

- Do not place the blood pressure cuff on the arm on the side on which a mastectomy or lymph node removal has been performed.
- Apply the blood pressure cuffs so that the blood circulation of the patient is not compromised.
- Do not apply the blood pressure cuffs on spots that have weak circulation or on extremities on which intravenous routes lie.
- High cuff pressures may be unpleasant for sensitive patients. Keep the patient's general condition under observation during the measurement.

# WARNING! Incorrect measurement

- Inflating the blood pressure cuff may lead to occasional malfunctions of other medical electrical devices used on the same limb.
- Use only blood pressure cuffs from seca.
- Before every measurement, ensure that the blood pressure mea-► surement tubes and connections are free of damage and airtight.
- Ensure that the arm on which the blood pressure cuff is attached does not move during the measurement.
- Ensure that the blood pressure cuff is applied according to the ► printed mark "Artery".




#### CAUTION! Incorrect measurement

If the blood pressure cuff is too mall, the measured blood pressure values will be too high. If the blood pressure cuff is too large, the measured blood pressure values will be too small.

Always use a blood pressure cuff that is the correct size.

# Preparing the blood pressure measurement

- 1. Ensure that the patient adopts the following position:
  - Sitting comfortably
  - Legs not crossed
  - Feet flat on the floor
  - Back and arm supported
- 2. Apply the blood pressure cuff to the patient's non-dominant arm as shown in the illustration below:
  - a) Note the labeling on the blood pressure  $cuff \rightarrow Markings$  on the blood pressure cuff
  - b) Use correct size of blood pressure cuff: End of cuff inside the adjustment range when put on
  - c) Position the blood pressure cuff at the level of the right atrium
  - d) Ensure correct fit of the blood pressure cuff
  - e) Route compressed air tube so that it cannot kink

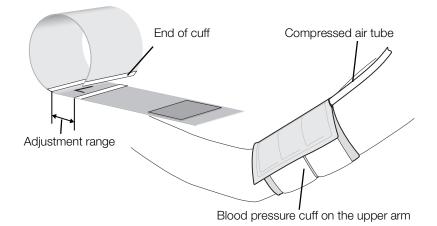

# NOTE

Get the patient to sit quietly for 5 minutes before recording measured values. The patient should remain relaxed and not talk during the measurement.

You have the following options for continuing:

- ► → Starting a single measurement
- ► → Starting a multiple measurement

#### NOTICE!

#### Faulty operation, missing measuring results

Results from multiple measurements cannot be transmitted to EMR systems or the **seca analytics 115** PC software.

Switch to Single measurement, if necessary, if you want to transmit blood pressure values to an EMR system or the seca analytics 115 PC software → Modifying presets.

#### NOTE

Your administrator will specify whether Single measurement or Multiple measurement is available when the device starts up. You can modify this and other presets for the current measurement procedure: → Modifying presets. After this, the settings specified by the administrator will be active again.

#### Starting a single measurement

- Press the button on the monitor.
- Blood pressure measurement starts:



Current cuff pressure is displayed.

The symbols for measurement procedure and measuring mode flash (in this case: Single measurement, inflation).

Measurement stops automatically as soon as valid blood pressure values are detected.



The values for systolic/diastolic blood pressure **SYS/DIA** and mean arterial pressure **MAP** are displayed.

You have the following options for continuing:

- ► Measure more vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

#### NOTE

• Using the Start button, you can cancel and restart blood pressure measurement at any time.

 If inflation fails to deliver a measured value, the device automatically switches to deflation. If necessary, the device re-pumps several times during the deflation (re-pumping: Increasing cuff pressure by approx. 50 mmHg followed by incremental releasing of that pressure). If no measured value is delivered even after several re-pumping steps, the procedure is canceled.

#### Starting a multiple measurement

Press the button on the monitor.
 The first part-measurement starts (in this case: 6 part-measurements).



Current cuff pressure is displayed.

The symbols for measurement procedure and measuring mode flash (in this case: Inflation).

The part-measurement stops automatically as soon as valid blood pressure values are detected.



The values for systolic/diastolic blood pressure **SYS/DIA** and mean arterial pressure **MAP** are displayed.

The waiting time until the next part-measurement starts.

The next part-measurement starts automatically.

Once all part-measurements are present, mean blood pressure (mean value) is displayed.



Disregarded part-measurements are displayed struck through (in this case: Measurement 1).

You have the following options for continuing:

- ► Measure more vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

#### NOTE

- Use the Start button to cancel the mean measurement and continue or to cancel and re-start.
- If inflation does not deliver a measured value, the device automatically switches to deflation.
- If necessary, the device re-pumps several times during the deflation (re-pumping: Increasing cuff pressure by approx. 50 mmHg followed by incremental releasing of that pressure). If no measured value is delivered even after several re-pumping steps, the procedure is canceled.

# **Modifying presets**

# NOTE

Your settings apply only to the current measurement procedure. If you stop the measurement procedure → Stopping a measurement procedure, the presets defined by the administrator become active again.

1. Ensure that the blood pressure cuff is not applied.

### 2. Press the NIBP field.

The **NIBP settings** dialog window opens. Tł

ne	presets	are	displayed.
----	---------	-----	------------

NIBP settings			
Туре	_		
● 🔶 Inflation	0	ŝ	Deflation
Starting pressure			
180 mmHg 🔹			
Procedure			
O • Single measurement	۲	:	Multiple measurement
Cancel			Confirm

- 3. Press the desired measurement method.
  - Inflation, continue at step 5.
  - Deflation, continue at step 4.
- 4. Adapt initial pressure if necessary.
- 5. Press the desired measuring sequence:
  - Single measurement
  - Multiple measurement or if a measuring profile has been configured for the multiple measurement - name of the measuring profile

#### NOTE

If you have questions about configuring the multiple measurement, contact your administrator.

- 6. Press the **Confirm** button. The dialog window closes. Changed settings are adopted for the current measurement.
- 7. Start the blood pressure measurement as described in the section entitled  $\rightarrow$  Preparing the blood pressure measurement.

Measuring temperature, oral/ axillary (COVIDIEN™ FILAC™ 3000, blue)

- → Starting an oral/axillary temperature measurement
- → Modifying presets



- The decision to use this device with children, pregnant or lactating patients is the responsibility of the user.
- Before every measurement, ensure that the measurement mode and the measuring method have been correctly selected.
- For each temperature measurement, use a new probe cover to reduce the risk of cross-contamination, nosocomial infections and inplausible measurements.
- Only use probe covers that are approved for the thermometer being used.
- Always use the probe cover directly with the thermometer from the cover box on the device.
- ► Ensure that the probe covers engage correctly on the thermometer.
- Probe covers are only intended for an individual measurement, they are not reusable and not sterile. Do not disinfect or sterilize the probe covers. Dispose of the according to national regulations and the regulations of your institute.
- Only use properly functioning thermometers. If you find damage, do not use the thermometer. Use a suitable replacement.
- When not in use, store the thermometer in the corresponding holder on the machine.
- If the measurements seem implausible, check and evaluate the vital signs of the patient using alternative means. Then check the measurement function of the device with the help of the section "Troubleshooting".

# WARNING!

# Patient hazard, incorrect measurement

- Ensure that in "Direct" measuring mode, oral temperature measurements do not last longer than 3 minutes and axillary measurements do not last longer than 5 minutes.
- Only perform oral/axillary measurement with devices that are equipped with a blue temperature probe and a blue probe holder.
- ► For devices with COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 measurement equipment, only use COVIDIEN<sup>TM</sup> temperature probes and probe covers.
- For axillary temperature measurements, ensure that the temperature probe, with probe cover, makes direct contact with the skin of the patient and does not come into contact with clothing or other objects.

# Starting an oral/axillary temperature measurement

- 1. Remove the temperature probe (blue) from the probe holder (blue).
- 2. Pick up a probe cover:




- a) Insert the probe in a probe cover in the pack
- b) Allow the probe cover to engage audibly with the probe
- c) Remove probe and probe cover from the pack
- d) Ensure that the probe cover is undamaged
- 3. Position the temperature probe as shown in the illustration:

Oral measurement:

Axillary measurement:



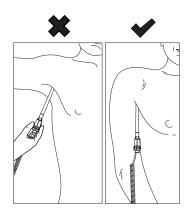

Measurement starts automatically.


The measured value and the symbol for the measurement procedure (in this case: Predictive) flash until a valid measured value is obtained.



The temperature value is displayed until you stop the measurement procedure  $\rightarrow$  Stopping a measurement procedure.




4. Discard the probe cover and dispose of it in line with your institution's policy.

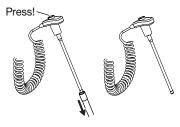

### NOTE

You can only perform a further temperature measurement if you discard the probe cover and push the temperature probe back into the probe holder completely.

5. Push the temperature probe into the probe holder completely.

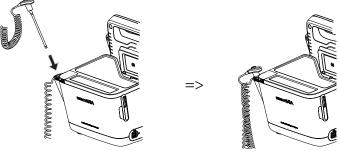

You have the following options for continuing:

- ► Measure further vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

#### Modifying presets

#### NOTE

Your settings apply to the current measurement procedure. If you stop the measurement procedure  $\rightarrow$  Stopping a measurement procedure, the presets defined by the administrator become active again.

- 1. Ensure that the temperature probe is pushed into the probe holder completely.
- 2. Press the **TEMP** field.

The **Temperature settings** dialog window opens. The presets are displayed.

O Temperature settings
● ▶ Predictive
O 🌡 Direct
rel
O 🛉 Axillary
Cancel Confirm

- 3. Press the desired measurement method.
  - Predictive
  - Direct
- 4. Press the desired measuring position.
  - Oral
  - Axillary
- 5. Press the **Confirm** button.

The dialog window closes.

Changed settings are adopted for the current measurement.

6. Perform a temperature measurement as described in the section entitled
 → Starting an oral/axillary temperature measurement.

# Measuring temperature, rectal (COVIDIEN™ FILAC™ 3000 red)

- → Starting a rectal temperature measurement
- → Modifying presets

#### WARNING! Patient hazard, incorrect measurement

- The decision to use this device with children, pregnant or lactating patients is the responsibility of the user.
- Before every measurement, ensure that the measurement mode and the measuring method have been correctly selected.
- For each temperature measurement, use a new probe cover to reduce the risk of cross-contamination, nosocomial infections and inplausible measurements.
- Only use probe covers that are approved for the thermometer being used.
- Always use the probe cover directly with the thermometer from the cover box on the device.
- ► Ensure that the probe covers engage correctly on the thermometer.
- Probe covers are only intended for an individual measurement, they are not reusable and not sterile. Do not disinfect or sterilize the probe covers. Dispose of the according to national regulations and the regulations of your institute.
- Only use properly functioning thermometers. If you find damage, do not use the thermometer. Use a suitable replacement.
- When not in use, store the thermometer in the corresponding holder on the machine.
- If the measurements seem implausible, check and evaluate the vital signs of the patient using alternative means. Then check the measurement function of the device with the help of the section "Troubleshooting".

# WARNING!

# Patient hazard, incorrect measurement

- Ensure that rectal temperature measurements in "Direct" measuring mode do not last longer than 5 minutes.
- Only perform rectal measurement with devices that are equipped with a red temperature probe and a red probe holder.
- ► For devices with COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 measurement equipment, only use COVIDIEN<sup>TM</sup> temperature probes and probe covers.
- For rectal temperature measurement use some lubricant on the temperature probe. Too much lubricant can distort the measurement result.
- For rectal temperature measurements, do not insert the temperature probes deeper than about 19 mm (3/4 inch) for adults and about 13 mm (1/2 inch) for children.

### Starting a rectal temperature measurement

- 1. Remove the temperature probe (red) from the probe holder (red).
- 2. Pick up a probe cover:



- a) Insert the probe in a probe cover in the pack
- b) Allow the probe cover to engage audibly with the probe
- c) Remove probe and probe cover from the pack
- d) Ensure that the probe cover is undamaged
- 3. Apply a little lubricant.
- 4. Guide the temperature probe into the patient's rectum:
  - For adults: 12 19 mm
  - ► For children: 6 13 mm

Measurement starts automatically.


The symbol for the measurement procedure (in this case: Predictive) flashes until a valid measured value is obtained.



The temperature value is displayed until you stop the measurement procedure  $\rightarrow$  Stopping a measurement procedure.




5. Discard the probe cover and dispose of it in line with your institution's policy.

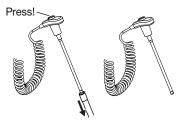

### NOTE

You can only perform a further temperature measurement if you discard the probe cover and push the temperature probe back into the probe holder completely. 6. Push the temperature probe into the probe holder completely.

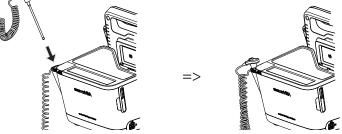

You have the following options for continuing:

- ► Measure further vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure  $\rightarrow$  Stopping a measurement ► procedure

#### Modifying presets

#### NOTE

Your settings apply to the current measurement procedure. If you stop the measurement procedure → Stopping a measurement procedure, the presets defined by the administrator become active again.

- 1. Ensure that the temperature probe is pushed into the probe holder completely.
- 2. Press the **TEMP** field.

The Temperature settings dialog window opens. The presets are displayed.

- 3. Press the desired measurement method.
  - Predictive
  - Direct
- 4. Press the Confirm button. The dialog window closes.

The modified settings are adopted.

5. Start the temperature measurement as described in the section entitled  $\rightarrow$  Starting a rectal temperature measurement.

### Measuring in-ear temperature (COVIDIEN™ GENIUS®3)

#### DANGER! Patient hazard

Choking on probe covers can lead to serious injury.

# WARNING!

### Patient hazard, incorrect measurement

- The decision to use this device with children, pregnant or lactating patients is the responsibility of the user.
- Before every measurement, ensure that the measurement mode and the measuring method have been correctly selected.
- For each temperature measurement, use a new probe cover to reduce the risk of cross-contamination, nosocomial infections and inplausible measurements.
- Only use probe covers that are approved for the thermometer being used.
- Always use the probe cover directly with the thermometer from the cover box on the device.
- Ensure that the probe covers engage correctly on the thermometer.
- Probe covers are only intended for an individual measurement, they are not reusable and not sterile. Do not disinfect or sterilize the probe covers. Dispose of the according to national regulations and the regulations of your institute.
- Only use properly functioning thermometers. If you find damage, do not use the thermometer. Use a suitable replacement.
- When not in use, store the thermometer in the corresponding holder on the machine.
- If the measurements seem implausible, check and evaluate the vital signs of the patient using alternative means. Then check the measurement function of the device with the help of the section "Troubleshooting".


# WARNING!

### Patient hazard, incorrect measurement

- If the thermometer is not placed in the auditory canal correctly, this may lead to permanent injury.
- Do not use the in-ear thermometer if the patient's auditory canal is blocked with blood, cerebrospinal fluid or other discharge.
- Do not use the in-ear thermometer if the patient's auditory canal is blocked with wax or a foreign body.
- Grommets or tympanostomy tubes have no negative impact on measuring accuracy. To prevent problems for the patient, do not resume in-ear temperature measurement any sooner than one week after an operation.
- Only use probe covers intended for your in-ear thermometer. Other probe covers may falsify the measuring results.
- Ensure that the probe tip seals the auditory canal. If this is not the case, measuring results will be falsified.
- Severe scarring of the eardrum may falsify the measurement, resulting in values which are too low being measured.


# CAUTION! Incorrect measurement

- Once there is a probe cover on the thermometer, do not point the probe tip at objects which generate heat – such as hands, computers or windows. This will lead to falsified measuring results.
- Use a new probe cover for every temperature measurement. The surface of the probe cover must be smooth, with no holes, cracks or creases.
- Using the thermometer without a probe cover will lead to falsified measuring results.
- Ensure that the measuring window of the ear thermometer is clean, dry and undamaged. Contamination, for example fingerprints, ear wax or dust will impair the transparency of the measuring window with the result that low values will be measured.
- Patients who wear removable hearing aids should remove them from the ear at least 10 minutes before the measurement. Implants do not generally have any effect on in-ear temperature.
- At low outdoor temperatures, the patient should wait at room temperature for a while before the measurement so as not to falsify the measuring results.
- If the thermometer has been stored outside the ambient temperature range quoted in the "Technical data" section, wait at least 30 minutes before using it until the thermometer has adapted to room temperature.
- Ear drops or other medications applied to the ear can falsify the measuring results. If possible, take the measurement on the other, non-treated ear.
- Temperature measurements on the left or right ear can lead to different results. Always perform follow-up measurements on the same ear.
- ► Wait at least two minutes before performing a follow-up measurement on the same ear.
- 1. Take the in-ear thermometer out of the SmartBucket.
- 2. Make sure that the lens of the measuring head is clean.

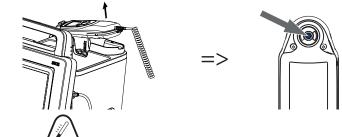

3. Press the *b*utton to switch on the in-ear thermometer.

- 4. Pick up a probe cover:
  - a) Push the measuring head firmly into a probe cover in the magazine
  - b) Ensure that the probe cover engages audibly with the measuring head
  - c) Remove the probe cover and thermometer from the magazine
  - d) Ensure that the probe cover is undamaged

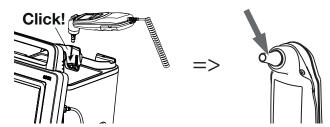

The system is ready to measure when the displays of both the monitor and the in-ear thermometer are showing dashes, the current measuring position and the thermometer icon as shown in the illustration below.



5. Introduce the measuring head into the patient's auditory canal as shown in the illustration.



6. Measure the patient's temperature:



- b) Wait until you hear a triple acoustic signal
- c) Take the measuring head out of the patient's ear

The monitor displays the temperature value.

The in-ear thermometer display likewise shows the temperature value and the Discard probe cover symbol.



The temperature value is displayed on the monitor until you stop the measurement procedure  $\rightarrow$  Stopping a measurement procedure.


#### NOTE

The monitor always displays measured values in the unit set on the monitor. If required, the values submitted by the in-ear thermometer are converted automatically.

7. Press the  $\bigcirc$   $\checkmark$  button to discard the probe cover.



- 8. Dispose of the probe cover in line with your institution's policy.
- 9. Press the in-ear thermometer into the SmartBucket holder until you feel it engage.

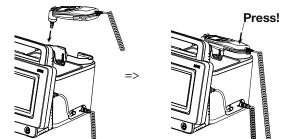

You have the following options for continuing:

- ► Measure further vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- ► Stop the measurement procedure → Stopping a measurement procedure

# **Entering temperature manually**

### NOTICE!

#### Faulty operation, missing measuring results Manual temperature entries cannot be transmitted to the

seca analytics 115 PC software.

 Enter these values manually in the seca analytics 115 PC software as well.

#### NOTE

- The option of entering temperatures manually is available on device versions delivered **without** integrated temperature measuring equipment. On other device versions, your administrator can deactivate the integrated temperature measuring equipment in order to allow manual temperature input.
- Follow the safety precautions in the instructions for use for your thermometer.
- 1. Measure body temperature as described in the instructions for use for your thermometer.
- 2. In the Vital signs tab, press the **TEMP** field:



#### The Temperature: manual entry dialog window is displayed:

Temperature: manual entry		
The oral	O 🝞 Ear	
O 🛉 Axillary	O 🐺 Skin	
O 📲 Rectal	O ₽•1 Contactless	
Cancel	Confirm	

- 3. State the measuring method:
  - a) Press the appropriate button
  - b) Press the **Confirm** button

A numerical keypad appears:

Temp. (°C)	×
4 5 6	
7 8 9	

# 4. Enter body temperature:

#### a) Enter the value



The value entered appears in the **TEMP** field of the Vital signs tab.

No patient fil		Login
In the user rogget I SpO₂ No Sense	or connected.	≡Q
	NIBP mmHg Sys/dia • 🤶	
		$\sim$
	MAP	
	₱ 🕑 PR 1/min ♥ SpO₂ % ▲	
37	<b>′.5</b>	$\sim$
🛓 kg	<u>1</u> m	명 (1

You have the following options for continuing:

- ► Measure more vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

#### Reading off pulse rate

#### → Interrogate pulse rate source

→ Modifying presets (seca measuring equipment only)




# Patient hazard, incorrect measurement

A pulse rate determined on the basis of blood pressure or oxygen saturation is susceptible to artifacts.

 To obtain an exact value, determine pulse rate by means of ECG or palpation.

Depending on device configuration, pulse rate is determined on the basis of blood pressure or oxygen saturation.

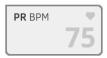

Pulse rate is displayed until you stop the measurement procedure → Stopping a measurement procedure.



## Interrogate pulse rate source

- 1. Press the **PR** field.
  - The **PR settings** dialog window opens. The source of the pulse rate (NIBP or SpO<sub>2</sub>) is displayed.
- 2. Press the **Confirm** button.
  - The dialog window closes.

### Modifying presets (seca measuring equipment only)

# NOTE

These settings apply to the current measurement procedure. If you stop the measurement procedure  $\rightarrow$  Stopping a measurement procedure, the presets defined by the administrator become active again.

- 1. Ensure that neither the blood pressure cuff nor the SpO<sub>2</sub> sensor are applied.
- 2. Press the PR field.

The **PR settings** dialog window opens.

The preset is displayed (in this case: Standard).

PR
Source: SpO <sub>2</sub>
Standard
O ≑♥ Sensitive
Cancel Confirm

3. Press the desired measuring mode:

seca measuring equipment			
Mode	Measuring range	Motion tolerance	
Standard	0 - 240 min <sup>-1</sup>	High	
Sensitive	20 - 300 min <sup>-1</sup>	Low	

#### 4. Press the **Confirm** button.

The dialog window closes. Changed settings are adopted for the current measurement.

 Start a blood pressure measurement or an SpO<sub>2</sub> measurement as described in the sections entitled → Preparing the blood pressure measurement and → Starting an SpO<sub>2</sub> measurement.

### Measuring oxygen saturation (SpO<sub>2</sub>)

- → Starting an SpO<sub>2</sub> measurement
- → Modifying presets



#### WARNING! Patient hazard, incorrect measurement

- ► The device has **no** alarm function. Never leave the patient unobserved during a measurement.
- The pulse oximeter is not an apnea monitoring device.
- The pulse oximeter must not be used to analyze arrhythmia.
- Incorrectly-applied sensors may lead to injuries at the point of application. Follow the Instructions for Use from the sensor manufacturer.
- Apply the blood pressure cuff and SpO<sub>2</sub> sensor to different extremities to avoid falsifying measuring results.
- Apply the intravenous pressure catheter and SpO<sub>2</sub> sensor to different extremities to avoid falsifying measuring results.
- Red and infrared light at fixed wavelengths is used for SpO<sub>2</sub> measurement. These wavelengths can affect other optical applications. Information about the wavelengths used can be found in the Instructions for Use for the sensor used.
- As with all medical equipment, carefully route patient cabling to reduce the possibility of patient entanglement or strangulation.
- Do not place the pulse oximeter or accessories in any position that might cause it to fall on the patient.
- Do not start or operate the pulse oximeter unless the setup was verified to be correct.
- Do not use the pulse oximeter during magnetic resonance imaging (MRI) or in an MRI environment.
- Do not use the pulse oximeter if it appears or is suspected to be damaged.
- Explosion hazard: Do not use the pulse oximeter in the presence of flammable anesthetics or other flammable substance in combination with air, oxygen-enriched environments, or nitrous oxide.
- To ensure safety, avoid stacking multiple devices or placing anything on the device during operation.
- To protect against injury, follow the directions below:
  - Avoid placing the device on surfaces with visible liquid spills.
  - Do not soak or immerse the device in liquids.
  - Do not attempt to sterilize the device.
  - Use cleaning solutions only as instructed in this instructions for use.
  - Do not attempt to clean the device while monitoring a patient.
- To protect from electric shock, always remove the sensor and completely disconnect the pulse oximeter before bathing the patient.
- If any measurement seems questionable, first check the patient's vital signs by alternate means and then check the pulse oximeter for proper functioning. Follow the section entitled "Troubleshooting".
- Inaccurate SpO<sub>2</sub> readings may be caused by the following conditions:
  - Improper sensor application and placement
  - Elevated levels of COHb or MetHb: High levels of COHb or MetHb may occur with a seemingly normal SpO<sub>2</sub>. When elevated levels of COHb or MetHb are suspected, laboratory analysis (CO-Oximetry) of a blood sample should be performed.
  - Elevated levels of bilirubin
  - Elevated levels of dyshemoglobin

- Vasospastic disease, such as Raynaud's, and peripheral vascular disease
- Hemoglobinopathies and synthesis disorders such as
- thalassemias, Hb s, Hb c, sickle cell, etc. – Hypocapnic or hypercapnic conditions
- Hypocaphic or hypercaphic cond
   Sovere anomia
- Severe anemiaVery low arterial perfusion
- Extreme motion artifact
- Abnormal venous pulsation or venous constriction
- Severe vasoconstriction or hypothermia
- Arterial catheters and intra-aortic balloon
- Intravascular dyes, such as indocyanine green or methylene blue
- Externally applied coloring and texture, such as nail polish, acrylic nails, glitter, etc.
- Birthmark(s), tattoos, skin discolorations, moisture on skin, deformed or abnormal fingers. etc.
- Skin color disorders
- Interfering substances: Dyes or any substance containing dyes that change usual blood pigmentation may cause erroneous readings.
- The pulse oximeter should not be used as the sole basis for medical decisions. It must be used in conjunction with clinical signs and symptoms.
- SpO<sub>2</sub> is empirically calibrated in healthy adult volunteers with normal levels of carboxyhemoglobin (COHb) and methemoglobin (MetHb).
- Do not adjust, repair, open, disassemble, or modify the pulse oximeter or accessories. Injury to personnel or equipment damage could occur. Return the pulse oximeter for servicing if necessary.

# M WARNING!

# Patient hazard, damage to device

- ► For devices equipped with Masimo SET<sup>®</sup> SpO<sub>2</sub> measurement equipment, only use Masimo sensors and patient cables.
- For devices equipped with seca SpO<sub>2</sub> measurement equipment, only use seca sensors and patient cables.

# CAUTION! Patient hazard, incorrect measurement

- Do not use damaged sensors or damaged patient cables, for example with exposed optics.
- Do not place the pulse oximeter where the controls can be changed by the patient.
- Electrical shock and flammability hazard: Before cleaning, always ► turn off the device and disconnect from any power source.
- When patients are undergoing photodynamic therapy they may be sensitive to light sources. Pulse oximetry may be used only under careful clinical supervision for short time periods to minimize interference with photodynamic therapy.
- ► Do not place the pulse oximeter on electrical equipment that may affect the device, preventing it from working properly.
- If SpO<sub>2</sub> values indicate hypoxemia, a laboratory blood sample should be taken to confirm the patient's condition.
- If the Low Perfusion or Low Signal Quality message is frequently displayed, find a better perfused monitoring site. In the interim, assess the patient and, if indicated, verify oxygenation status through other means
- Change the application site or replace the sensor and/or patient cable when a "Replace sensor" and/or "Replace patient cable", or a persistent poor signal quality message (such as "Low SIQ") is displayed on the host monitor. These messages may indicate that patient monitoring time is exhausted on the patient cable or sensor.
- If using pulse oximetry during full body irradiation, keep the sensor ► out of the radiation field. If the sensor is exposed to the radiation, the reading might be inaccurate or the device might read zero for the duration of the active irradiation period.
- Variation in measurements may be profound and may be affected by sampling technique as well as the patient's physiological conditions. Any results exhibiting inconsistency with the patient's clinical status should be repeated and/or supplemented with additional test data. Blood samples should be analyzed by laboratory instruments prior to clinical decision making to completely understand the patient's condition.
- Do not submerge the pulse oximeter in any cleaning solution or at-tempt to sterilize by autoclave, irradiation, steam, gas, ethylene oxide or any other method. This will seriously damage the pulse oximeter.
- Electrical Shock Hazard: Carry out periodic tests to verify that leakage currents of patient-applied circuits and the system are within acceptable limits as specified by the applicable safety standards. The summation of leakage currents must be checked and in compliance with IEC 60601-1 and UL60601-1. The system leakage current must be checked when connecting external equipment to the system. When an event such as a component drop of approximately 1 meter or greater or a spillage of blood or other liquids occurs, retest before further use. Injury to personnel could occur.
- To minimize radio interference, other electrical equipment that emits radio frequency transmissions should not be in close proximity to the pulse oximeter.
- Replace the cable or sensor when a replace sensor or when a low SIQ message is consistently displayed while monitoring consecutive m


### NOTE:

**OTE:** A functional tester cannot be used to assess the accuracy of the pulse oximeter. • A functional tester cannot be used to assess the accuracy of the pulse oximeter.

- High-intensity extreme lights (such as pulsating strobe lights) directed on the sensor, may not allow the pulse oximeter to obtain vital sign readings.
- When using the Maximum Sensitivity setting, performance of the "Sensor Off" detection may be compromised. If the device is in this setting and the sensor becomes dislodged from the patient, the potential for false readings may occur due to environmental "noise" such as light, vibration, and excessive air movement.
- Do not loop the patient cabling into a tight coil or wrap around the device, as this can damage the patient cabling.
- Additional information specific to the Masimo sensors compatible with the pulse oximeter, including information about parameter/ measurement performance during motion and low perfusion, may be found in the sensor's directions for use (DFU).
- Cables and sensors are provided with X-Cal<sup>TM</sup> technology to minimize the risk of inaccurate readings and unanticipated loss of patient monitoring. Refer to the Cable or Sensor DFU for the specified duration of the patient monitoring time.

#### Starting an SpO<sub>2</sub> measurement

- Apply the SpO<sub>2</sub> sensor according to the sensor manufacturer's instructions for use.
  - a) Ensure that the SpO<sub>2</sub> sensor is the correct size
  - b) Prepare measuring point (e.g. remove jewelry or nail varnish)
  - c) Apply the SpO<sub>2</sub> sensor to the measurement point (in this case: Softclip sensor)

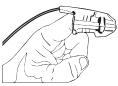

Measurement starts automatically.



The symbol for the measurement method flashes (in this case: Normal) until a valid measured value is obtained.



The SpO<sub>2</sub> value is displayed.

You have the following options for continuing:

- ► Measure more vital signs → Measuring vital signs
- Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

#### NOTE

If your device is equipped with Masimo SET<sup>®</sup> pulse oximetry, perfusion index (PI) is displayed in addition to oxygen saturation. This helps you assess perfusion at the measurement point and to find a better measurement point if necessary.

# Modifying presets

# NOTE

Your settings apply to the current measurement procedure. If you stop the measurement procedure  $\rightarrow$  Stopping a measurement procedure, the presets defined by the administrator become active again.

1. Ensure that no  $SpO_2$  sensor has been applied to the patient.

#### 2. Press the $\mathbf{SpO}_2$ field.

The **SpO**<sub>2</sub> settings dialog window opens.

The presets are displayed (in this case: Masimo SET® pulse oximetry).

SpO <sub>2</sub>
O 🕿 Stable
🖲 🟊 Normal
O ∧ the Sensitive
Cancel Confirm

3. Press the desired sensitivity:

Masimo SET <sup>®</sup> SpO <sub>2</sub> module		
Mode	Indication	
Normal	<ul><li>Normal perfusion</li><li>Mild perfusion disorders</li></ul>	
Adaptive Probe Off Detection (APOD)	Vigorous patient movements	
Maximum	<ul> <li>Poor perfusion</li> <li>Severely disrupted signal, for example due to indoor lighting or direct sunlight</li> </ul>	

seca SpO <sub>2</sub> module		
Mode	Motion tolerance	
Stable	High	
Normal	Normal	
Sensitive	Low	

#### 4. Press the **Confirm** button.

The dialog window closes. The changed settings are used for the current measurement.


Start the SpO₂ measurement as described in the section entitled
 → Starting an SpO₂ measurement.

The device provides several options for adding the parameters weight and height. Your administrator specifies which of the options you can use. Further information is available here:  $\rightarrow$  Adding weight and height.

# Weight and height

### Stopping a measurement procedure

Stop the current measurement procedure before you start a new one.

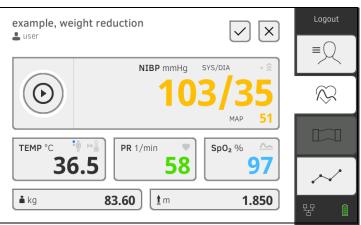

You have the following options:

- Press the button: Save measurement procedure
- ► Assign anonymous measurement to a seca patient file: → Assigning anonymous measurement procedure to a seca patient file
- Press the button: Discard measurement procedure

Assigning anonymous measurement procedure to a seca patient file

- To assign an anonymous measurement procedure to a patient file, proceed as follows:
- 1. Log in  $\rightarrow$  Logging in.
- 2. Press the Patient tab.

You have the following options for continuing

- $\rightarrow$  Calling up a seca patient file
- $\rightarrow$  Creating a seca patient file
- 3. Confirm the **Assign measurement?** message. The measurement procedure is assigned to the patient file.



The measurement procedure is saved.

### 6.4 Measuring bioimpedance

- → User qualification for bioimpedance measurement
- → Preparing a bioimpedance measurement
- → Performing a measurement (with WiFi direct connection to the measuring mat)
- → Performing a measurement (without WiFi direct connection to the measuring mat)
- $\rightarrow$  Connecting the measuring mat

### WARNING! Hazard to patient, malfunction, damage to device

Set up the device so that it cannot fall on patients.

- Route the cables and tubes of the measurement accessory so that ► the patient cannot become entangled or be strangled.
- ► The device has **no** alarm function. Never leave the patient unobserved during a measurement.
- Always connect only one individual patient to the device for each measurement procedure.

Observe the points below in order to obtain meaningful and comparable bioimpedance measurements:

- Amount of time patient should lie **before** each measurement: Approx. 10 minutes
- If possible, arrange follow-up measurements at the same time of day

User qualification for bioimpedance Bioimpedance measurements may only be carried out by persons who have been familiarized with the functions of the device according to the regulations of the respective institution.

> To perform a bioimpedance measurement, at least the following sections of these instructions for use must be observed in addition to the information in this section:

- → Basic safety precautions
- → Contraindications
- → Hygiene treatment
- Preparing a bioimpedance measurement


measurement



- 1. Ensure that the measuring mat is correctly suspended in the magnetic catch of the monitor  $\rightarrow$  Suspending the measuring mat in the magnetic catch.
- 2. Switch on the system  $\rightarrow$  Switching on.
- 3. Log yourself in  $\rightarrow$  Logging in.
- 4. Call up a patient file  $\rightarrow$  Calling up a seca patient file.



5. Add the basic parameters in the seca patient file by pressing the relevant button:

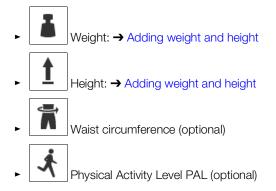

### NOTE

- To perform a bioimpedance measurement, at least the basic parameters weight and height must be acquired. If the basic parameters waist circumference and physical activity level are not entered, the following parameters cannot be calculated:
  - Visceral Adipose Tissue (VAT)
  - Total energy expenditure (TEE)
- As soon as you have added the basic parameters required, the "BIA" tab is activated.

Once the basic parameters have been added, the device submits the updated seca patient file to the measuring mat. You have the following options for continuing:

- → Performing a measurement (with WiFi direct connection to the measuring mat)
- ➤ Performing a measurement (without WiFi direct connection to the measuring mat)



Press the key: Save basic parameters (without starting bioimpedance measurement)


If a WiFi direct connection between the monitor and the measuring mat is **active**, the measuring mat and the monitor communicate continuously. As a result, some operating steps can be performed on both the monitor and the measuring mat.

### NOTE


If you are unsure whether the WiFi direct connection between the monitor and the measuring mat is active, contact your administrator.

- ✓ Bioimpedance measurement prepared: → Preparing a bioimpedance measurement.
- 1. Press the BIA tab.

Performing a measurement (with WiFi direct connection to the measuring mat) 2. Wait until the position buttons on the monitor are active.

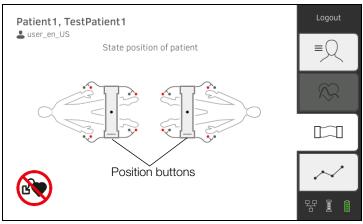


3. Attach the electrodes to the extremities as shown.

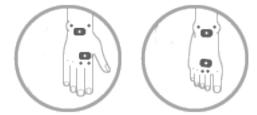


- 4. Take the measuring mat out of the magnetic catch on the monitor.
- 5. Connect the measuring mat to the electrodes → Connecting the measuring mat.
- 6. Perform the following steps on the monitor or on the measuring mat:a) Enter patient position: Press the relevant position button





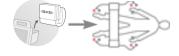


7. Watch measurement progress on the monitor:

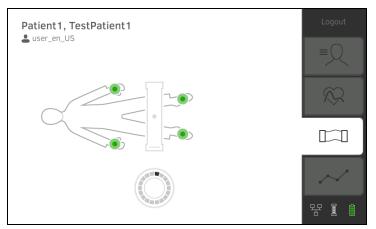

	Monitor					
	Measurement progress	Electrode symbol	Progress symbol			
1.	Electrodes being tested					
2.	Measurement in progress Measuring results will be saved on the measuring mat		$\bigcirc$			
3.	Measuring results will be submitted to the monitor					
4.	Measurement end: Measuring results have been submitted to the monitor		Off			

The measurement procedure is complete when the **Measurement** successful message appears.

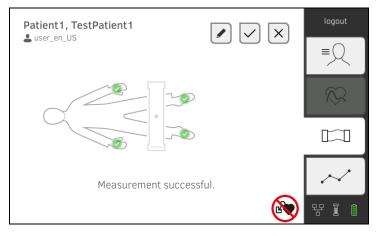

You have the following options for continuing:



- Press the button: Save measurement
- ► Press the Analysis tab: → Analyzing measurement
- ► Press the Vital signs tab: → Measuring vital signs




- Press the button: Discard measurement
- 8. Remove the electrode cables from the electrodes.
- 9. Remove the electrodes from the patient.

Performing a measurement (without WiFi direct connection to the measuring mat) If **no** WiFi direct connection between the monitor and measuring mat is **active**, some operating steps can only be carried out on the measuring mat. Measuring mat and monitor communicate via an infrared connection. The measuring mat must be suspended in the magnetic catch of the monitor before and after a measurement to enable data to be transmitted.

### NOTE


If you are unsure whether the WiFi direct connection between the monitor and the measuring mat is active, contact your administrator.

- ✓ Bioimpedance measurement prepared: → Preparing a bioimpedance measurement.
- 1. Press the BIA tab.

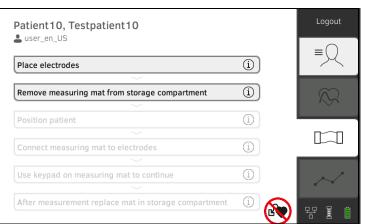


### NOTE

If you press the operating steps on the screen, additional information will be displayed to assist you with the bioimpedance measurement. 2. Attach the electrodes to the extremities as shown.

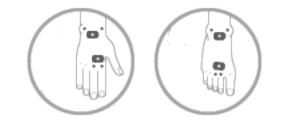

- 3. Take the measuring mat out of the magnetic catch.
- Connect the measuring mat to the electrodes → Connecting the measuring mat.

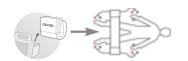


5. Use the keypad of the measuring mat for the following steps: a) Enter patient position: Press the relevant position button



button: Start the measurement procedure b) Press the 6. Observe measuring progress on the measuring mat:

	Measuring mat					
	Measurement progress	Electrodes	Position button			
1.	<ul><li>Patient position is given</li><li>Electrodes being tested</li></ul>	On	On			
2.	<ul> <li>Measurement in progress</li> <li>Measuring results will be stored on the measuring mat</li> </ul>	Flashing	On			
3.	Measurement end: Measuring results are stored on the measuring mat, waiting for transmission to the monitor	Off	Flashing			

- 7. Remove the electrode cables from the electrodes.
- 8. Remove the electrodes from the patient.
- 9. Suspend the measuring mat in the magnetic catch of the monitor  $\rightarrow$  Suspending the measuring mat in the magnetic catch. The updated seca patient file is transmitted to the monitor. The measurement procedure is complete when the Measurement successful message appears.

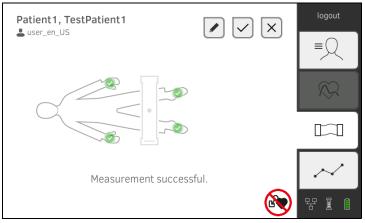

You have the following options for continuing:


- Press the button: Add comment
- Press the button: Save measurement
- ► Press the Analysis tab: → Analyzing measurement
- ► Press the Vital signs tab: → Measuring vital signs
- button: Discard measurement Press the

### Connecting the measuring mat

The measuring mat is intended for bioimpedance measurement by the 8point method (measurement of the whole body) with the patient in a lying position. Measurement by the 4-point method (right half of the body) is likewise possible.

1. Lay the measuring mat on the patient's knees.


- ► The side with the writing on it should be facing upwards
- The keypad should be facing the user

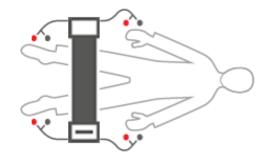

#### NOTE

If you wish to measure using the 4-point method, attach the electrode cables on the keypad for the measuring mat to the hand and foot electrodes of the right half of the body.

- 2. Connect the measuring mat to the electrodes.
  - Arrange the electrode cables so that they do not cross over one another
  - Do not route electrode cables over or underneath the patient
  - Place the push-button adapter on the electrodes (black: Proximal, red: Distal)

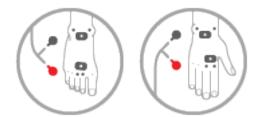

- 3. Ask the patient to do the following during the measurement:
  - ► Hold arms and legs away from the body
  - ► Lie still
  - Do not touch metal parts on bed surface
- 4. Continue with the measurement as described in the relevant section:
  - ► Step 6. of the measurement with active WiFi direct connection
    - ► Step 5. of the measurement without active WiFi direct connection

### 6.5 Adding weight and height

<ul> <li>→ Receiving weight and height (seca 360° proximity)</li> <li>→ Receiving weight and height (seca 360° wireless)</li> <li>→ Entering weight and height manually</li> </ul>	
You can add the parameters weight and height <ul> <li>In the Patient tab</li> <li>In the Vital signs tab</li> </ul>	as follows:
Steve, James Luser_en_US T T T T T	Logout =
First name Steve Surname James Date of birth 10.03.1976	
k     1       i     i	811
No patient file loaded No user logged in SpO <sub>2</sub> No Sensor connected. NIBP mmHg SYS/DIA	
MAP       TEMP °C       Image: PR 1/min       Sp02 %	
Your administrator can set up the device so that weight	

Your administrator can set up the device so that weight and height can be received by compatible seca measuring devices:

- ► → Receiving weight and height (seca 360° proximity)
- ► → Receiving weight and height (seca 360° wireless)


### NOTE

If you are uncertain of which options you can use, contact your administrator.

You can also enter weight and height manually:  $\rightarrow$  Entering weight and height manually.


# Receiving weight and height (seca 360° proximity)

If a corresponding network connection is set up, weight and height can be submitted from a seca measuring device via LAN or WiFi.

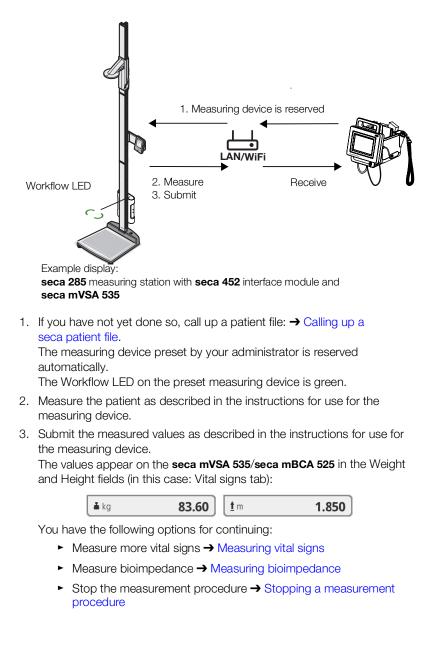

# Receiving weight and height (seca 360° wireless)

If your seca scale or your seca length measuring device is equipped with a **seca 360° wireless** module you can transmit weight and height wirelessly.

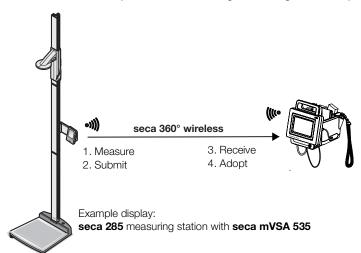

- 1. Measure the patient as described in the instructions for use for the measuring device.
- 2. Submit the measured values as described in the instructions for use for the measuring device.
- 3. On the **seca mVSA 535/seca mBCA 525** press the Weight or Height field. A numerical keypad appears (in this case: Weight):

	Weight (kg)	×
-	360°	
The value	e received appears in the field.	
4. Adopt the	e value:	
a) Pre	ess the <b>360°</b> field.	
3, 110		



The adopted values appear in the Weight and Height fields (in this case: Vital signs tab):




You have the following options for continuing:

- ► Measure more vital signs → Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

# Entering weight and height manually

You can enter weight and height manually or adopt the values from the previous measurement.

- 1. Press the Patient tab.
- 2. If you have not yet done so, select a patient file.
- 3. Press the Weight or Height field.
  - A numerical keypad appears (in this case: Weight):

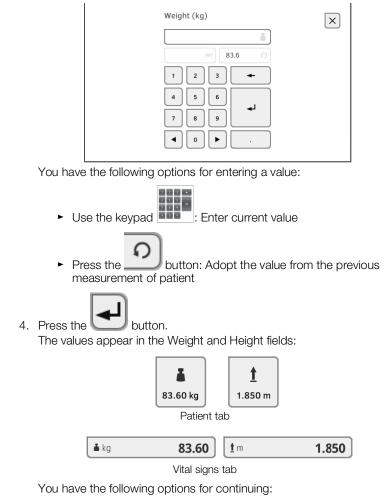

- Measure more vital signs  $\rightarrow$  Measuring vital signs
- ► Measure bioimpedance → Measuring bioimpedance
- Stop the measurement procedure → Stopping a measurement procedure

### 6.6 Analyzing measurement

- → Viewing current measuring results
- → Analyzing vital signs (history)
- → Analyzing a bioimpedance measurement

### NOTE

- In order to be able to view analyses, you must assign the current measurement to a seca patient file → Stopping a measurement procedure or call up a seca patient file → Calling up a seca patient file.
- This section is restricted to navigation in the Analysis tab. For details about analysis parameters and modules, see the sections entitled
   → Analysis parameters and → Analysis modules.

### Viewing current measuring results

In order to view a summary of the current measuring results, proceed as follows:

1. Press the Analysis tab.

The module overview is displayed.

example, weight reduction Last measurement: 12.01.2016, 16:01	:00	Logout
Development/growth	Energy	3
Function/rehabilitation	Fluid	
Health risk	Vital signs	$\sim$
		율

You have the following options for continuing:


► To view an analysis module: continue at step 2.



 Press the button: submit a results report to the seca analytics 115 PC software for printing (seca directprint function of the seca analytics 115 PC software) 2. Press an analysis module. A parameter overview of the current measuring results is displayed:



- You have the following options for continuing:
  - ► → Analyzing vital signs (history)
  - ► → Analyzing a bioimpedance measurement

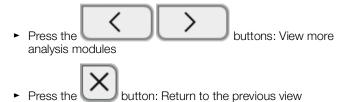

Analyzing vital signs (history)

To view the history of an analysis parameter, proceed as follows:

1. Call up a parameter overview  $\rightarrow$  Viewing current measuring results.

examp Vital sig	le, weight reduction * 01.08.1961	×	Logout
Last mea	surement: 12.01.2016, 16:01:00		≡Q
NIBP	Blood pressure	129 mmHg 30 mmHg	
PR	Pulse rate	70 1/min	$\bigotimes$
SpO <sub>2</sub>	Oxygen saturation	-	
TEMP	Temperature	-	,
	Summary of modules		$\sim$
Health risi			뫄 ()

 Press the parameter whose history you wish to view.
 All measurements for the selected analysis parameter are displayed. The latest measurement is selected automatically:

example, we Vital signs NIBP		<b>ion</b> *01.0	08.1961 X
12.01.2	<b>016</b> 16:01:00	129 mmHg 70 mmHg	
12.12.2	<b>015</b> 19:26:00	149 mmHg 91 mmHg	
21.11.2	<b>015</b> 17:14:00	135 mmHg 83 mmHg	
03.10.2	<b>015</b> 13:51:00	139 mmHg 89 mmHg	
History: <b>1</b> selecte	d		$\checkmark$

You have the following navigation options:

- Press the drop-down menu: Filter measuring results by period. Then continue at step 5.
- ► To select values for the history individually: continue at step 4.
- Press the button: Add a comment
- Press the button: Return to the previous view

### NOTE


Just a single comment can be added to each measured result. Existing comments will be overwritten.

- 3. Press the \_\_\_\_\_ checkboxes for all the measurements you want to look at in the history.
- 4. Press the button.

The history is displayed (in this case three measurements)



You have the following navigation options:

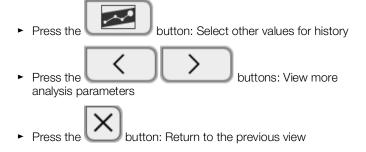

# Analyzing a bioimpedance measurement


- → Calling up detail view
- → Viewing the history of an analysis parameter

### Calling up detail view


1. Call up a parameter overview  $\rightarrow$  Viewing current measuring results.

	<b>ble, weight reduction</b> * 01.08.196		Logout
	n/rehabilitation asurement: 12.01.2016, 16:01:00		≡Q
FFM FFMI	Fat-Free Mass Fat-Free Mass Index	72.52 kg 26.6 kg/m <sup>2</sup>	
FM FMI	Fat Mass Fat Mass Index	36.58 kg 13.4 kg/m <sup>2</sup>	$\sim$
всс	Body Composition Chart	13.4 kg/m <sup>2</sup> 26.6 kg/m <sup>2</sup>	
ѕмм	Skeletal muscle mass	36.1 kg	
Energy	Summary of modules	Fluid	·~· 뚝 ()

2. In the parameter overview, press an analysis parameter. The detail view is displayed:

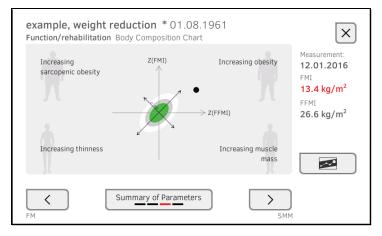

You have the following navigation options:

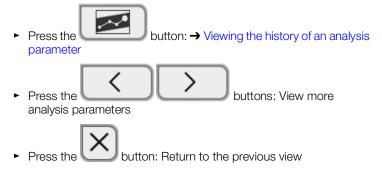


### Viewing the history of an analysis parameter

To view the history of an analysis parameter, proceed as follows:

1. Call up a parameter overview  $\rightarrow$  Viewing current measuring results.

	<b>ble, weight reduction</b> * 01.08.196 n/rehabilitation	i ×	Logout
Last mea	asurement: 12.01.2016, 16:01:00 Fat-Free Mass Fat-Free Mass Index	72.52 kg 26.6 kg/m <sup>2</sup>	= <u>Q</u>
FM FMI	Fat Mass Fat Mass Index	36.58 kg 13.4 kg/m <sup>2</sup>	$\bigotimes$
всс	Body Composition Chart	13.4 kg/m <sup>2</sup> 26.6 kg/m <sup>2</sup>	
ѕмм	Skeletal muscle mass	36.1 kg 🔒	
<b>C</b> Energy	Summary of modules	Fluid	문 (

2. Press the parameter whose history you wish to view. The detail view of the parameter is displayed:

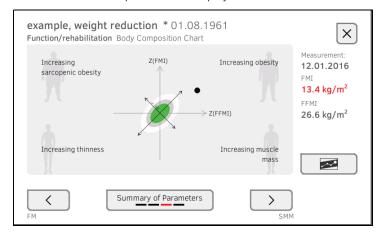

3. Press the button.

All measurements for the selected analysis parameter are displayed. The latest measurement is selected automatically.

Health r	<b>ble, anore</b> isk Body Com Select all			×
~	22.08.2012	10:18:00	2.9 kg/m <sup>2</sup> 13.3 kg/m <sup>2</sup>	
	25.05.2012	10:35:00	2.7 kg/m <sup>2</sup> 13.2 kg/m <sup>2</sup>	
	24.02.2012	11:40:00	1.6 kg/m <sup>2</sup> 12.9 kg/m <sup>2</sup>	
	11.01.2012	11:15:00	1.7 kg/m <sup>2</sup> 12.9 kg/m <sup>2</sup>	
History:	1 selected			$\checkmark$

You have the following options for continuing:


- Press the drop-down menu: Filter measuring results by period (a maximum of 5 measuring results are displayed). Then continue at step 5.
- ► To select values for the history individually: continue at step 4.
- Press the button: Add a comment
- Press the button: Return to the previous view

#### NOTE


Just a single comment can be added to each measured result. Existing comments will be overwritten.

- 4. Press the \_\_\_\_\_ checkboxes for all measurements (maximum: 5) which you want to look at in the history.
- 5. Press the button.

The history is displayed (in this case: three measurements)

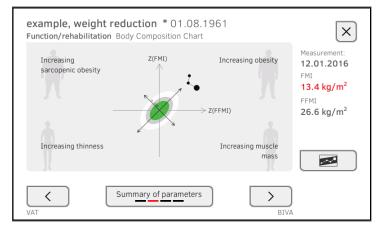

You have the following options for continuing:

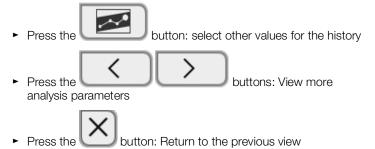

# 7. OPERATION WHEN CONNECTED TO AN EMR SYSTEM

- → Switching on the device
- → Measuring
- → Logging in with EMR user ID
- → Calling up patient data from the EMR system
- → Saving measurements in the EMR system

Your administrator can set up the device so that it communicates with your EMR system. In this case, the following additional functions are available:

- Log in with EMR user ID
- Call up patient data from EMR system on the device
- Submit measuring results to the EMR system

### NOTICE!

### Faulty operation, implausible measuring results

- Connection of the device to your EMR system is based on the technical and regulatory situation in your institution. In the individual case, this may lead to the device behaving differently to the way described in these instructions for use. If you have any questions, contact your administrator.
- The Measure bioimpedance function is not available if the device is connected to an EMR system.

### 7.1 Switching on the device

### NOTE

To prevent faulty input, we recommend entering IDs with the aid of a barcode scanner; information about compatible barcode scanners can be found here:  $\rightarrow$  Optional accessories and spare parts.

- 1. Ensure that a barcode scanner is connected to the free USB interface of the device.
- 2. Press the ON/OFF button of the monitor. The LED of the ON/OFF button is white.
- 3. Wait until the Vital signs tab is displayed:

No patient f			Login
L No user logg	NIBP mmHq		≡Q
$\odot$	SYS/DIA		$\otimes$
TEMP °C	MAP		
		PI	$\sim$
🛓 kg	) <b><u>t</u> m</b>	)	뭥. ()

You have the following options for continuing:


- ► → Measuring with an active connection to an EMR system
- ► → Measuring with an interrupted connection to an EMR system

### 7.2 Measuring

- → Measuring with an active connection to an EMR system
- → Measuring with an interrupted connection to an EMR system

# Measuring with an active connection to an EMR system



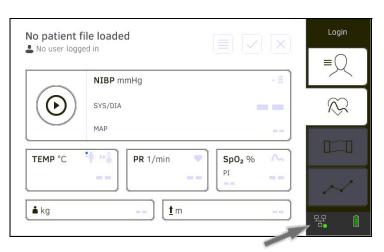


If the device has an active connection to the EMR system, proceed as described in the following sections:

- 1. → Scanning EMR user ID (recommended)
- 2. → Scanning patient ID (recommended)
- 3.  $\rightarrow$  Measuring vital signs
- 4.  $\rightarrow$  Adding weight and height
- 5.  $\rightarrow$  Saving measurements in the EMR system

The device provides the option of performing measurements with an interrupted connection to the EMR system. The measurements are temporarily saved on the device and can be submitted as soon as the connection to the EMR system is active again.

### NOTE

If you are not sure if this function is enabled, contact your administrator.

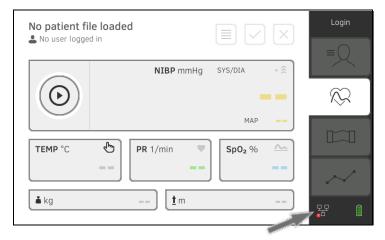

Measuring with an interrupted connection to an EMR system



To perform a measurement with an interrupted connection to an EMR system, proceed as follows:

- 1. Log yourself in:
  - a) Scan EMR user ID
  - b) Confirm status message

The User ID not verified message appears.

No patient fil		Logout
	NIBP mmHg SYS/DIA + இ	≡Q
$\odot$		$\sim$
TEMP °C	MAP	
		~~
🛓 kg	<u>t</u> m	모문 🔒

- 2. Call up a patient file:
  - a) Scan patient ID
    - b) Confirm status message

The patient ID is displayed:

FN22222258		Logout
	NIBP mmHg SYS/DIA • 🔶	$\sim$
$\odot$		$\bigotimes$
	MAP	
TEMP °C	• PR 1/min • SpO <sub>2</sub> %	
		$\sim$
<b>å</b> kg	) <u>t</u> m	동문 Î

- 3. Perform the measurement as described in the following sections:
  - ► → Measuring vital signs
  - ► → Adding weight and height
  - ► → Stopping a measurement procedure

The measurement is saved in the **Unsubmitted measurements** list.

- 4. Log in again when the connection to the EMR system is active again.
- 5. Proceed as described in this section: → Using the Unsubmitted measurements list.

### 7.3 Logging in with EMR user ID

If the device is connected to your EMR system, you must log in to the device using your EMR user ID. You have the following options:

- → Scanning EMR user ID (recommended)
- → Entering EMR user ID manually
- 1. Scan your EMR user ID.

Your EMR user ID is displayed in the — input field.

 Enter your EMR system password. The Vital signs tab is displayed. You can call up patient data from the EMR system → Calling up patient data from the EMR system.

### Entering EMR user ID manually

Scanning EMR user ID

(recommended)

1. Press the Login button.

seca	*	Login
		$\otimes$
	$\checkmark$	$\sim$
		명 (1

The login window is displayed.

2. Enter your EMR user ID.

Your EMR user ID is displayed in the input field:

 Enter your EMR system password. The Patient tab is displayed.
 You can call up activated the FMR support

You can call up patient data from the EMR system  $\rightarrow$  Calling up patient data from the EMR system.

### 7.4 Calling up patient data from the EMR system

You have the following options for calling up patient data from the EMR system:

- → Scanning patient ID (recommended)
- → Manually entering patient ID

Scanning patient ID (recommended)

If you logged in from the Vital signs tab using a scanner ( $\rightarrow$  Logging in with EMR user ID), the Vital signs tab continues to be shown after login:



1. Scan the patient ID.

The patient data are shown in the Patient information dialog window.


First name		
Albert		
Surname <b>Normal</b>		
Date of birth: Gender: Ethnicity: Patient ID:	02.01.1988 Male Caucasian FNSECA1	
		<ul> <li>✓</li> </ul>

- 2. Ensure that the patient data displayed are correct.
- 3. Press the button.

The patient data is transmitted to the device and displayed in the Vital signs tab.

### Manually entering patient ID

If you logged in using a keypad (→ Entering EMR user ID manually), the Patient tab is shown after login:

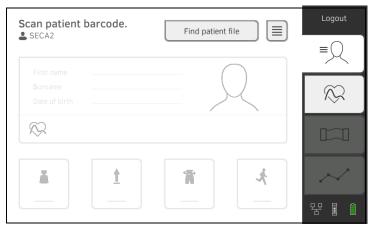

1. Press the Find patient file button.



2. Enter patient ID using the keypad The patient data are shown in the Patient information dialog window.

(i) Patient	information	
First name Albert		
Surname Normal		
Date of birth: Gender: Ethnicity: Patient ID:	02.01.1988 Male Caucasian FNSECA1	
		$\checkmark$

3. Ensure that the patient data displayed are correct.



The patient data is transmitted to the device and displayed in the Vital signs tab.

### 7.5 Saving measurements in the EMR system

- → Submitting the measurement directly to the EMR system
- → Saving measurement temporarily and submitting it later
- → Using the Unsubmitted measurements list

# Submitting the measurement directly to the EMR system

To submit a measurement directly to the EMR system, proceed as follows:

- 1. In the Vital signs tab, press the button.
- 2. In the dialog window, press the **Submit** button.

(i) Measurements
Save or submit measurement? "submit": Measurement will be submitted to your EMR. "save": Measurement will be saved on this device only, not sent.
Submit Cancel Save

The measurement is submitted to the EMR system and assigned to the EMR patient file.

The measurement procedure is complete.

The patient data are removed from the display.

You can temporarily save a measurement on the device, for example to have the measuring results assessed by a third party. The measurement can then be submitted to the EMR system. This ensures that only plausible measuring results are saved in the EMR system.

- 1. In the Vital signs tab, press the button.
- 2. In the dialog window, press the **Save** button.

(i) Measurements
Save or submit measurement? "submit": Measurement will be submitted to your EMR. "save": Measurement will be saved on this device only, not sent.
Submit Cancel Save

The measurement is assigned to the patient ID and saved temporarily on the device.

The measurement procedure is complete.

The measurement can be assessed in the Unsubmitted measurements list and submitted to the EMR system  $\rightarrow$  Using the Unsubmitted measurements list.

# Saving measurement temporarily and submitting it later

# Using the Unsubmitted measurements list

- → Confirm offline measurement (EMR system connection)
- → Viewing details
- → Submitting the measurement
- → Deleting a measurement

In the **Unsubmitted measurements** list, you can view and assess the details of a measurement before submitting the measurement to the EMR system. The list is available in the Patient and Vital signs tabs.

A measurement appears in the list under the following conditions:

- In the Measurements dialog window, press the Save button → Saving measurement temporarily and submitting it later.
- The measurement cannot be submitted because the connection to the EMR system is interrupted.

To open the **Unsubmitted measurements** list, proceed as follows:

- 1. Log in → Scanning EMR user ID (recommended).
- 2. Ensure that the Patient and Vital signs tabs are active.



The Unsubmitted measurements list is displayed:

		<b>~</b>	
FN2222225852	10:19:54	å ↑	
careaware2, jira12372	13.07.2020		
FN2222225852	10:18:50		
FN2222225852	13.07.2020	$(\mathbf{I})$	
FN2222225852	10:14:24		
FN2222225852	13.07.2020	N- @	

### NOTE

Measurements marked yellow (offline measurement) contain unconfirmed patient data – because the connection to the EMR system was interrupted, for example.


### Confirm offline measurement (EMR system connection)

Measurements marked yellow (offline measurement) contain unconfirmed patient data – because the connection to the EMR system was interrupted, for example. These measurements have to be confirmed before you can view details or submit the measurement to your EMR system.



- 1. Ensure that the connection to the EMR system is active.
- 2. Press an offline measurement (marked yellow).

SECA Select all			×
FN222225852	10:19:54	å 1	
careaware2, jira12372	13.07.2020		
FN222225852	10:18:50		
FN222225852	13.07.2020	$\odot$	
FN222225852	10:14:24		
FN222225852	13.07.2020	∿ @	
easurements: 0 selected		=	

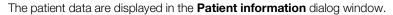

02.01.1988 Male Caucasian FNSECA1	
	Male Caucasian

3. Ensure that the patient data displayed are correct.



The yellow marking is no longer displayed in the **Unsubmitted measurements** list.

The details of the measurement are displayed  $\rightarrow$  Viewing details.

### **Viewing details**

- 1. Press a measurement.
  - The details of the measurement are displayed:

Result	Value	Unit .	
Average (NIBP)	<b>120</b> SYS <b>100</b> MAP <b>92</b> DIA	mmHg	
Pulse rate	86	1/min	
SpO <sub>2</sub>	98	%	
Temperature	37.0	°C	

### NOTE

Measurements marked blue contain mean values. If you press the measurement, you can view the individual results.

2. Press the button to close Detail view.

### Submitting the measurement



- 1. Ensure that the connection to the EMR system is active:
- 2. Press the checkboxes of the measurements to be submitted:

Select all			
FN222225852	10:19:54	<u>گ</u> گ	
careaware2, jira12372	13.07.2020		
FN222225852	10:18:50		
FN222225852	13.07.2020	٩	
FN222225852	10:14:24		
FN222225852	13.07.2020	∿ @	
surements: 0 selected		1 I I I I I I I I I I I I I I I I I I I	

3. Press the button.

The measurements are submitted to the EMR system. The measurements are deleted from the list.

### NOTE

In the EMR system, measurements from the **Unsubmitted measurements** list are not assigned to the user who performed the measurements, but to the user who submitted them.

### Deleting a measurement

1. Press the checkboxes \_\_\_\_\_ of the measurements to be deleted:

	Unsul		nts		×
		Select all FN222225852	10:19:54	<u>گ</u> 1	
		careaware2, jira12372	13.07.2020 10:18:50 13.07.2020	•	
		FN2222225852 FN2222225852	10:14:24 13.07.2020		$\overline{)}$
	Measure	ements: 0 selected		Î	$\checkmark$
2.	Press th	he button.	s are deleted		

# 8. HYGIENE TREATMENT

→ Cleaning

→ Disinfecting

- → Removing/fitting probe holder (devices with temperature probe)
- → Removing/fitting the magazine holder (devices with in-ear thermometer)
- → Sterilizing

### WARNING! Electric shock

The device is not de-energized when the on/off key is pressed and the display goes out. Use of fluids on the device may cause electric shock.

- Ensure that the device is switched off before performing any hygiene treatment.
- Disconnect the power supply connector before performing any hygiene treatment.
- Before each hygiene treatment, take the rechargeable battery out of the device (if present and removable).
- ► Ensure that no fluids penetrate the device.

### NOTICE!

### Damage to device

Inappropriate detergents and disinfectants may damage the sensitive surfaces of the device.

- Use only disinfectants free of chlorine and alcohol which are explicitly suitable for acrylic sheet and other sensitive surfaces (active ingredient: quaternary ammonium compounds, for example).
- ► Do not use caustic or abrasive detergents.
- Do not use organic solvents (e.g. white spirit or petroleum spirit).
- Use disinfectants including the active ingredient 70 % isopropanol only for measuring accessories for measuring vital signs.

# 8.1 Cleaning

Clean the device and its accessories as described in the table:

Component (depending on version)	Interval	Cleaning
seca mBCA 525: Monitor with storage compartment seca mVSA 535: Monitor with SmartBucket	As required	<ol> <li>Remove all the measuring accessories (measuring devices and consumables) from the device (depending on version)</li> <li>→ Removing the probe holder</li> <li>→ Removing the magazine holder</li> <li>Moisten a soft cloth with a soap solution</li> <li>Wipe over all surfaces</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>
Measuring mat and electrode cables	As required	<ol> <li>Moisten a soft cloth with a mild soap solution</li> <li>Clean measuring mat and electrode cables</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>
Blood pressure cuff and compressed air tube	As required	<ol> <li>Moisten a soft cloth with a mild soap solution</li> <li>Clean blood pressure cuff and compressed air tube</li> <li>Rinse thoroughly with water</li> <li>Allow to dry at room temperature</li> </ol>

Component (depending on version)	Interval	Cleaning
Temperature probe (red/blue) with cable	As required	<ol> <li>Discard probe cover and dispose of it</li> <li>Moisten a soft cloth with a mild soap solution</li> <li>Clean temperature probe</li> <li>Shake out temperature probe so that no liquid remains in it</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>
Probe holder (red/blue)	As required	<ol> <li>A Removing the probe holder</li> <li>Moisten a cotton bud with a mild soap solution</li> <li>Wipe over all the surfaces of the probe holder</li> </ol>
In-ear thermometer	After every use	<ul> <li>In-ear thermometer and cable:</li> <li>1. Do <b>not</b> discard probe cover</li> <li>2. Moisten a soft cloth with a mild soap solution: Ratio of water to soap solution: 20:1, temperature: Max. 55 °C (130 °F)</li> <li>3. Wring out the cloth so that no excess liquid can penetrate the in-ear thermometer</li> <li>4. Wipe over in-ear thermometer and cable</li> <li>5. Dry the in-ear thermometer and cable using a lint-free cloth</li> <li>6. Discard probe cover</li> </ul>
	As required	<ul> <li>Measuring head and lens:</li> <li>1. Carefully remove all foreign particles using a cloth moistened with alcohol (70 % isopropanol)</li> <li>2. Wipe dry the lens at the tip of the measuring head with a lint-free cloth (e.g. spectacle-cleaning cloth)</li> <li>3. Make sure that the lens at the tip of the measuring head does not have any fingerprints or marks on it</li> <li>4. Allow the in-ear thermometer to dry completely in air</li> </ul>
Magazine holder for probe covers (in-ear thermometer)	As required	<ol> <li>→ Removing the magazine holder</li> <li>Moisten a soft cloth or cotton bud with a mild soap solution</li> <li>Wipe over the surfaces of the magazine holder</li> <li>Allow the magazine holder to dry completely in air</li> </ol>
$SpO_2$ sensor with cable	Follow the manufacturer's instructions for use	
Patient cable for SpO <sub>2</sub> sensor	Follow the manufacturer's instructions for use	

# 8.2 Disinfecting

- 1. Follow the instructions for use for the disinfectant.
- 2. Disinfect the device and its accessories as described in the table:

Component (depending on equipment)	Interval	Disinfecting	
<b>seca mBCA 525</b> : Monitor with storage compartment <b>seca mVSA 535</b> : Monitor with SmartBucket	As required	<ol> <li>Remove all the measuring accessories (measuring devices and consumables) from the device (depending on version)</li> <li>→ Removing the probe holder</li> <li>→ Removing the magazine holder</li> <li>Moisten a soft cloth with disinfectant (active ingredient: quaternary ammonium compounds)</li> <li>Wipe over all surfaces</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>	
Measuring mat and electrode cables	<b>Before</b> and <b>after</b> a measurem ent	<ol> <li>Moisten a soft cloth with disinfectant (active ingredient: quaternary ammonium compounds)</li> <li>Wipe over measuring mat and electrode cables</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>	
Blood pressure cuff and compressed air tube	As required	<ol> <li>Moisten a soft cloth with disinfectant (active ingredient: 70 % isopropanol)</li> <li>Wipe over cuff and compressed air tube</li> <li>Rinse thoroughly with water</li> <li>Allow to dry at room temperature</li> </ol>	
Temperature probes (red/blue) with cable	As required	<ol> <li>Discard probe cover and dispose of it</li> <li>Moisten a soft cloth with disinfectant (active ingredient: 70 % isopropanol)</li> <li>Wipe over temperature probe</li> <li>Shake out temperature probe so that no liquid remains in it</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>	
Probe holder (red/blue)	As required	<ol> <li>1. → Removing the probe holder</li> <li>2. Moisten a cotton bud with disinfectant (active ingredient: 70 % isopropanol)</li> <li>3. Wipe over the inner surfaces of the probe holder</li> <li>4. Allow to air-dry for approx. 30 minutes</li> </ol>	
In-ear thermometer, cable, measuring head and lens	As required	<ol> <li>Discard probe cover and dispose of it</li> <li>Wipe over in-ear thermometer several times with a cloth moistened with alcohol (70 % isopropanol) so that all surfaces are visibly moist for at least one minute</li> <li>Make sure that the lens at the tip of the measuring head does not have any fingerprints or marks on it</li> <li>Allow the in-ear thermometer to dry completely in air</li> </ol>	
Magazine holder for probe covers (in-ear thermometer)	As required	<ol> <li>→ Removing the magazine holder</li> <li>Moisten a cotton bud with disinfectant (active ingredient: quaternary ammonium compounds)</li> <li>Wipe over surfaces</li> <li>Allow to air-dry for approx. 30 minutes</li> </ol>	
SpO <sub>2</sub> sensor with cable	Follow the manufacturer's instructions for use		
Patient cable for SpO <sub>2</sub> sensor	Follow the manufacturer's instructions for use		

# 8.3 Sterilizing

The device and accessories may not be sterilized.

#### 8.4 Removing/fitting probe holder (devices with temperature probe)




#### WARNING! Patient hazard

The color of the probe holder indicates whether a device is intended for oral/axillary or rectal temperature measurement. This distinction can no longer be made once the probe holder has been removed. Confusing the probe holders can lead to cross-contamination.


• Ensure that the probe holder is fitted back in the device from which it was removed following a hygiene treatment.

### 1. Open the covering cap.

2. Remove the probe holder.

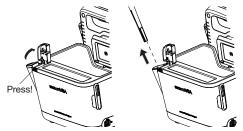

1. Insert the probe holder in the SmartBucket as shown in the illustration below.

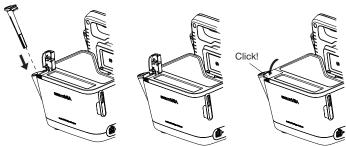

2. Close the cover cap until you hear it engage.


### Fitting the probe holder

Removing the probe holder


## 8.5 Removing/fitting the magazine holder (devices with in-ear thermometer)

### Removing the magazine holder

- 1. Lift the magazine holder with a finger until the magazine holder comes out of its catch.
- 2. Remove the magazine holder.

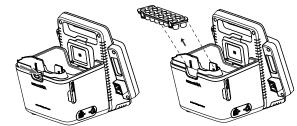

- 1. Insert the magazine holder in the SmartBucket as shown in the illustration below.
- 2. Push down the magazine holder until you hear it engage.

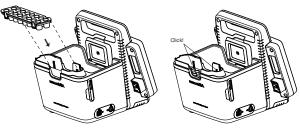

### Fitting the magazine holder

# 9. FUNCTION CHECK

9.1 Device

Perform a function check prior to each use.

A complete function check includes:

- visual inspection for mechanical damage
- · checking the alignment of the device
- visual and function check of the display elements
- function check of all the controls shown in the section entitled "Overview"
- function check of optional accessories

If you notice any faults or deviations during the function check, first try to resolve the error with the aid of the section entitled "Troubleshooting" in this document.



If you notice any faults or deviations during the function check which cannot be resolved with the aid of the section entitled "Troubleshooting" in this document, you may not use the device.

- Have the device repaired by seca Service or by an authorized service partner.
- Follow the section entitled "Servicing" in this document.

#### **COVIDIEN™ GENIUS®3 in-ear thermometer** 9.2

For the COVIDIEN<sup>™</sup> GENIUS<sup>®</sup>3 in-ear thermometer, the manufacturer CardinalHealth<sup>™</sup> recommends a function check if one of the following situations applies:

- In-ear thermometer was not used in accordance with purpose
- In-ear thermometer was dropped
- In-ear thermometer was stored below -25 °C or above +55 °C

# 10.SERVICING

### 10.1 Device

The measuring technology of this device must be checked every two years. We recommend servicing the whole device as part of this check.

### NOTICE!

### Incorrect measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or by an authorized service partner.
- You can find service partners in your area at www.seca.com or by ► sending an e-mail to service@seca.com.

17-10-05-353-002h\_2023-01 B

## 10.2 COVIDIEN™ GENIUS®3 in-ear thermometer

For the COVIDIEN<sup>™</sup> GENIUS<sup>®</sup>3 in-ear thermometer, the manufacturer CardinalHealth<sup>™</sup> recommends calibrating regularly every 12 months. The COVIDIEN<sup>™</sup> Genius Checker/Calibrator is required for calibration. If you do not have a COVIDIEN<sup>™</sup>Genius Checker/Calibrator available, get in touch with your contact at CardinalHealth<sup>™</sup>.

# **11.TROUBLESHOOTING**

- → Monitor
- → Measuring mat
- → Bioimpedance measurement
- → Vital signs measurement
- → Network connection
- → Connection to the seca analytics 115 software
- → EMR system connection via seca connect 103 software
- → seca 360° proximity connection

## 11.1 Monitor

Fault	Cause	Remedy
Fault		-
Monitor cannot be	No power supply	Provide power supply Provide power supply and charge
switched on	Rechargeable battery discharged	rechargeable battery
	Rechargeable battery faulty	Replace rechargeable battery
<b>T</b>	Device on standby	<ul><li>Touch the touchscreen display</li><li>Press the ON/OFF button</li></ul>
Touchscreen display remains off	Device not switched on	Switch on the device
	No power supply	Check whether power is being supplied
	Touchscreen display faulty	Inform seca Service
Touchscreen display not reacting	Device in undefined state following implausible input	<ul> <li>Switch off the device (press and hold the ON/OFF button for approx. 15 seconds)</li> <li>Switch the device back on</li> </ul>
Image on touchscreen display faulty	Touchscreen display faulty	Inform seca Service
	Password overwritten on last synchronization with the <b>seca analytics 115</b> PC software	<ul> <li>Use the new password</li> <li>If new password not known, contact administrator</li> </ul>
		Use the old password
Password not accepted	Following a backup, the old passwords are valid again	<ul> <li>Administrator:</li> <li>If you no longer know the old password, assign a new password</li> <li>Have the administrator password reset by seca Service</li> </ul>
The Vital signs tab is not active on a seca mVSA 535	USB connecting cable of the SmartBucket not connected to the monitor	<ul> <li>Connect the USB connecting cable of the SmartBucket to the monitor</li> <li>Restart the device</li> </ul>
	SmartBucket not activated	Administrator: Activate SmartBucket: → Managing system components in the administrator menu
The Vital signs tab is not active on a mBCA	The Vital signs tab is only displayed on a <b>seca mVSA 535</b>	Retrofit SmartBucket → Optional accessories and spare parts Note the serial number of your mBCA

Fault	Cause	Remedy
	No seca patient file prepared	Prepare seca patient file → Preparing a bioimpedance measurement
BIA tab not active	Measuring mat not activated	Administrator: Activate measuring mat: → Managing system components in the administrator menu
	No measuring mat present	<ul> <li>Administrator, if desired:</li> <li>Retrofit measuring mat</li> <li>Activate measuring mat: → Managing system components in the administrator menu</li> </ul>
Patient and user data can not be restored manually	USB stick contains several backup copies	Ensure that there is only one folder on the USB stick with the designation <ddmmyyyy_hhmm>_seca_monitor_b ackup (if necessary, rename other folders)</ddmmyyyy_hhmm>
	Backup copy on the USB stick has been renamed	Ensure that backup copies are saved in the folder with the designation <ddmmyyyy_hhmm>_seca_monitor_b ackup (if necessary, rename folders)</ddmmyyyy_hhmm>
Patient and user data cannot initially be encrypted after software update	There is not enough memory on the monitor for initial encryption	<ul> <li>Initial manual encryption by administrator:</li> <li>→ Exporting patient and user data manually</li> <li>→ Resetting the device</li> <li>→ Restoring patient and user data manually</li> <li>If necessary, make previous device settings (for example, WiFi connection) manually</li> </ul>

# 11.2 Measuring mat

Fault	Cause	Remedy
Measuring mat cannot be switched on	Rechargeable battery discharged	Suspend measuring mat in storage compartment of monitor and charge rechargeable battery
	Measuring mat faulty	Replace measuring mat
	Rechargeable battery discharged	Suspend measuring mat in storage compartment of monitor and charge rechargeable battery
Charge statue   ED is red	Rechargeable battery faulty	Replace measuring mat
Charge status LED is red	Inductive charging interface of monitor covered, e.g. by other measuring accessories	First suspend the measuring mat in the magnetic catch of the monitor, then stow other measuring accessories
	Inductive charging interface faulty	Inform seca Service
WiFi LED is red	No WiFi connection to monitor	Administrator: Check settings for WiFi connection and correct if necessary
	WiFi module of measuring mat faulty	Replace measuring mat
One or more Electrode contact LEDs are red	Electrode cables not connected to electrodes	Ensure that all electrode cables are engaged in the push-buttons of the electrodes
	Electrodes defective	Replace electrodes
	Electrode cables or measuring mat faulty	Replace measuring mat

Fault	Cause	Remedy
Both "patient position" LEDs light up red	<ul> <li>Firmware update necessary</li> <li>Calibration necessary</li> <li>Measuring mat faulty</li> </ul>	Inform seca Service
One or more LEDs on the measuring mat not on	Measuring mat faulty	Replace measuring mat
seca patient files cannot be transmitted to the measuring mat	No WiFi connection	<ul> <li>Suspend the measuring mat in the storage compartment of the device</li> <li>Call up seca patient file again, data will be transmitted via the infrared interface</li> </ul>
	Infrared interface faulty	Inform seca Service
No WiFi connection	Distance between measuring mat and monitor is too big	<ul> <li>Reduce distance</li> <li>→ Performing a measurement (without WiFi direct connection to the measuring mat)</li> </ul>

# **11.3 Bioimpedance measurement**

#### → General

→ Deviating measuring results

ieneral		
Fault	Cause	Remedy
Bioimpedance measurement not available	BIA measuring mat deactivated	Administrator: Activate measuring mat: Further information is available here: → Retrofitting the measuring mat (seca mVSA 535 only)
measurement not available	Device is connected to an EMR system via the <b>seca connect 103</b> software	Not a malfunction. Bioimpedance measurement not possible when connected via <b>seca connect 103</b>
	Patient position set on device does not match actual patient position	Ensure that the patient position set on the device and the actual position match
	Patient moved during the measurement	Request the patient not to move during the measurement and repeat the measurement
	Patient touches metal parts on bed surface	Request the patient not to touch metal parts during the measurement and repeat the measurement
Results of bioimpedance	The patient's arms or legs are not held away from the body	Ask the patient to hold their arms and legs away from the body
measurement deviate significantly from expected results	Electrode cables incorrectly assigned	Ensure that the electrode cables are connected to the correct electrodes to suit patient position
	Electrodes defective	Replace electrodes
	Electrode cables or measuring mat faulty	Replace measuring mat
	Incorrect seca patient file called up	Transmit measurement to seca analytics 115 PC software and assign to the correct seca patient file there
	Measuring results are correct and medically justified	See section $\rightarrow$ Deviating measuring results

# General

17-10-05-353-002h\_2023-01 B

Fault	Cause	Remedy
Some analysis modules are not shown in the Analysis tab	List of analysis modules which can be displayed has been restricted by administrator	Administrator: Modify the analysis modules which can be displayed
Value of an analysis parameter is shown in red	Value outside the normal range determined for that analysis parameter	<ul> <li>Repeat the measurement to rule out measurement error</li> <li>If the value remains outside the normal range on a repeat measurement, take the measured value into account as the examination continues</li> </ul>

## **Deviating measuring results**

An automatic plausibility check is performed with each bioimpedance measurement. If measuring results are considered implausible, the device displays an appropriate message. Implausible measuring results can have the following causes:

- Faulty measurement performed → Bioimpedance measurement
- Medical reasons:
  - Pronounced edema
  - Severe cachexia

The device provides the option of checking deviating measuring results in detail and saving or discarding the measurement procedure thereafter.


## NOTICE!


#### Inconsistent measuring results

If you save implausible measured values unchecked, the patient's state of health cannot be evaluated correctly.

- Save implausible measured values only if you have sufficient specialist knowledge:
  - Knowledge in bioimpedance measurement technology
  - Knowledge in the interpretation of bioimpedance measuring results

When measuring results deviate, the measurement is interrupted after the plausibility check:

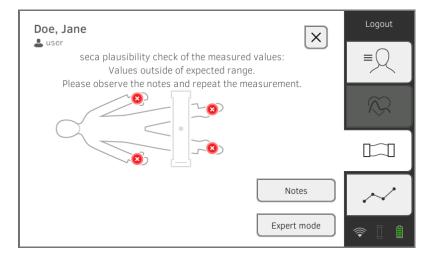

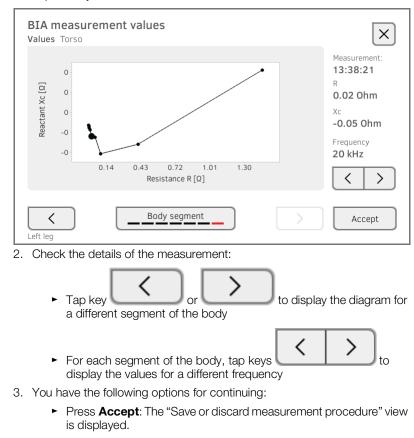

1. Tap on **Expert mode** to check details of the measurement.

 Press the key: The "Discard measurement procedure" view is displayed.

# **11.4 Vital signs measurement**

- → General
- → Blood pressure measurement
- → COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 temperature measurement
- → COVIDIEN<sup>TM</sup> GENIUS<sup>®</sup>3 temperature measurement
- → Manual temperature input
- → SpO<sub>2</sub> measurement

#### General

Fault	Cause	Remedy
	USB connecting cable of the SmartBucket not connected to the monitor	<ul> <li>Connect the USB connecting cable of the SmartBucket to the monitor</li> <li>Restart the device</li> </ul>
Vital signs tab is not displayed	SmartBucket not activated	Administrator: Activate SmartBucket: → Managing system components in the administrator menu
seca mVSA 535 is being operated on a wheeled stand: Measuring results for vital signs are implausible	<ul> <li>Original seca USB connecting cable not being used</li> <li>USB extension cable also in use</li> </ul>	<ul> <li>Use original seca USB connecting cable (in scope of delivery for the wheeled stand)</li> <li>Do not use a USB extension cable</li> </ul>

#### **Blood pressure measurement**

Fault	Cause	Remedy
	Excessive patient movement	Ask the patient to move as little as possible
	Incorrect blood pressure cuff used	<ul> <li>Use correct size of blood pressure cuff</li> <li>Only use seca blood pressure cuffs</li> </ul>
Implausible measuring results	Blood pressure cuff not applied correctly	Correctly apply blood pressure cuff, see instructions for use for blood pressure cuff
	Blood pressure cuff applied to an extremity to which there is intravenous access	Apply blood pressure cuff to a different extremity
Insufficient cuff pressure	Incorrect blood pressure cuff used	<ul> <li>Use correct size of blood pressure cuff</li> <li>Only use seca blood pressure cuffs</li> </ul>
	Blood pressure cuff or compressed air tube leaking	Dispose of blood pressure cuff, use replacement
	Pump in device faulty	Do not continue using device and have it repaired by seca Service
No option to select Multiple measurement in the NIBP settings dialog window	Administrator has configured a measuring profile. The name of the measuring profile is displayed.	<ul> <li>Not a malfunction</li> <li>Press the name of the measuring profile</li> <li>→ Starting a multiple measurement.</li> </ul>
Blood pressure values do not appear in the EMR system or in the seca analytics 115 PC software	Multiple measurement was performed	<ul> <li>For current measurement:</li> <li>→ Modifying presets and → Starting a single measurement</li> <li>Administrator:</li> <li>Specify Individual measurement as preset</li> <li>→ Presets for blood pressure</li> </ul>

## COVIDIEN™ FILAC™ 3000 temperature measurement

Fault	Cause	Remedy
No temperature measurement possible	Temperature module of the SmartBucket not activated	Administrator: Activate temperature module: System\SmartBucket tab
	Special patient condition, such as hypothermia	<ul> <li>Assess patient vital signs using alternative means</li> <li>Switch from predictive measurement mode to direct mode</li> </ul>
	Rectal measurement performed with blue temperature probe	Only perform rectal measurement with red temperature probe
	Oral/axillary measurement performed with red temperature probe	Only perform oral/axillary measurement with blue temperature probe
Implausible measuring results, temperature measurement unsuccessful	Patient activity before oral temperature measurement: • Physical exertion • Eating/drinking • Brushing teeth • Smoking	Only perform oral temperature measurement about 20 minutes after any of these activities
	Set measuring position does not match actual measuring position	<ul> <li>Select measuring position to suit the probe used</li> <li>Set correct measuring position on device</li> </ul>
	No probe cover used	<ul> <li>Disinfect the temperature probe</li> <li>Disinfecting</li> <li>Use probe cover</li> </ul>
	Temperature probe faulty	Dispose of temperature probe, use replacement
Temperature probe cannot be pushed completely into the probe holder	Probe cover not discarded	<ul> <li>Carefully withdraw temperature probe and probe cover from the probe holder</li> <li>Discard probe cover</li> <li>Push the temperature probe into the probe cover</li> </ul>

# COVIDIEN™ GENIUS®3 temperature

## measurement

Fault	Cause	Remedy
No temperature measurement possible	Temperature module of the SmartBucket not activated	Administrator: Activate temperature module: System\SmartBucket tab
	Patient temperature exceeds measuring range of in-ear thermometer	Assess patient vital signs using alternative means
	Patient temperature undershoots measuring range of in-ear thermometer	

Fault	Cause	Remedy
	Ambient temperature exceeds permitted range	<ul> <li>Modify ambient temperature</li> <li>Perform measurement at a location with</li> </ul>
	Ambient temperature undershoots permitted range	an ambient temperature within the permitted range
	No probe cover used	<ul> <li>Disinfect the in-ear thermometer</li> <li>→ Disinfecting</li> <li>Use probe cover</li> </ul>
Measurement does not	Probe cover not properly located on measuring head	Ensure that probe cover engages audibly with the measuring head
start	Probe cover damaged	Dispose of damaged probe cover, use new one
Temperature module of the SmartBucket not activated		Activate temperature module (administrator rights required): → Retrofitting the SmartBucket (seca mBCA 525 only)
Measured result unexpectedly high	Probe cover damaged	Dispose of damaged probe cover and use new one
Measured result unexpectedly low	<ul> <li>Lens of measuring head blocked</li> <li>Opening in probe cover blocked</li> </ul>	<ul><li>Clean measuring head</li><li>Dispose of probe cover, use new one</li></ul>
unexpectedly low	Patient's auditory canal blocked	Clean auditory canal
	Probe cover damaged	Dispose of damaged probe cover and use new one
Implausible measuring results	Measuring position on in-ear thermometer in wrong place	Correct setting on in-ear thermometer (see instructions for use for in-ear thermometer)
	In-ear thermometer faulty	Dispose of in-ear thermometer, use replacement
Different temperature units on monitor and display	Setting of unit on monitor and on in-ear thermometer is not synchronized automatically. If necessary, the monitor converts measuring results automatically.	<ul> <li>Press the °C/°F button on the in-ear thermometer</li> <li>Change units on the monitor (administrator rights required)</li> </ul>
	Internal memory: Checksum error	<ul> <li>Put on a new probe cover and repeat measurement.</li> <li>If the error occurs again, contact seca Service</li> </ul>
	Thermometer no longer calibrated	<ul><li>Do not continue using device</li><li>Contact seca Service</li></ul>

## Manual temperature input

Fault	Cause	Remedy
Manual temperature input not possible	Measured values outside permitted range	Observe permitted temperature range, Technical data: → Manual temperature input
	Device has integrated temperature measurement (probe or in-ear thermometer)	<ul> <li>Not a malfunction</li> <li>Use the measuring equipment for the device</li> </ul>
	Temperature module of the SmartBucket activated	Administrator: Deactivate temperature module: <b>System\SmartBucket</b> tab
Manual temperature entries do not appear in the seca analytics 115 PC software	Manual temperature entries are not synchronized with the <b>seca analytics 115</b> PC software	Not a malfunction • Enter values in the <b>seca analytics 115</b> PC software manually

#### SpO<sub>2</sub> measurement

Fault	Cause	Remedy	
	Intra-aortic balloon pump affecting pulse rate	Check pulse rate using ECG	
	Sensor damp	<ul><li>Dry sensor</li><li>Use dry sensor</li></ul>	
	Sensor not applied correctly	Apply sensor correctly, see instructions for use for sensor	
	Strong ambient light	Cover application site with opaque material	
Implausible measuring results	Electromagnetic interference	<ul> <li>Switch off devices in proximity, isolate interfering device</li> <li>Align interfering device differently or set up in a different location</li> <li>Increase the distance between this device and the interfering device</li> </ul>	
	Poor perfusion	<ul> <li>Assess patient vital signs using alternative means</li> <li>Apply blood pressure cuff to location with better perfusion</li> </ul>	
N	Masimo SET <sup>®</sup> only: Sensor service life expired	Use new Masimo SET® sensor.	
Measurement not possible	Sensor or patient cable defective	Dispose of sensor or patient cable, use spare part	
Measurement not possible	Sensor or patient cable defective	Dispose of sensor or patient cable, use spare part	
Pulse is not found or is lost	Sensor too rigid	<ul><li>Use suitable size of sensor</li><li>Apply sensor to a different finger</li></ul>	
	Strong ambient light	Cover application site with opaque material	
	Poor perfusion	<ul> <li>Assess patient vital signs using alternative means</li> <li>Apply sensor to a location with better perfusion</li> </ul>	

## **11.5 Network connection**

Fault	Cause Remedy	
No WiFi connection	WiFi function of device is deactivated	Administrator: Activate WiFi
	Distance between monitor and WiFi router is too big	<ul> <li>Reduce distance</li> <li>Transmit measuring results to seca analytics 115 PC software or seca connect 103 via LAN</li> </ul>
	WiFi not available in your institution	Transmit measuring results to seca analytics 115 PC software or seca connect 103 via LAN
No network connection	Firewall: ports required are not enabled	Administrator: Enable required ports in firewall → Monitor interfaces and network ports
	Firewall/gateway configuration does not permit use of LAN and WiFi in parallel	Administrator: Deactivate one of the two transmission options on the device.

# 11.6 Connection to the seca analytics 115 software

Fault Cause Remedy			
Data transmission cannot be set up between device and seca analytics 115	Software versions not compatible	Use a compatible version of seca analytics 115 → Compatibility	
and a start file and found	No seca patient file set up yet	Create seca patient file → Creating a seca patient file	
seca patient file not found when searching for a patient on the device	seca patient file is not assigned to the user in <b>seca analytics 115</b>	Check whether the seca patient file can be assigned to the user in <b>seca analytics 115</b>	
	Windows firewall port-blocking is active, required ports are blocked	Administrator: Enable the ports required	
No access to seca patient database of the seca analytics 115 PC software	No network connection set up between the device and the PC on which the <b>seca analytics 115</b> PC software is installed	<ul> <li>Administrator:</li> <li>→ Setting up a LAN connection to the network (stationary operation)</li> <li>→ Setting up a WiFi connection (mobile operation)</li> <li>→ Setting up a connection to the seca analytics 115 PC software</li> </ul>	
	The PC on which the <b>seca analytics 115</b> PC software is installed is not switched on	Switch on PC	
	Automatic synchronization is deactivated on the device	Administrator: → Exporting patient and user data manually	
Print function on device not available	Software versions not compatible	Use a compatible version of the <b>seca analytics 115</b> PC software: 1.4 Build 6657 or higher	

Fault	Cause	Remedy	
	PC printer not switched on	Switch on PC printer	
	The PC on which the <b>seca analytics 115</b> PC software is installed is not switched on	Switch on PC	
Results report not being printed	No network connection set up between device and <b>seca analytics 115</b> PC software	Administrator: Set up LAN connection	
	No connection set up between	Administrator: Set up connection between	
	seca analytics 115 PC software and PC	seca analytics 115 PC software and PC	
	printer	printer	
Blood pressure values do not appear in the seca analytics 115 PC software	Multiple measurement was performed	User: • → Modifying presets • → Starting a single measurement Administrator: • Specify Individual measurement as preset → Presets for blood pressure	
Manual temperature entries do not appear in the seca analytics 115 PC software	Manual temperature entries are not synchronized with the <b>seca analytics 115</b> PC software	Not a malfunction <ul> <li>Enter values in the seca analytics 115 PC software manually</li> </ul>	

# 11.7 EMR system connection via seca connect 103 software

For more information on using the **seca connect 103** software, see the **seca 103/452** system instructions for use.

Fault	Cause Remedy		
Data transmission cannot be set up between device and seca connect 103	Software versions not compatible	Administrator: Use a compatible version of the seca connect 103 software → Compatibility	
	BIA measuring mat activated. EMR system connections are possible only with the BIA measuring mat deactivated	Deactivate measuring mat, more information is available here: → Retrofitting the measuring mat (seca mVSA 535 only)	
Unable to find user ID	No connection to the server	Administrator: Check server connection	
	User ID not recognized by the server	Ensure that user ID is valid	
Unable to find patient ID	No connection to the server	Administrator: Check server connection	
	Patient ID not recognized by the server	Ensure that patient ID is valid	
	Scanner not connected correctly	Ensure that the USB stick is correctly located in the USB interface	
Unable to scan IDs	Barcode scanner incompatible	Use compatible barcode scanner	
	Scanner defective	Replace scanner	
There is no request for a password following input of the EMR user ID. The Vital signs tab is displayed immediately	Not a malfunction: The device connection to the EMR system has been configured so that no password needs to be entered.	Administrator: Modify configuration if necessary → Making workflow settings	

Fault Cause Remedy		Remedy	
	No patient file available in the EMR system	Create patient file directly in the EMR system	
Unable to find EMR system	Error scanning patient ID	Scan patient ID again	
patient data	No network connection to the EMR system	Administrator: Check the network connection and set up again if necessary	
Login using EMR user ID not possible	EMR system connection interrupted, device set up so that no measurements are possible	Administrator: If possible, allow measurements with the EMR system connection interrupted → Making workflow settings	
Blood pressure values do not appear in the EMR system	Multiple measurement was performed	<ul> <li>For current measurement:</li> <li>→ Modifying presets and → Starting a single measurement</li> <li>Administrator:</li> <li>Specify Individual measurement as prese</li> <li>→ Presets for blood pressure</li> </ul>	
	Network cable not plugged in correctly	Check whether the network cable is connected correctly and that the connectors are firm in their sockets	
	Network cable defective	Replace network cable	
Not possible to submit measuring results to the	Device outside WiFi range	Bring device into WiFi range	
EMR system	No network connection to the EMR system	Administrator: Check the network connection and set up again if necessary	
	Network connection interrupted	If network connection is active → Using the Unsubmitted measurements list	

# 11.8 seca 360° proximity connection

Fault	Cause	Remedy
Weight or height values do not appear in the display	No <b>seca 360° proximity</b> connection set up	Administrator: → Setting up a seca 360° proximity connection
	Network connection interrupted	<ul> <li>LAN: Ensure that the network cable is correctly plugged in and is not damaged</li> <li>WiFi: Reduce distance from router</li> </ul>
	seca scale/measuring rod incorrectly configured	<ul> <li>Administrator: Configure seca scale/ measuring rod as follows:</li> <li>Scan user ID: Required</li> <li>Scan patient ID: Required</li> <li>Confirmation of measurement on seca measuring device: Required</li> <li>Port used: Port configured for communi- cation with the seca connect 103 software (default: 22020) → Setting up a connec- tion to the seca analytics 115 PC software</li> <li>Follow seca 103/452 system instructions for use</li> </ul>
	No measurement has yet been performed using the seca scale/ measuring rod	Perform measurement as described in the instructions for use for the seca scale/seca measuring rod
	seca scale/measuring rod has not submitted a value	Submit value as described in the instructions for use for the seca scale/seca measuring rod
An error message appears in the Weight or Height field	An error occurred on the seca scale/ measuring rod	<ul> <li>Eliminate error as described in the instructions for use for the seca scale/seca measuring rod.</li> <li>Follow seca 103/452 system instructions for use</li> <li>If you are unable to eliminate the error, contact seca Service.</li> </ul>

# 11.9 seca 360° wireless connection

Fault	Cause	Remedy	
Weight or height values do not appear in the display	<b>seca 360° wireless</b> wireless module of the connected scale/measuring rod not activated	Administrator: → Setting up a seca 360° wireless	
	No <b>seca 360° wireless</b> wireless connection set up	connection	
	Scale/measuring rod not wireless- capable (do not contain a <b>seca 360° wireless</b> wireless module)	Enter measured values manually: → Preparing a bioimpedance measurement	

# **12. TECHNICAL DATA**

- → Monitor
- → Monitor interfaces and network ports
- → Measuring mat
- → Bioimpedance measurement
- → Vital signs measurement
- → Analysis parameters
- $\rightarrow$  Analysis modules
- → Standards and directives

## 12.1 Monitor

Dimensions, weights		
Monitor with storage compartment (seca mBCA 525)		
Dimensions (seca mBCA 525)		
Depth	230 mm	
• Width	252 mm	
• Height	262 mm	
Net weight (seca mBCA 525)	approx. 2 kg	
Monitor with SmartBucket (seca	mVSA 535)	
Dimensions, empty (seca mVSA 535 for temperature probe)		
Depth	278 mm	
• Width	254 mm	
• Height	262 mm	
Dimensions, empty (seca mVSA 535 for in-ear thermometer)		
Depth	278 mm	
• Width	252 mm	
• Height	262 mm	
Net weight (seca mVSA 535)	approx. 3 kg	

Further technical data (all models)		
Ambient conditions, operation		
<ul> <li>Temperature (with COVIDIEN<sup>™</sup> FILAC<sup>™</sup> 3000)</li> </ul>	+10 °C to +40 °C (50 °F to 104 °F)	
<ul> <li>Temperature (with COVIDIEN<sup>™</sup> GENIUS<sup>®</sup>3)</li> </ul>	+16 °C to +33 °C (60.8 °F to 91.4 °F)	
Air pressure	700 hPa – 1060 hPa	
Humidity	20 % – 80 %, non-condensing	
Ambient conditions, storage		
Temperature	-10 °C to +55 °C (14 °F to 131 °F)	
Air pressure	700 hPa – 1060 hPa	
Humidity	15 % – 95 %, non-condensing	
Ambient conditions, transport		
Temperature	-10 °C to +55 °C (14 °F to 131 °F)	
• Air pressure 700 hPa – 1060		
Humidity	15 % – 95 %, non-condensing	
Setup location, maximum altitude above mean sea level 3		
Display type	7" touchscreen display	
Power supply for monitor, input		
• Type	Internal power supply unit, IEC 60320	
	C13	
Power supply voltage	100 V ~ to 240 V ~	
Power supply frequency	50 Hz – 60 Hz	
Current consumption	0.85 A	
Mobile power supply	Lithium-ion battery	
Voltage	11.25 V	
Capacity	2950 mAh	
Range (seca mVSA 535/seca mBCA 525): full brightness, new		
rechargeable battery)	approx. 5 h	

Further technical data (all models)		
Power consumption		
<ul> <li>Standby (touchscreen display off, ON/OFF button green)</li> </ul>	< 5 W	
<ul> <li>Operation (ON/OFF button white)</li> </ul>	< 9 W	
Operation (charging rechargeable battery for monitor and measuring	< 35 W	
mat, on/off button is white)		
Medical device in accordance with Directive 93/42/EEC	Class IIa	
IEC 60601-1:		
Insulated device, protection class:	II	
Type of protection seca mBCA 525	IP20	
Type of protection seca mVSA 535	IP21	
Duty cycle	Continuous duty	
	seca 360° wireless	
	2 x USB 2.0 (max 500 mA)	
	LAN: Ethernet (10/100 Base-T)	
Interfaces	WiFi: 2.4 GHz (WPA, WPA2 PSK,	
	WPA2 Enterprise PEAP RADIUS)	
	Infrared	
	Inductive charging of measuring mat	
	rechargeable battery	
Compatible printer	Microsoft <sup>®</sup> Windows <sup>®</sup> -compatible printer	
	via seca analytics 115 PC software	

# 12.2 Monitor interfaces and network ports

## Interfaces and network ports

Interfaces	Protocol	Data transmission rate	Factory setting
WiFi:	IEEE 802.11 b/g/n	Up to 72.2 Mbps	On
LAN:	IEEE 802.3u	100 Mbit/s	On
TCP port:	Transmission Control Protocol	n/a	20010, Off
UDP port:	User Datagram Protocol	n/a	20011
File Transfer port:	File Transfer Protocol	n/a	20012
seca connect 103 port:	Proprietary	n/a	22020
USB (2 ports):	USB 2.0	480 Mbit/s per port	On
seca 360° wireless:	Proprietary	50 kb/s	Off

## Recommended WiFi settings

This table contains settings for optional WiFi performance.

Settings	Recommended settings	Consequences in the event of different settings
Authentication/	WPA2 Personal	No network connection, other encryption methods
encryption	WPA2 Enterprise (EAP-TLS)	not supported
Frequency band	<ul><li>Single band 2.4 MHz</li><li>IEEE 802.11 b/g/n</li></ul>	-
Data transmission rate	Up to 72.2 Mbps	-
Network configuration	• DHCP     • UDP	-
Firewall/ ports to be opened	<ul> <li>TCP port: 20010</li> <li>UDP port: 20011</li> <li>File transfer port: 20012</li> <li>seca connect 103 port: 22020</li> </ul>	No synchronization
Separate VLAN	No special requirements	_
QoS	No special requirements	_
VoiP	No special requirements	_
WiFi multimedia	No special requirements	_
Network latency	No special requirements	_
IT support	No special requirements	_

Settings	Recommended settings	Consequences in the event of different settings
Redundant	No special requirements	
power supply	No special requirements	_

# 12.3 Measuring mat

Measuring mat		
Dimensions		
• Depth	783 mm	
Width	170 mm	
• Height	20 mm	
Net weight	approx. 1 kg	
Ambient conditions, operation		
Temperature	+10 °C to +40 °C (50 °F to 104 °F)	
Air pressure	700 hPa – 1060 hPa	
Humidity	20 % – 80 %, non-condensing	
Ambient conditions, storage		
Temperature	-10 °C to +60 °C (14 °F to 140 °F)	
Air pressure	700 hPa – 1060 hPa	
Humidity	15 % – 95 %, non-condensing	
Ambient conditions, transport		
Temperature	-10 °C to +60 °C (14 °F to 140 °F)	
Air pressure	700 hPa – 1060 hPa	
Humidity	0 % – 95 %, non-condensing	
Setup location, maximum altitude above mean sea level	3000 m	
Power supply	Lithium-ion battery	
Range (measurements)	approx. 5 h	
Medical device in accordance with Directive 93/42/EEC	Class Ila	
FDA classification	Class II Medical Device	
IEC 60601-1: Medical electrical device, type BF	$\mathbf{\dot{\mathbf{x}}}$	
Type of protection	IP44	
Duty cycle	Continuous duty	

# **12.4 Bioimpedance measurement**

- → Measuring method
- → Clinical studies
- → Accuracy of predictive formulas

## Measuring method

Measuring method		
Measuring method	8-point bioimpedance measurement 4-point bioimpedance measurement	
Measuring frequencies	(measure right half of body) 1; 2; 5; 10; 20; 50; 100; 200; 500 kHz	
Measured values	Impedance (Z), resistance (R), reactance ( $X_c$ ), phase angle ( $\phi$ )	
Phase angle measuring range	0° to 20°	
Impedance measuring range	10 Ω to 1000 Ω	
Measuring segments	Right arm, left arm, right leg, left leg, right half of body, torso	
Measuring current	100 µA (+20 %, -50 %)	
Measuring time	max. 30 s	

Measuring method		
Accuracy		
(8-point bioimpedance measurement, frequencies 5 kHz and 50 kHz		
Segments: Right half of body, left half of body)		
<ul> <li>Impedance (at phase angle 0°)</li> </ul>	±5 Ω	
• Phase angle (at phase angle 0°, impedance 200 $\Omega$ to 1000 $\Omega$ )	0.5°	

## **Clinical studies**

Clinical studies form the scientific basis for analyzing body composition using this device. The study results are integrated in the device software in the form of reference values.

For some analysis parameters, the reference used depends on the patient's ethnicity. The device automatically uses ethnicity-dependent references to suit the corresponding entry in the seca patient file  $\rightarrow$  Preparing a bioimpedance measurement.

Details about the clinical studies can be found on our website, www.seca.com.

## Accuracy of predictive formulas

Standard deviation (SEE) for predictive formulas in this device <sup>a</sup>					
Ethnicity:	Caucasian	Afro-American Asian		South and Central American	Other
Parameter	SEE	SEE	SEE	SEE	SEE
FFM	2.50 kg	2.21 kg	2.54 kg	2.62 kg	2.49 kg
TBW	2.01	1.81	1.41	1.4	1.71
ECW	1.11	0.91	0.91	0.7	0.91
SMM left arm	0.19 kg	0.28 kg	0.19 kg	0.16 kg	0.21 kg
SMM right arm	0.22 kg	0.30 kg	0.21 kg	0.17 kg	0.23 kg
SMM left leg	0.81 kg	0.71 kg	0.81 kg	0.83 kg	0.79 kg
SMM right leg	0.68 kg	0.66 kg	0.76 kg	0.71 kg	0.70 kg
SMM total	1.8 kg	2.0 kg	1.7 kg	1.7 kg	1.8 kg
VAT	0.5	0.6	0.61	1.21	0.81

a. In the USA, a study was conducted on 130 healthy adults of different ethnicities. The aim of the study was to validate the parameters determined using seca formulas against clinically established reference methods. The results of this comparative study are shown in the table above. The table shows the standard deviation (SEE) by ethnicity for the parameters determined using seca's in-house formulas.

# 12.5 Vital signs measurement

- → Blood pressure measurement
- → COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 temperature measurement
- → COVIDIEN<sup>TM</sup> GENIUS<sup>®</sup>3 temperature measurement
- → Manual temperature input
- → Masimo SET<sup>®</sup> SpO<sub>2</sub> measurement
- $\rightarrow$  seca SpO<sub>2</sub> measurement

## **Blood pressure measurement**

seca blood pressure module		
	Oscillometric	
	<ul> <li>Possible to switch between inflation</li> </ul>	
	and deflation	
Measuring method	<ul> <li>Single measurement</li> </ul>	
	Multiple measurement can be config-	
	ured: Max. 6 measurements in max.	
	30 minutes	
Initial cuff pressure, deflation	Can be set: 80 mmHg – 280 mmHg	
	(260 mmHg cannot be set)	
Maximum cuff pressure	300 mmHg	
Measuring range for blood pressure		
Inflation:		
Systolic BP	77 mmHg – 200 mmHg	
Diastolic BP	45 mmHg – 190 mmHg	
Mean arterial pressure	56 mmHg – 193 mmHg	
Deflation:		
Systolic BP	25 mmHg – 280 mmHg	
Diastolic BP	10 mmHg – 220 mmHg	
Mean arterial pressure	15 mmHg – 260 mmHg	
Accuracy (under laboratory conditions, verified with CuffLink patient	max. ± 3 mmHg / 2 %,	
simulator from Fluke)	the larger value in each case applies	
Measuring accuracy for blood pressure (determined by the manufacturer		
of the measuring module in clinical trial in accordance with		
DIN EN ISO 81060)		
Inflation:		
Mean deviation of systolic BP	0.36 mmHg	
Standard deviation of systolic BP	4.27 mmHg	
Mean deviation of diastolic BP	-0.12 mmHg	
Standard deviation of diastolic BP	3.78 mmHg	
Deflation:		
Mean deviation of systolic BP	0.10 mmHg	
Standard deviation of systolic BP	3.24 mmHg	
Mean deviation of diastolic BP	-0.20 mmHg	
Standard deviation of diastolic BP	2.95 mmHg	
Pressure transducer:	5	
Accuracy	±1 mmHg	
Resolution	1 mmHg	
Leakage rate	< 3 mmHg/min	
Limit value for pressure	300 mmHg	
• Switch off and release pressure at (first fault)	> 330 mmHg	
Measuring time for blood pressure:		
Normal	15 – 20 s	
Maximum (adults)	90 s	

seca blood pressure module		
Pulse rate:		
<ul> <li>Measuring range for inflation</li> </ul>	45 min <sup>-1</sup> – 200 min <sup>-1</sup>	
<ul> <li>Measuring range for deflation</li> </ul>	30 min <sup>-1</sup> – 240 min <sup>-1</sup>	
<ul> <li>Accuracy (under laboratory conditions, verified with CuffLink patient</li> </ul>	max. $\pm 3 \text{ min}^{-1} / 3 \%$ ,	
simulator from Fluke)	the larger value in each case applies	
IEC 60601-1: Medical electrical device, type BF (defibrillation-protected)	-  <b>†</b>  -	

## COVIDIEN™ FILAC™ 3000 temperature measurement

COVIDIEN <sup>™</sup> FILAC <sup>™</sup> 3000 temperature module		
Measurement modes	Direct, predictive	
Measuring position:		
Blue probes	Oral, axillary	
Red probes	Rectal	
Measuring range		
Direct mode	30 °C – 43 °C (86 °F – 109.4 °F)	
Predictive mode	35.5 °C – 42 °C (95.9 °F – 107.6 °F)	
Measuring time (following application at measuring position)		
Direct:		
All measuring positions	60 – 120 sec	
Predictive:		
Oral, no fever	3 – 5 sec	
Oral, fever	8 – 10 sec	
Axillary	8 – 12 sec	
• Rectal	10 – 14 sec	
Switchover time from predictive mode to direct mode		
<ul> <li>Measuring position not detected (following removal from probe cover)</li> </ul>	60 sec	
No stable temperature value obtained (following application)	70 sec	
Accuracy (water bath):		
Direct mode	± 0.1 °C ( ± 0.2 °F)	
Predictive mode	± 0.1 °C (± 0.2 °F)	
IEC 60601-1: Medical electrical device, type BF	$\mathbf{\dot{\mathbf{x}}}$	
Type of protection against ingress of liquids	IP21	

Clinical accuracy <sup>a b</sup>					
Measuring position: Oral Axillary Rectal					
d (age group I)	-0.44	-0.01	0.09		
L <sub>A</sub> (age group I)	1.01	0.86	0.99		
d (age group II)	-0.21	-0.04	0.12		
L <sub>A</sub> (age group II)	0.75	0.65	0.67		
σ <sub>r</sub>	0.34	0.28	0.28		

a. The clinical accuracy of the COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 was determined in a clinical study in accordance with EN 80601-2-56. Clinical bias  $\overline{d}$  and limits of agreement L<sub>A</sub> are quoted for the age group and measuring position in question. Clinical repeatability  $\sigma_i$  is independent of age. The reference body sites of the reference thermometer used in the clinical study correspond to the measuring positions quoted.

b. The age of the test subjects in age group I is between 3 and 4 years. The age of the subjects in age group II is 5 and above.

## **COVIDIEN™ GENIUS®3 temperature**

## measurement

COVIDIEN <sup>™</sup> GENIUS <sup>®</sup> 3 in-ear thermometer		
Measuring method	Direct mode	
Measuring range (in-ear)	33 °C - 42 °C (91.4 °F - 107.6 °F)	
Measuring time	Less than 2 sec	
Resolution	0.1 °C; 0.1 °F	
Accuracy • Ambient temperature: 16 °C - 33 °C (60,8 °F - 91,4 °F) Target temperature: 33 °C - 42 °C (91,4 °F - 107,6 °F)	± 0.3 °C (± 0.5 °F)	
EN 60601-1: Medical electrical device, type BF	<b>*</b>	
Type of protection against ingress of liquids	IP22	

## Manual temperature input

Manual temperature input		
Device versions	Versions without integrated	
	temperature measurement	
Measuring methods	Oral, axillary, rectal,	
Medsuning methods	in-ear, on the skin, contactless	
Temperature range	32 °C – 44 °C (89.6 °F – 111.2 °F)	

## Masimo SET<sup>®</sup> SpO<sub>2</sub> measurement

#### General technical data


Masimo SET <sup>®</sup> SpO <sub>2</sub> module <sup>a b c d e f g</sup>		
Measurement	Functional oxygen saturation	
Measuring method	Spectrophotometry (red/infrared)	
LED wavelength:		
Red	660 nm	
Infrared	905 nm	
Maximum light output	15 mW	
This information may be of particular interest to clinicians		
Measuring time:		
Device switched on, sensor not applied	≤ 12 sec	
Device switched off, sensor applied	≤ 8 sec	
Measuring range:		
• SpO <sub>2</sub>	0 % - 100 %	
Pulse rate	25 min <sup>-1</sup> – 240 min <sup>-1</sup>	
Perfusion index	0.02 % – 20 %	
Measuring accuracy:	70.0/ 100.0/	
Measuring range	70 % – 100 %	
<ul> <li>SpO<sub>2</sub> (no patient movement)</li> </ul>	70 % – 100 % ± 2 digits <sup>h</sup>	
	0 % – 69 % not specified	
<ul> <li>SpO<sub>2</sub> (patient movement)</li> </ul>	70 % – 100 % ± 3 digits	
	0 ‰ – 69 % not specified	
Pulse rate (no patient movement)	$25 \text{ min}^{-1} - 240 \text{ min}^{-1} \pm 3 \text{ digits}$	
Pulse rate (patient movement)	25 min <sup>-1</sup> – 240 min <sup>-1</sup> ± 5 digits	
Low perfusion performance		
Pulse amplitude	> 0.02 %	
• Transmission	> 5 %	
• Oxygen saturation (SpO <sub>2</sub> )	$\pm 2$ digits	
Pulse rate	± 3 digits	
Resolution:		
• Oxygen saturation (SpO <sub>2</sub> )	1 %	
Pulse rate	1 min <sup>-1</sup>	

## Masimo SET<sup>®</sup> SpO<sub>2</sub> module<sup>a b c d e f g</sup>

IEC 60601-1: Medical electrical device, type BF

a. The accuracy of the Masimo SET<sup>®</sup> technology with Masimo sensors was validated with no movement in human blood studies with induced hypoxia on healthy adult male and female volunteers with light to dark skin pigmentation in the range from 70 – 100 % for SpO<sub>2</sub> compared to a laboratory CO oximeter and ECG monitor. This fluctuation corresponds to ± 1 standard deviation. Plus or minus one standard deviation covers 68 % of the population.

- b.The accuracy of the Masimo SET<sup>®</sup> technology with Masimo sensors was validated with movement in human blood studies with induced hypoxia on healthy adult male and female volunteers with light to dark skin pigmentation, 2 to 4 Hz rubbing and tapping movements being performed at an amplitude of 1 to 2 cm and a non-repetitive movement being performed at 1 to 5 Hz at an amplitude of 2 to 3 cm in the range from 70 100 % for SpO<sub>2</sub> compared to a laboratory CO oximeter and ECG monitor. This fluctuation corresponds to ± 1 standard deviation and thus covers 68 % of the population.
- c.The accuracy of the Masimo SET<sup>®</sup> technology in the case of poor perfusion was validated on the test bench compared to a Biotek Index 2<sup>™</sup> simulator and the Masimo simulator with signal strengths of over 0.02 % and a transmission of over 5 % for saturations in the range from 70 to 100 %. This fluctuation corresponds to ± 1 standard deviation. Plus or minus one standard deviation covers 68 % of the population.
- d.The accuracy of the pulse rate of the Masimo SET<sup>®</sup> technology with Masimo sensors was validated on the test bench for the range 25 -240 min-1 compared to a Biotek Index 2<sup>™</sup> simulator. This fluctuation corresponds to ± 1 standard deviation. Plus or minus one standard deviation covers 68 % of the population.
- e. The precise data can be found in the directions for use (DFU) for the sensors. Unless otherwise stated, the measuring position for reusable sensors should be changed at least every 4 hours and for adhesive sensors, at least every 8 hours.
- f.The sensor accuracy quoted applies in conjunction with the Masimo technology with a Masimo patient cable used for LNOP sensors, RD SET sensors, the LNCS sensors or the M-LNCS sensors. The numbers stand for ARMS (accuracy root mean square). Because pulse oximeter measurements are statistically distributed measuring procedures, only about two-thirds of the measurements are expected to be in a range of ± ARMS compared to the reference value. Unless stated otherwise, accuracy for SpO<sub>2</sub> of 70 % to 100 % is quoted. The accuracy of pulse rate is quoted from 25 to 240 min-1.
- g. Masimo M-LNCS, LNOP, RD SET, and LNCS sensor types have the same optical and electrical properties and may differ only in application type (adhesive/non-adhesive/hook & loop), cable lengths, optical component locations (top or bottom of sensor as aligned with cable), adhesive material type/size, and connector type (LNOP 8-pin modular plug, RD 15-pin modular plug, LNCS 9-pin, cable-based, and M-LNCS 15-pin, cable-based). All sensor accuracy information and sensor application instructions are provided with the associated sensor directions for use.
- h.Digit: Numerical value by which the last place of a value displayed can deviate from the actual measured value; used to state the accuracy of a measuring device (example: displayed SpO<sub>2</sub> value 70 %, accuracy  $\pm$  2 digits; actual value is between 68 % and 72 %).

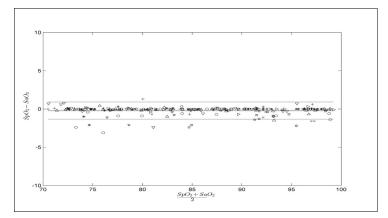

#### Accuracy of Masimo SET® DCI/DCIP sensors

Measured values		
Measuring	A <sub>BMS</sub>	
range	RMS	
90-100 %	0.60 %	
80-90 %	0.54 %	
70-80 %	0.67 %	
Total value		
70-100 %	2 %	

#### **Masimo Patent Information**

Masimo Patents: www.masimo.com/patents.htm

## **No Implied License Statement**

Possession or purchase of this device does not convey any express or implied license to use the device with unauthorized sensors or cables which would, alone or in combination with this device, fall within the scope of one or more of the patents relating to this device.

#### seca SpO<sub>2</sub> measurement

seca SpO $_2$ module <sup>a b</sup>		
Measurement	Functional oxygen saturation	
Measuring method	Spectrophotometry (red/infrared)	
LED wavelength:		
Red	660 nm	
Infrared	900 nm	
Maximum light output	5 mW	
This information may be of particular interest to clinicians		
Measuring range:		
• SpO <sub>2</sub>	0 % – 100 %	
Pulse rate (default)	30 min <sup>-1</sup> – 240 min <sup>-1</sup>	
Pulse rate (enhanced)	20 min <sup>-1</sup> – 300 min <sup>-1</sup>	
Measuring accuracy:		
<ul> <li>SpO<sub>2</sub> (no patient movement)</li> </ul>	60 % - 70 % ± 3 A <sub>rms</sub>	
	60 % - 100 % ± 2 A <sub>rms</sub>	
	<60 % not specified	
<ul> <li>SpO<sub>2</sub> (patient movement)</li> </ul>	$70 \% - 100 \% \pm 3 A_{rms}^{C}$	
	<70 % not specified	
<ul> <li>Pulse rate (no patient movement)</li> </ul>	≤ 2 min <sup>-1</sup>	
Pulse rate (patient movement)	-	
Resolution:		
<ul> <li>Oxygen saturation (SpO<sub>2</sub>)</li> </ul>	1 %	
Pulse rate	1 min <sup>-1</sup>	
IEC 60601-1: Medical electrical device, type BF	$\mathbf{\mathbf{\hat{x}}}$	

a.Validated by clinical tests in which the sensor values measured were compared to those of the arterial CO oximetry of healthy adults over the specified range of functional oxygen saturation.

b.As the measurements of pulse oximeters are statistically distributed, only approximately two-thirds of these measurements will probably fall in the range from  $\pm A_{RMS}$  (accuracy root mean square) of the value measured by CO oximeters.

c.Tested with a Fluke Index II Oximeter Tester (all movement patterns).

# 12.6 Analysis parameters

## NOTE

These instructions for use describe the maximal available functional scope of the device. The actual functional scope of your device may be less than this.

Analysis parameter	Display	Analysis module
	<ul> <li>Normal range display of R and X<sub>c</sub> in coordinate</li> </ul>	
Bioimpedance vector analysis	system in relation to height	Fluid
(BIVA)	• 50 %, 75 %, 95 % percentiles in the form of	Health risk
	tolerance ellipses	
Body mass index (BMI)	Absolute in kg/m <sup>2</sup>	Development/growth
Extracellular water (ECW)	Absolute in I	Fluid
Fat-free mass (FFM)	Absolute in kg	Function/rehabilitation
Fat mass (FM)	Absolute in kg	Energy
	<ul> <li>Relative to weight in %</li> </ul>	Function/rehabilitation
Total energy expenditure (TEE)	Absolute in MJ/d or kcal/d	Energy
Total body water (TBW)	Absolute in I	Fluid
Weight (W)	Absolute in kg	Development/growth
Height (H)	Absolute in m	Development/growth
Extracellular water (ECW)/total body water (TBW)	Relative in %	Fluid (international)
Energy stored in the body (E <sub>body</sub> )	Absolute in MJ or kcal	Energy
Body composition chart (BCC):	Absolute in kg/m <sup>2</sup>	
Mass indices	<ul> <li>Normal range display</li> </ul>	Function/rehabilitation
Fat-free mass index (FFMI)	• 50 %, 75 %, 95 % percentiles in the form of	Health risk
Fat mass index (FMI)	tolerance ellipses	
Phase angle ( $\boldsymbol{\varphi}$ )	<ul> <li>Absolute in degrees</li> </ul>	Health risk
Filase aligie (ψ)	<ul> <li>Normal range display</li> </ul>	
Reactance (X <sub>c</sub> )	Absolute in ohms	Fluid
		Health risk
Resistance (R)	Absolute in ohms	Fluid
		Health risk
Resting energy expenditure (REE)	Absolute in MJ/d or kcal/d	Energy
Skeletal muscle mass (SMM)	<ul> <li>Absolute in kg</li> </ul>	Function/rehabilitation
· · ·	Normal range display	
Visceral adipose tissue (VAT)	Absolute in I	Health risk
Blood pressure, non-invasive (NIBP)	Absolute in mmHg	Vital signs
Body temperature (TEMP)	Absolute in °C	Vital signs
Pulse rate (PR)	Absolute in min <sup>-1</sup> (based on NIBP or SpO <sub>2</sub> )	Vital signs
Oxygen saturation (SpO <sub>2</sub> )	Relative in %	Vital signs

# 12.7 Analysis modules

## NOTE

These instructions for use describe the maximal available functional scope of the device. The actual functional scope of your device may be less than this.

Analysis module	Description	Analysis parameter
Development/growth	Supports the monitoring of weight changes	<ul><li>Weight</li><li>Height</li><li>Body mass index (BMI)</li></ul>
Energy	<ul> <li>Determination of energy expenditure and energy reserves</li> <li>Required: Weight, height, PAL</li> </ul>	<ul> <li>Fat mass (FM)</li> <li>Energy stored in the body (E<sub>body</sub>)</li> <li>Resting energy expenditure (REE)</li> <li>Total energy expenditure (TEE)</li> </ul>
Function/rehabilitation	<ul> <li>Determining fitness level</li> <li>Assessing the success of a training program</li> <li>Required: Weight, height</li> </ul>	<ul> <li>Fat-free mass (FFM)</li> <li>Fat mass (FM)</li> <li>Body composition chart (BCC)</li> <li>Skeletal muscle mass (SMM)</li> </ul>
Fluid	<ul> <li>Determination of fluid status</li> <li>Required: Weight, height</li> </ul>	<ul> <li>Total body water (TBW)</li> <li>Extracellular water (ECW)ECW/ TBW [%]</li> <li>Bioimpedance vector analysis (BIVA)</li> </ul>
Health risk	<ul> <li>Summary of body composition</li> <li>Estimate of health risk</li> <li>Required: Weight, height, waist circumference</li> </ul>	<ul> <li>Phase angle (φ)</li> <li>Visceral adipose tissue (VAT)</li> <li>Bioimpedance vector analysis (BIVA)</li> <li>Body composition chart (BCC)</li> </ul>
Vital signs Overview of vital signs to support a diagnosis		<ul> <li>Blood pressure (NIBP)</li> <li>Body temperature (TEMP)</li> <li>Pulse rate (PR)</li> <li>Oxygen saturation (SpO<sub>2</sub>)</li> </ul>

# 12.8 seca 360° wireless system

seca 360° wireless		
Maximum number of wireless groups	3	
	1 baby scale	
	1 measuring station (or 1 personal scale and 1 measuring rod)	
Maximum configuration per wireless group	1 mVSA/1 mBCA	
	1 PC with seca 360° wireless USB adapter 456	
	and seca analytics 115 PC software	
Number of channels per wireless group	3	
Type of channel assignment	Automatic (recommended)	
rype of channel assignment	Manual	
Channel numbers	0 - 99	
Minimum spacing of channel numbers	30	
<ul> <li>Frequency band</li> </ul>	2.433 GHz - 2.480 GHz	
<ul> <li>Transmission power</li> </ul>	< 10 mW	
Maximum range	10 m	

This device complies with the following standards and directives:

- IEC 60601-1 (Medical electrical equipment Part 1: General requirements for basic safety and essential performance)
- IEC 60601-1-2 (Medical Electrical Equipment Part 1-2: General Requirements For Basic Safety And Essential Performance – Collateral Standard: Electromagnetic Compatibility – Requirements And Tests)
- ISO 80601-2-56 (Medical Electrical Equipment Part 2-56: Particular Requirements For Basic Safety And Essential Performance Of Clinical Thermometers For Body Temperature Measurement.)
- IEC 80601-2-30 (Medical Electrical Equipment Part 2-30: Particular Requirements For The Basic Safety And Essential Performance Of Automated Non-Invasive Sphygnomanometers)
- ISO 80601-2-61 (Medical Electrical Equipment Part 2-61: Particular Requirements For Basic Safety And Essential Performance Of Pulse Oximeter Equipment)

# **13.OPTIONAL ACCESSORIES AND SPARE PARTS**

Optional accessories and spare parts	Article number
SmartBucket seca mVSA 526	Version overview at
(mBCA seca mBCA 525 compatible from serial number 1000000090505)	www.seca.com
Bioimpedance measurement:	
seca mBCA 531 measuring mat	531-20-00-001
Blood pressure measurement:	
Cuff, size XS	490-0024-001
• Cuff, size S	490-0025-001
• Cuff, size M	490-0026-001
• Cuff, size L	490-0027-001
• Cuff, size XL	490-0028-001
Cuff, size XXL	490-0029-001
<ul> <li>Extension for compressed air tube (1.3 m)</li> </ul>	490-0033-001
Extension for compressed air tube (3.0 m)	490-0034-001
Masimo SET <sup>®</sup> SpO <sub>2</sub> measurement:	Obtainable directly from the
Sensors and patient cables from the Masimo RD SET® product line	manufacturer, see
Not compatible: Sensors for neonates	www.masimo.com
seca SpO <sub>2</sub> measurement:	
<ul> <li>Finger Clip SF7500 (adults)</li> </ul>	490-0006-001
<ul> <li>Soft Sensor SC7500 (adults)</li> </ul>	490-0007-001
<ul> <li>Soft Sensor SCM7500 (children)</li> </ul>	490-0008-001
Patient cable XT6500	490-0012-001
Temperature measurement:	
<ul> <li>COVIDIEN<sup>™</sup> FILAC<sup>™</sup> 3000 blue for oral/axillary measurement</li> </ul>	68-90-00-044-009
<ul> <li>COVIDIEN<sup>™</sup> FILAC<sup>™</sup> 3000 red for rectal measurement</li> </ul>	68-90-00-045-009
<ul> <li>COVIDIEN<sup>™</sup> GENIUS<sup>®</sup>3 in-ear thermometer</li> </ul>	68-90-00-143-009
Push-button electrodes for affixing to patient:	
Pack of 100	68-90-00-043-009
Push-button electrodes for affixing to patient:	400 0000 001
Pack of 450	490-0022-001
Probe covers for COVIDIEN <sup>™</sup> FILAC <sup>™</sup> 3000:	490-0015-001
100 packs, pack of 20 probe covers	490-0015-001
Probe covers for COVIDIEN <sup>™</sup> GENIUS <sup>®</sup> 3:	400 0010 001
22 packs, pack of 6 magazines (16 probe covers per magazine)	490-0016-001

Optional accessories and spare parts	Article number
Software: • seca analytics 115 • seca connect 103	For details of application-specific license packages go to www.seca.com
seca 475 wheeled stand for seca mBCA 525 Holder for SmartBucket (retrofit for 475-00-00-009) seca 475 wheeled stand for seca mVSA 535	475-00-009 490-0017-009 475-05-35-009
seca 432 carry case	432-00-00-009
Scanner (medical device): • Honeywell Xenon 1900H (2D) • Datalogic Gryphon I (GD4430 HC (2D)	Cannot be ordered through seca

# **14.COMPATIBLE SECA PRODUCTS**

Product	seca 360° wireless [Serial number to]	seca 360° proximity	Article number
	Measuring stations	;	
seca 285/seca 284	No	Yes, with accessories <sup>a</sup>	For details of country-specific
seca 287/seca 286	restriction		versions, go to
seca 787	_	Yes, with accessories <sup>a.</sup>	www.seca.com
seca 797		Yes	
	Multifunctional scale	25	
seca 651	No	Yes	
seca 650	restriction		_
seca 655	No	Yes	
seca 654	restriction 1000000026211-		-
seca 635	1000000028211-		
Seca 055	1000000737903		
seca 634	1000000027487-		
000000	1000000757963		
	1000000027015-		
seca 645	1000000760426		
seca 644	1000000027016-		
	1000000760426		
	1000000021683-		
seca 657	1000000753707		For details of country-specific
			versions, go to
seca 656	1000000026289-		www.seca.com
	1000000753707 1000000022821-	Yes, with accessories <sup>a.</sup>	
seca 665	1000000022821-		
Seca 000	1000000750440		
seca 664	1000000027014-		
	1000000750446		
	100000020483-		
seca 677	1000000743340		
seca 676	1000000024369-		
	1000000743340		
	1000000017288-		
seca 685	1000000748046		
	1000000017405		
seca 684	1000000017495-		
	1000000748046		

Product	seca 360° wireless [Serial number to]	seca 360° proximity	Article number
	Column scales		
seca 704	5704209100721- 10000000723845	Yes, with accessories <sup>a.</sup>	For details of country-specific versions, go to
seca 703 <sup>b</sup>	5703209102764- 1000000723845		www.seca.com
	Chair scales <sup>b.</sup>		
seca 954 (1309007)	1000000005919- 1000000736608		
<b>seca 954</b> (1309377)	1000000011074- 10000000736608		For details of country-specific
<b>seca 959</b> (7021002)	1000000014301- 1000000736608	Yes, with accessories <sup>a.</sup>	versions, go to www.seca.com
<b>seca 959</b> (7021092)	1000000014426- 10000000736608		
seca 963	1000000045451- 10000000736608		

a.seca 452 external interface module (firmware version Q2\_2020\_Build\_615 or higher) required b.seca 452 external interface module can only be retrofitted by authorized service technicians

# 15. DISPOSAL

- → Measuring mat and device
- → Batteries and rechargeable batteries
- → Consumables

## 15.1 Measuring mat and device



Do not dispose of the device with household waste. The device must be disposed of properly as electronic waste. Comply with the national provisions applicable in your country. For further information contact our service department at:

#### service@seca.com

## 15.2 Batteries and rechargeable batteries



Spent (rechargeable) batteries should not be discarded with household waste, regardless of whether they contain harmful substances or not. As a consumer you are obliged by law to dispose of (rechargeable) batteries via the collection points set up by the municipal authorities or the retail sector. Only discard (rechargeable) batteries when fully discharged.

## 15.3 Consumables



Do not dispose of disposable items such as probe covers in household waste. Used probe covers must be treated as infectious biological waste. Observe the regulations in your institution and your respective national regulations. 17-10-05-353-002h\_2023-01 B

# **16.WARRANTY**

We offer a two-year warranty from the date of delivery for defects attributable to faulty material or poor workmanship. This excludes all moveable parts such as (rechargeable) batteries, cables, power supply units, etc. Defects which are covered by the warranty shall be rectified free of charge for customers on production of the sales receipt. No further claims can be accepted. The costs of shipment in both directions shall be borne by the customer where the device is not located at the customer's premises. In the event of any damage during shipment warranty claims can only be asserted where the complete original packaging was used for shipment and the device was secured inside in the same manner as in the original packaging. You should therefore keep all packaging.

The warranty shall become null and void where the device is opened by persons not expressly authorised to do so by seca.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

Details on the warranty for measuring accessories, such as blood pressure cuffs,  $SpO_2$  sensors or thermometers, can be found at www.seca.com.

# **17. DECLARATION OF CONFORMITY**

**C E** <sub>0123</sub>

seca gmbh & co. kg hereby declares that the product meets the terms of the applicable European directives. The unabridged declaration of conformity can be found at: www.seca.com.

# FOR ADMINISTRATORS: CONFIGURING seca mVSA 535 / seca mBCA 525

- → Preparing configuration
- → User accounts
- → Making settings for measuring mode
- → Managing system components
- → Setting up peripherals
- → Connecting to an EMR system
- → Factory settings
- → Instructions for use for seca mVSA 535/seca mBCA 525

#### NOTE

This document describes the maximal equipment of the **seca mVSA 535/seca mBCA 525** product family: measurement of blood pressure, temperature, oxygen saturation and bioimpedance. Depending on the actual equipment of your device, some of this information may not be relevant to your device. Pay attention to the information in this document which is relevant to your device.

#### NOTE

- This part of the user documentation contains information about configuring the device for measuring mode and for integrating it in a network.
- Integrating this device in a network containing other devices may lead to previously unknown risks to patients, operators or third parties. It is the responsibility of the operating company to determine, analyze, evaluate, and control these risks.
- Note the instructions on IT security in our White Paper titled "Cyber Security". The document can be found as a download in the Support sector at www.seca.com.
- The functions described in this part of the user documentation are accessible only to users with administrator rights.
- Follow the information in the instructions for use → Instructions for use for seca mVSA 535/seca mBCA 525.

# **1. PREPARING CONFIGURATION**

- → Administrator login
- → Configuration options

## 1.1 Administrator login

## NOTICE!

#### Faulty configuration

For devices with bioimpedance measurement: During configuration, data are transmitted to the measuring mat via the infrared interface. The infrared interface is located in the magnetic catch of the monitor.

- ► Ensure that the measuring mat is correctly suspended in the magnetic catch of the monitor throughout the entire configuration process → Suspending the measuring mat in the magnetic catch.
- 1. Switch on the device. The measuring mat – if there is one – switches on automatically.

#### 2. Log in as the administrator:

- ► Initial login: → Activating initial user accounts
- ► Log in routinely: continue at step 3.

3. Press the **Login** button.

	Seca	Login = Q
	The login window is displayed.	
4.	Press the input field. The list of user accounts is displayed.	
5.	Press your user account. Your user account is displayed in the input field.	
6.	Press the input field.	
7.	Enter your password using the keypad	
	Administer user database	Logout
		Users
	Search	System
	Name Role	Settings
	admin Admin	Peripherals
	service Service	Reset
	user Physician	
		<pre></pre>

# 1.2 Configuration options

- → Network functions
- → Overview of access rights

## **Network functions**

• = p	oossible, -	- = not	possible
-------	-------------	---------	----------

Function	LAN	WiFi	seca 360° wireless	Infrared
Receiving patient weight from	•a.	•a.	•	_
scale/measuring rod	••••	•	•	-
Receiving patient height from	•a.	a. a.	•	
measuring rod/measuring station	•ui •ui	•		-
Communication		•		•
between monitor/measuring mat	-	•	-	•
Using seca directprint				
(seca analytics 115 PC software	•	•	-	-
function)				
Synchronizing seca patient files				
and user accounts with the	•	٠	-	-
seca analytics 115 PC software				
Connecting to the EMR system via	•	•		
the seca connect 103 software	•		-	-

a.seca 360° proximity connection: Available for seca measuring devices with an internal interface module or **seca 452** external interface module (firmware version Q2\_2020\_Build\_615 or higher)

## **Overview of access rights**

• = possible, - = not possible

Function	Administrator	User
Creating seca patient files	-	•
Calling up seca patient files	-	•
Entering basic parameters (weight, height,		•
waist circumference, PAL)	-	•
Editing seca patient files	-	•
Deleting seca patient files	-	•
Exporting a seca patient file	-	•
Performing measurements	-	•
Viewing examination results	-	•
Printing examination results	-	•
Analysis parameters: Adding comments	-	•
Managing the patient database	-	•
Managing the user database	•	-
Modifying default settings (e.g. time, date)	•	-
Modifying units for measured values	•	-
Modifying analysis parameters which can be	•	_
displayed		_
Setting up network connections	•	-
Configuring automatic database	•	_
synchronization		_
Configuring automatic export	•	-
Importing backup from USB memory stick	•	-
Restoring factory settings	•	-
Resetting the user interface	•	-
Exporting backup to USB memory stick	•	-
Updating the monitor software	•	-

# 2. USER ACCOUNTS

- → Activating initial user accounts
- → Working with user accounts

## 2.1 Activating initial user accounts

The following user accounts are available on the device initially:

- Admin: (Configure and manage device)
- User: (Perform measurements and manage them).

The user accounts have to be activated in order to be able to use the device:

- → Changing the password for the Admin user account
- → Assigning a password to the User user account

#### 1. Press the **Login** button.

The login window is displayed.

	_	Login
seca		
		$\otimes$
	0 1	
	$\checkmark$	$\sim$
		목 ()

- 2. Press the input field. The list of user accounts is displayed.
- 3. Press the Admin user account. The account is displayed in the input field.
- 4. Press the input field.
- 5. Enter the initial administrator password "1357". You are prompted to change the password.
- 6. Enter a new administrator password.

#### NOTICE!

#### Data access by unauthorized persons

An insecure password may allow unauthorized persons to access patient data or device settings.

- Select a password to satisfy your institution's security requirements.
- 7. Enter the password again.
  - The login window is displayed.
- 8. Activate the User user account → Assigning a password to the User user account.

# Changing the password for the Admin user account

Assigning a password to the User user account

#### 1. Log in using the Admin user account.

2. Press the Users tab.

Administer user databas	e		Logout
			Users
Search	9		System
Name	Role	-	Settings
admin	Admin		Peripherals
			Reset
user	Physician		

- 3. Select the User user account.
- 4. Enter a password.

#### NOTICE!

#### Data access by unauthorized persons

An insecure password may allow unauthorized persons to access patient data or device settings.

- ► Select a password to satisfy your institution's security requirements.
- 5. Enter the password again.
- 6. Press the **Save** button. The password has been saved.

The User user account can be used.

7. Configure the device as required for your specific situation → For administrators: Configuring seca mVSA 535 /seca mBCA 525.

## 2.2 Working with user accounts

- → Setting up user accounts
- → Editing user accounts
- → Deleting user accounts

## Setting up user accounts

User accounts can be created only in the **seca analytics 115** PC software. To make new user accounts available on the device, it must be synchronized with the PC software  $\rightarrow$  Activating automatic synchronization.

**Editing user accounts** 

#### To edit a user account, proceed as follows:

1. Press the Users tab.

Administer user database				Logout
				Users
Search		9		System
Name	~	Role	-	Settings
admin		Admin		Peripherals
				Reset
user		Physician		

You have the following navigation options:

- Desired entry visible: Continue at step 3.
- Desired entry not visible: Continue at step 2.
- 2. Search for the desired user account in the list:



b) Enter the user name using the keypad 
 A hit list is displayed.

- Press the desired entry. The selected user account is displayed.
- 4. Change the user data to the extent required by pressing the relevant input field:
  - Specify role
  - Change password
  - Select display language

NOTICE!

#### Data access by unauthorized persons

An insecure password may allow unauthorized persons to access patient data or device settings.

- ► Select a password to satisfy your institution's security requirements.
- Use the user accounts without password protection only for special applications (e.g. configuring interfaces with EMR systems). seca Service will be pleased to assist if you have any questions relating to interface configuration.

#### NOTE

- Specification of the **Assistance** or **Physician** role is relevant for use of the user account in the **seca analytics 115** PC software. Details on this can be found in the administrator manual for the **seca analytics 115** PC software.
- The display language is specified for each user individually. If no user is logged in, the user interface is displayed in the system language
   → Making regional settings.
- 5. Ensure that the **Password protected** field is activated (default setting).
- 6. Press the **Save** button.

The changes will be saved.

The user account is synchronized with the **seca analytics 115** PC software.

#### NOTE

If automatic synchronization is activated, changes you make to user accounts on the device are automatically adopted in the **seca analytics 115** PC software → Activating automatic synchronization.

#### **Deleting user accounts**

User accounts can only be deleted in the **seca analytics 115** PC software. For deletion to take effect on the device, it must be synchronized with the PC software  $\rightarrow$  Activating automatic synchronization.

# 3. MAKING SETTINGS FOR MEASURING MODE

- → Making regional settings
- → Setting display brightness and volume
- → Calibrating the touchscreen display
- → Setting units of measurement
- → Deactivating analysis modules
- → Making presets for vital signs measurement

#### Making regional settings 3.1



# **CAUTION!**

Loss of data, misinterpretation of measurements Incorrect settings for date and time can lead to misinterpretation of measurements.

- ► Standalone operation: Ensure that the date and time information on the device is correct.
- ► Network operation: Ensure that the date and time information in the seca PC software is correct. These settings are adopted by the device.
- 1. Press the Settings tab.

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
		뭑, 自

#### 2. Press the Regional settings button.

Regional settings	×	Logout
Sustem Janguage		Users
System language	Decimal separator	System
U.S. English 📃	· · · · · ·	Gystein
Date		Settings
07.07.2020	dd.mm.yyyy	Peripheral
Time		Reset
20:46:10	24 h 🚽	
Name style		
First name Surname	With separator	뫊. 1

- 3. Specify the regional settings by pressing the relevant input field:
  - Select system language
  - Select decimal separator
  - Enter date
  - Select date format
  - ► Enter time
  - Select time format
  - Select naming convention
  - Activate/deactivate name separator

The measuring mat - if there is one - switches on automatically.

#### NOTE

1. Press the Settings tab.

- The user interface is shown in the system language if no user is logged in to the device. If a user is logged in, the user interface is displayed in the individual display language → User accounts.
- Settings you make in this tab are active directly. You do not need to save or confirm them.

# 3.2 Setting display brightness and volume

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
		뮹, ①

#### 2. Press the **Display and volume** button.

Display and volume	X	Logout
	$\Box$	Users
Calibrate touchscreen		System
Display brightness		Settings
<	>	Peripheral
Warning and info tones		Reset
<	$\rightarrow$	
Key tones		
$\langle$	$\rightarrow$	程, 1



- Set display brightness
- Set volume for warning and information sounds
- Set volume for button sounds

#### NOTE

Amended settings can immediately be seen/heard each time a button is pressed. You do not need to save or confirm them.

# 3.3 Calibrating the touchscreen display

1. Press the **Settings** tab.

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
		뫄. ()

#### 2. Press the **Display and volume** button.

Display and volume	X	Logout
	$\frown$	Users
Calibrate touchscreen		System
Display brightness	·	Settings
<		Peripheral
Warning and info tones		Reset
<		
Key tones		
<	$\left \right>$	윢. ()

- 3. Press the **Calibrate touchscreen** button.
- 4. Confirm that you wish to proceed. The calibration display appears:

+	
Press the	+ symbol.

5. Press the symbol. The symbol changes position.

- Press the symbol again.
   The symbol changes its position again.
- 7. Repeat step 6. until you get a request to confirm calibration.
- 8. Confirm calibration.
  - The touchscreen display is calibrated.

#### NOTE

If calibration is not confirmed, the procedure re-starts after a few seconds.


#### 3.4 Setting units of measurement



# Patient hazard

In order to avoid misinterpretations, measuring results for medical use must be displayed and used in SI units (weight: kilogrammes, length: metres) only. Some devices offer the ability to display measuring results in other units. This is only an additional function.

- Use the results exclusively in SI units.
- ► The use of measuring results in non-SI units is the sole responsibility of the user.

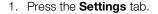

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
vitai sigiis		
		뫄.

#### 2. Press the Units button.

Units		×	Logout
Desting an entry of the set	W/-i-let	$\frown$	Users
Resting energy expenditure	Weight		System
Length	kg Blood pressure		Settings
metric	mmHg	-	Peripherals
Energy	Temperature		Reset
– LW	°C	-	

- 3. Make the desired settings:
  - Reference for resting energy expenditure
  - Unit for energy
  - Unit for weight
  - Unit for height
  - Unit for temperature
  - Unit for blood pressure

#### NOTE

Settings you make in this tab are active directly. You do not need to save or confirm them.

# 3.5 Deactivating analysis modules

	Settings		Logout
	Display and volume	Regional settings	System
			Settings
	Units	Analysis modules	Peripheral
	Vital signs		Reset
			뭥. (1)
2. Pr	ess the Analysis modules	s button.	
	Analysis modules		Logout Users
	Development/growth	✓ Vital signs	System
	Energy		Settings
	Function/rehabilitation		Peripherals
	Fluid		Reset
	Health risk		
3. Pr to Th	ess the checkboxes deactivated analysis modules are activ deactivate. Analyzing measurement.	of all the analysis	modules you would lik displayed in the analys
. –			
de	<ul> <li>reactivate analysis module eactivated analysis modules ne reactivated analysis mod Analyzing measurement.</li> </ul>	s again.	
<b>→</b>	Analyzing measurement.		
→			
<b>→</b>	NOTE • If the analysis modules E basic parameters Wais Level will not be interro	t circumference a	nd Physical Activity

# 3.6 Making presets for vital signs measurement

- → Presets for blood pressure
- → Presets for pulse rate (seca measuring equipment only)
- $\rightarrow$  Presets for SpO<sub>2</sub>
- → Presets for temperature (COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 only)
- → Selecting color mode for Vital signs tab

In the **Settings** tab, you can set up the settings for blood pressure, temperature and  $\text{SpO}_2$  measurements preferred in your institution.

The user can modify the settings during a measurement procedure → Measuring vital signs. The presets will be active again after the end of the measurement procedure.

For the Vital signs tab, you can select a color mode to enable the device to be read perfectly whatever the illumination conditions. This function is not available for other tabs. This setting cannot be changed during a measurement procedure.

#### Presets for blood pressure

- → Making general settings
- → Configuring multiple measurement

#### NOTE

This is where to specify default settings which the user can modify for the respective current measurement procedure. The settings from the default settings then become active again.

#### Making general settings

1. Press the **Settings** tab.

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
		명, 1

2. Press the Vital signs button.

Vital signs	×	Logout
		Users
NIBP	PR	System
		Settings
SpO <sub>2</sub>	ТЕМР	Peripheral
		Reset
Color mode		
Standard -		
		垊 ()

# 3. Press the **NIBP** button.

The presets are displayed.

NIBP settings			×
Туре			
O â Inflation	ÿ	Deflation	
Starting pressure			
Procedure			
O • Single measurement	:	Multiple measurement	

- 4. Press the desired measurement method.
  - Inflation, continue at step 6.
  - Deflation, continue at step 5.
- 5. Adapt Starting pressure if necessary.
- 6. Press the desired measuring sequence:
  - Single measurement
  - Multiple measurement → Configuring multiple measurement

#### NOTICE!

#### Faulty operation, missing measuring results

Results from multiple measurements cannot be transmitted to EMR systems or the **seca analytics 115** PC software.

- Specify Single measurement as a preset if blood pressure values are to be transmitted to an EMR system or the seca analytics 115 PC software.
- 7. Press the **Confirm** button.

The dialog window closes.

The modified settings are adopted.

#### Configuring multiple measurement

# NOTE

Settings you make here can**not** be changed by the user.

1. Press the **Settings** tab.

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
		琚. ()

2. Press the **Vital signs** button.

Vital signs	Logout
	Users
NIBP PR	System
	Settings
SpO <sub>2</sub> TEMP	Peripheral
	Reset
Color mode	
Standard	
	육. ()

3. Press the **NIBP** button. The presets are displayed.

NIBP settings			×
Туре			
O â Inflation	ž	Deflation	
Starting pressure			
Procedure			
O • Single measurement	:	Multiple measurement	

Press the button.
 The Configure multiple measurement dialog window appears:

Configure multiple	measurement	×
Use measurements 1 - 4 - 2 - 5 - 3 - 6 -	Meas. profile (name) Delay (minutes) O0:00 Interval (minutes) 05:42	
Total (minutes): 30:00 Maximum total must not e	exceed 30 minutes.	

- 5. Specify the number of part-measurements:
  - Maximum six part-measurements
  - Deactivated part measurements at the start and during the measuring sequence are performed but not used for forming the mean value
  - Deactivated part-measurements at the end of the measuring sequence are not completed: The running time of the measurement is reduced.

Example	Measuring sequence
$1 - 4 \checkmark$ $2 \checkmark 5 \checkmark$ $3 \checkmark 6 \checkmark$	Factory setting: 6 part- measurements Part-measurement 1 is discarded
$1 - 4 - $ $2 \checkmark 5 - $ $3 \checkmark 6 \checkmark$	Example: 6 part-measurements Part-measurements 1, 4 and 5 are discarded
$1 - 4 \checkmark$ $2 \checkmark 5$ $3 \checkmark 6$	Example: 4 part-measurements • Part-measurement 1 is discarded • Part-measurements 5 and 6 are not performed

6. Specify a start delay (waiting time before part-measurement 1):

- Format: mm:ss
- Min: 00:00, Max:10:00
- 7. Specify the interval (waiting time between part-measurements)
  - Format: mm:ss
  - Min: 01:35, max: 14:15 minutes

8. Assign a name to the measuring profile.

Configure multiple	×	
Use measurements	Meas. profile (name)	
1 - 4 -	Example 1	
2 🖍 5 – 3 🖌 6 🖍	Delay (minutes) 02:00 Interval (minutes) 04:00	
Total (minutes): 23:30 Maximum total must not (	exceed 30 minutes.	

9. Press the button.

The measuring profile is saved.

The name of the measuring profile appears in the **NIBP settings** dialog window instead of the text **Multiple measurement**.

1. Press the **Settings** tab.

Settings	Logout
	Users
Display and volume Regional settings	System
	Settings
Units Analysis modules	Peripheral
	Reset
Vital signs	
	뫄. ()

## 2. Press the Vital signs button.

Vital signs X	Logout
	Users
NIBP PR	System
	Settings
SpO <sub>2</sub> TEMP	Peripheral
	Reset
Color mode	
Standard 🚽	
	뭥. (1

# Presets for pulse rate (seca measuring equipment only)

3. Press the **PR** button.


© PR
Source: SpO <sub>2</sub>
Standard
O ≑♥ Sensitive
Cancel Confirm

4. Press the desired measuring mode (seca measuring equipment only):

seca measuring equipment			
Mode	Measuring range	Motion tolerance	
Standard	0 - 240 min <sup>-1</sup>	High	
Sensitive	20 - 300 min <sup>-1</sup>	Low	

Press the **Confirm** button. The dialog window closes. The modified settings are adopted.

#### 1. Press the **Settings** tab.

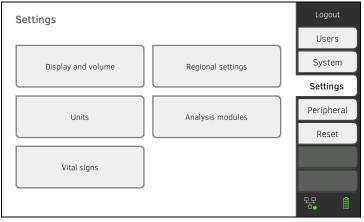

#### 2. Press the Vital signs button.

Vital signs X	Logout
	Users
NIBP PR	System
	Settings
SpO <sub>2</sub> TEMP	Peripheral
	Reset
Color mode	
Standard	
	暏. 1

Presets for  $SpO_2$ 

3. Press the **SpO<sub>2</sub>** button.

The presets are displayed (in this case: Masimo SET<sup>®</sup> pulse oximetry):

SpO <sub>2</sub>
O 🚣 Stable
🖲 🟊 Normal
O Att Sensitive
Cancel Confirm

4. Press the desired sensitivity:

#### NOTE

The Maximum setting (Masimo SET<sup>®</sup> pulse oximetry) is not available as a preset. Select this setting for each measurement directly  $\rightarrow$  Measuring oxygen saturation (SpO<sub>2</sub>).

Masimo SET <sup>®</sup> SpO <sub>2</sub> module		
Mode	Indication	
Normal	<ul><li>Normal perfusion</li><li>Mild perfusion disorders</li></ul>	
Adaptive Probe Off Detection (APOD)	Vigorous patient movements	
Maximum	<ul> <li>Poor perfusion</li> <li>Severely disrupted signal, for example due to indoor lighting or direct sunlight</li> </ul>	

seca SpO <sub>2</sub> module	
Mode	Motion tolerance
Stable	High
Normal	Normal
Sensitive	Low

#### 5. Press the **Confirm** button.

The dialog window closes. The modified settings are adopted.

# Presets for temperature (COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 only)

#### 1. Press the **Settings** tab.

Settings	Logout
	Users
Display and volume Regional settings	System
	Settings
Units Analysis modules	Peripheral
	Reset
Vital signs	
	垊. ()

2. Press the Vital signs button.

Vital signs	×	Logout
		Users
NIBP	PR	System
		Settings
SpO <sub>2</sub>	ТЕМР	Peripheral
		Reset
Color mode		
Standard -		
		程 1

- 3. Ensure that the temperature probe is pushed into the probe holder completely.
- 4. Press the **TEMP** field.

The presets are displayed (in this case: COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 blue):

•	Predictive
0	Direct
•	Oral
0 🛉	Axillary

- 5. Press the desired measurement method.
  - Predictive
  - Direct
- 6. Press the desired measuring position (COVIDIEN<sup>TM</sup> FILAC<sup>TM</sup> 3000 blue only).
  - Oral
  - Axillary
- Press the **Confirm** button. The dialog window closes. The modified settings are adopted.

Selecting color mode for Vital signs 1. Press the Settings tab. tab

Settings		Logout
		Users
Display and volume	Regional settings	System
		Settings
Units	Analysis modules	Peripheral
		Reset
Vital signs		
		뮹, ()

#### 2. Press the **Vital signs** button.

Vital signs X		Logout
		Users
NIBP	PR	System
		Settings
SpO <sub>2</sub>	TEMP	Peripheral
		Reset
Color mode		
Standard -		
		뮴, 1

- 3. Press the **Color mode** input field.
- 4. Select a color mode.
  - Standard
  - · Color, day
  - Color, night
- 5. Log off.
- 6. Press the Vital signs tab. The Vital signs tab is displayed in the selected color mode.

# 4. MANAGING SYSTEM COMPONENTS

- → Viewing system information
- $\rightarrow$  Updating the monitor software
- → Updating the measuring mat software
- → Retrofitting the SmartBucket (seca mBCA 525 only)
- → Retrofitting the measuring mat (seca mVSA 535 only)
- → Retrofitting the in-ear thermometer (seca mVSA 535 only)
- → Data export and backup
- → Power management

## 4.1 Viewing system information

1. Press the **System** tab.

System information	Logout
	Users
Monitor Measuring mat	System
	Settings
SmartBucket Data management	Peripherals
	Reset
Power management	
	명 🛛 🌒

2. Select a system component.

The system information for the selected system component is displayed (in this case: **SmartBucket**).

SmartBucket	Logout
	Users
SmartBucket active	System
Details SmartBucket Software update SmartBucket	Settings
Information SmartBucket	Peripherals
	Reset
Product information: simulated smartbucket	
	· 문 [] 自

You have the following options:

- View details
- ► Perform software update (not for SmartBucket) → Updating the monitor software

## 4.2 Updating the monitor software

This function allows you to update the software of the device. Current software packages can be found at www.seca.com.

#### NOTICE! Data loss

- Export seca patient files and user accounts manually before updating the software (for operation without seca analytics 115 PC software).
- Before updating the software, ensure that all the data on the device have been synchronized with the seca analytics 115 PC software (for operation with seca analytics 115 PC software).
- 1. Press the **System** tab.

System information	Logout
	Users
Monitor Measuring mat	System
	Settings
SmartBucket Data management	Peripherals
	Reset
Power management	
	목 [] (1)

#### 2. Press the Monitor button.

Monitor	Logout
	Users
Details Monitor Software update Monitor	System
Device name	Settings
Example 2	Peripheral
Information Monitor	Reset
Serial number: 76543217654321	
Software version: 1.0.1153 INT	
	뫄 (

- 3. Press the **Software update Monitor** button.
- 4. Select the source for the software update:
  - USB memory stick: continue at step 5.
    - Network: Continue at step 6.

active.

 Connect the USB memory stick to the monitor. The software package is automatically transmitted to the device. Following successful transmission, the Start software update button is

#### Press the Start software update button. The software package will be installed. Following successful installation, the device restarts automatically.

#### NOTE

The software package for the monitor also contains data for the measuring mat. If this data is more current than the software on the measuring mat, the corresponding message will appear on screen. Then proceed as described in the section entitled  $\rightarrow$  Updating the measuring mat software.

#### 4.3 Updating the measuring mat software

In the following cases, it may be necessary to update the software of the measuring mat:


- Monitor software has been updated (seca mVSA 535, seca mBCA 525)
  - → Updating the monitor software
- System has had a measuring mat added (seca mVSA 535)
   → Retrofitting the measuring mat (seca mVSA 535 only)
- Measuring mat has been replaced with a new one (seca mVSA 535, seca mBCA 525)

If the measuring mat software is out of date, the corresponding message will appear on the monitor. In this case, proceed as outlined below:

1. Press the **System** tab.

System information	Logout
	Users
Monitor Measuring mat	System
	Settings
SmartBucket Data management	Peripherals
	Reset
Power management	
	목 🛛 🌒

2. Press the Measuring mat button.

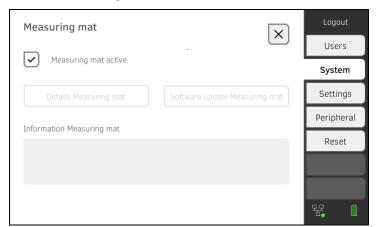

- 3. Press the Software update Measuring mat button.
- 4. Follow the on-screen instructions.
- 5. Press the **Start software update** button. The software update will be installed.

# 4.4 Retrofitting the SmartBucket (seca mBCA 525 only)

medical Body Composition Analyzer **seca mBCA 525** from serial number 1000000090505 can be retrofitted with the SmartBucket **seca mVSA 526** to measure vital signs  $\rightarrow$  Optional accessories and spare parts.

- 1. Retrofit the SmartBucket as described in the relevant assembly instructions.
- 2. Activate the SmartBucket in the device:

a) Press the System tab

	Logout
	Users
Measuring mat	System
	Settings
Data management	Peripherals
	Reset
	뀸 🛛 📋

- b) Press the SmartBucket button
- c) Activate the SmartBucket active checkbox
- d) If SmartBucket contains a temperature module: Activate the **Temp. module active** checkbox

SmartBucket	Logout
	Users
SmartBucket active	System
Details SmartBucket Software update SmartBucket	Settings
Information SmartBucket	Peripherals
anormation Sinartoucket	Reset
Product information: simulated smartbucket	
	꾹 [] (1

3. Follow the on-screen instructions.

#### NOTICE! Malfunction

If your device is connected to an EMR system, the Measure bioimpedance function can**not** be used.

- Do not retrofit a measuring mat.
- Ensure that the **Measuring mat active** checkbox is deactivated.

**seca mVSA 535** can be retrofitted with the **seca mBCA 531** measuring mat to perform bioimpedance measurements  $\rightarrow$  Optional accessories and spare parts.

- 1. Start up the measuring mat as described in the corresponding assembly instructions.
- 2. Activate the measuring mat in the device:

# a) Press the **System** tab

System information	Logout
	Users
Monitor Measuring r	nat System
	Settings
SmartBucket Data manage	Peripherals
	Reset
Power management	
	목 🛛 🗎

- b) Press the Measuring mat button
- c) Activate the Measuring mat active checkbox



3. Follow the on-screen instructions.

# 4.6 Retrofitting the in-ear thermometer (seca mVSA 535 only)

**seca mVSA 535** without temperature measurement can be retrofitted with the COVIDIEN<sup>TM</sup> GENIUS<sup>®</sup>3 in-ear thermometer  $\rightarrow$  Optional accessories and spare parts.

- Connect the in-ear thermometer as described in the section entitled → Connecting the in-ear thermometer.
- Activate the temperature module in the device:
   a) Press the **System** tab

System information	Logout
	Users
Monitor Measuring mat	System
	Settings
SmartBucket Data management	Peripherals
	Reset
Power management	
Fower management	
	맘 🛛 🌒

b) Press the SmartBucket button

c) Activate the Temp. module active checkbox

SmartBucket	Logout
	Users
SmartBucket active	System
Details SmartBucket Software update SmartBucket	Settings
Information SmartBucket	Peripherals
	Reset
Product information: simulated smartbucket	
	뫔 🛛 🌢

3. Follow the on-screen instructions.

# 4.7 Data export and backup

- → Exporting patient and user data manually
- → Restoring patient and user data manually

Exporting patient and user data manually

This function allows you to export seca patient files and user accounts in order to create a backup copy, for example.

1. Press the **System** tab.

System information		Logout
		Users
Monitor	Measuring mat	System
		Settings
SmartBucket	Data management	Peripherals
		Reset
Power management		
		무무 🗋 🗎

#### 2. Press the Data management button.

Data management	×	Logout
		Users
Export system log	Export	System
		Settings
Backup patient data	Backup	Peripherals
Restore patient data	Restore	Reset
seca analytics 115 Last synchronization: -	C	

- 3. Connect a USB memory stick to the monitor.
- 4. Press the **Backup** button.
- 5. Enter a password to encrypt the data.

#### NOTICE!

#### Data access by unauthorized persons

An insecure password may allow unauthorized persons to access patient data.

- Select a password to satisfy your institution's security requirements.
- 6. Enter the password again to confirm it.

The data are exported to the USB memory stick.

7. Archive the data in line with your institution's policy.

# Restoring patient and user data manually

This function allows you to restore externally backed-up seca patient files and user accounts.

# NOTICE!

#### Data loss

When you restore externally backed-up data, the current data on the device will be overwritten.

- Export seca patient files and user accounts manually before restoring older data manually (operation without seca analytics 115 PC software).
- Before restoring manually, ensure that all the data on the device have been synchronized with the seca analytics 115 PC software (operation with seca analytics 115 PC software).
- 1. Load the archived seca patient files and user accounts onto a USB memory stick.
- 2. Press the **System** tab.

System information	Logout
	Users
Monitor Measuring mat	System
	Settings
SmartBucket Data management	Peripherals
	Reset
Power management	
	목 [] (1)

#### 3. Press the **Data management** button.


Data management	×	Logout
	$(\land)$	Users
Export system log	Export	System
		Settings
Backup patient data	Backup	Peripherals
Restore patient data	Restore	Reset
seca analytics 115		
Last synchronization: -	5	<b>₹</b>

- 4. Connect the USB memory stick to the monitor.
- 5. Press the **Restore** button.
- 6. Enter the password you assigned on export to decrypt the data. The data will be imported.

#### 4.8 Power management

You can specify from which battery charge status (in %) the device shuts down and switches off.

- 1. Press the **System** tab.
- Press the **Power management** button. The current setting is displayed (default setting: 50 %).

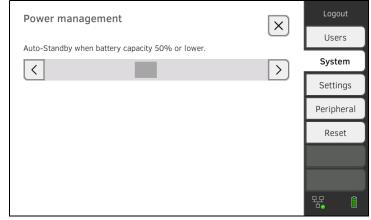

- 3. Set the desired battery charge status from which the device is to shut down and switch off.
  - Press the button to switch off the device at a lower battery charge status (minimum: 10 %).
  - Press to switch off the device at a higher battery charge status (maximum: 100 %).

# 5. SETTING UP PERIPHERALS

- → Setting up a LAN connection to the network (stationary operation)
- → Setting up a WiFi connection (mobile operation)
- → Setting up a connection to the seca analytics 115 PC software
- → Setting up a connection to the seca connect 103 software
- → Setting up a seca 360° proximity connection
- → Setting up a seca 360° wireless connection


# 5.1 Setting up a LAN connection to the network (stationary operation)

- → Introduction
- → Activating the LAN connection
- → Deactivating the LAN connection

For stationary use, for example in a treatment room at your practice, you can connect the monitor to your LAN network to exchange data with the **seca analytics 115** PC software or the **seca connect 103**.

A requirement for exchanging data with the **seca analytics 115** PC software is that the **seca analytics 115** PC software is installed on a server or standalone PC and the network configuration (UDP or DHCP) is known.

A requirement for exchanging data with the **seca connect 103** software is that the **seca connect 103** software is installed on a server and the configuration of the **seca connect 103** software (server IP and connect port) is known.

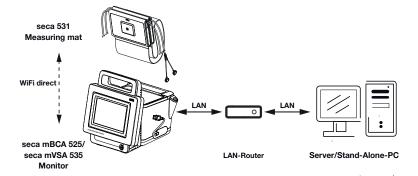

For devices with bioimpedance measurement: If you connect the device to your network via LAN cable, the measuring mat can communicate with the monitor via the factory-activated **WiFi direct (BIA mat)** connection. This allows data to be exchanged even when the measuring mat is not suspended in the magnetic catch of the monitor.

#### Introduction

# Activating the LAN connection

1. Press the **Peripherals** tab.

Peripherals		Logout
		Users
LAN	WiFi	System
		Settings
seca 360° wireless	1360° wireless seca connect	Peripheral
seca 360° proximity	Certificates	
		뫊 ()
Press the <b>LAN</b> button.		
		Logout

# 2.

Set LAN connection		×	Logout
		$( \ \ )$	Users
LAN active	DHCP		System
IP address	Default gateway		Settings
172.16.0.245	172.16.0.254		
Netmask			Peripherals
			Reset
255.255.255.0			
MAC address:			

- 3. Press the LAN active checkbox. The LAN function is activated.
- 4. Make the setting applicable to your network:
  - ► To set up connection manually, continue at step 5.
  - ► To set up automatic connection: Press the **DHCP** button and continue with  $\rightarrow$  Setting up a connection to the seca analytics 115 PC software

Set LAN connection		×	Logout
		(	Users
LAN active	UHCP		System
IP address	Default gateway		Settings
172.16.0.245	172.16.0.254		Peripherals
Netmask			Reset
255.255.255.0			
MAC address:			

- 5. Make the settings relevant to your network:
  - Enter the IP address of the monitor (last three digits must be different from the PC address)
  - Enter netmask (must match the netmask of the PC)
  - Enter default gateway (if available)

Set LAN connection		X	Logout
		$\frown$	Users
LAN active	ОНСР		System
IP address	Default gateway		Settings
172.16.0.245	172.16.0.254		Peripherals
Netmask			
255.255.255.0			Reset
MAC address:			
			<pre> </pre>

# 모모

The symbol is displayed on the monitor. The LAN connection is set up.

#### NOTE

Settings you make in this dialog window are active directly. You do not need to save or confirm them.

- 6. Continue based on your device variant:
  - ► For devices with bioimpedance measurement, continue at step 7.
  - For devices without bioimpedance measurement, continue at step 10.
- 7. Ensure that the measuring mat is correctly suspended in the magnetic catch of the monitor.
- 8. In the **Peripherals** tab, press the **WiFi** button.
- 9. In the WiFi mode drop-down menu, press the desired mode:
  - WiFi direct (BIA mat): The monitor and the measuring mat communicate with one another via a direct WiFi connection
  - WiFi off: The monitor and the measuring mat communicate with one another via the infrared interface

Set WiFi connection		Logout
ViFi mode		Users
WiFi direct (BIA mat)		System
WiFi client (network)		Settings
WiFi direct (BIA mat)	scan WiFi	Peripherals
WiFi off		Reset
<b>1AC address:</b> 00:07:80:a9:e2:d7		

- NOTE
- If the WiFi off setting is selected, the measuring mat must be suspended in the magnetic holder of the monitor before and after every measurement so that data can be exchanged via the infrared interface.
- If a BIA measuring mat is communicating with the monitor in WiFi direct (BIA mat) mode, only the LAN connection is available for connection to the network.



10. Press the button to confirm your entry.

11. You have the following options for continuing:


- ► → Setting up a connection to the seca analytics 115 PC software
- ► → Setting up a connection to the seca connect 103 software

#### **Deactivating the LAN connection**

\_ ما ما ز بر

1. Press the **Peripherals** tab.

	Peripherals			
				Users
	LAN	WiFi		System
				Settings
	seca 360° wireless	seca connect		Peripheral
				Reset
	seca 360° proximity	Certificates		
				뮥, ()
2.	Press the <b>LAN</b> button.			
	Set LAN connection		X	Logout
				Users

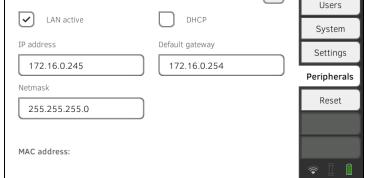

- 3. Press the LAN active checkbox. The LAN function is deactivated. The connection data are deleted.
- 4. To reactivate the LAN function, proceed as outlined below:
  - a) Press the LAN active checkbox
  - b) -> Activating the LAN connection

# 5.2 Setting up a WiFi connection (mobile operation)


- → Introduction
- → Activating the WiFi connection
- → Deactivating the WiFi connection

Introduction

For mobile use, for example on the ward of a hospital, you can integrate the device in your network as a WiFi client to exchange data with the **seca analytics 115** PC software or **seca connect 103**.

A requirement for exchanging data with the **seca analytics 115** software is that the **seca analytics 115** PC software is installed on a server or standalone PC and access data for your WiFi network are known.

A requirement for exchanging data with the **seca connect 103** software is that the **seca connect 103** software is installed on a server and the configuration of the **seca connect 103** software (server IP and connect port) is known.

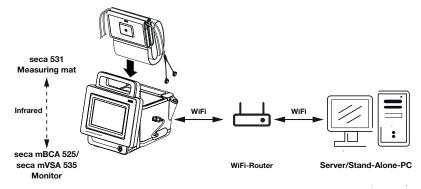

For devices with bioimpedance measurement: If you integrate the device in your network via WiFi, the measuring mat can only communicate with the monitor via the infrared interface.

#### NOTICE! Malfunction

The measuring mat and the monitor can communicate via the infrared interface only when the measuring mat is suspended in the magnetic catch of the monitor.

 Ensure that the measuring mat is suspended in the magnetic catch of the monitor before and after every measurement.

#### Activating the WiFi connection

- 1. Ensure that the measuring mat if present is correctly suspended in the magnetic catch of the monitor.
- 2. Press the **Peripherals** tab.

Peripherals	Logout
	Users
LAN WiFi	System
	Settings
seca 360° wireless seca connect	Peripheral
	Reset
seca 360° proximity Certificates	
	뮹. 1

- Make sure that the LAN connection to the network is **not** active → Deactivating the LAN connection.
- 4. Press the **WiFi** button.

Set WiFi connection		Logout
		Users
WiFi mode		System
SSID		Settings
	scan WiFi	Peripherals
		Reset
MAC address: 00:07:80:a9:e2:d7		

- 5. In the **WiFi mode** drop-down menu, press the **WiFi client (network)** setting.
- 6. Make the setting applicable to your WiFi network:
  - Visible network: Press the Scan WiFi button
  - ► Hidden network: Press the **Hidden network** button
- 7. Enter the SSID for your network:
  - ► Visible network: Select SSID from drop-down menu
  - Hidden network: Enter SSID manually
- 8. Enter the user name and password for the network.

#### NOTE

In some networks, no user name is required. In this case, the input field is grayed out.

- 9. Press the button to confirm your entry. The connection is established. The settings will be transmitted to the measuring mat if present.
  - The symbol , if measuring mat is present, and are displayed on the monitor. The device is connected to your network via WiFi.
- 10. You have the following options for continuing:
  - ► → Setting up a connection to the seca analytics 115 PC software
  - ► → Setting up a connection to the seca connect 103 software

### **Deactivating the WiFi connection**

1. Press the **Peripherals** tab.

Peripherals		Logout
		Users
LAN	WiFi	System
		Settings
seca 360° wireless	seca connect	Peripheral
		Reset
seca 360° proximity	Certificates	
		뫇. 1

2. Press the WiFi button.

Set WiFi connection		Logout
		Users
WiFi mode WiFi direct (BIA mat)		System
WiFi client (network)		Settings
WiFi direct (BIA mat)	scan WiFi	Peripherals
WiFi off		Reset
MAC address: 00:07:80:a9:e2:d7		

- 3. In the WiFi mode drop-down menu, press the WiFi off setting.
- Press the button to confirm your entry. The WiFi function is deactivated. The connection data are deleted.

# 5.3 Setting up a connection to the seca analytics 115 PC software


- → Introduction
- ightarrow Connecting the device automatically (UDP)
- $\rightarrow$  Connecting the device manually (TCP)
- → Exporting patient and user data manually
- → Setting up automatic export

If you connect the device to the **seca analytics 115** PC software, the following functions are available:

- Synchronize patient and user data automatically → Exporting patient and user data manually.
- **seca directprint**: Issue results reports directly from the device on a network printer → Viewing current measuring results.

A requirement is that the device is connected to the LAN or WiFi network of your institution.


- → Setting up a LAN connection to the network (stationary operation)
- → Setting up a WiFi connection (mobile operation)

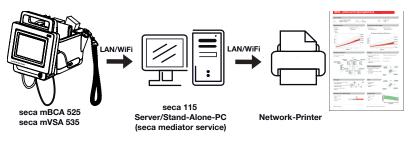

#### NOTE

- You specify the network printer for the **seca directprint** function directly in the **seca analytics 115** PC software.
- You should also follow the administrator manual for the seca analytics 115 PC software.

#### 1. Press the **Peripherals** tab.

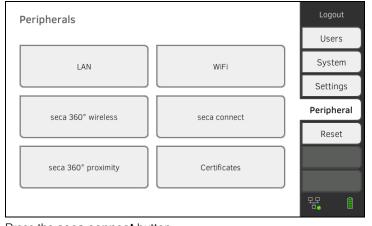

Press the seca connect button.
 The seca connect configuration dialog window opens.

# Connecting the device automatically (UDP)

Introduction

# 17-10-05-353-002h\_2023-01 B

# 3. In the **Connect Mode** drop-down menu, select the option **seca analytics 115 UDP**.

The recommended presets are displayed.

seca connect configuration	on 🗸 🗙	Logout
		Users
Connect Mode	seca analytics 115 UDP	System
		Settings
Connect Server IP		Peripherals
		Reset
Connect Port	20011	
	$\rightarrow$	


- 4. Adapt the presets if necessary.
- 5. Ensure these settings match those in the **seca analytics 115** PC software (menu: Extras\Settings\Synchronization).



- Press the button:
  Activating automatic synchronization
  Press the button:
  - Confirm settings, exit dialog

# Connecting the device manually (TCP)

1. Press the **Peripherals** tab.

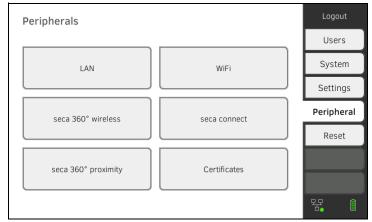

2. Press the **seca connect** button.

The seca connect configuration dialog window opens.

# 3. In the **Connect Mode** drop-down menu, select the option **seca analytics 115 TCP**.

The recommended presets are displayed.

seca connect configurati	on 🗸 🗙	Logout
		Users
Connect Mode	seca analytics 115 TCP	System
		Settings
Connect Server IP	0.0.0.0	Peripherals
Connect Port	20011	Reset
	$\rightarrow$	<pre></pre>

- 4. Adapt the presets if necessary.
- 5. In the **Connect Server IP** line, enter the appropriate IP address.

Configuration	IP address
	The IP address of the PC on which
seca analytics 115 PC software as client/server solution	the <b>seca analytics 115</b> PC software was installed with the <b>Server</b> or
as client/server solution	
	Complete options
seca analytics 115 PC software	IP address of the PC workstation
as a standalone solution	I address of the FO workstation

 In the Connect Port line, enter the port of the PC selected under 5. (default setting: 20010).

You have the following options for continuing:



If you are using the device in combination with the **seca analytics 115** PC software, you should activate automatic synchronization. The following data are then synchronized as soon as they are changed either on the device or in the PC software:

- seca patient files
- User accounts and passwords
- Automatic export settings

#### NOTICE! Malfunction

In order to be able to use automatic synchronization, a connection to the PC software must be set up  $\rightarrow$  Setting up a connection to the seca analytics 115 PC software.

 Ensure that the connection to the PC software is set up before you activate automatic synchronization.

Activating automatic synchronization


1. Press the **Peripherals** tab.

Peripherals		Logout
		Users
LAN	WiFi	System
		Settings
seca 360° wireless	seca connect	Peripheral
		Reset
seca 360° proximity	Certificates	
		垊. ()

- 2. Press the **seca connect** button.
- 3. In the **Connect Mode** drop-down menu, select the option seca analytics 115 TCP or seca analytics 115 UDP.
- 4. Press the button to reach the **seca analytics 115 configuration** page.

seca analytics 115 configuration		
Automatic synchronization		
Automatic export (CSV)		
Automatic export (EMR)		
File Transfer TCP Port 20012		
$\overline{\langle}$		

- 5. Press the **Automatic synchronisation** checkbox. Automatic synchronization is activated.
- 6. To adopt the settings, proceed as follows:

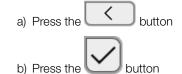

The settings are adopted.

#### NOTE

- The data are synchronized as soon as they are changed either on the device or in the PC software. No further settings are required.
- In the event of conflicts during synchronization, the data of the **seca analytics 115** PC software take priority. The device data will be overwritten.

#### Setting up automatic export

This function allows you to transmit measuring results to an EMR system automatically. The export takes place as soon as data are generated or amended on the device. To be able to use this function, an interface to the **seca analytics 115** PC software must be set up in your EMR system.


- 1. Establish which data format your EMR system accepts.
- 2. Press the **Peripherals** tab.

Peripherals		Logout
		Users
LAN	WiFi	System
		Settings
seca 360° wireless	seca connect	Peripheral
		Reset
seca 360° proximity	Certificates	
		못 1

- 3. Press the **seca connect** button.
- 4. In the **Connect Mode** drop-down menu, select the option **seca analytics 115 TCP** or **seca analytics 115 UDP**.
- 5. Press the button. The **seca analytics 115 configuration** page is displayed.

seca analytics 115 configuration		
Automatic synchronization		
Automatic export (CSV)		
Automatic export (EMR)		
File Transfer TCP Port 20012		
<		

- 6. Ensure that the Automatic synchronisation checkbox is activated.
- 7. Press the desired export format.
  - Automatic export (CSV)
  - Automatic export (EMR)
- 8. To adopt the settings, proceed as follows:

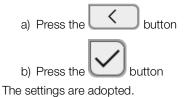


9. Set up an interface to the **seca analytics 115** PC software in your EMR system.

#### NOTE

- Settings you make in this dialog window are transmitted to the **seca analytics 115** PC software → Exporting patient and user data manually.
- Note the user documentation for the EMR system used.
- seca Service will be pleased to assist if you have any questions relating to interface configuration.

### 5.4 Setting up a seca 360° proximity connection

Use a **seca 360° proximity** connection to adopt measured values from compatible seca measuring devices (scale, measuring rod, measuring station).

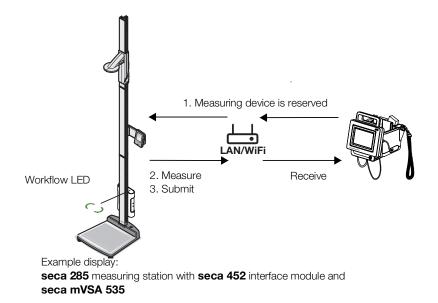

The following requirements must be met in order to connect:

- The seca measuring device is equipped with either a seca 452 external interface module or with an internal interface module (in each case firmware version Q2\_2020\_Build\_615 or higher).
- The seca measuring device is connected to your LAN or WiFi network.
- The seca measuring device is configured as follows:
  - Scan user ID: Required
  - Scan patient ID: Required
  - Confirmation of measurement on seca measuring device: Required
  - Port used: Port configured for communication with the seca connect 103 software (default: 22020) → Setting up a connection to the seca connect 103 software

#### NOTE

Information about how to configure your seca measuring device can be found in the **seca 103/452** system instructions for use.

Proceed as follows to connect the seca measuring device:

1. Press the **Peripherals** tab.


Peripherals		Logout
		Users
LAN	WiFi	System
		Settings
seca 360° wireless	seca connect	Peripheral
		Reset
seca 360° proximity	Certificates	
		윢. 1

#### Press the seca 360° proximity button. The seca 360° proximity dialog window opens

seca 360° proximity		X	Logout
			Users
Filter by organization ID		(+) (S)	System
Device name	IP	- ×	Settings
			Peripheral
			Reset
			뮹. ()

- 3. Search for available devices:
  - Press the button: Search for devices on the network

Press the button: Enter the IP address of a device
 Available devices are displayed.

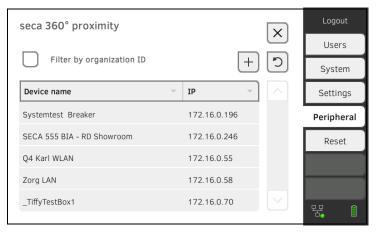

#### NOTE

- If you have assigned organization IDs to your devices, you can filter them accordingly and thus shorten the list of results.
- Information about how to assign organization IDs can be found in the **seca 103/452** system instructions for use.
- 4. Press the desired device in the list:

seca 360° proximity	>	Logout
		Users
Filter by organization ID		System
Device name	- IP -	Settings
Systemtest Breaker	172.16.0.196	Peripheral
SECA 555 BIA - RD Showroom	172.16.0.246	Reset
Q4 Karl WLAN	172.16.0.55	
Zorg LAN	172.16.0.58	
_TiffyTestBox1	172.16.0.70	

The device is shown blue in the list.

The connection is set up.

In measuring mode, the seca measuring device is reserved as soon as a patient file is called up. The device is released again once the measurement is stopped  $\rightarrow$  Receiving weight and height (seca 360° proximity).


### 5.5 Setting up a seca 360° wireless connection

- → Introduction
- → Activating/deactivating the seca 360° wireless module
- → Setting up a seca 360° wireless connection

#### Introduction

If you use devices from the **seca 360° wireless** system in your institution, e.g. a measuring station, you can connect them to the **seca mVSA 535**/ **seca mBCA 525**. You can submit the parameters height and weight directly to the device and adopt them in a seca patient file.

On the seca mVSA 535, the values also appear in the Vital signs tab.

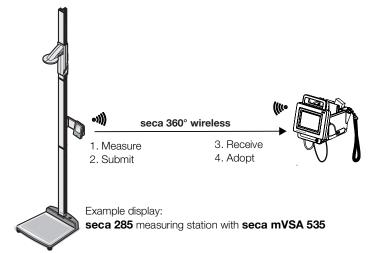


In order to be able to communicate with devices from the seca 360° wireless system, the seca 360° wireless module of the seca mVSA 535/ seca mBCA 525 must be activated and a wireless group set up.

#### NOTE

- The device can only receive data, not transmit it, via the **seca 360° wireless** connection.
- Follow the instructions in this section to achieve a stable connection of the desired **seca 360° wireless** devices and to prevent the undesired integration of other devices.
- You should also follow the instructions for use for the **seca 360° wireless** system devices being used.

# Activating/deactivating the seca 360° wireless module


1. Press the **Peripherals** tab.

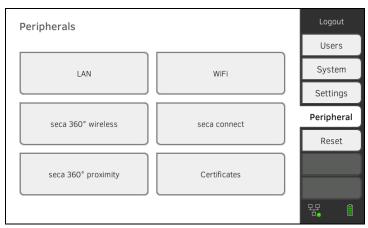

### 2. Press the **seca 360° wireless** button.

seca 360° wireless				×	Logout
					Users
Send/receive on			Set up	J	System
Device type	~	w/less gp. –	Channels -		Settings
					Peripheral
					Reset
					뫄. ()

- 3. Press the **Send/receive on** checkbox. The **seca 360° wireless** module is activated.
- 4. To deactivate the seca 360° wireless module, press the Send/receive on checkbox again.
- Ensure that the seca 360° wireless modules of the seca mVSA 535/ seca mBCA 525 and of all devices you want to connect to the seca mVSA 535/seca mBCA 525 are activated.
- 2. Switch off all the devices you want to connect to the seca mVSA 535/ seca mBCA 525.
- 3. Press the **Peripherals** tab.

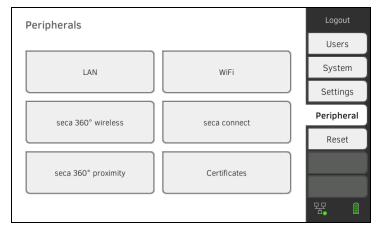

Setting up a seca 360° wireless connection

4. Press the seca 360° wireless button.



#### 5. Press the Set up button.

6. Press one of the three wireless groups.

The device searches for **seca 360° wireless** devices which are switched on.

The device will suggest three wireless channels.

seca 360° wireless	$\left  \times \right $
Wireless group	Channel 1 Channel 2 Channel 3
O Wireless group 0	03 - 60 - 98 -
Wireless group 1	
O Wireless group 2	
${\mathbb I}_{{\mathfrak g}_{{\mathfrak g}_{\mathcal G}}^{\sigma_{\mathfrak g}}}^{\sigma_{\mathfrak g}}}$ Number of devices found:	
	Finish

#### NOTICE!

#### Incorrect device assignment

Only one example of a device category (e.g. personal scale or measuring rod) can be integrated in each wireless group.

► Take note of the Technical Data in the section entitled → seca 360° wireless system of the instructions for use.

#### NOTICE!

#### Data transmission interference

The proposed channel numbers can be edited. This function is intended purely for special reception situations. Devices may be assigned to the wrong wireless groups or unreliable wireless connections may arise.

- Only change the channel numbers if the proposed channel numbers do not result in a reliable wireless connection.
- Ensure that the new channel numbers are not being used for other wireless groups.
- Ensure that the channel numbers are at least 30 apart from one another.
- 7. Switch on all **seca 360° wireless** devices you wish to incorporate in the wireless group.

When devices are detected, a beep will be heard. Detected devices will be displayed on the monitor.

Press the Finish button once all seca 360° wireless devices have been detected.
 The seca 360° wireless connection is set up.


17-10-05-353-002h\_2023-01 B

# 6. CONNECTING TO AN EMR SYSTEM

- → Information about user and patient data
- → Assigning a clear device name
- → Setting up a connection to the seca connect 103 software
- → Configuring data transfer to the EMR system
- → Use of certificates
- → Summary: Device characteristics when connected to EMR system

You can connect the device to compatible EMR systems via the **seca connect 103** software.

seca recommends performing this integration exclusively in collaboration with seca Service and the manufacturer of your EMR system.

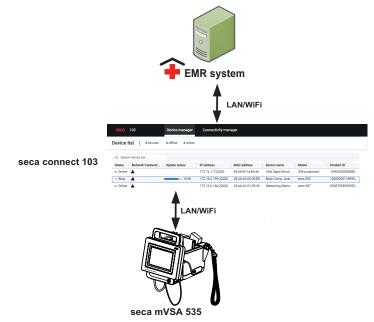

The following functions are available when you connect to an EMR system:

- Log in to device with EMR user ID
- Call up EMR system patient data on the device
- Submit measuring results to the EMR system

Information about the measuring sequence when you connect to an EMR system can be found here:  $\rightarrow$  Operation when connected to an EMR system.

### 6.1 Information about user and patient data

When connecting to an EMR system, the following changes apply to the management of user and patient data:

- User IDs for medical staff can only be set up in the EMR system.
- Patient files can only be managed in the EMR system.
- User IDs for administrators and service technicians can only be set up on the device.

17-10-05-353-002h\_2023-01 B

# 6.2 Assigning a clear device name

For the device to be connected to an EMR system, it needs to be assigned a clear device name. Under this device name, the device transmits a heartbeat to the network every 30 seconds.

To assign a device name, proceed as follows:

- 1. Log in as administrator.
- 2. Press the **System** tab.

System information	Logout
	Users
Monitor Measuring mat	System
	Settings
SmartBucket Data management	Peripherals
	Reset
Power management	
	· 문 🛛 🇯

#### 3. Press the **Monitor** button.

The **Monitor** dialog window is shown:

Monitor	Logout
	Users
Details Monitor Software update Monitor	System
Device name	Settings
Example 2	Peripheral
Information Monitor	Reset
Serial number: 76543217654321	
Software version: 1.0.1153 INT	
	윢, ()

- 4. Enter a clear device name in line with the naming conventions of your institution:
  - a) Press the Device name input field



# 6.3 Setting up a connection to the seca connect 103 software

- → Setting up a seca 360° proximity connection
- → Making workflow settings

The device must be connected to the **seca connect 103** software in order to be able to communicate with your EMR system.

The following requirements must be met for this:

- The BIA measuring mat is deactivated → Retrofitting the measuring mat (seca mVSA 535 only)
- The device is connected to your LAN or WiFi network
- The server address and port for the **seca connect 103** are known
- The EMR system is connected to the **seca connect 103** software. Information on this can be found in the **seca 103/452** system instructions for use (from version 2.0 or higher)

#### NOTE

- Follow the information in this section of the document → Monitor interfaces and network ports.
- Information on connecting the EMR system to the seca connect 103 software can be found in the seca 103/452 system instructions for use (from version 2.0 or higher)

#### 1. Press the **Peripherals** tab.

Peripherals		Logout
		Users
LAN	WiFi	System
		Settings
seca 360° wireless	seca connect	Peripheral
		Reset
seca 360° proximity	Certificates	
		뫊. 1

#### 2. Press the **seca connect** button.

The seca connect configuration dialog window opens.

**Entering connection data** 

190 • seca mVSA 535/seca mBCA 525

# 3. In the **Connect Mode** drop-down menu, select the option **seca connect 103**.

seca connect configura	tion 🗸 🗙	Logout
		Users
Connect Mode	seca connect 103 -	System
		Settings
Connect Server IP	0.0.0.0	Peripherals
		Reset
Connect Port	22020	
	$\rightarrow$	

- 4. Enter the connection data for the **seca connect 103** software:
  - Enter the IP address of the server on which the seca connect 103 is installed
  - Enter the port for the seca connect 103 (recommended: 22020)


You have the following options for continuing:



Making workflow settings

You can specify the following settings for the measuring sequence and communication with the EMR system:

- Password settings for the user ID
- Scan sequence for user and patient ID
- Permit offline measurements if the EMR system connection is interrupted (recommended for mobile use or difficult WiFi reception conditions)
- Enter an organization ID to assign the device to a department in your institution, for example
- 1. Press the Peripherals tab.

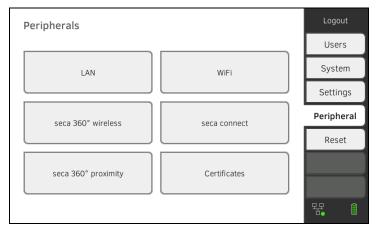

- 2. Press the **seca connect** button.
- 3. Ensure that in the **Connect Mode** drop-down menu the option **seca connect 103** is selected.

4. Press the button. The **seca connect 103: Workflow** page is displayed.

	seca connect 103: Wo	orkflow			
	User/patient ID: permit any scan sequence				
	Allow offline measure	ements			
	User ID	required, without password			
	Organization ID	791430			
	<				
5.	Specify the settings for	the scan sequence for user and patient ID:			
	► Sequenc	e for ID scan fixed: 1. User, 2. Patient			
		e for ID scan flexible			
6.		rements are to be possible even if the connection			
		terrupted (offline measurements):			
	▶ □ Do not a	llow offline measurement			
	► Allow off	ine measurements			
	NOTE				
		ts are temporarily saved in the <b>Unsubmitted</b>			
		. When the connection to the EMR system is easurements can be called up, evaluated and sub-			
	mitted to the EMR s	system $\rightarrow$ Using the Unsubmitted measurements			
7	list. Under <b>User-ID</b> specify	whether a password is required for login:			
<i>.</i>		be entered (recommended)			
	No password required				
8.	If desired, enter an <b>Organization ID</b> to assign the device to a department				
	in your institution, for ex	ample.			
9.	Adopt the settings:				
	a) Press the	button			
	b) Press the	button			

The settings are saved.

# 6.4 Configuring data transfer to the EMR system

seca recommends performing the data transfer to the EMR system only in collaboration with seca Service and the manufacturer of your EMR system.

#### NOTE

More information on this can be found in the **seca 103/452** system instructions for use (from version 2.0 or higher).

# 6.5 Use of certificates

The device is prepared for the use of certificates. If you wish to use this function, please contact seca Service.

Cert	Certificates		1	X	Logout
				$\frown$	Users
	Type of certificate	Valid from	Valid thru		System
	seca connect 103 CA	27.06.2019	24.06.2029		Settings
	seca connect 103 Client	27.06.2019	24.06.2029		Peripheral
	WiFi 802.1X	11.02.2019	11.02.2020		Reset
	Proximity Server				
1					暏. 1

# 6.6 Summary: Device characteristics when connected to EMR system

Function	Standalone mode (factory setting)/ connection to seca analytics 115	Connection to EMR system via seca connect 103
User identification (measure)	seca user account, role: Physician	EMR user ID
User identification (manage)	seca user account, role: Admin	Device-specific user account, role: Admin
Login	Select from list	<ul><li>Scan barcode</li><li>Manual</li></ul>
Password	Manual	Manual
Patient identification	Select from list	<ul><li>Scan barcode</li><li>Enter manually</li></ul>
Source of patient data	Device     seca analytics 115	EMR system
Create/edit patient data on device	Possible	Not possible
Storage location for	<ul> <li>Device database</li> </ul>	<ul> <li>Unsubmitted measurements list</li> </ul>
measurements	<ul> <li>seca analytics 115</li> </ul>	<ul> <li>EMR system</li> </ul>
Unsubmitted measurements list	Not available	Available

# 7. FACTORY SETTINGS

- → Overview of factory settings
- → Resetting the device
- → Resetting the user interface
- → Exporting the system log/audit trail
- → Enabling VNC access

# 7.1 Overview of factory settings

In the **Reset** tab, you can reset the device to the following factory settings:

Function	Setting
Administrator password	1357
Display language	English
Date format:	
International	dd.mm.yyyy
Time format:	
International	24 h
Naming convention:	
International	Surname, first name
Name separator	Decimal point
Display brightness	100 %
Volume for warning and	
information sounds	70 %
Volume for button sounds	70 %
Rechargeable battery capacity,	≤ 50 %
automatic switch-off at:	≤ 50 %
Weight:	
International	kg
Height:	
International	m
Blood pressure:	
Unit	mmHg
Presets	Inflation, single measurement
Temperature:	
Unit	C°
COVIDIEN <sup>TM</sup> FILAC <sup>TM</sup> 3000	
Blue	Oral measurement, predictive
	measurement
Red	Predictive measurement
Pulse rate:	
Unit	min <sup>-1</sup>
Preset	
(seca measuring equipment only)	Default setting
Oxygen saturation:	~
Unit	%
	Normal
Decimal separator:	0
International	Comma
Energy:	MJ
Reference resting energy	FAO/WHO/UNU
expenditure: LAN	
	On
Connection data	None

Function	Setting
WiFi client (device <-> network):	Off
Connection data	None
Direct WiFi (device <->	
measuring mat):	
seca mBCA 525	On
seca mVSA 535	Off
Device name	[serial number]
Integration mode	Standalone
seca 360° wireless:	Off
Connection data	None
seca software	
seca connect 103:	
Communication server IP	None
Connect port	22020
seca software	
seca analytics 115:	Off
UDP	Off
Communication server IP	None
TCP port	20010
UDP port	20011
File transfer port	20012
Active analysis modules	
seca mBCA 525	All
seca mVSA 535	Vital signs

# 7.2 Resetting the device

Use the **Reset device** function to reset the device to factory settings  $(\rightarrow \text{Overview of factory settings})$ . seca patient files and user accounts are deleted from the device in this process.

#### NOTE

If seca patient files and user accounts are to be retained, proceed as described in the section entitled  $\rightarrow$  Resetting the user interface.

1. Press the Reset tab.

Reset	Logout
D	
VNC access	Users
Reset device Reset GUI	System
	Settings
	Peripheral
	Reset
	$\sim$
	뀸 🛛 🃋

2. Ensure that seca patient files and user accounts have been exported or synchronized using the **seca analytics 115** PC software.

#### 3. Press the **Reset device** button.

seca patient files and user accounts will be deleted. The device will be reset to factory settings  $\rightarrow$  Overview of factory settings. Use the **Reset GUI** function to reset the user interface (GUI = Graphical User Interface) of the device to factory settings  $\rightarrow$  Overview of factory settings. seca patient files and user accounts will be retained in this process.

#### NOTE

If all seca patient files and user accounts are to be removed from the device, proceed as described in the section entitled  $\rightarrow$  Resetting the device.

1. Press the Reset tab.

VNC access	Users
Reset device Reset GUI	System
	Settings
	Peripheral
	Reset
	$\sim$
	명 🛛 🋈

#### 2. Press the Reset GUI button.

The user interface will be reset to factory settings  $\rightarrow$  Overview of factory settings.

seca patient files and user accounts will be retained.

# 7.4 Exporting the system log/audit trail

This function allows you to export the system log/the audit trail and make it available to seca Service for support purposes, for example.

- 1. Press the **System** tab.
- 2. Press the Data management button.

Data management	×	Logout
		Users
Export system log	Export	System
		Settings
Backup patient data	Backup	Peripherals
Restore patient data	Restore	Reset
seca analytics 115 Last synchronization: -	C	

- 3. Press the **Export** button.
- 4. Select the export destination:
  - ► USB memory stick: continue at step 5.
  - ► Network: Continue at step 6.

- 5. Connect a USB memory stick to the monitor.
- 6. Press the **Export system log** button. The system log/audit trail is exported.

#### NOTE

If you have selected Network as the export destination, the system log in the installation directory of the **seca analytics 115** PC software (server or standalone PC) will be exported to the Program-Data\seca\LogExports folder.

# 7.5 Enabling VNC access

With a VNC connection, you can reproduce the user interface of the device on a PC screen and remotely control the device from the PC. The requirement is that a VNC viewer is installed on the PC.

1. Press the **Reset** tab.

Reset	Logout
VNC access	Users
Reset device Reset GUI	System
	Settings
	Peripheral
	Reset
	$\sim$
	명 🛙 🗎

- 2. Press the **VNC access** checkbox.
- 3. Restart the device.
  - The VNC service of the device will be started.
- 4. Set up the VNC connection using the VNC viewer of your PC.

# Medical Measuring Systems and Scales since 1840

seca gmbh & co. kg Hammer Steindamm 3–25 22089 Hamburg · Germany T +49 40 20 00 00 0 F +49 40 20 00 00 50 E info@seca.com

seca operates worldwide with headquarters in Germany and branches in: seca france seca united kingdom seca north america seca schweiz seca zhong guo seca nihon seca mexico seca austria seca polska seca middle east seca suomi seca américa latina seca asia pacific seca danmark seca benelux seca lietuva and with exclusive partners in more than 110 countries. All contact data at seca.com