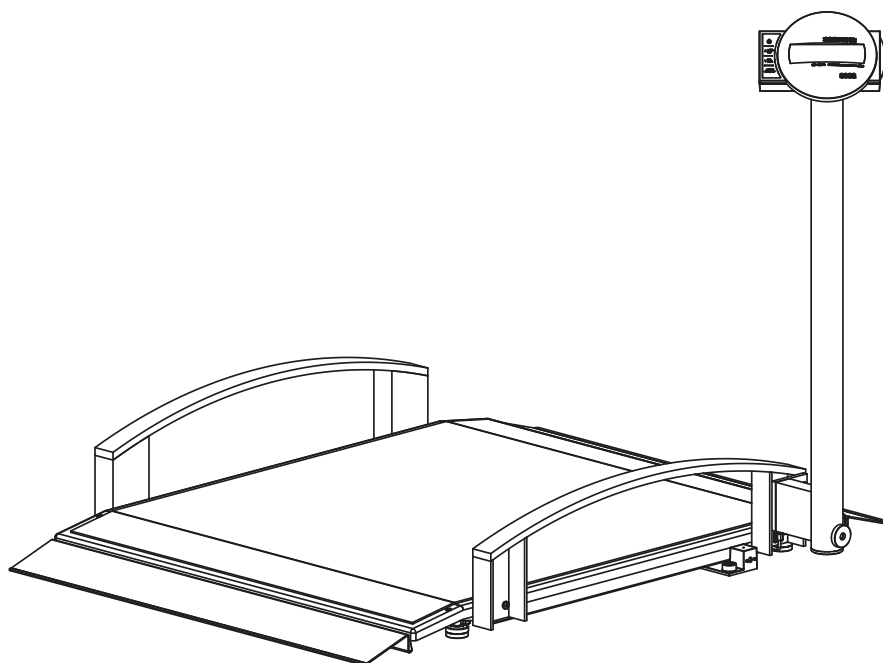
seca 665/seca 664

Instructions for use

17-10-07-678-100_2025-08S

Deutsch – Gebrauchsanweisung	3
English – Instructions for use	34
Français – Mode d’emploi.....	64
Italiano – Istruzioni per l’uso	95
Español – Instrucciones de uso	126
Português – Manual de instruções de utilização	157
Polski – Instrukcja obsługi.....	188

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TABLE OF CONTENTS

1 About this document	35	6 Operation.....	51
1.1 Representation in text	35	6.1 Safety information	51
1.2 Representation in diagrams	35	6.2 Weighing.....	51
1.3 Devices in this document	35	Switching on the scale	51
Devices in this document	35	Weighing a patient.....	52
2 Description of device	36	Taring off additional weight (TARE)	52
2.1 Intended use	36	Keep the measuring result on the	
2.2 Contraindications	36	display (HOLD)	52
2.3 Clinical benefit	36	Determining Body Mass Index	53
2.4 Patient target group.....	36	Switching the weighing range	
2.5 User qualification.....	36	(verified models)	53
Assembly	36	Switching off the scale	54
Operation.....	36	6.3 Other functions (menu).....	54
Administration/network operation	36	Deleting saved values automatically	
2.6 Functional description	36	(ACLR).....	54
Measuring weight	36	Saving additional weight permanently	
External interface module (optional		(Pt)	54
accessory)	36	Activating Autohold function (AHold).....	55
seca connect 103 software (optional		Activating/deactivating beeps (bEEP)	55
accessory)	36	Setting filtering (FIL).....	55
3 Safety information	37	Non-verified models: Switching the	
3.1 Safety information in these instructions		weight unit (Unit)	56
for use.....	37	Restoring factory settings (rESet)	56
3.2 Basic safety information	37	7 Hygiene treatment	57
Handling device.....	37	7.1 Cleaning	57
Preventing electric shock	38	7.2 Disinfecting.....	57
Preventing injuries and infections.....	38	7.3 Sterilizing.....	57
Preventing damage to device.....	39	8 Function check	57
Handling measuring results	39	9 Troubleshooting.....	58
Handling packaging material.....	40	10 Servicing	59
Handling batteries and rechargeable		10.1 Verified scales	59
batteries.....	40	10.2 Non-verified scales.....	60
4 Overview	41	11 Technical data.....	60
4.1 Controls	41	11.1 General technical data	60
4.2 Symbols in the display	42	11.2 Weight measurement	61
4.3 Markings on the device and on the type		12 Optional accessories and spare parts.....	62
plate.....	43	13 Disposal	62
4.4 Menu structure	45	13.1 Disposing of the device.....	62
4.5 Navigating in the menu.....	45	13.2 Disposing of batteries and rechargeable	
5 Starting up device.....	46	batteries.....	62
5.1 Scope of delivery	46	14 Warranty	63
5.2 Assembling device	46	15 Declaration of conformity	63
Fitting the column.....	46		
Connecting the display cable.....	47		
Fitting the ramp	48		
Final steps	48		
5.3 Establishing power supply	48		
Connecting the battery block.....	48		
Connecting the power supply unit	48		
5.4 Setting up the data connection (optional)	49		
5.5 Making the scale operational	49		
Setting up the column	49		
Aligning the scale	49		
5.6 Transporting the scale.....	50		
Lowering the column	50		
Moving the scale	50		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

1.2 Representation in diagrams

Symbol	Description
	Indicates directions of movement of the device or of device components
	Navigation path in menu displays
	• Correct action • Correct result of action
	• Incorrect action • Incorrect result of action
	End of a procedure, e.g. the installation of a part

1.3 Devices in this document

This document applies to the following devices:

- **seca 665:** Verified scale
- **seca 664:** Non-verified scale

2 DESCRIPTION OF DEVICE

2.1 Intended use

The multifunctional scale supports physicians in decision-making regarding diagnosis or therapy, based on weight or weight changes.

2.2 Contraindications

No contraindications are known.

2.3 Clinical benefit

The multifunctional scale supports physicians in decision-making regarding diagnosis or therapy, based on the measured weight.

By that, the multifunctional scale only provides an indirect clinical benefit.

The physician must perform additional examinations for an accurate diagnosis.

2.4 Patient target group

The scale is intended for persons of any age (with the exception of babies) who

- do not exceed the maximum weight capacity of the scale and
- use a wheelchair or
- are able to stand independently on the scale with the support of a handrail.

2.5 User qualification

Assembly	Devices shipped partly assembled may only be assembled by sufficiently qualified persons (such as specialist dealers, hospital technicians or seca Service technicians).
-----------------	--

Operation	The device may only be operated by persons with formal training in the healthcare sector or in medicine.
------------------	--

Administration/network operation	The device may only be set up and incorporated in a network by experienced administrators or hospital technicians.
---	--

2.6 Functional description

Measuring weight	Weight calculation is performed by four load cells.
-------------------------	---

External interface module (optional accessory)	Data can be sent to the seca connect 103 software via a WiFi or LAN connection with the aid of the seca 452 external interface module (optional).
---	---

seca connect 103 software (optional accessory)	The seca connect 103 software obtains measurement data from an internal or external interface module and sends them to compatible seca measuring devices or to third-party EMR systems.
---	--

3 SAFETY INFORMATION

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

CAUTION!

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

DANGER!
Risk of explosion

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
 - oxygen
 - flammable anesthetics
 - other flammable substances/air mixtures

CAUTION!
Patient hazard, damage to device

- ▶ Additional devices connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the requirements of standards for medical systems (see IEC 60601-1-1 or Section 16 of edition 3.1 of IEC 60601-1 respectively). Anyone connecting additional devices to medical electrical devices is considered a system configurator and therefore responsible for ensuring that the system complies with the requirements of standards for systems. This also applies to additional devices recommended by seca. Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**CAUTION!****Patient hazard, malfunction**

- ▶ Keep medical electrical devices such as high-frequency surgical equipment at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ Keep high-frequency equipment such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to www.seca.com.

Preventing electric shock**WARNING!****Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

Preventing injuries and infections**WARNING!****Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited motor skills when they are getting up, e.g. from a wheelchair.
- ▶ Ensure that the patient does not step directly onto or off the edges of the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

**WARNING!****Danger of slipping**

- ▶ Ensure that the weighing platform is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

Preventing damage to device

WARNING!

Risk of infection

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Should the patient have contagious diseases, subject the device to a hygiene treatment as described in the relevant section of these instructions for use immediately afterwards.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.
- ▶ Subject the device to a hygiene treatment at regular intervals.

NOTICE!

Damage to device

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ If you are not using the device for an extended period of time, remove batteries or battery packs. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions.
- ▶ Do not use the device over 3000 m above mean sea level. Electronic components might become damaged.
- ▶ Store the device only in the intended storage conditions.
- ▶ Use only cleaning agents and disinfectants that match the details in the section entitled "Hygiene treatment".
- ▶ Ensure that maximum capacity is not exceeded.

Handling measuring results

CAUTION!

Patient hazard

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- ▶ Only use measuring results in SI units.
- ▶ The user takes sole responsibility for the use of measuring results in non-SI units.

NOTICE!**Inconsistent measuring results**

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

Handling packaging material**WARNING!****Risk of suffocation**

Packaging materials made of plastic film (bags) present a risk of suffocation.

- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

Handling batteries and rechargeable batteries**WARNING!****Injury**

Batteries and rechargeable batteries contain harmful substances which may explode and so escape if not handled properly.

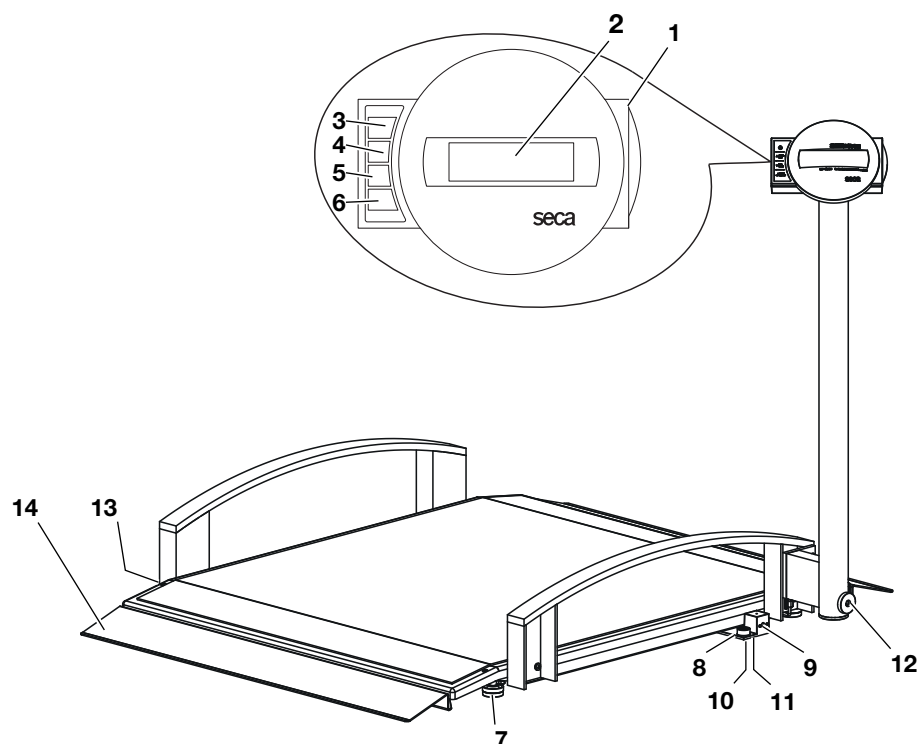
- ▶ Do not try to recharge batteries.
- ▶ Do not expose batteries/rechargeable batteries to heat.
- ▶ Do not burn batteries/rechargeable batteries.
- ▶ If harmful substances are leaking out, avoid contact with the skin, eyes, and mucous membranes. Rinse affected areas of the body with plenty of clean water and seek medical help at once.

NOTICE!**Damage to device and malfunction due to incorrect handling**

- ▶ Use only the type of battery/rechargeable battery quoted in this document.
- ▶ Always replace all batteries/rechargeable batteries at the same time.
- ▶ Do not short-circuit batteries/rechargeable batteries.
- ▶ If you do not use the device for an extended period of time, remove the batteries/rechargeable batteries. This prevents acid from leaking into the device.
- ▶ If harmful substances have penetrated the device, do not continue using it. Have the device checked by an authorized seca service partner and repaired if necessary.

4 OVERVIEW


4.1 Controls

Item	Control	Function
1	Display housing	Central control and display element
2	Display	Display element for measuring results and to configure the device
3		Switches the scale on and off
4		During weighing: - Press briefly: Activate Hold function - Press and hold: Activate Tare function In the menu: - Select submenu, select menu item
5		During weighing: - Press briefly: Activate BMI function - Press and hold: Call up menu In the menu: - Select submenu, select menu item
6		In the menu: - Confirm selected menu item - Save set value
7	Foot screw	4 pcs, for precise alignment
8	Spirit level	Indicates whether the device is horizontal
9	Power supply connection	Serves to connect the optional power supply unit

Item	Control	Function
10	Display connection	Serves to connect the display head
11	Battery pack compartment	Contains the battery block
12	Locking screw	To secure the column
13	Caster	Two pieces, for transporting short distances
14	Ramp	To access the scale e.g. with a wheelchair

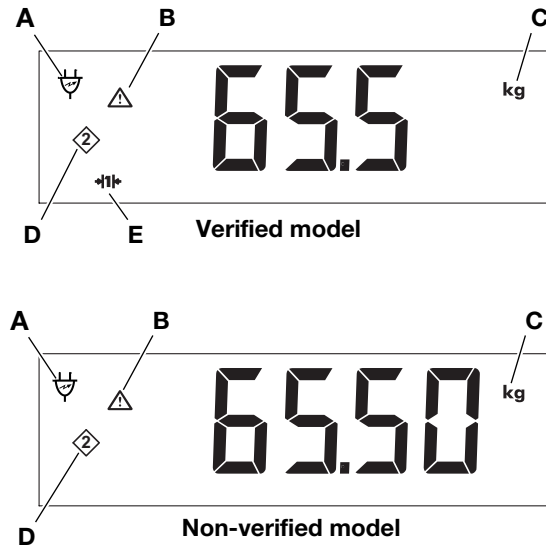4.2 Symbols in the display


NOTE

In the following chapters of these instructions for use, the display of the verified model is shown, unless a distinction between “verified/non-verified” is required.

Item	Control	Function
A		Operation with power supply unit, battery block is charged
B		Non-verifiable function active (for technical reasons, symbol is also displayed on non-verified scales)
C	kg lbs	Unit in which weight is displayed (lbs only on non-verified models)
D		Memory currently in use
E	→ 1 ← → 2 ←	Weighing range currently in use (only verified models)


4.3 Markings on the device and on the type plate

Markings on the device and on the type plate	
Symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identifier (product identification number)
	Article number
	Serial number
GAL	Value in m/s ² (model-dependent) • Gravitational acceleration on earth • Depends on the intended location
ProdID	Product identification number
Mat.No.	Variant number
Approval Type	Type designation of design approval
	Follow instructions for use
	Medical electrical device, Type B
IP20	Type of protection to IEC 60529: • Protection against ingress of solid foreign bodies with a diameter of over 12.5 mm • Protection against access with fingers • No protection from ingress of water
e	Value in units of mass (verified models) Used to classify and verify a scale
d	Value in units of mass (non-verified models) States the difference between two consecutive display values
	Active weighing range
	Class III scale to directive 2014/31/EU
	Device complies with EU directives • M: Conformity label in compliance with directive 2014/31/EU for non-automatic scales (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the CE-label was applied (verified models) • 0102: Notified Body for Metrology (verified models) • 0123: Notified Body for Medical Devices
	Medical device in accordance with regulation (EU) 2017/745
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.

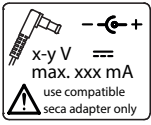Markings on the device and on the type plate	
Symbol	Meaning
	Device complies with United Kingdom directives • M: Conformity label in compliance with UK directive SI 2016 no. 1152 for non-automatic scales (NAWIR) (verified models) • 24: (Example: 2024) Year in which declaration of conformity was implemented and the UKCA label was applied (verified models) • xxxx: Notified Body for Medical Devices of the United Kingdom • yyyy: Notified Body for Metrology of the United Kingdom (verified models)
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
	Importer/representative in Switzerland: seca ag (schweiz) Medizinische Waagen und Messsysteme Schönematt Str. 2 CH-4153 REINACH
	Type plate on the power supply connection socket • Power supply voltage required in V • Maximum current consumption in mA • : Note polarity of device connector • : Operate device with direct current • : Use compatible seca power supply units only
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

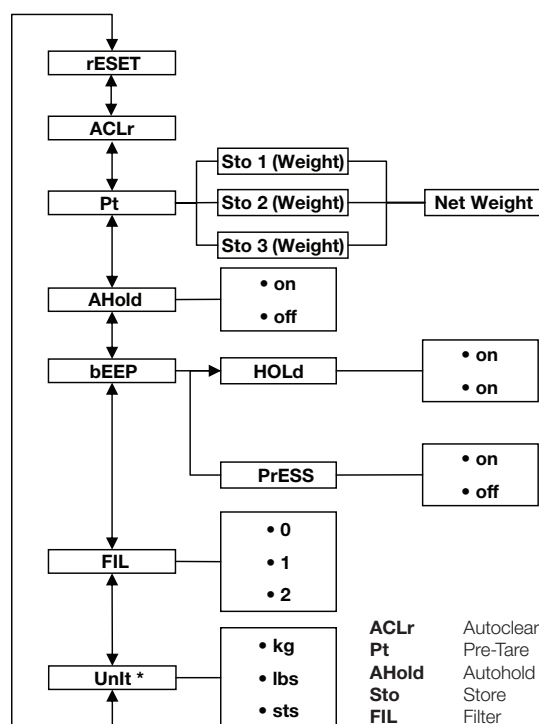4.4 Menu structure

* Unverified scales

4.5 Navigating in the menu

AHOLD

FIL

FIL 0

FIL 2

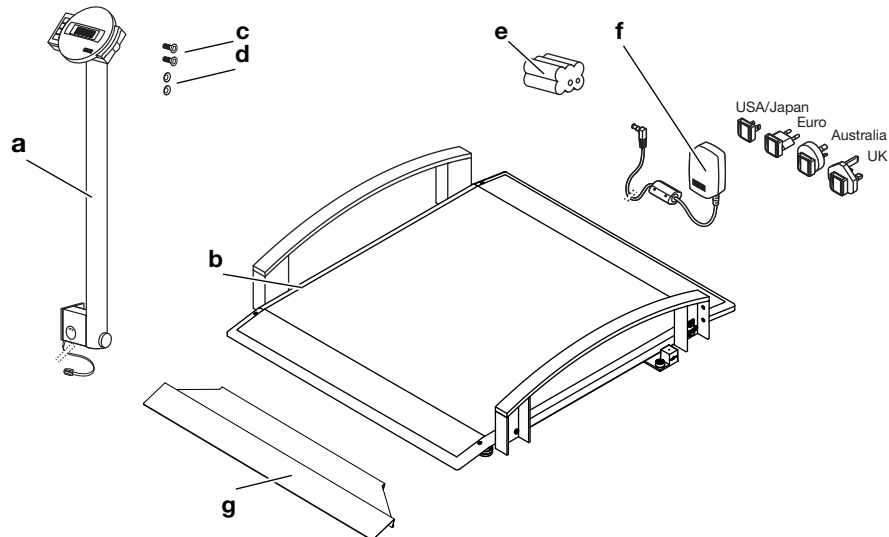
1. Switch on the scale.
2. Hold down the arrow key ▼ until the menu is called up.
⇒ The last menu item selected is displayed (in this case: Autohold **AHOLD**).
3. Press one of the arrow keys until the desired menu item appears in the display (in this case: filtering **FIL**).
4. Confirm the selection by pressing the ↵ key.
⇒ The current setting for the menu item or a submenu is displayed (in this case: level **0**).
5. To change the setting, press the ▲ or the ▼ key until the desired setting is displayed (in this case: level **2**).
6. Confirm the selection by pressing the ↵ key.
⇒ The menu closes.
7. To make more settings, call up the menu again and proceed as described.

NOTE

If no key is pressed for approx. 24 seconds, the menu will close.

5 STARTING UP DEVICE


5.1 Scope of delivery

Item	Component	Pcs.
a	Column with display housing and display cable	1
b	Weighing platform	1
c	Hex head socket screw, countersunk head M8 x 16 mm	2
d	Serrated lock washer, Ø 8.4 mm	2
e	Power supply unit with adapters (depending on model: Power supply unit with Euro connector)	1
f	Battery block	1
g	Ramp	2
–	Hex socket wrench, not shown	1
–	Screwdriver, not shown	1
–	Instructions for use, not shown	1

5.2 Assembling device

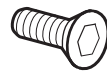
Fitting the column You need the following connecting elements for this assembly step:


2x hex head socket screw

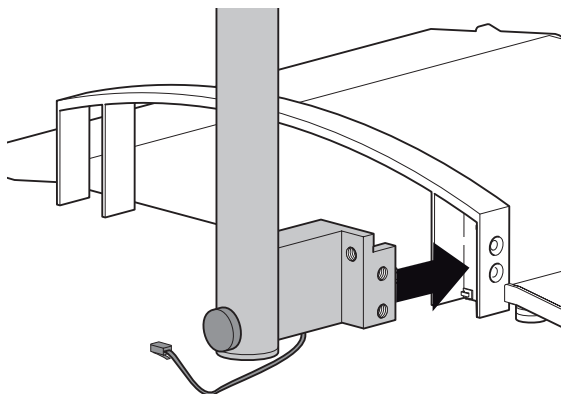2x serrated lock wa

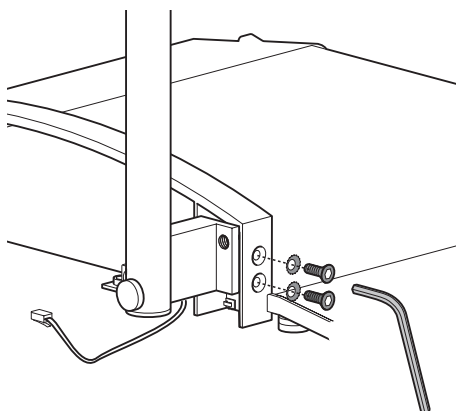1. Route the display cable so that it cannot become pinched when fitting the column.
2. Attach the column to the rail as shown in the figure above.

3. Put a serrated lock washer on each of the screws.
4. Put the screws in the bores of the rail and the column.
5. Tighten the screws (10 Nm).

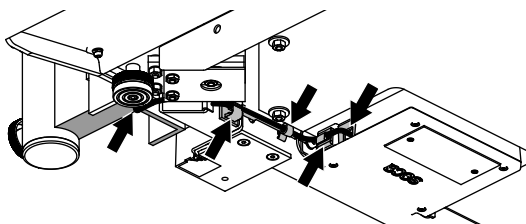
Connecting the display cable

NOTICE!

Malfunction as a result of errors in installation

If cables are strained during installation, e.g. with sharp bends or kinked connectors, this may result in faulty displays and failure of the display.

- Route all cables to prevent sharp bends and kinked connectors.
- Provide strain relief by routing all cables in the relevant holders.

1. Lift the scale so that the underneath of the weighing platform is accessible.

NOTE

You can support the scale on the rail for this installation work.

2. Guide the display cable under the weighing platform to the electronics box as shown in the figure.
3. Connect the display cable to the electronics box.
4. Press the factory-fitted cable clip attached to the display cable into the hole in the frame of the scale until you feel it engage.

5. Press the display cable into the cable clips on the frame of the scale until you feel them engage.
6. Set the scale upright.

Fitting the ramp

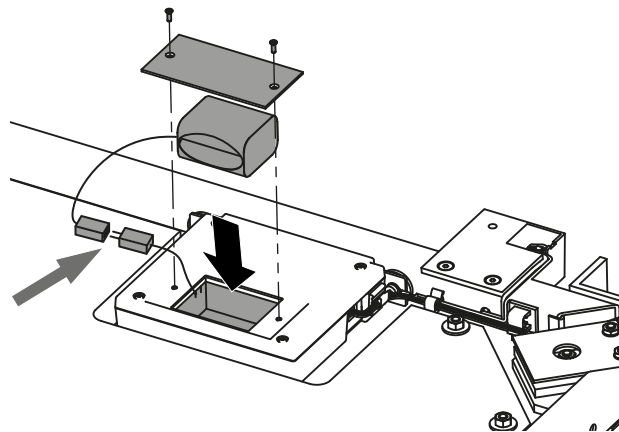
- Fit the supplied ramp as described in the relevant assembly instructions.

Final steps

1. Ensure that all components are correctly located.
2. Ensure that all cables are correctly routed.
3. Ensure that all screws are fully tightened.

5.3 Establishing power supply

Connecting the battery block

1. Loosen the battery compartment screws.
2. Remove the lid of the battery compartment.
3. Remove the battery block from the battery compartment.
4. Remove the connector cable from the battery compartment.
5. Connect the connector cable to the battery block.
6. Insert the battery block into the battery compartment.
7. Screw the battery compartment lid onto the battery compartment.

Connecting the power supply unit

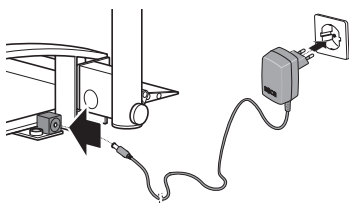
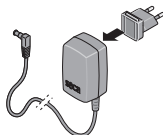
WARNING!

Personal injury or damage to the device as a result of incorrect power supply units

Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- Use only original seca power supply units with a controlled 12 V output voltage.

1. If required, insert the power supply connector necessary for your power supply into the power supply unit.


2. Insert the device connector of the power supply unit into the connecting socket of the scale.
3. Plug the power supply unit into a power supply socket.
4. Allow the scale to charge on the network for at least 24 hours when it is first charged so that the battery block is completely charged.

5.4 Setting up the data connection (optional)

If you want to set up a data connection via **seca connect 103** and **seca 452**, contact your hospital technician, your IT administrator or seca Service.

5.5 Making the scale operational

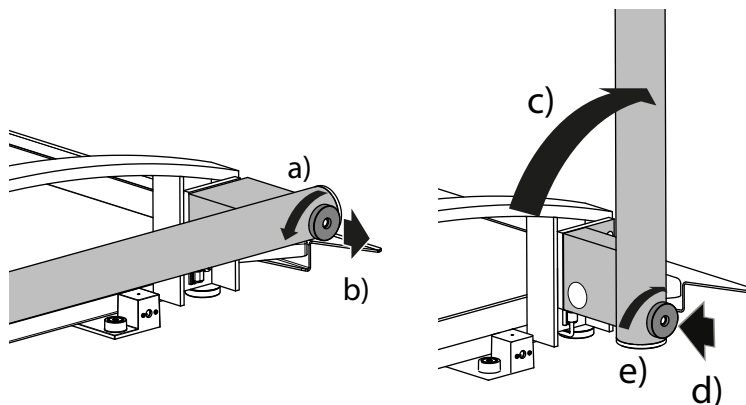Setting up the column

1. Place the scale on a solid, level surface.
2. Set up the column:
 - a) Undo the locking screw
 - b) Keep the locking screw pulled out
 - c) Set up the column
 - d) Engage the locking screw
 - e) Tighten the locking screw

Aligning the scale

CAUTION!

Faulty measurement as a result of devices being set up incorrectly

Soft floors (wooden boards, for example) give under the patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- Select a setup location with a level, stable floor in order to achieve precise measuring results.
- Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- Align the device before using it and every time it changes location.

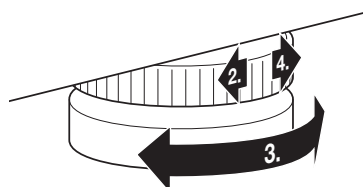
CAUTION!

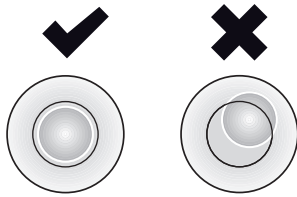
Faulty measurement as a result of devices being set up incorrectly

If the weighing platform or attachments (e.g. ramps, handrails, rail parts) come into contact with walls or objects, the measuring result will be falsified.

- Locate the device at a sufficient distance from walls and objects.

1. Place the scale on a firm, level surface.
2. Undo the knurled wheels.


3. Align the device by turning the foot screws.
The air bubble in the spirit level must be located in the exact center of the circle.
4. Tighten the knurled wheels in the direction of the arrow.
⇒ The foot screws are secured against being adjusted.

5.6 Transporting the scale

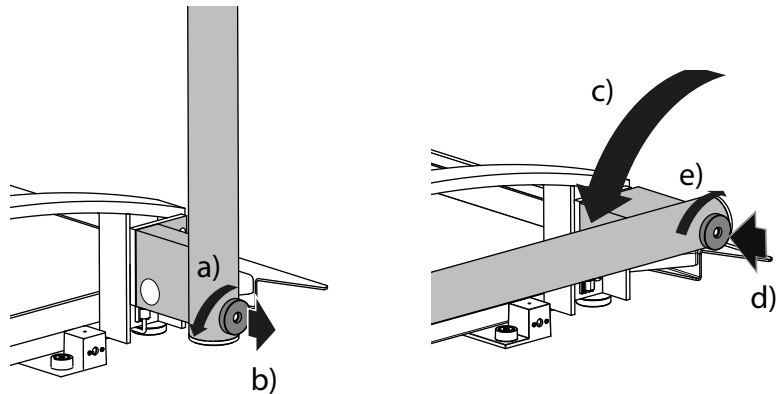Lowering the column


1. Fold the column down:
 - a) Undo the locking screw
 - b) Keep the locking screw pulled out
 - c) Fold down the column
 - d) Engage the locking screw
 - e) Tighten the locking screw

Moving the scale

WARNING! Injury from device falling over

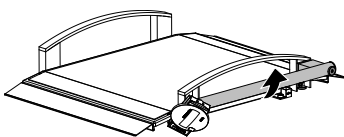- Always transport the scale with two persons.
- Transport the scale carefully to prevent it from tipping over or colliding with other people.
- Keep the scale in a safe place and prevent it from tipping over. Do not leave the scale unsecured in an upright position.

1. Disconnect the power supply unit from the power supply socket.
2. Disconnect the device connector of the power supply unit from the connection socket of the scale.
3. Tilt the device until it can be moved freely on the casters.
4. Move the device in this position to the desired set up or storage location.

6 OPERATION

6.1 Safety information

WARNING!

Injury from falling

- ▶ Keep an eye on ramps, edges and the guardrail when pushing the handling aid onto the weighing platform.
- ▶ Make sure that the brakes on patient handling aids are engaged during the weighing procedure.

WARNING!

Injury from falling

Patients in motorized wheelchairs might drive onto the weighing platform too quickly or at the wrong angle and fall.

- ▶ Help patients in motorized wheelchairs onto the weighing platform. These patients should not drive onto the weighing platform unassisted.

WARNING!

Injury from falling/damage to device

- ▶ If the patient handling aid has a quick-locking mechanism or similar parts close to the floor, ensure that such parts do not get caught on the scale or scratch the weighing platform.

WARNING!

Faulty measurement (patient handling aid)

- ▶ Make sure that all wheels of the patient handling aid are standing on the surface sensitive to weighing. Make sure that the patient handling aid does not touch the wall or any other objects.
- ▶ Ensure that the wheels of patient handling aids are not positioned at the transitions between the weighing platform and ramps (additionally with the bed scale: between the two weighing platforms), so producing a force shunt (incorrect measured values).

CAUTION!

Faulty measurement

- ▶ Help patients in (motorized) wheelchairs onto the scale in order to position the wheelchair correctly. Patients should not perform measurement themselves.

CAUTION!

Personal injury/faulty measurement

- ▶ Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

6.2 Weighing

Switching on the scale

- ▶ Press the  key.
 - ⇒ All the elements of the display are shown briefly, and then **SECA** appears in the display.
 - ⇒ The scale is operational when **0.0** (non-verified model: **0.00**) appears in the display.

Weighing a patient

CAUTION!

Injury from falling

Persons with limited mobility may fall when stepping on or off the weighing platform.

- ▶ Support people with limited mobility when they step on and off the scale.
- ▶ Move people who cannot stand without assistance onto the weighing platform using a suitable patient handling aid.

✓ The weighing platform has no load.

1. Start the weighing procedure:
 - ▶ Move the patient onto the weighing platform with a suitable patient handling aid
 - ▶ Ask the patient to step onto the weighing platform
2. Ask the patient not to move.
3. Read off the measuring result.

NOTE

To weigh patients with limited mobility, refer to section → [Taring off additional weight \(TARE\)](#), page 52. To weigh patients who are not mobile, refer to section → [Saving additional weight permanently \(Pt\)](#), page 54.

Taring off additional weight (TARE)

Use the **TARE** function to prevent an additional weight affecting the patient's weight value.

This function is suitable for patients with limited mobility who cannot stand for the entire weighing procedure. You can start by weighing a wheelchair, for example, and taring off its unladen weight. The patient can then be weighed in a sitting position.

NOTICE!

Faulty measurement as a result of force shunt

If an additional weight is touching the surface on which the scale is standing, the weight will not be measured correctly.

- ▶ Make sure that additional weights are only placed on the scale's weighing platform.

1. Switch on the scale.
2. Place the wheelchair on the scale.
3. Hold down the ▲ **hold/tare** arrow key until the **NET** message appears in the display.
4. Wait until the display stops flashing and **0.0** appears instead.
⇒ The weight of the wheelchair is tared off.
5. Guide the patient onto the weighing platform and support them to sit in the wheelchair.
6. Ask the patient not to move.
⇒ The patient's weight is displayed.
7. Read off the measuring result.
8. To deactivate the **TARE** function, press the **hold/tare** arrow key until the **NET▲** message is no longer displayed or turn off the scale.

NOTE

The maximum weight which can be displayed is reduced by the weight of the objects which have been tared.

Keep the measuring result on the display (HOLD)

When the **Hold** function is activated, the weight is still displayed after the load has been removed from the scale. This enables you to attend to the patient before recording the weight value.

✓ The scale has no load.

1. Switch on the scale.
2. Weigh the patient as described in the section entitled → [Weighing a patient](#), page 52.

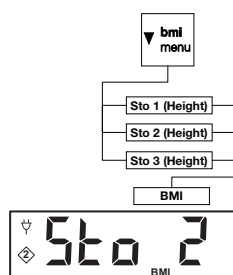
3. Press the **▲ hold/tare** arrow key briefly.
4. Wait until the weight value has stopped flashing.
 - ⇒ The **△** symbol (non-verifiable function) and the **HOLD** message are displayed.
 - ⇒ The patient can leave the weighing platform and the weight value will continue to be displayed.
5. To deactivate the function, press the **▲ hold/tare** arrow key briefly.
 - ⇒ The **△** symbol and the **HOLD** message are no longer displayed.
 - ⇒ The weight value is no longer displayed.

NOTE

If the autohold function is activated, the weight value is automatically displayed permanently until the scale switches itself off or is switched off, see → [Activating Autohold function \(AHOLD\)](#), page 55.

Determining Body Mass Index

Body Mass Index correlates height and weight. To do this, the patient's height must be entered manually. The device calculates the BMI from the entered value and the measured weight.

You can save heights for a maximum of three patients so that the values are quickly available for repeat measurements.

✓ The scale has no load.

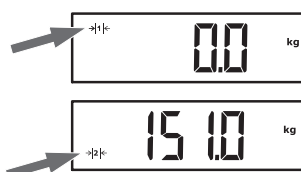
1. Switch on the scale.
2. Press the **▼ bmi/menu** arrow key briefly.
 - ⇒ The message **BMI** appears.
 - ⇒ The last memory used is displayed (in this case, memory 2).
3. You have the following options for continuing:
 - Adopt memory
 - Select different memory: Press the **▲** or the **▼** key
4. Confirm your setting by pressing the **↵** key.
 - ⇒ Arrows flash in the display.
 - ⇒ The last height saved to the selected memory is displayed.
5. You have the following options for continuing:
 - Adopt value
 - Change value: Press the **▲** or the **▼** key
6. Confirm your setting by pressing the **↵** key.
 - ⇒ The height entered is saved and can be called up again for calculating a new BMI.

NOTE

Make a note of the memory and assigned height so that you can call up the value again for calculating a new BMI.

7. Weigh the patient as described in the section entitled "Weighing a patient".
 - ⇒ The patient's BMI is automatically calculated and displayed.
8. To deactivate the BMI function, briefly press the **↵** key.

Switching the weighing range (verified models)

The scale has two weighing ranges. In weighing range 1 (→|1|←), graduations in the weight display are finer. In weighing range 2 (→|2|←), you can use the maximum capacity of the scale.

After the scale is switched on, weighing range 1 is active.

If a particular weight value is exceeded, the scale automatically switches to weighing range 2.

- To switch back to weighing range 1, completely remove the weight from the scale.
 - ⇒ Weighing range 1 is active again.

Switching off the scale

- Press the  key.

NOTE

For battery pack/battery mode only: The device switches off automatically after a short time if there is no load on the scale.

6.3 Other functions (menu)

Deleting saved values automatically (ACLR)

Out-of-date measuring results lead to incorrect calculation of BMI. You can set the device so that the measuring results are deleted automatically after 5 minutes.

NOTE

This function is activated at the factory on some models. The function can be deactivated.

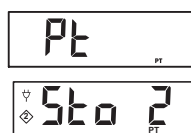
1. From the menu, select the **ACLR** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
⇒ The menu closes.

Saving additional weight permanently (Pt)

The **Pre-Tare** function is suitable for patients who are not mobile and need to be weighed in a wheelchair, for example. You can save the unladen weight of the wheelchair independently of a weighing procedure. During the weighing procedure, you can call up the unladen weight of the wheelchair and have it automatically deducted from the measuring result.

The device has three memories for weight values. You can save various weight values and call them up individually depending on the initial situation, so that they are automatically deducted from the measuring result.

1. From the menu, select the **Pt** menu item.
2. Confirm your selection.
⇒ The last memory used is displayed.
3. You have the following options for continuing:
 - Adopt memory
 - Select different memory: Press the ▲ or the ▼ key
4. Confirm your selection.
⇒ Arrows flash in the display.
⇒ The additional weight saved to the selected memory is displayed.
5. You have the following options for continuing:
 - Adopt value
 - Change value: Press the ▲ or the ▼ key

NOTE

When you enter the value **0**, the function is switched off. The message **Pt** is no longer shown in the display.

6. Confirm your selection.
7. Move the patient onto the scale.
⇒ The patient's weight is displayed.
⇒ The saved additional weight has been deducted automatically.
8. To deactivate the function, select the **Pt** menu item from the menu again.

9. Confirm your selection.
 - ⇒ The function is deactivated.
 - ⇒ The menu closes.

NOTE

When you switch the scale off, the function is switched off. The message **Pt** is no longer displayed when you switch the scale on again.

Activating Autohold function (AHOLD)

When you activate the **Autohold** function, the measuring result is displayed for every weighing procedure after the weight has been removed from the scale. It is then no longer necessary to activate the **Hold** function manually.

NOTE

This function is activated at the factory on some models. The function can be deactivated.

AHOLD

1. From the menu, select the **AHOLD** menu item.
2. Confirm your selection.
 - ⇒ The current setting is displayed.
3. Select the desired setting:
 - **On**
 - **OFF**
4. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Activating/deactivating beeps (bEEP)

bEEP

PrESS

You can activate beeps for a number of device functions:

1. From the menu, select the **bEEP** menu item.
2. Confirm your selection.
3. Select a menu item:
 - **PrESS**: beep when key is pressed
 - **HOLD**: beep when weighing value is stable
4. Confirm your selection.
 - ⇒ The current setting is displayed.
5. Select the desired setting:
 - **On**
 - **OFF**
6. Confirm your selection.
 - ⇒ The setting is adopted.
 - ⇒ The menu closes.

Setting filtering (FIL)

Setting the filter (**FIL** = filter) allows you to reduce interference when determining weight. The selected setting affects the sensitivity with which the weight display reacts to patient movements and the time until the **Hold** function displays a weight value permanently (**Hold** value).

Setting (FIL)	Sensitivity	Determination of Hold value
0	Sensitive	Slow
1	Medium	Medium
2	Insensitive	Fast

NOTE

With the **0** setting and patients who are not very steady on their feet, it is possible that no weight value is displayed permanently despite the **Hold** function being activated.

The greatest deviation between weight value displayed and actual value occurs at setting **2**.

F IL

1. From the menu, select the **FIL** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select a filtering level.
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Non-verified models: Switching the weight unit (UnIt)

You can select the unit in which weight is to be displayed.

CAUTION! **Patient hazard**

To prevent misinterpretations, measuring results for medical purposes must only be displayed and used in SI units (kilograms/grams, meters/centimeters). Some devices have the option of displaying measuring results in different units. This is purely an additional function.

- Only use the measuring results in SI units.
- You as the user take sole responsibility for the use of measuring results in non-SI units.

Un It

gr

LbS

1. From the menu, select the **UnIt** menu item.
2. Confirm your selection.
⇒ The current setting is displayed.
3. Select the unit in which weight is to be displayed:
 - Kilograms/grams (**gr**)
 - Pounds (**lbs**)
 - Stones (**sts**)
4. Confirm your selection.
⇒ The setting is adopted.
⇒ The menu closes.

Restoring factory settings (rESEt)

You can restore the factory settings for the functions below:

Function	Factory setting
Autohold (AHOLd)	model-depen- dent
Acoustic signal (PrESS)	OFF
Acoustic signal (HOLd)	On
Filtering (FIL)	0
Autoclear (ACLr)	model-depen- dent
Pre-tare (Pt)	0 kg
Height for Body Mass Index (bMi)	170 cm
Unit, weight (UnIt)	kg (gr)

rESEt

1. From the menu, select the **rESEt** menu item.
2. Confirm your selection.
⇒ The menu closes.
3. Switch off the scale.
⇒ Factory settings are restored and are available when the scale is switched on.

7 HYGIENE TREATMENT

WARNING! **Electric shock**

The device is not de-energized when the ON/OFF button is pressed and the display goes out. Use of fluids on the device may cause an electric shock.

- Before each hygiene treatment, ensure that the device is switched off.
- Disconnect the power supply connector before each hygiene treatment.
- Take the battery block out of the device before every hygiene treatment.
- Ensure that no fluids penetrate the device.

WARNING! **Risk of infection**

- Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and impair its operability.

- Do not use aggressive or abrasive cleaning agents.
- Do not use organic solvents (e.g. white spirit or petroleum spirit).

7.1 Cleaning

- If required, moisten a soft cloth with a mild soap solution and wipe the device over with it.

7.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use of the disinfectant.
3. Disinfect the device:
 - Moisten a soft cloth with disinfectant and wipe down the device with it.
 - Comply with the intervals, see table:

Interval	Component
Before and after each measurement	Weighing platform
As required	Keyboard, display, display column

7.3 Sterilizing

The device must not be sterilized.

8 FUNCTION CHECK

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled “Overview”
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the “Troubleshooting” section in this document.

CAUTION!

Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the “Troubleshooting” section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled “Servicing” in this document.

9 TROUBLESHOOTING

NOTICE!

Malfunction

If the device is equipped with the optional **seca 452** external interface module and is connected to the **seca connect 103** software, error messages and error symbols which are not described here may occur.

- Follow the system instructions for use for **seca 103/452**.

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service.

Fault	Cause	Remedy
No weight is displayed with a load on the scale.	The device has no power supply.	• Switch on device. • Connect the power supply unit.
 is displayed.	Battery pack capacity is decreasing	Recharge the battery pack soon
bAtt is displayed.	Battery pack is discharged	Charge battery pack
0.0 (verified models)/ 0.00 (non-verified models) is not displayed before weighing takes place.	A load was already on the device before it was switched on.	• Remove the load from the device. • Switch off the device, then switch it back on again.
----- is displayed before weighing takes place.	A load was already on the device before it was switched on.	• Remove the load from the device. • Switch off the device, then switch it back on again.
The weight display flashes.	Consistent weight value not yet detected.	• One of the following functions is active: Hold, Autohold, Tare. Wait until the device has detected a consistent weight value. The display stops flashing. • Take the load off the device, wait until 0.0/0.00 is displayed and measure again.
Implausible measuring results.	Patient moved too much during the measurement.	Take the load off the device, wait until 0.0/0.00 is displayed and repeat the measurement procedure.
One display segment is displayed continuously or not at all.	The display is defective.	Inform seca Service.

Fault	Cause	Remedy
StOP is displayed.	Maximum capacity has been exceeded.	Remove the load from the device.
tEMP is displayed.	The ambient temperature of the scale is too high or too low.	- Operate the device only in the intended ambient conditions (General technical data). - After transporting or storing the device, wait approx. 15 minutes before switching it on.
The device no longer reacts when keys are pressed.	Device is in an undefined state following implausible input.	- Take power supply unit out of the power supply socket. - Wait approx. 1 minute. - Plug the power supply unit into the power supply socket; the scale and the multifunctional display will switch on automatically.
	The keypad is defective.	Inform seca Service.

10 SERVICING

10.1 Verified scales

CAUTION!

Faulty measurements as a result of verification being performed poorly or not at all

- ▶ Have verification performed only by authorized persons.
- ▶ Always have verification performed if one or more quality seals are damaged.

seca recommends having your device serviced prior to verification.

CAUTION!

Faulty measurements as a result of poor servicing

- ▶ Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- ▶ You can find a service partner in your vicinity at www.seca.com or by emailing service@seca.com.

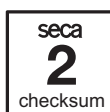
Have authorized persons perform verification in line with national legal regulations.

Verification is necessary whenever one or more quality seals are damaged or the contents of the verification counter no longer match the number on the applicable verification sticker. If quality seals are damaged, contact seca Service directly.

Verifications may only be performed by authorized agencies. To guarantee this, the scale is equipped with a verification counter which records each change in verification-related data.

If you want to check whether the scale has been properly verified, proceed as follows:

1. Ensure that the device is switched off.
2. Press and hold any key and start the scale.
⇒ The current content of the verification counter flashes in the display for a few seconds.
3. Compare the content of the verification counter output with the number given on the verification counter seal.

Both numbers have to match for the verification to be valid. If the verification sticker and the verification counter do not match, the scale must be verified. Please contact your service partner or seca Service. Once the scale has been verified, a new, updated verification counter sticker is used to identify the verification counter reading. The person authorized to perform verification secures this verification with an additional seal. The verification counter sticker can be ordered from seca Service.

10.2 Non-verified scales

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find your local service partner at www.seca.com.

11 TECHNICAL DATA

11.1 General technical data

Parameter	Value
Dimensions, total: • Depth • Width (incl. ramps) • Height (operation/transport)	1100 mm 1176 mm 912 mm/202 mm
Dimensions, weighing platform: • Depth • Width • Height	800 mm 965 mm 57 mm
Weight (incl. ramps)	approx. 32 kg
Height of digits, display	25 mm
Ambient conditions, operation • Temperature • Air pressure • Humidity	+10 °C to +40 °C / +50 °F to 104 °F 700 – 1060 hPa 30 % – 80 % no condensation
Ambient conditions, storage/transport • Temperature • Air pressure • Humidity	-10 °C to +65 °C / +14 °F to 149 °F 700 – 1060 hPa 0 % – 95 % no condensation
Medical device in accordance with regulation (EU) 2017/745	Class I with measuring function
IEC 60601-1: Medical electrical device, Type B	
Type of protection in accordance with IEC 60529	IP20
Duty cycle	Continuous duty
Service life, from commissioning ^a	at least 10 years
Application parts according to IEC 60601-1	Weighing platform, display column, display housing, film keypad, display

17-10-07-678-100_2025-06S

Parameter	Value
Battery block - Supply voltage - Maximum running time	7.2 V approx. 2400 weighing operations
Power supply unit - Supply voltage - Current consumption - Insulated device, protection class II (IEC 60601-1) - Mains voltage - Power supply frequency	12 V approx. 25 mA ☐ 100 V – 240 V 50 Hz – 60 Hz

^a Minimum service life when used as intended and serviced in a timely manner. seca products usually have a significantly longer service life.

11.2 Weight measurement

Verified model	
Verification in line with directive 2014/31/EU	Class III
Maximum capacity - Weighing range 1 - Weighing range 2	200 kg 300 kg
Minimum capacity - Weighing range 1 - Weighing range 2	2 kg 4 kg
Graduation - Weighing range 1 - Weighing range 2	100 g 200 g
Tare range	300 kg (subtractive)
Accuracy on initial verification - Weighing range 1: < 50 kg - Weighing range 1: 50 kg to 200 kg - Weighing range 2: < 100 kg - Weighing range 2: 100 kg to 300 kg	± 50 g ± 100 g ± 100 g ± 200 g

Non-verified model	
Maximum capacity	360 kg/800 lb/57 st
Minimum capacity	1 kg
Graduation	50 g/0.1 lb
Tare range	360 kg/800 lb/57 st (subtractive)
Accuracy < 67 kg 67 kg to 360 kg < 147.7 lb 147.7 lb to 800 lb < 10.55 st 10.55 st to 57 st At ambient temperature of 20° C and humidity of 45 %	± 100 g ± 0.15 % ± 0.2 lb ± 0.15 % ± 0.2 lb ± 0.15 %

12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Power supply unit with Euro connector: 230 V~ /50 Hz/ 12 V= / 130 mA	68 32 10 252
Switch-mode power supply unit with adapters: • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.98 A	68 32 10 265 009
Switch-mode power supply unit with adapters: • Input: 100 V – 240 V~, 50 Hz – 60 Hz • Output: 12 V= / 0.6 A	68 32 10 270 009
NiMH battery block: 7.2 V= / , 1300 mAh	68 22 12 721 009
Ramp seca 470	470 0000 009
Accessories for integration via WiFi/LAN • External Interface module seca 452 • Software seca connect 103 • Barcode scanner (medical device)	452 0000 009 Will be provided in the course of integration projects Cannot be ordered from seca; for recommendations, see system instructions for use for seca 103/452

13 DISPOSAL

13.1 Disposing of the device

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service or an authorized service partner.

13.2 Disposing of batteries and rechargeable batteries

Do not dispose of used batteries and rechargeable batteries in household waste, regardless of whether they contain harmful substances or not. As a consumer, you are legally obliged to dispose of batteries and rechargeable batteries via local authority collection points or trade collection points. Only dispose of batteries and rechargeable batteries once they are completely discharged.

14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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