seca Printer module

(D)	<i>Bedienungsanleitung</i>	3
(GB)	<i>Instruction manual</i>	8
(F)	<i>Mode d'emploi</i>	13
(I)	<i>Manuale di istruzioni</i>	18
(E)	<i>Manual de instrucciones</i>	23
(DK)	<i>Betjeningsvejledning</i>	28
(S)	<i>Bruksanvisning</i>	33
(N)	<i>Bruksanvisning</i>	38
(FIN)	<i>Käyttöohje</i>	43
(NL)	<i>Bedieningshandleiding</i>	48
(P)	<i>Instruções de utilização</i>	53
(GR)	<i>Οδηγίες χειρισμού</i>	58
(J)	取扱い説明書	63
(CN)	操作说明	68

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
Best.Nr.: 12-05-02-206

1. Connecting the scale

If your scale is equipped with a printer in the display unit, a more powerful mains power unit will be required as the power supply. In this case, use seca mains power unit 68-32-10-263 supplied.



- The mains power unit can be adapted to a wide variety of socket systems using the adapters. Change the adapter of the mains power unit if required: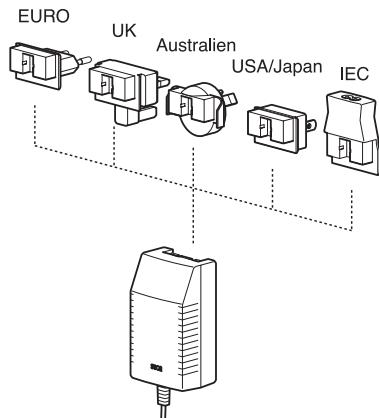

pull the adapter off the mains power unit from the side.

Then plug the correct adapter onto the mains power unit. Ensure that the adapter is properly engaged and securely located.

- Connect the mains power unit to the scale.
- Connect the mains power unit to the socket.

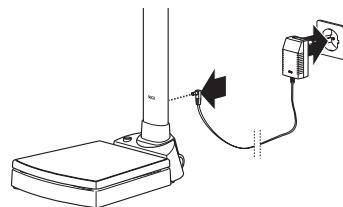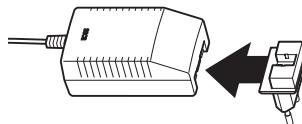2. Clock function

Displaying time

After the scale has been switched on, time is displayed for about 2 seconds.

If a printer is connected, the time and date of the weighing operation is printed out in addition to the weight itself.

Depending on the country to which it is supplied, the display can be in either 24-hour clock format (e.g. 19:45) or in 12-hour format (e.g. P 7:45).

If **TIME** appears in the display and 3 alarm signals are heard after the scale is switched on, this means that the scale has been without a power supply for a prolonged period, so the clock needs to be reset.

Setting the clock

- To reach setting mode, press the **FUNC/SET (BMI)** key for about 2 seconds.

Depending on the version of the scale, the year will appear immediately or additional menu items will appear.

If additional menu items appear, use the arrow keys to select the **TIME** menu item and then press the **FUNC/SET (BMI)** key to reach the settings menu. The year appears in the display.

- Use the arrow keys to set the correct year.
- Press the **FUNC/SET (BMI)** key to confirm the year information entered. This simultaneously switches you to the next value you can set. Change this value too using the arrow keys and confirm the entry using the **FUNC/SET (BMI)** key.

Set all the values for the clock correctly in the same way. The sequence of values is year, month, day, hour, minute.

On clocks with a 12-hour display, **A** appears on the left of the display for morning times and **P** for afternoon and evening times.

- After you have confirmed the minutes using the **FUNC/SET (BMI)** key, you exit the settings menu automatically and the clock continues to run at the time set.

If you also require seconds to be set accurately, then proceed as follows:

- set a minute which is still to come, but do not confirm it using the **FUNC/SET (BMI)** key. As soon as the set minute starts, press the **FUNC/SET (BMI)** key. At that point, the clock will start running at 00 seconds and the set minute.


3. Printer function

Insert paper


The printer uses thermal paper. You can order this either directly from seca (see “Spare parts and accessories” on page 12) or from a specialist supplier. The rolls of paper may not exceed the following dimensions: ø 28 mm, width 57 mm.

To insert the roll of paper, proceed as follows:


- Lift the transparent flap of the printer a little way until the paper dispenser flips up with a click.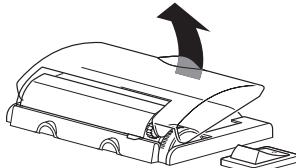

- Now open the paper dispenser fully by pulling it forwards from the rear.
- Remove the packaging from the thermal paper.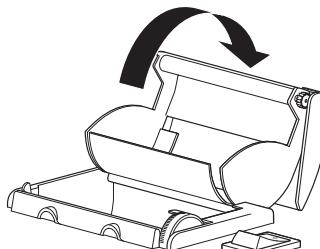


- Unroll a little of the paper.
- Insert the roll in the paper dispenser so that the printable outside of the paper is facing the tearing edge.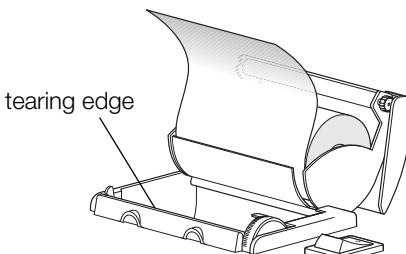

- Push the paper dispenser of the printer closed. Ensure that the flap engages with an audible click.

Warning!

The flap must be closed (engaged) correctly, otherwise the printing unit may be damaged.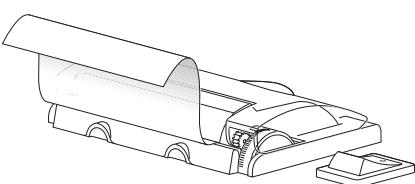

Weighing with printout

- Switch the scale on if necessary.
- Switch on the printer at the on/off switch.
- Get the patient to stand on the scale. The patient should then stand still. The measured weight is printed out.
- If you also want to measure height (models **763/764** only), perform a measurement (see "Measuring height" in the operating instructions for the scale).


Then press the **Data Valid** key. The height and calculated BMI value are then printed out.

- When the load is removed from the scale, date and time are also printed. Then tear off the printout towards the tearing edge.

Note:

If the printout is in 12-hour format, morning times are marked **AM** and afternoon times **PM**.

- If you do not require any further printout, switch the printer off again at the on/off switch.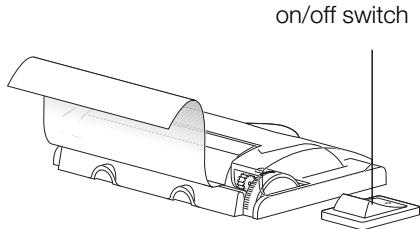

What do I do if ...

... the scale switches off when attempting to print?

- Too weak a mains power unit is connected. Connect seca mains power unit 68-32-10-263 supplied (see "Connecting the scale" on page 8).

... the printer does not print during weighing?


- Is the printer switched on?
- Is the flap of the paper dispenser completely closed?

... PRPEr appears in the display?

- The paper is finished. Insert a new roll.
- Is the flap of the paper dispenser closed?

... the printer prints during weighing, but there is nothing to be seen on the paper?

- The paper has been put in the wrong way round. Insert the roll the right way round.

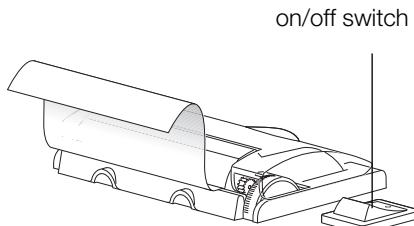... the printer prints out during weighing but only special characters appear on the printout?

- The printer was switched off during the previous printing operation. Switch off the scale and restart. Only switch off the printer after printing.

... the printout is incomplete (there is no date and the printout has not been fed out far enough)?

- The load must first be removed from the scale before printing can finish. None of the scale functions should be activated either.

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