

Instructions for use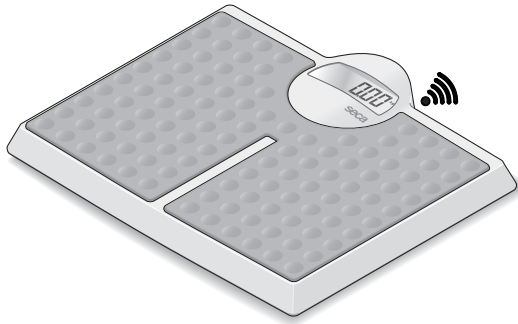


seca 813

seca robusta bt


1. SICHERHEITSHINWEISE • SAFETY INFORMATION • CONSIGNES DE SÉCURITÉ • INDICAZIONI PER LA SICUREZZA • INDICACIONES DE SEGURIDAD • INSTRUÇÕES DE SEGURANÇA • ZASADY BEZPIECZEŃSTWA • SIKKERHEDSHENVISNINGER • SÄKERHETSANVISNINGAR • SIKKERHETSINSTRUKSJONER • TURVALLISUUSOHJEET • VEILIGHEIDSI-NSTRUCTIES • ΥΠΟΔΕΙΞΕΙΣ ΑΣΦΑΛΕΙΑΣ • ПРАВИЛА БЕЗОПАСНОСТИ • 安全上のご注意 • GÜVENLİK UYARILARI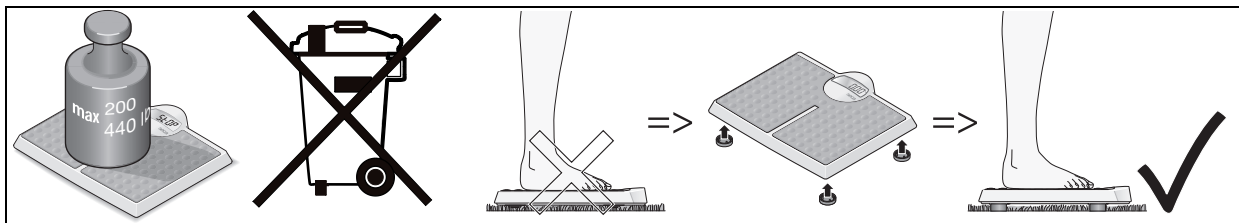


2. BEVOR ES RICHTIG LOSGEHT... • BEFORE YOU GET STARTED ... • AVANT DE COMMENCER... • PRIMA DI COMINCI-ARE VERAMENTE... • ANTES DE EMPEZAR... • ANTES DE INICIAR A UTILIZAÇÃO DEVE... • PRZYGOTOWANIA WAGI DO PRACY • INDEN MAN KOMMER RIGTIGT I GANG ... • INNAN DU KAN SÄTTA IGÅNG... • FØR DU SETTER I GANG... • ENNEN KUIN ALOITAT... • VOOR HET GEBRUIK... • ΠΡΙΝ ΑΠΌ ΤΗ ΘΈΣΗ ΣΕ ΛΕΙΤΟΥΡΓΙΑ... • ПЕРЕД ТЕМ, КАК ПОЛЬЗОВАТЬСЯ ВЕСАМИ... • ご使用になる前に... • TAM OLARAK BAŞLAMADAN ÖNCE...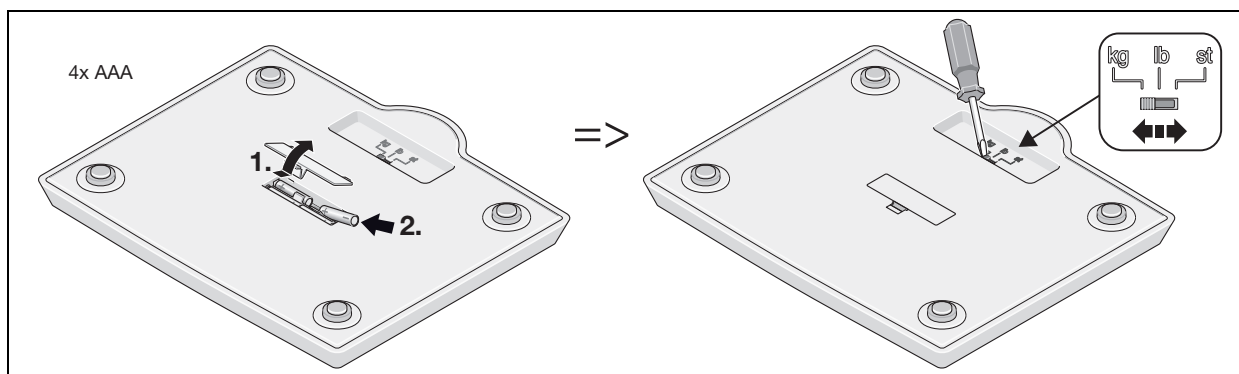


3. **WAAGE AKTIVIEREN NACH EINSETZEN DER BATTERIE • ACTIVATING THE SCALE AFTER INSERTING THE BATTERY • ACTIVER LA BALANCE APRÈS AVOIR INSÉRÉ LA PILE • ATTIVARE LA BILANCIA DOPO AVER INSERITO LA BATTERIA • ACTIVAR LA BÁSCULA TRAS INSERTAR LA PILA • ATIVAR A BALANÇA APÓS A COLOCAÇÃO DAS PILHAS • AKTYWACJA WAGI I INSTALACJA BATERII • AKTIVERING AF VÆGTEN OG ISÆTNING AF BATTERIET • AKTIVERING AV VÅGEN EFTER ATT BATTERIET HAR SATTES IN • AKTIVERE VEKT ETTER INNSETTING AV BATTERIET • VAA'AN AKTIVOINTI PARISTON ASETTAMISEN JÄLKEEN • WEEGSCHAAL ACTIVEREN NADAT DE BATTERIJ IS GEPLAATST • ΕΝΕΡΓΟΠΟΙΗΣΗ ΖΥΓΑΡΙΑΣ ΜΕΤΑ ΑΠ'Ο ΤΟΠΟΘΕΤΗΣΗ ΜΠΑΤΑΡΙΑΣ • АКТИВАЦИЯ ВЕСОВ ПОСЛЕ ТОГО, КАК ВСТАВЛЕНА БАТАРЕЯ • 乾電池を挿入した後、体重計をオンにします • PILLER TAKILDIKTAN SONRA TARTININ ÇALIŞTIRILMASI**

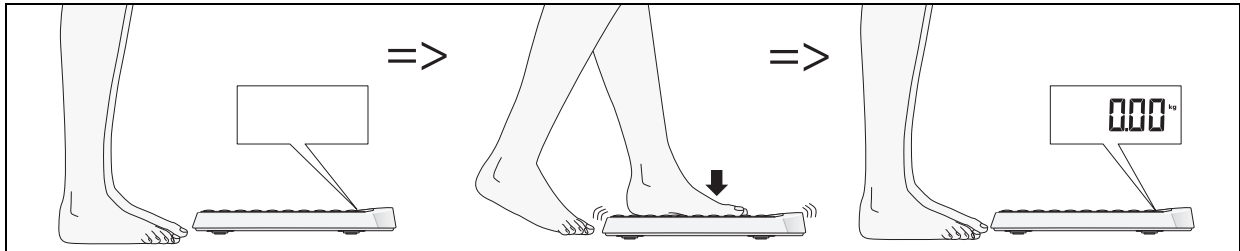
4. **WIEGEN (OHNE APP) • WEIGHING (WITHOUT THE APP) • PESER (SANS L'APPLICATION) • PESATURA (SENZA APP) • PESAR (SIN APLICACIÓN) • PESAR (SEM APP) • WAŻENIE (BEZ APLIKACJI) • VEJNING (UDEN APP) • VÄGNING (UTAN APP) • VEIING (UTEN APP) • PUNNITUS (ILMAN SOVELLUSTA) • WEGEN (ZONDER APP) • ΖΥΓΙΣΜΑ (ΧΩΡΙΣ ΕΦΑΡΜΟΓΗ) • ВЗВЕШИВАНИЕ (БЕЗ ПРИЛОЖЕНИЯ) • 体重測定 (アプリ不使用) • TARTMA (UYGULAMA OLMADAN)**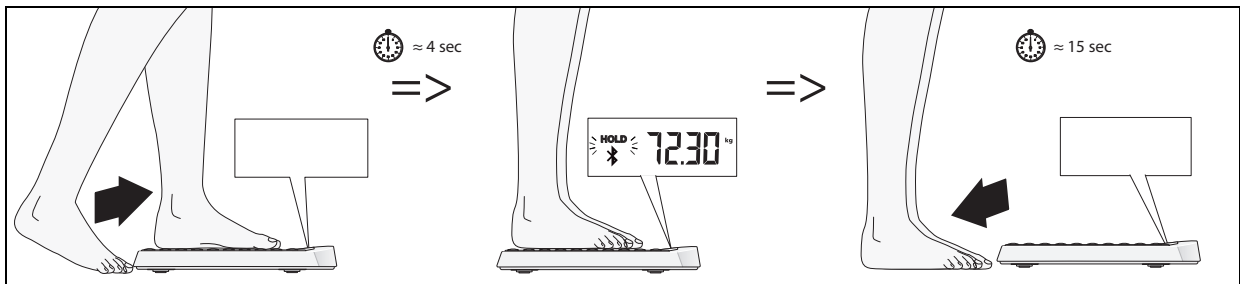

5. WIEGEN (MIT APP) • WEIGHING (WITH THE APP) • PESER (AVEC L'APPLICATION) • PESATURA (CON APP) • PESAR (CON APLICACIÓN) • PESAR (COM APP) • WAŻENIE (Z APLIKACJĄ) • VEJNING (MED APP) • VÄGNING (MED APP) • VEIING (MED APP) • PUNNITUS (SOVELLUKSEN KANSSA) • WEGEN (MET APP) • ΖΥΓΙΣΜΑ (ΜΕ ΕΦΑΡΜΟΓΗ) • ВЗВЕШИВАНИЕ (С ПРИЛОЖЕНИЕМ) • 体重測定 (アプリ使用) • TARTMA (UYGULAMA ILE)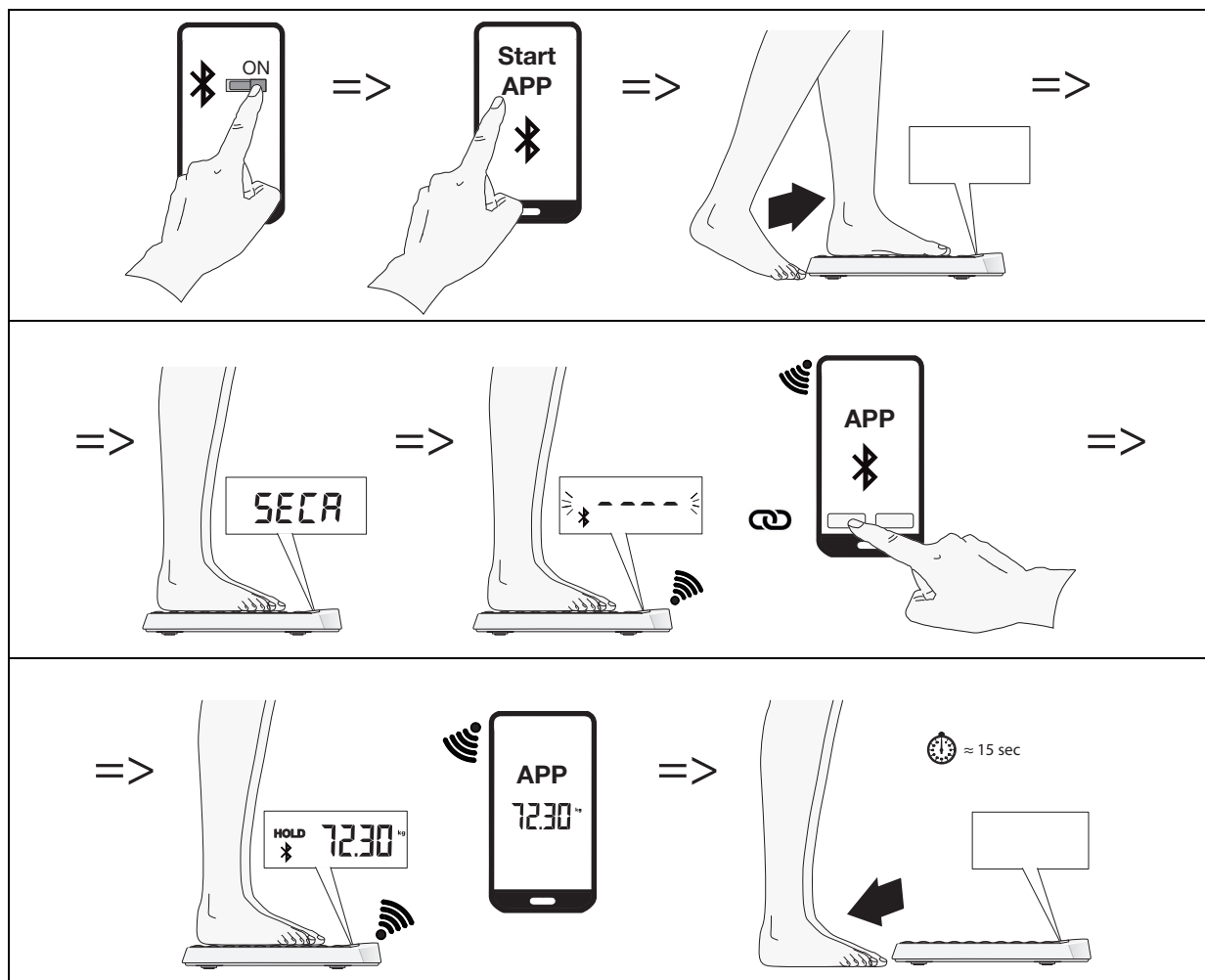

6. BATTERIEN AUSTAUSCHEN • REPLACING THE BATTERIES • REMPLACER LES PILES • SOSTITUIRE LE BATTERIE • CAMBIAR LAS PILAS • TROCAR AS PILHAS • WYMIANA BATERII • UDSKIFTNING AF BATTERIERNE • BYTTE AV BATTERIER • SKIFTE UT BATTERIER • PARISTOJEN VAIHTAMINEN • BATTERIJEN VERVANGEN • ANTIKATAΣΤΑΣΗ ΜΠΑΤΑΡΙΩΝ • ЗАМЕНА БАТАРЕЙ • 乾電池を交換してください • PILLERIN DEĞİŞTIRILMESİ

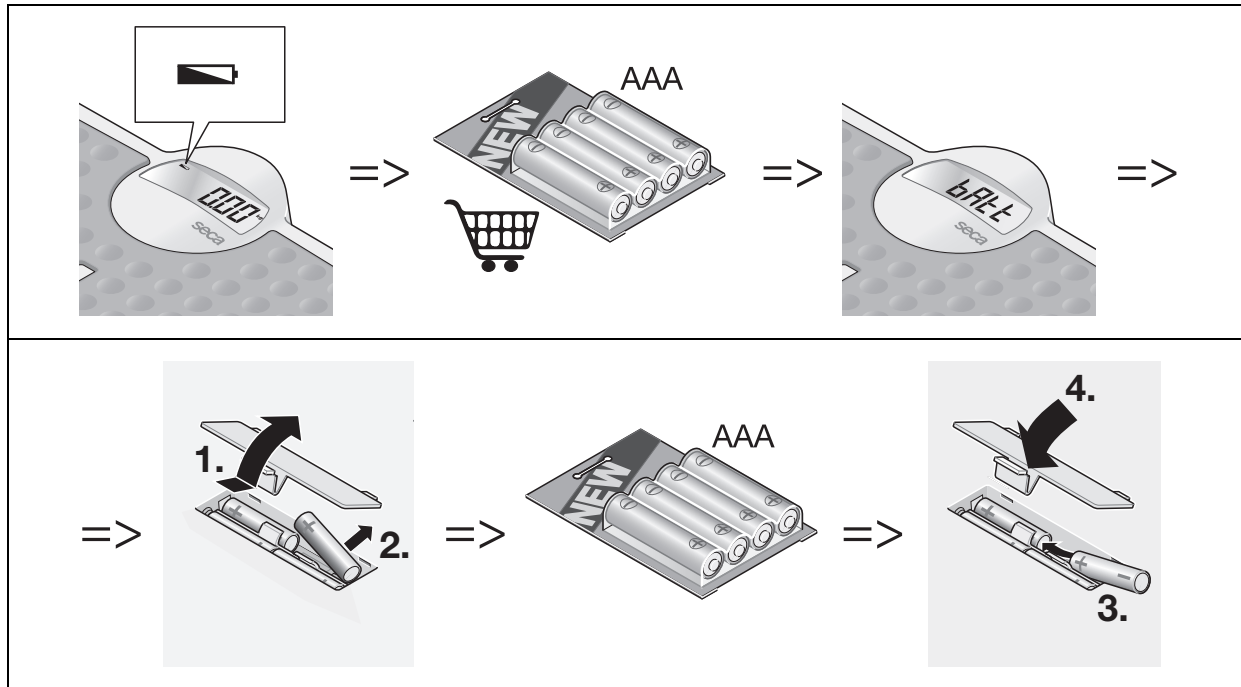DE

WARNUNG!

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EN

WARNING!

Personal injury as a result of improper handling

Batteries and rechargeable batteries contain harmful substances which may explode if not handled properly.

- ▶ Do not try to recharge batteries.
- ▶ Do not expose (rechargeable) batteries to heat.
- ▶ Do not burn (rechargeable) batteries.
- ▶ If acid is leaking out, avoid contact with the skin, eyes and mucous membranes. Rinse affected areas with plenty of clean water and seek medical help at once.

NOTICE!

Damage to device and malfunctions with improper handling

- ▶ Only use the type of (rechargeable) battery specified in this document.
- ▶ When replacing (rechargeable) batteries, always replace a complete set at a time.
- ▶ Do not short-circuit (rechargeable) batteries.
- ▶ If you do not use the device for a long period of time, remove the batteries (incl. rechargeable batteries). This prevents acid from leaking into the device.
- ▶ If acid leaked into the device, discontinue use. Have the device checked by an authorised seca Service partner and repaired if necessary.

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EN - Warranty

We offer a two-year warranty from the date of delivery for defects attributable to faulty material or poor workmanship. This excludes all moveable parts such as (rechargeable) batteries, cables, power supply units, etc. Defects which are covered by the warranty shall be rectified free of charge for customers on production of the sales receipt. No further claims can be accepted. The costs of shipment in both directions shall be borne by the customer where the device is not located at the customer's premises. In the event of any damage during shipment warranty claims can only be asserted where the complete original packaging was used for shipment and the scales were secured inside in the same manner as in the original packaging. You should therefore keep all packaging.

The warranty shall become null and void where the device is opened by persons not expressly authorised to do so by seca.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

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10. DECLARATION OF CONFORMITY FOR USA AND CANADA

FCC Warning Statement

Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

FCC Interference Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna. Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Canada ISED Regulatory Compliance Statement

This device contains licence-exempt transmitter(s) that comply with Innovation, Science and Economic Development Canada's licence-exempt RSS(s).

Operation is subject to the following two conditions:

- This device may not cause interference.
- This device must accept any interference, including interference that may cause undesired operation of the device.

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This equipment complies with IC RSS-102 radiation exposure limits set forth for an uncontrolled environment. This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

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CAN ICES-3 (B)/NMB-3(B)

**Supplier's Declaration of Conformity
47 CFR § 2.1077 Compliance Information**

Unique Identifier:
seca robusta 813 3221 139
seca robusta 813 3221 009

Responsible Party - U.S. Contact Information

seca north america
13604 Benson Avenue
Chino, CA 91710
USA
Phone: +1 800 542 7322; Email: info.us@seca.com

FCC Compliance Statement

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

The *Bluetooth*[®] word mark and logos are registered trademarks owned by the Bluetooth SIG, Inc. and any use of such marks by seca gmbh & co. kg is under license. Other trademarks and trade names are those of their respective owners.

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