
seca 874 u+

Instructions for Use

17-10-06-646-100d_2026-02S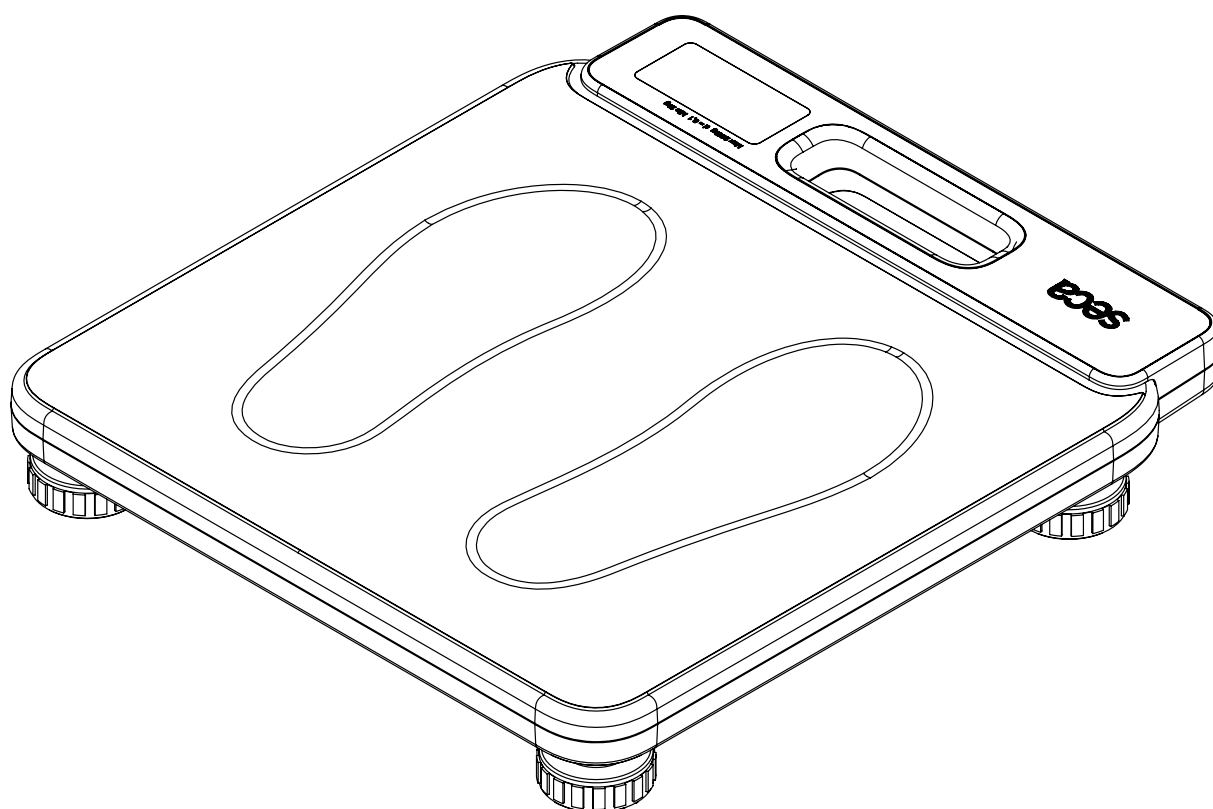

English – Instructions for Use.....	3
Français – Mode d’emploi.....	23
Español – Instrucciones de uso	44
.....العربية – تعليمات الاستخدام	65

CONTENTS

1 About this document	4	10 Servicing	20
1.1 Representation in text	4	11 Technical data.....	20
2 Description of device	4	11.1 General technical data	20
2.1 Intended use.....	4	11.2 Weight measurement	21
2.2 User qualification.....	4	12 Optional accessories and spare parts.....	21
2.3 Patient target group.....	4	12.1 Changing feet	21
2.4 Contraindications	4	13 Disposal	22
2.5 Clinical benefit	4	13.1 Disposing of the device.....	22
2.6 Functional description	5	13.2 Disposing of batteries and rechargeable batteries.....	22
3 Safety precautions.....	5	14 Warranty	22
3.1 Safety information in these instructions for use	5	15 Declaration of conformity	22
3.2 Basic safety information	5		
Handling device	5		
Preventing electric shock	6		
Preventing injuries and infections.....	6		
Preventing damage to device.....	7		
Handling measuring results	7		
Handling packaging material	8		
Handling batteries and rechargeable batteries.....	8		
4 Overview	9		
4.1 Controls	9		
4.2 Symbols in the display	10		
4.3 Labels	10		
5 Starting up device.....	12		
5.1 Scope of delivery.....	12		
5.2 Establishing power supply	12		
Inserting batteries.....	12		
Connecting the power pack (optional)	13		
Connecting the seca 441 Solar Panel (optional).....	13		
5.3 Setting up and aligning the device	13		
6 Operation.....	14		
6.1 Switching the device on and off.....	14		
Switching on the device	14		
Switching off the device	14		
Switching on the display (standby).....	14		
Rotating the display.....	14		
6.2 Weighing.....	15		
Weighing a patient.....	15		
Weighing babies/toddlers.....	15		
6.3 Performing serial measurements with adults and children.....	16		
6.4 Bluetooth (for model with Bluetooth only)	16		
Switching Bluetooth on/off	16		
Initially connecting to a Bluetooth- compatible device (pairing mode)	17		
7 Hygiene treatment	18		
7.1 Cleaning.....	18		
7.2 Disinfecting.....	18		
7.3 Sterilizing	18		
8 Function check	19		
9 Troubleshooting.....	19		

1 ABOUT THIS DOCUMENT

1.1 Representation in text

Symbol	Description
✓	Requirement for actions
►	Action
1. 2.	Actions with specified sequence
a) b)	Steps of an action with specified sequence
⇒	Result of an action
• •	First level of a list
– –	Second level of a list

2 DESCRIPTION OF DEVICE

2.1 Intended use

The electronic flat scale supports physicians in decision-making regarding diagnosis or therapy based on weight.

2.2 User qualification

The device may only be used by persons with sufficient expertise.

2.3 Patient target group

The device is intended for persons of any age who can stand independently and who do not exceed the maximum weight capacity of the device. Babies and toddlers can be weighed with the **2 in 1** function.

2.4 Contraindications

No contraindications are known.

2.5 Clinical benefit

The electronic flat scale supports physicians in decision-making regarding diagnosis or therapy based on the measured parameter (indirect clinical benefit).

2.6 Functional description

Weight on this electronic personal scale is detected by four load cells.

The **2 in 1** function can be used to determine the weight of babies and toddlers. The child is held in an adult's arms during weighing to do this.

Model with Bluetooth: The device can use Bluetooth to transmit the measured weight value and current battery status to a smartphone (or other mobile device).

3 SAFETY PRECAUTIONS

3.1 Safety information in these instructions for use

DANGER!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries will occur.

WARNING!

Used to identify an extremely hazardous situation. If you fail to take note of this information, serious irreversible or fatal injuries may result.

CAUTION!

Used to identify a hazardous situation. If you fail to take note of this information, minor to moderate injuries may result.

NOTICE!

Used to identify possible incorrect usage of the device. If you fail to take note of this information, you may damage the device, or the measuring results may be incorrect.

NOTE

Includes additional information about use of the device.

3.2 Basic safety information

Handling device

- ▶ Please take note of the information in these instructions for use.
- ▶ Keep the instructions for use in a safe place. The instructions for use are a component of the device and must be available at all times.
- ▶ In the interest of patient safety, you and your patients are obliged to report serious events that occur in connection with this product to the manufacturer and to the authority responsible in your country.

DANGER!

Risk of explosion

- ▶ Do not use the device in an environment in which one of the following gases has accumulated:
 - oxygen
 - flammable anesthetics
 - other flammable substances/air mixtures

**CAUTION!****Patient hazard, damage to device**

- ▶ Have servicing carried out regularly as described in the relevant section of this document.
- ▶ Technical modifications may not be made to the device. The device does not contain any user-serviceable parts. Only have servicing and repairs performed by an authorized seca service partner. You can find your local service partner at www.seca.com.
- ▶ Use only seca original accessories and spare parts, otherwise seca will not grant any warranty.

**CAUTION!****Patient hazard, malfunction**

- ▶ Keep medical electrical devices such as high-frequency surgical equipment at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ Keep high-frequency equipment such as cellphones at a minimum distance of approx. 1 meter to prevent faulty measurements.
- ▶ The actual transmission output of HF devices may require minimum distances of more than 1 meter. For details, go to www.seca.com.

Preventing electric shock**WARNING!****Electric shock**

- ▶ Set up devices which can be operated with a power supply unit so that the power supply socket is within easy reach and the power supply can be disconnected quickly.
- ▶ Ensure that your local electricity supply matches the details on the power supply unit.
- ▶ Never touch the power supply unit with wet hands.
- ▶ Do not use extension cables and multiple outlets.
- ▶ Ensure that cables are not crushed or damaged by sharp edges.
- ▶ Ensure that cables do not come into contact with hot objects.
- ▶ Do not operate the device at an altitude of more than 3000 m above sea level.

Preventing injuries and infections**WARNING!****Injury from falling**

- ▶ Ensure that the device is steady and level.
- ▶ Route connector cables (if present) so that neither users nor the patient can trip over them.
- ▶ Assist people with limited motor skills when they are getting up, e.g. from a wheelchair.
- ▶ Ensure that the patient does not step directly onto or off the edges of the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

**WARNING!****Danger of slipping**

- ▶ Ensure that the weighing platform is dry before the patient steps onto it.
- ▶ Ensure that the patient has dry feet before stepping onto the weighing platform.
- ▶ Ensure that the patient steps onto and off the weighing platform slowly and safely.

**WARNING!****Risk of infection**

- ▶ Before and after every measurement, wash your hands to reduce the risk of cross-contamination and nosocomial infections.
- ▶ Should the patient have contagious diseases, subject the device to a hygiene treatment as described in the relevant section of these instructions for use immediately afterwards.
- ▶ Ensure that the patient does not have any open wounds or infectious skin alterations which may come into contact with the device.
- ▶ Subject the device to a hygiene treatment at regular intervals.

Preventing damage to device

NOTICE!**Damage to device**

- ▶ Ensure that fluids never get inside the device. These can destroy the electronics.
- ▶ For devices with power supply operation: Switch off the device before you disconnect the power supply unit from the power supply socket.
- ▶ For devices with power supply operation: If the device is not to be used for an extended period, disconnect the power supply unit from the power supply socket. Only then is the device de-energized.
- ▶ For devices with battery or rechargeable battery operation: If you are not using the device for an extended period of time, remove batteries or rechargeable batteries. Only then is the device de-energized.
- ▶ Do not drop the device.
- ▶ Do not subject the device to shocks or vibrations.
- ▶ Perform a function check before each use as described in the corresponding section in this document. Do not operate the device if it is not working properly or is damaged.
- ▶ Do not place the device in direct sunlight and ensure that it is not placed in the direct proximity of a heat source. The excessive temperatures could damage the electronics.
- ▶ Avoid rapid temperature fluctuations. If the device is transported so that a temperature difference of over 20 °C occurs, the device must be left to stand for at least 2 hours before it is switched on, otherwise condensation may form; this may damage the electronics.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Store the device only in the intended storage conditions outlined in the section entitled "Technical data".
- ▶ Use only cleaning agents and disinfectants which match the details in the section entitled "Hygiene treatment".
- ▶ For scales: Ensure that maximum capacity is not exceeded.

Handling measuring results

NOTICE!**Inconsistent measuring results**

- ▶ Before you save and continue using measured values determined using this device (e.g. in seca software or in an EMR system), ensure that the measured values are plausible.
- ▶ If measured values have been sent to seca software or to an EMR system, ensure before continuing to use them that the measured values are plausible and assigned to the correct patient.

Handling packaging material

WARNING!

Risk of suffocation

Packaging materials made of plastic film (bags) present a risk of suffocation.

- ▶ Store packaging material out of the reach of children.
- ▶ If the original packaging material is no longer available, only use plastic bags with safety holes to reduce the risk of suffocation. Use recyclable materials if possible.

NOTE

Store the original packaging material for future use (e.g. returning for servicing).

Handling batteries and rechargeable batteries

WARNING!

Injury

Batteries and rechargeable batteries contain harmful substances which may explode and so escape if not handled properly.

- ▶ Do not try to recharge batteries.
- ▶ Do not expose batteries/rechargeable batteries to heat.
- ▶ Do not burn batteries/rechargeable batteries.
- ▶ If acid is leaking out, avoid contact with the skin, eyes, and mucous membranes. Rinse affected areas of the body with plenty of clean water and seek medical help at once.


NOTICE!

Damage to device and malfunction due to incorrect handling

- ▶ Use only the type of battery/rechargeable battery quoted in this document.
- ▶ Always replace all batteries/rechargeable batteries at the same time.
- ▶ Do not short-circuit batteries/rechargeable batteries.
- ▶ If you do not use the device for an extended period of time, remove the batteries/rechargeable batteries. This prevents acid from leaking into the device.
- ▶ If acid has penetrated the device, do not continue using it. Have the device checked by an authorized seca service partner and repaired if necessary.


4 OVERVIEW

4.1 Controls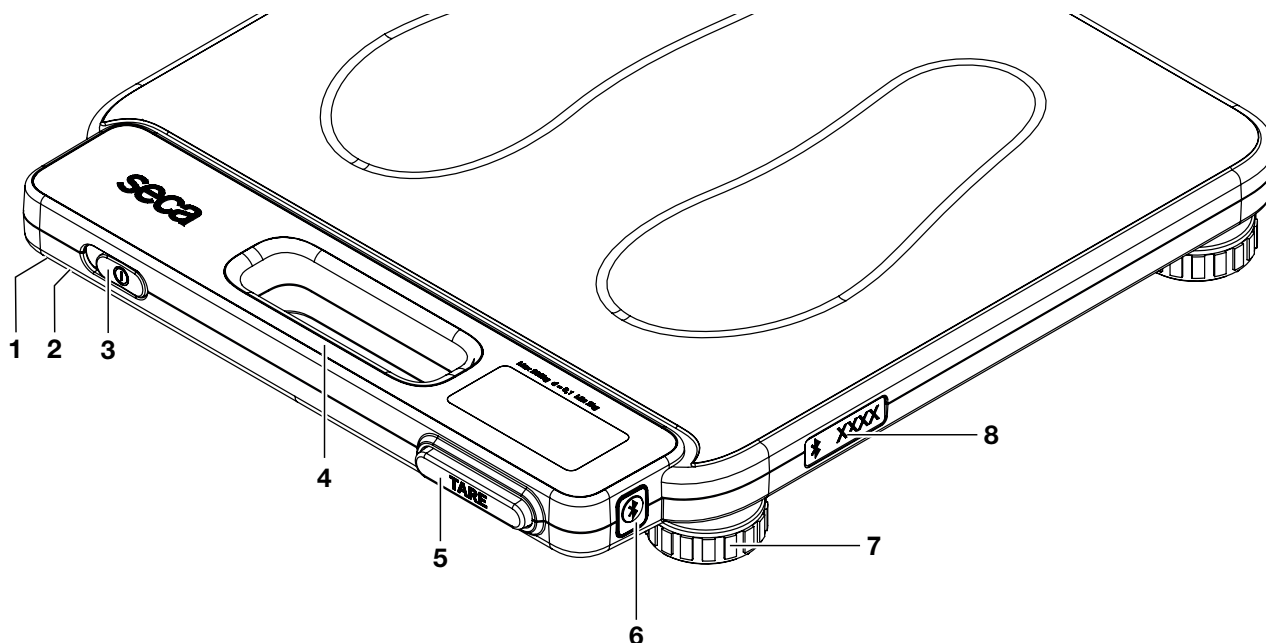

Item	Control	Function
1	Power supply connection (underside of the device)	- For connecting a power supply unit (optional) - For connecting the seca 441 Solar Panel (optional)
2	Battery compartment (underside of the device)	Takes 2 AA-type mignon batteries (AA/1.5 V)
3	On/off switch	Switches the scale on and off  : On  : Off
4	Handle	For transporting the device
5		- Switching on the display (standby) - For setting the weight display to zero - Rotating the display - Start the 2 in 1 function for weighing babies and toddlers
6		For switching Bluetooth on/off (on models with Bluetooth only)
7	Foot screw	4 pcs, for precise alignment
8		Bluetooth identification label (example), on models with Bluetooth only


4.2 Symbols in the display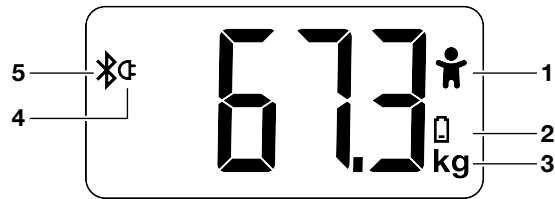

Item	Symbol	Meaning
1		2 in 1 function active
2		- Symbol lit up continuously: Batteries are low - Symbol flashing slowly: Batteries are very low - Symbol flashing quickly, device switches off: Batteries have discharged
3	kg	Weight in kilograms
4		Device is powered by power supply unit
5		Bluetooth active (on models with Bluetooth only)

4.3 Labels


Markings on the device and on the type plate	
Text/symbol	Meaning
	Name and address of manufacturer, date of manufacture
UDI	Unique Device Identification (product identification number as per Directive (EU) 2017/745)
	Article number
	Serial number
ProdID	seca product identification number
	Follow instructions for use
	Medical electrical device, Type BF
IP53	Type of protection to IEC 60529: - Protection from dust in harmful quantities (dust-proof) - Protection from splashing (angle < 60 degrees).
d	Value in units of mass States the difference between two consecutive display values
	Device complies with EU directives 0123 : Notified Body for Medical Devices
	Medical device in accordance with Regulation (EU) 2017/745

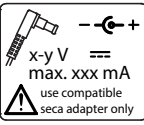Markings on the device and on the type plate	
Text/symbol	Meaning
	Device meets the requirements of the USA and Canada. Certified and tested by a licensing laboratory (NRTL) of TÜV SÜD Product Services GmbH.
	Device complies with United Kingdom directives XXXX : Notified Body for Medical Devices of the United Kingdom
	Importer/representative in the United Kingdom: seca Ltd 40 Barn Street B5 5QB Birmingham United Kingdom
	Bluetooth identification label (example), on models with Bluetooth only
	Symbol of the US Federal Communications Commission (FCC)
FCC ID	Device license number from the US Federal Communications Commission (FCC)
IC ID	Device license number from Industry Canada
	Type plate on the power supply connection socket • Power supply voltage required in V • Maximum current consumption in mA • : Note polarity of device connector • : Operate device with direct current • : Use compatible seca power supply units only
	Do not dispose of device in household waste

Markings on the packaging	
Symbol	Meaning
	Protect from moisture
	Arrows indicate top of product Transport and store in an upright position
	Fragile, do not throw or drop
	Permitted min. and max. temperature for transport and storage
	Permitted min. and max. humidity for transport and storage
	Permitted min. and max. air pressure for transport and storage
	Open packaging here
	Packaging material can be disposed of through recycling programs

5 STARTING UP DEVICE

5.1 Scope of delivery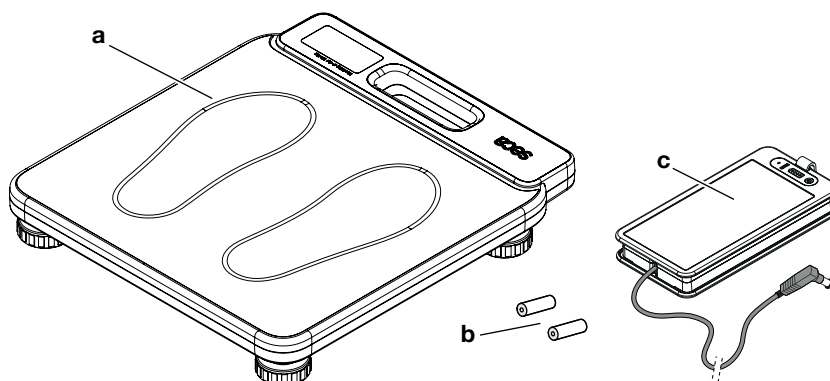


Item	Component	Pcs.
a	Personal scale	1
b	Battery	2
c	seca 441 Solar Panel (variant-dependent)	1
–	Instructions for use	1

5.2 Establishing power supply

Inserting batteries

You need 2 batteries (type AA/1.5 V, included in scope of delivery).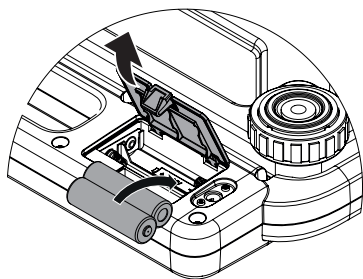

1. Open the cover of the battery compartment.

NOTICE!


Damage to device

Device may overheat and become damaged.

- Ensure the correct polarity of the batteries (markings in the battery holder).

2. Insert the batteries.
3. Close the battery compartment.

Connecting the power pack (optional)

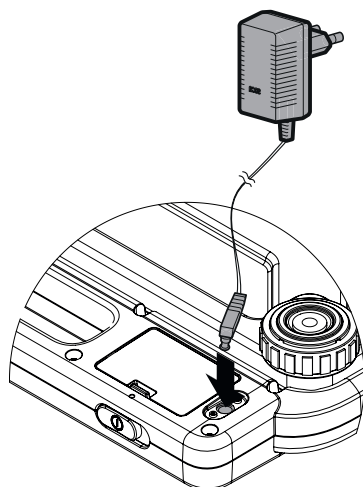The scale can be operated with a power pack (optional accessory).

WARNING!

Personal injury or damage to device as a result of incorrect power supply units
Conventional power supply units may deliver a higher voltage than is indicated on them. The measuring device may overheat, catch fire, melt or short-circuit.

- ▶ Use only original seca power supply units with a controlled 12 V output voltage.

1. Insert the connector required for your power supply into the power supply unit.
2. Insert the device connector of the power supply unit into the connecting socket of the scale.
3. Plug the power supply unit into a power supply socket.

NOTICE!

Faulty measurement

- ▶ Do not pull on the power cable during measurements.

Connecting the seca 441 Solar Panel (optional)

The scale can be operated with a **seca 441 Solar Panel** (→ [Optional accessories and spare parts, page 21](#)). You can find information about this in the separate instructions for use for the **seca 441 Solar Panel**.

5.3 Setting up and aligning the device

CAUTION!

Faulty measurement as a result of devices being set up incorrectly

Soft floors (wooden boards, for example) give under the patient's weight and falsify the measuring result. Incorrect ambient conditions or lack of alignment lead to measurement errors.

- ▶ Select a setup location with a level, stable floor in order to achieve precise measuring results.
- ▶ Use the device only in the intended ambient conditions outlined in the section entitled "Technical data".
- ▶ Align the device before using it and every time it changes location.

WARNING!

Faulty measurement

- ▶ Keep the surface underneath clean.
- ▶ Make sure that there are no objects on this surface that could impair measurement (e.g. stones).


WARNING!

Incorrect measured result

- ▶ Select a location where you can read the display easily.
- ▶ Avoid direct sunlight falling on the display while you are reading the measuring result.

1. Place the device on a firm, level surface.
2. Align the device by turning the foot screws.
3. Ensure that only the feet of the device are in contact with the ground.

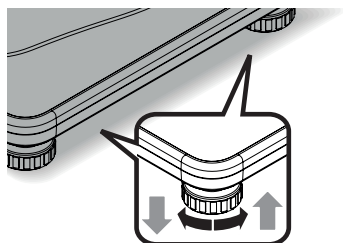6 OPERATION

CAUTION!

Personal injury/faulty measurement

- Before using the device each time, perform a function check as described in the corresponding section of these instructions for use.

NOTE

Heating of the tread area

With high temperatures the tread area adjusts to the ambient temperature. Sensitive patients might find this unpleasant.

6.1 Switching the device on and off

Switching on the device

- ✓ The scale has no load.

- Set the ON/OFF switch to "ON" position.

- ⇒ The scale is operational when **0.0** appears in the display.
- ⇒ If the scale is operated with a power supply unit, the  symbol appears in the display.

NOTE

The display switches off automatically after a short time if there is no load on the scale (when operated with batteries only).

Switching off the device

- Set the ON/OFF switch to "OFF" position.

NOTE

Switch off the device before transporting it so that the **TARE** key is not accidentally pressed during transport, so wasting power.

Switching on the display (standby)

- Briefly press the **TARE** key.

- ⇒ The scale is operational when **0.0** appears in the display.

NOTE

The display switches off automatically after a short time if there is no load on the scale (when operated with batteries only).

Rotating the display

- ✓ Device is switched on

1. Press and hold the **TARE** key (5 seconds).

- ⇒ The **SECA** display flashes.

2. Briefly press the **TARE** key.

- ⇒ The display rotates.
- ⇒ After 5 seconds of inactivity, **0.0** appears and the display remains in this position.

NOTE

If you briefly press the key again during these 5 seconds, the display will rotate back.

6.2 Weighing

Weighing a patient

NOTICE!

Damage to device

The display is sensitive and may be damaged under load

- Make sure that the patient does not step on the display.

- ✓ The scale is switched on.

- ✓ The scale has no load.

1. If the scale switched off automatically after the last measurement, press the **TARE** key.

⇒ The scale is operational when **0.0** appears in the display.

2. Ask the patient to step onto the scale.

⇒ The display flashes until a stable weight is measured.

3. Read off the measuring result.

⇒ With power supply operation: The weight remains visible in the display until a new load is placed on the scale.

In battery mode: The scale switches off after 1 minute.

Weighing babies/toddlers

The **2 in 1** function can be used to determine the weight of babies and toddlers. The child is held in an adult's arms during weighing to do this. Proceed as follows:

- ✓ The scale has no load.

1. Ask the adult to stand on the scale.

2. Read off the measuring result.

3. Briefly press the **TARE** key.

⇒ The  symbol appears in the display (flashing).

⇒ The adult's weight is saved.

⇒ **0.0** appears in the display.

4. Ask the adult to get onto the scale holding the child.

⇒ The  symbol appears continuously.

⇒ The child's weight is displayed (in battery mode: for 1 minute).

WARNING!

Faulty measurement as a result of initial weight changing

If the child is weighed together with a different adult, it will not be possible to determine the child's weight correctly.

- Ensure that the child is always weighed with the adult with whom the initial weight was determined.

- Ensure that the adult's weight does not change - as a result of their taking off a piece of clothing, for example.

5. Read off the measuring result.

6. Ask the adult to get off the scale with the child.


7. You have the following options for deactivating the **2 in 1** function:

- Load the scale.


- Press the **TARE** key.

- Switch off the device.


6.3 Performing serial measurements with adults and children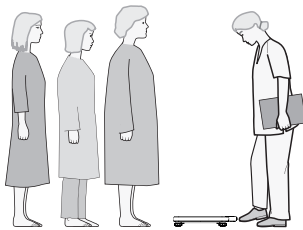

- ✓ The scale is switched on.
- ✓ The scale has no load.
- 1. If the scale switched off automatically after the last measurement, press the **TARE** key.
 - ⇒ The scale is operational when **0.0** appears in the display.
- 2. Ask the first patient to step onto the scale and hold still.
 - ⇒ The display flashes until a stable weight is measured.
 - ⇒ The weight value is displayed continuously (in battery mode: for 1 minute).

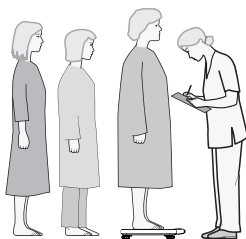
- 3. Read off the measuring result.
- 4. Ask the patient to step off the scale.
 - ⇒ The weight remains visible in the display until a new load is placed on the scale (in battery mode: for 1 minute).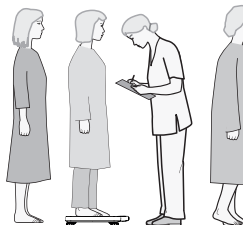

- 5. Repeat steps 2 to 4 for subsequent patients.

NOTE

If you wish to zero the display from time to time, press the **TARE** key.

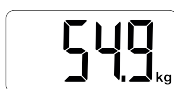6.4 Bluetooth (for model with Bluetooth only)

Switching Bluetooth on/off

Switching on Bluetooth

- ✓ The scale is switched on.
- 1. Briefly press the Bluetooth key.
 - ⇒ The Bluetooth  symbol flashes.
 - ⇒ **On** is displayed.

- ⇒ 0.0 is displayed when there is no load on the scale.
- ⇒ As soon as there is a Bluetooth connection, the Bluetooth symbol is displayed continuously.
- ⇒ The measuring results and battery status are transmitted to the connected device.

Switching off Bluetooth

- ✓ The scale is switched on.

1. Briefly press the Bluetooth key.

- ⇒ OFF is displayed.

- ⇒ The Bluetooth symbol disappears.
- ⇒ 0.0 is displayed when there is no load on the scale.


Initially connecting to a Bluetooth-compatible device (pairing mode)

You can use Bluetooth to connect the device to a smartphone (or other device). You set this connection up in pairing mode.

- ✓ The smartphone supports Bluetooth Low Energy and is in the vicinity.
- ✓ The Bluetooth app is opened on your smartphone.

1. Press and hold the Bluetooth key (approx. 2 seconds).
 - ⇒ Bluetooth symbol flashes quickly.
 - ⇒ Device is ready for connection for approx. 60 seconds.
2. Select the device (e.g. "seca 874 u+ 6349") on the smartphone.
The number of the device is shown on the Bluetooth identification label.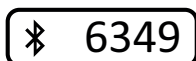

- ⇒ The Bluetooth symbol appears continuously.
- ⇒ The connection between the device and the smartphone is set up.
- ⇒ As soon as the device displays a stable measured value, it transmits the data (weight and battery status) to the smartphone.

NOTE

Data access by unauthorized persons

The measuring results are accessible to everyone who has access to the device via Bluetooth (exception: patient ID or name).

The device can only ever be connected with one mobile device at a time, not with several simultaneously.

Note the instructions on IT security in the seca White Paper entitled "Cyber Security" and in the "Bluetooth Recommendation". The documents can be found as a download in the Support sector at www.seca.com.

NOTE

Pairing mode needs to be activated once only in order to set up the connection to a smartphone. When the connection is set up successfully, the device then automatically connects to the smartphone as soon as it is in range.

7 HYGIENE TREATMENT

WARNING! **Electric shock**

The device is not de-energized when the On/off switch is pressed and/or the display goes out. Use of fluids on the device may cause an electric shock.

- ▶ Follow the steps below to de-energize the device:
 - switch off device (On/off switch).
 - disconnect power supply unit from power supply socket.
 - remove batteries.
- ▶ Ensure that no fluids penetrate the device.

WARNING! **Risk of infection**

- ▶ Subject the device to a hygiene treatment at regular intervals as described in this section.

NOTICE!

Damage to device

Unsuitable cleaning agents and disinfectants may damage the sensitive surfaces of the device and impair its operability.

- ▶ Do not use aggressive or abrasive cleaning agents.
- ▶ Do not use organic solvents (e.g. white spirit or petroleum spirit).

7.1 Cleaning

- ▶ If required, moisten a soft cloth with a mild soap solution and wipe over the device and carry case with it.

7.2 Disinfecting

1. Disinfect the device at regular intervals with a disinfectant suitable for sensitive surfaces and acrylic glass (e.g. 70 % ethanol).
2. Follow the instructions for use for the disinfectant.
3. Disinfect the device as follows:
 - ▶ Moisten a soft cloth with disinfectant and wipe down the device with it.
 - ▶ Comply with the intervals, see table:

Interval	Component
Before and after every measurement with direct skin contact	Weighing platform
As required	• Display • Carry case

7.3 Sterilizing

The device must not be sterilized.

8 FUNCTION CHECK

- Perform a function check before each use.

A complete function check includes:

- A visual inspection for mechanical damage
- A test of the alignment of the device
- A visual and function check of the display elements
- A function check of all the controls shown in the section entitled "Overview"
- A function check of the optional accessories

If you find faults or deviations during the function check, first try to remedy the fault with the aid of the "Troubleshooting" section in this document.

CAUTION!

Personal injury

If you find faults or deviations during the function check which you are unable to remedy with the aid of the "Troubleshooting" section in this document, you must not use the device.

- Have the device repaired by seca Service or an authorized service partner.
- Follow the section entitled "Servicing" in this document.

9 TROUBLESHOOTING

If faults occur when operating the device, first attempt to remedy them yourself using the following table. If the fault persists, contact seca Service or an authorized service partner.

Fault	Cause	Remedy
Display is blank	• Scale is switched off • Scale is on standby mode • Scale is not supplied with power (batteries or power supply unit)	• Check that the scale is switched on • If the scale is on standby mode, press the TARE key • Check that batteries are inserted (correctly) or that a power supply unit is connected
Scale switches itself on, e.g. during transport	TARE key pressed	Set the On/off switch to "OFF" when transporting
The scale displays a weight after transport or a battery replacement although no load has been placed on it	Scale has determined an incorrect zero point	Switch off the scale, then switch it back on again
0.0 flashes before weighing	Scale had load on it before being switched on	• Remove the load from the scale • Switch off the scale, then switch it back on again
---- displayed before weighing	Scale had load on it before being switched on	• Remove the load from the scale • Switch off the scale, then switch it back on again
Single segment is illuminated continuously or not at all	Display faulty	Inform seca Service
Display:  lit continuously	Battery voltage is dropping	Replace the batteries

Fault	Cause	Remedy
Display:  flashing slowly	Battery voltage is very low	Replace the batteries
Display:  flashing quickly (for 3 seconds), device switches off	Batteries discharged	Replace the batteries
Display: StOP	Maximum capacity exceeded	Remove the load from the scale
Display: Er + figure	–	• Switch off the scale, then switch it back on again • If the problem persists, inform seca Service

10 SERVICING

The product needs to be set up carefully and serviced regularly. Depending on how frequently the product is used, seca recommends servicing at intervals of 3 to 5 years.

CAUTION!

Faulty measurements as a result of poor servicing

- Have servicing and repairs carried out exclusively by seca Service or an authorized service partner.
- You can find your local service partner at www.seca.com.

11 TECHNICAL DATA

11.1 General technical data

Dimensions	
• Depth	400 mm
• Width	360 mm
• Height	61 mm
Net weight	Approx. 3.7 kg (8.2 lbs)
Ambient conditions, operation	
• Temperature	0 °C to +45 °C
• Air pressure	700 hPa – 1060 hPa
• Humidity	0 % – 95 % no condensation
Ambient conditions, storage/transport	
• Temperature	-20 °C to +65 °C
• Air pressure	700 hPa – 1060 hPa
• Humidity	0 % – 95 % no condensation
Height of digits	26 mm
Power supply	
• Power supply unit (optional)	
– Supply voltage	12 V
• Batteries	
– Supply voltage	3 V
– Battery type	2 x type AA, 1.5 V, not rechargeable
• seca 441 Solar Panel (optional)	5 V

17-10-06-646-100d_2026-02S

Maximum current consumption	Typically 15 mA
Medical device in accordance with Regulation (EU) 2017/745	Class I with measuring function
EN 60601-1: Medical electrical device, Type BF	
Application part in line with IEC 60601-1	Tread area
Type of protection	IP53
Duty cycle	Continuous duty
Bluetooth interface (depending on model)	Bluetooth Low Energy (The "Bluetooth Recommendation" can be retrieved in the Download area for the device at www.seca.com .)


11.2 Weight measurement

Minimum capacity	2 kg
Maximum capacity (depending on version):	
874 1009 659	200 kg
874 2709 659	200 kg
874 3209 659	200 kg
874 3909 659	250 kg
874 4109 659	250 kg
Accuracy	± 100 g (± 0.15 %)
Graduation	100 g

12 OPTIONAL ACCESSORIES AND SPARE PARTS

Accessory/spare part	Article number
Carry case	12 05 01 231 009
Power supply:	
• Switch-mode power supply unit: 100-240 V~ / 50-60 Hz / 12 V= / min. 0.6 A	68 32 10 265 009 or 68 32 10 270 009
• seca 441 Solar Panel	441 0000 008
Spare parts kit: 4 feet incl. fixing screws	01 10 04 016 009

12.1 Changing feet

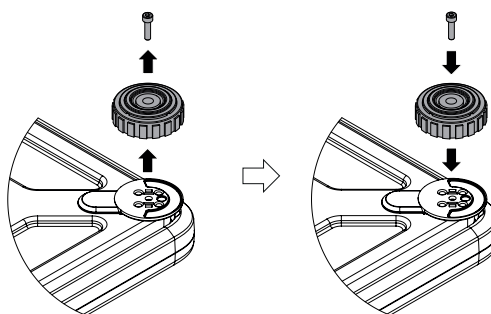13 DISPOSAL

13.1 Disposing of the device

Do not dispose of the device in your household waste. The device must be properly disposed of as electronic scrap. Follow your respective national regulations. For more information, please contact seca Service or an authorized service partner.

13.2 Disposing of batteries and rechargeable batteries

Do not dispose of used batteries and rechargeable batteries in household waste, regardless of whether they contain harmful substances or not. As a consumer, you are legally obliged to dispose of batteries and rechargeable batteries via local authority collection points or trade collection points. Only dispose of batteries and rechargeable batteries once they are completely discharged.

14 WARRANTY

There is a two-year warranty period from delivery for defects attributable to poor materials or workmanship. All movable parts, e.g. batteries, cables, power supply units, rechargeable batteries etc. are exempt. Defects which come under the warranty will be repaired for the customer free of charge against proof of purchase. Additional claims cannot be considered. Costs of transport to and from seca are the responsibility of the customer if the device is located somewhere other than the customer's headquarters. In the event of transport damage, claims under warranty can only be made if the complete original packaging was used for transport and the device was secured and fastened in it according to its originally packaged condition. You should therefore keep all packaging parts.

The warranty will be voided if the device is opened by persons not expressly authorized by seca to do so.

In the event of a warranty issue, please contact your local seca office or the dealer from whom you ordered the product.

15 DECLARATION OF CONFORMITY

seca gmbh & co. kg hereby declares that the product complies with the terms of the applicable European directives and regulations. The unabridged declaration of conformity can be found at www.seca.com.

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