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

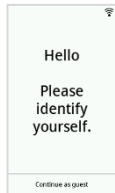
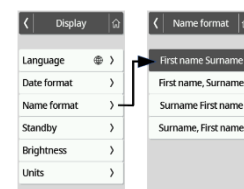
GB The following supplemental chapters include new and updated functions that vary from the descriptions in the currently published Instructions for Use.

6.1 Activating the multifunctional display

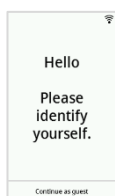New screensaver for devices connected to an EMR system:

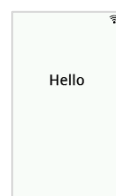
New screensaver for devices connected to the **seca analytics 125** software and for stand-alone devices:


6.6 Measuring bioimpedance

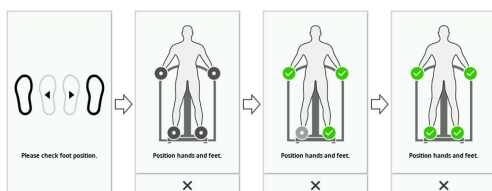New display showing the patient's electrode contact test:

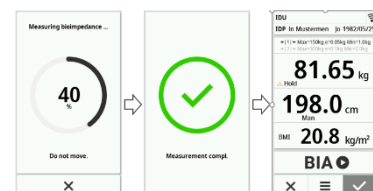
New display showing bioimpedance measurement progress:


6.7 Completing the measurement procedure

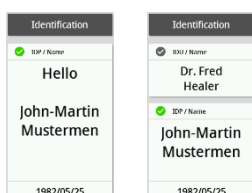New display to check that the scanned IDs are correct:

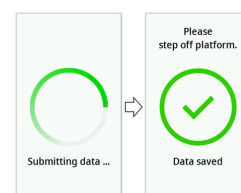New display confirming measurement results are being sent and that the data have been successfully transferred to the EMR system or **seca analytics 125**:


7.3 Adjusting display settings

Setting name format

Device mode	Function available
Basic	–
Advanced	•
Expert	•
Service	•

The format in which the patient's and user's name is displayed can be set.

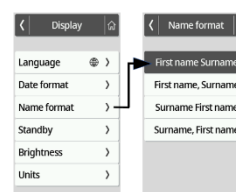- Press the key.
The **Settings** menu is displayed.
- Press the or key until the **Display** menu item is displayed.
- In the **Display** menu, select the **Name format** item.
- Press the desired date format. The setting is active.
- To exit the menu, press the key.

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