# seca **755**

# Mechanical column scales with BMI display



- Quick and easy ascertainment of BMI.
- Simultaneous evaluation of nutritional condition.
- Clearly designed dial and low platform.



#### seca 755:

## The intelligent all-round solution.

his dial column scale can do something other mechanical scales can't: determine the Body Mass Index (BMI) at the same time as weighing. The BMI is recognized by the World Health Organization as being the indicator of one's individual nutritional condition. When fitted with the measuring rod seca 224, the dial scale becomes an all-in-one mechanical weighing and measuring station. The elegant all-round solution is especially suitable for use in the general medical sector in doctors' surgeries and hospitals. At the same time, it is a comfortable instrument for health-care workers all around the world. When installed in pharmacies and fitness studios, it provides an exclusive additional service for customers.



#### Quick and easy ascertainment and evaluation of BMI.

Underweight, normal weight, or overweight? For the first time, the seca 755 answers these questions mechanically: when the patient mounts the scale, a "BMI disc" is set in motion just below the very precise weight displayed in the large dial. The disc is visible in a small window next to which a height scale is printed. The BMI can now be quite easily read off from this height scale. The colored sectors of the BMI disc provide information on how the ascertained BMI is to be interpreted, on whether the patient is underweight, overweight or of normal weight. If there is uncertainty in regards to the patient's height, this can be determined precisely in just one step with the optional integrable telescopic measuring rod seca 224. It is hardly possible to be quicker and more precise.

#### Clearly designed dial and low platform.

When designing this elegant scale, special attention was paid to the ergonomics. The dial which is tilted slightly towards the observer is particularly large and clearly laid out, making it easier to read. The seca 755 is easy to mount as the platform is very low. The tread area is so large that even tall and heavyweight patients have sufficient space. The tread area is also covered with a nonslip rubber mat for safe footing.

#### Mobile with integrated transport castors.

Thanks to the integrated transport castors, the seca 755 is easy to move around and has no problems with uneven surfaces due to its adjustable levelling feet.



The separately available measuring rod seca 224 permits simultaneous weighing and measuring.





It is easy to determine BMI with the colorcoded dial.

# seca **755**

### Technical data

- Capacity: 350 lbs / 160 kg, 160 kg
- Graduation: 1 lbs / 500 g, 500 g
- Dimensions (WxHxD): 14.2 x 39.4 x 24.7" / 360 x 1,000 x 628 mm
- Dimensions/platform (WxHxD): 13.2 x 3.1 x 13.6" / 335 x 80 x 345 mm
- Weight: 39.9 lbs / 18.1 kg
- Functions: BMI, reset-to-zero function
- Optional: measuring rod seca 224

Toll free +1 800 542 7322 • Fax +1 888 705 7397 • info.us@seca.com • www.seca.com North America West • seca corp. • 13601 Benson Avenue • Chino, CA 91710 USA